


The Canfield Patient Positioner

Parts List:

- Allen Key
- 12 Screws
- 2 Cinch Beads
- 3 Equal Lengths of Cord
- 2 Side Plates
- 2 Narrow Flat Bars
- 1 Wide Flat Bar

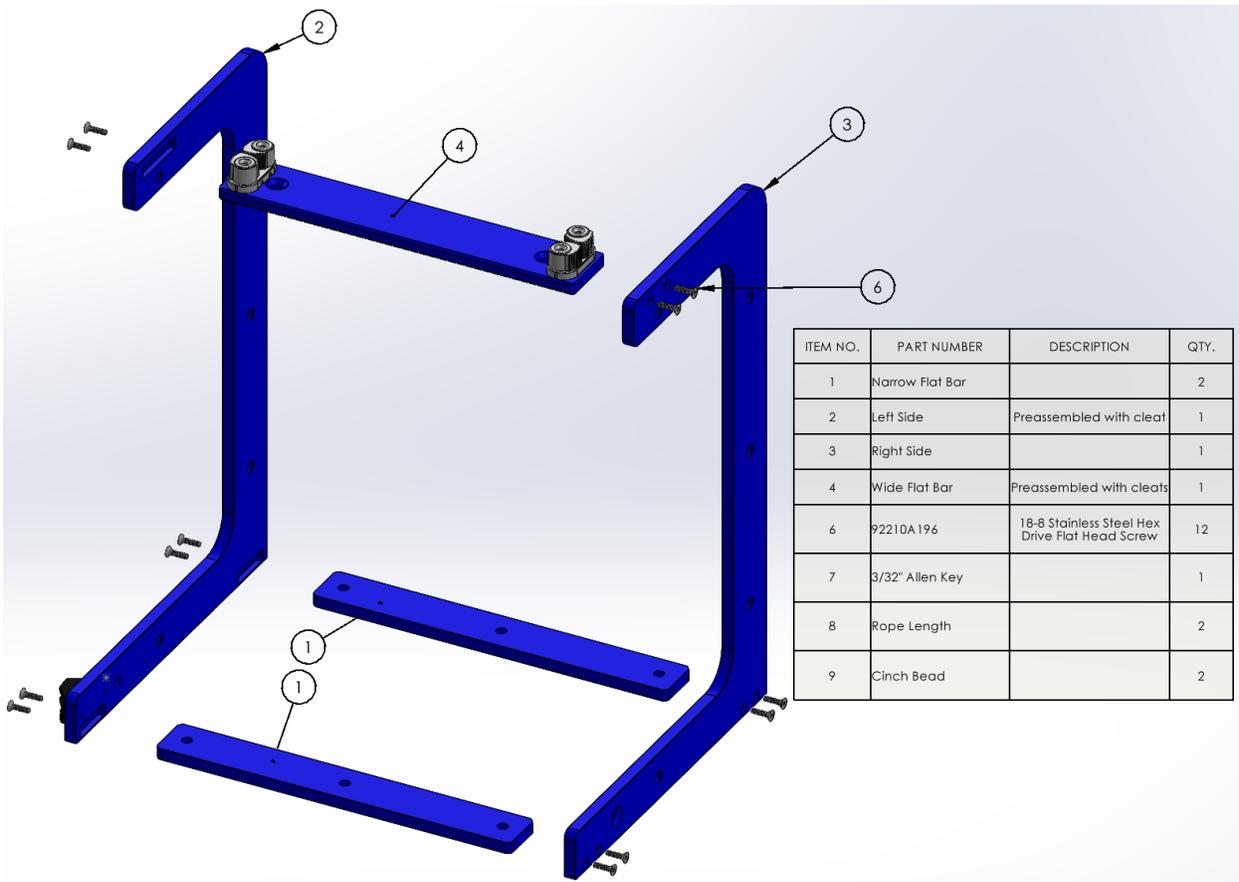

Assembly Instructions:

1. Unpack all components



2. Insert the narrow flat bar into one of the bottom slots on the side plate with the black cleat attached
3. Start the two screws by hand but do not tighten
4. Repeat steps 2 and 3 for the other narrow flat bar
5. Insert the wide flat bar into the upper slot of the same side plate with the logo and the cleats on top
6. Start the two screws by hand but do not tighten
7. Align the second side plate and insert all three flat bars in their corresponding slots
8. Start the remaining six screws by hand but do not tighten
9. Progressively begin tightening all twelve screws until the screws heads are flush
 - a. If the screws are overly tight, back them off and retighten gently

b. The assembly should look like the photo below



10. Weave the cord through the top right hole in the wide flat bar

11. Bring the end of the cord up through the left hole and lock both ends in their respective cleats

12. When using the cinch bead, feed the center of the cord through the bead



13. When the cinch bead is not in use, place one bead on the cord and rest it on the knot



14. The second cord should be fed from the right side so the knot remains on the right side

15. The end will now weave through the left side plate and lock into the lower cleat

