


START ASSEMBLY HERE

Determining your temple attachment

Test fit the clip (Bag 3) on your temple, if it fits the temple see Figure 1, if not see Figure 2 on the right.

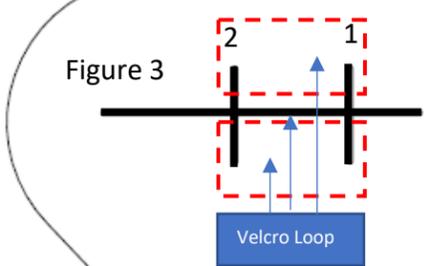

Setting the Clip Location:

The black markings on the left and right of these instructions are a template to set the location of the Velcro Loop (Bag 2)

For Standard Field Telescopes:


Pull off the paper backing on the Velcro loop in (Bag 2) and place the Velcro along line number 1 as shown in Figure 3 with the blue arrow.

For Expanded Field Telescopes:

Pull off the paper backing on the Velcro loop and place it along line number 2 as shown in Fig.3

To adjust height of shield place Velcro above, in middle or below the center line (blue arrows)

Figure 1

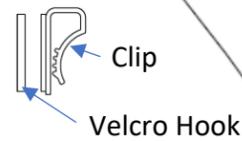

Remove the paper backing from the clip, then remove the paper backing from the Velcro hook (Bag 4) and attach it to the clip as seen in Figure 1.

Place the clip on the temple by pushing the clip down on the top of the temple (Figure 12)

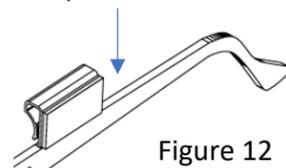


Figure 12

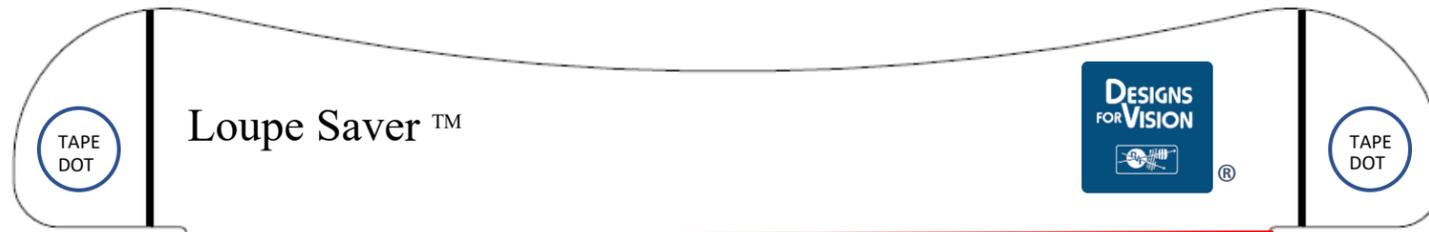


Figure 4

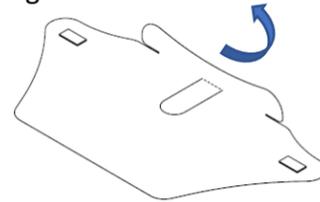


Figure 5

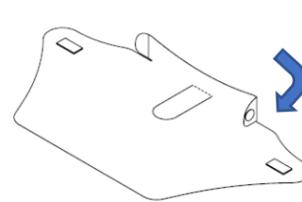


Figure 6

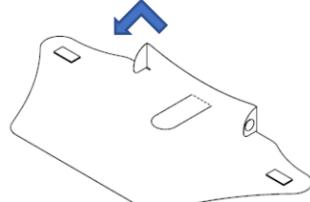


Figure 7

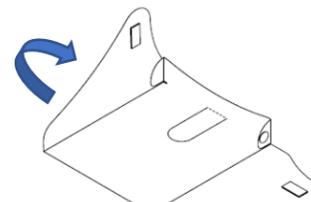


Figure 10

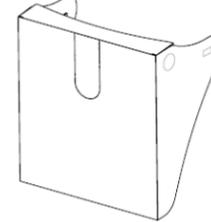


Figure 9


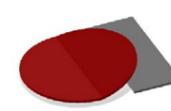


Figure 8

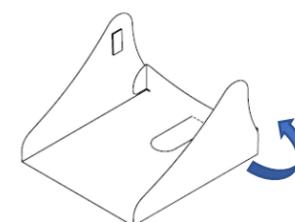

If using a LED light, **PUSH** on this panel to remove, if desired

Assembling your Loupe Saver™ Face Shield

- Place the face shield on top of these instructions, they are an assembly template.
- Place 1 tape dot (Bag #1) in each blue circle on the top of the template
- Once the dots are in place, flip the shield over and put it back on the template.
- Set your Velcro Loop location, see left side of template.
- Once they are in position, start bending your shield by following these steps.
- The first bend is along the red line on the top, see Figure 4, The next two bends are indicated by the two black lines on each side of the top shown in Figures 5 and 6
- Next, fold the left side along the blue line, then the right side along the blue line shown in Figures 7 and 8
- Once the two sides are folded, remove the red paper backing on the glue dots Figure 9, (removing backing may require a sharp edge to get started) and attach the top flap to the side flaps to complete the assembly

The complete assembly is shown in Figure 10
Once assembled, attach the velcro loops on the shield to the clips or Velcro hooks on your frame. To adjust the shield move the clips/Velcro hooks forward or back to accommodate your light and loupes

Visit: www.DesignsForVision.com for Disinfection Guidelines

PARTS LIST

- Bag 1: 24 dots
- Bag 2: 24 Velcro Loops
- Bag 3: 6 Temple Clips
- Bag 4: 6 Velcro Hooks

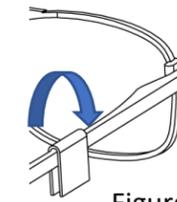


Figure 2

Thin Temple Frames

Wrap 1 Velcro hook (Bag 4) around each temple as shown in Figure 2.

Once the shield is fully assembled, the Velcro Loops will attach to the hooks on the temples as seen in Figure 11


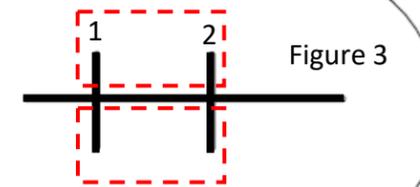


Figure 3

Figure 11

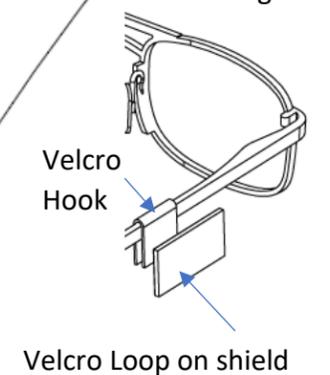

Velcro Hook
Velcro Loop on shield