
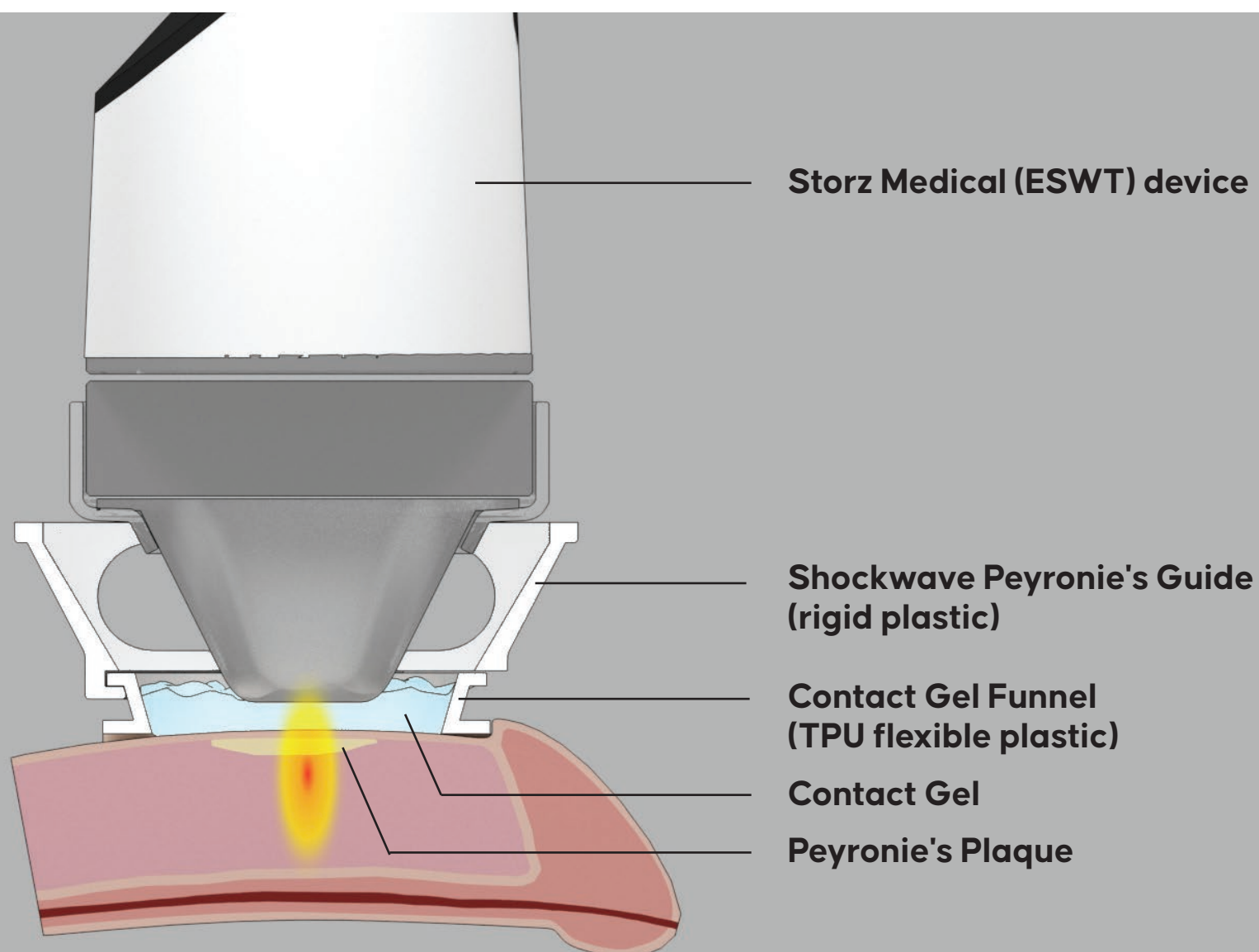
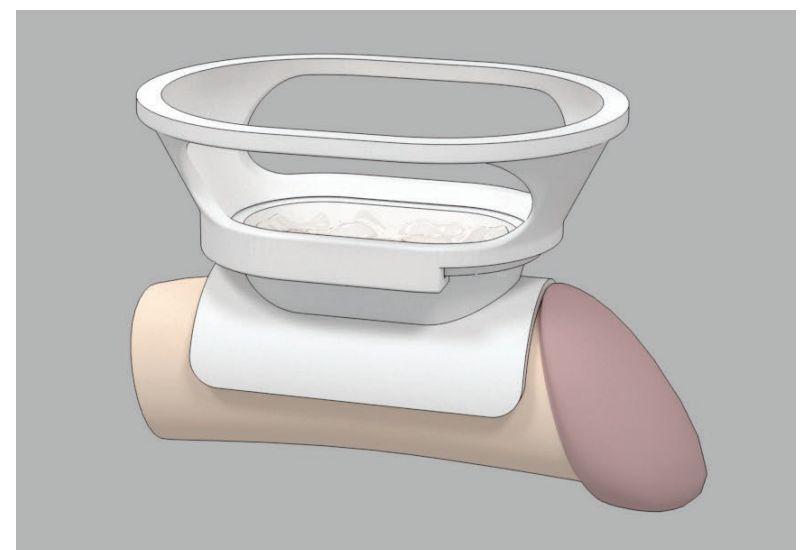


Peyronie's Shockwave Guide


- **Stops contact gel from dispersing**
- **Greater therapy accuracy**
- **Raises the shockwave focal point to better target plaque**

Storz Medical Focused Extracorporeal Shockwave Therapy (ESWT) device is a common therapy for dispersal of Peyronie's plaque. As you are aware the therapeutic effectiveness of the ESWT device with the long Stand-off is 0 - 30mm with a focal point of approximately 15mm.


The **Peyronie's Shockwave Guide** was developed to raise the Stand-off between 5mm - 10mm from the penis, depending on the model of **Contact Gel Funnel** used, to raise the focal point of the shockwave to better target the plaque. The device also keeps the contact gel from dispersing and increases accuracy by keeping the ESWT device constrained to the precise plaque area.

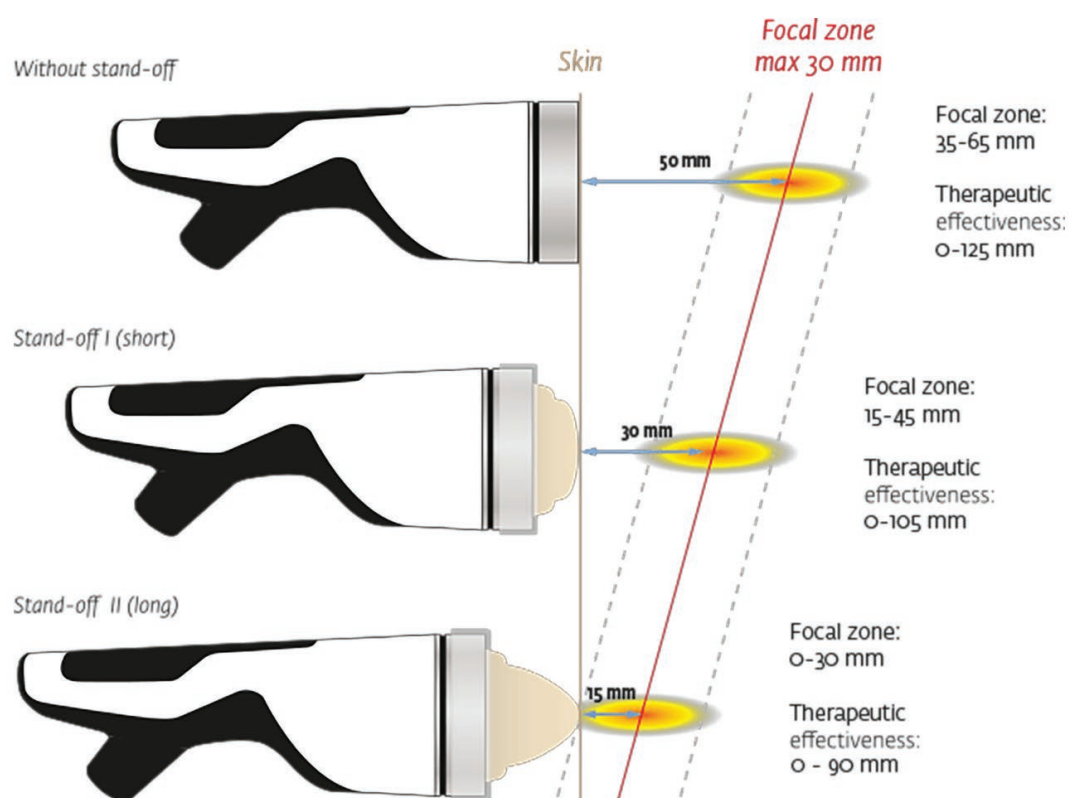


Instructions for Use

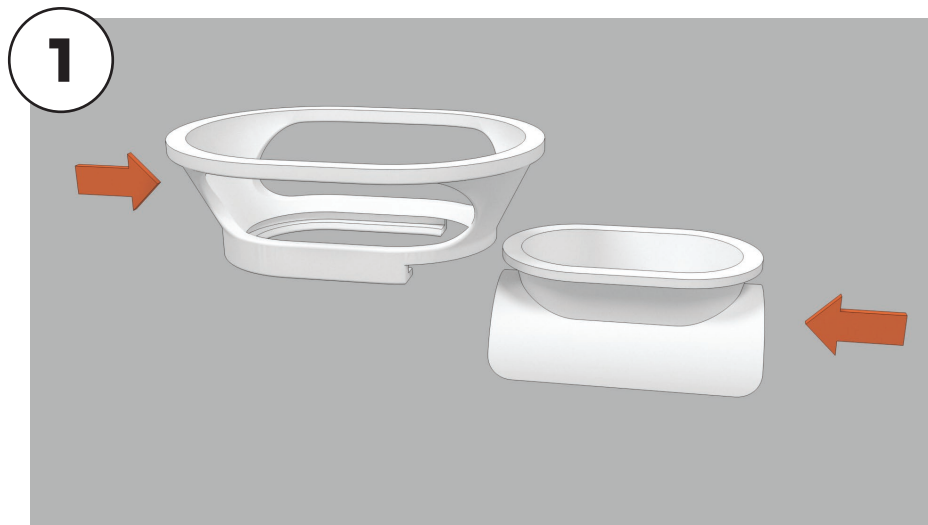


Assemble **Peyronie's Shockwave Guide** and **Contact Gel Funnel**.

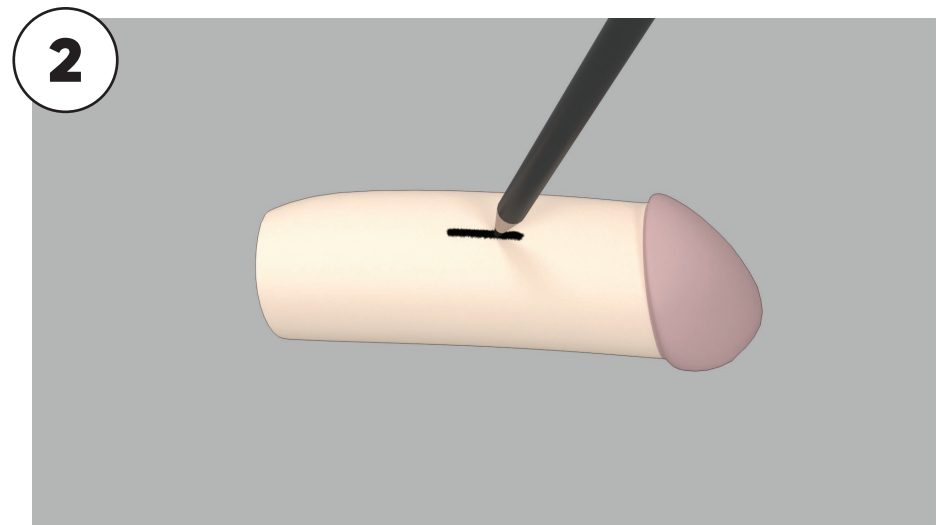


Mark the location of the plaque using an eyeliner pencil or similar.

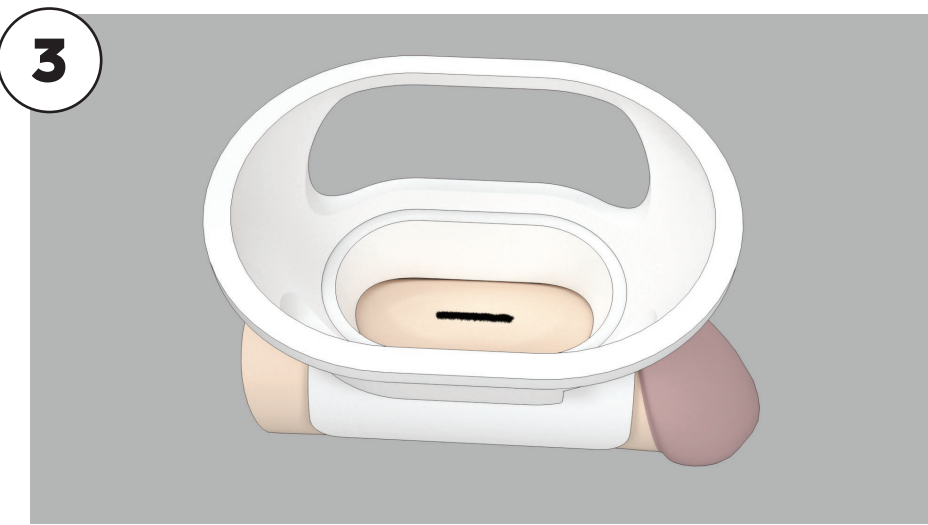


Place the assembly over the plaque location with the mark centred in both directions. Hold this in place manually during the procedure.

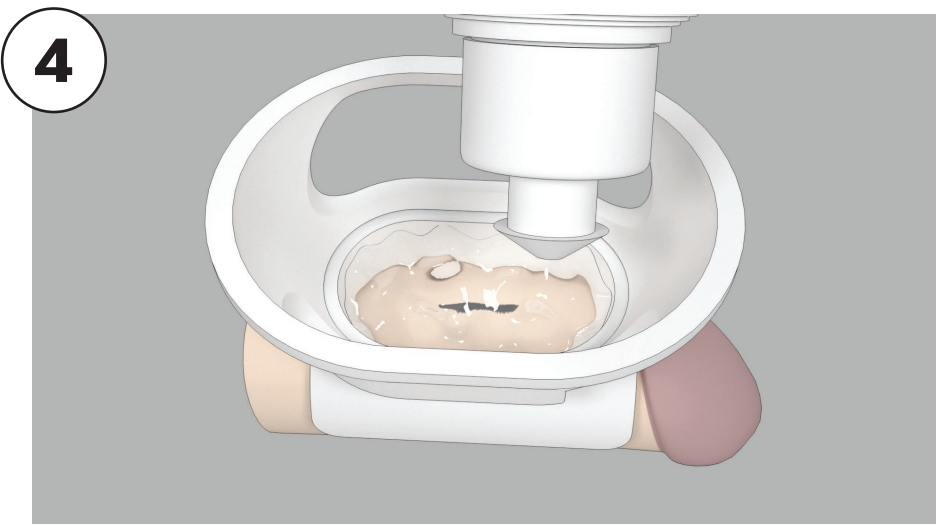


Fill the funnel with the appropriate amount of contact gel.

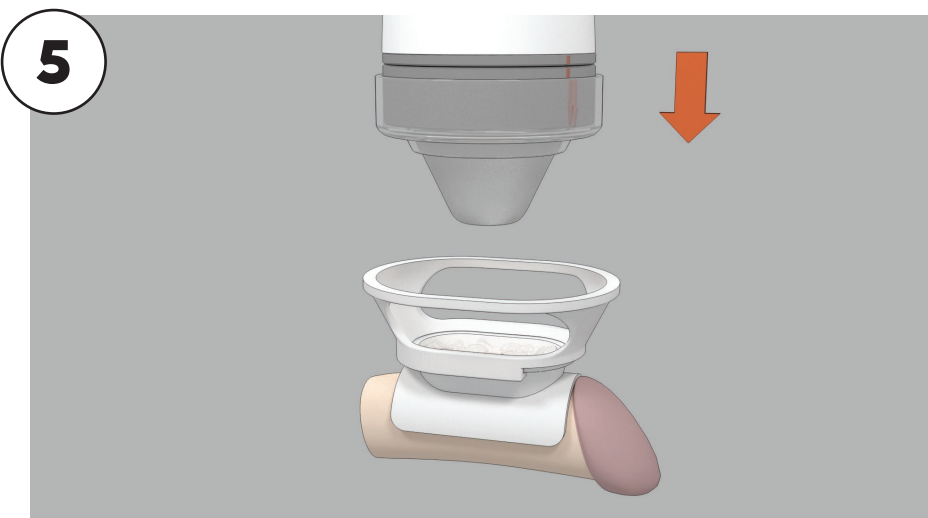

Insert the Storz Medical ESWT device into the **Peyronie's Shockwave Guide**.

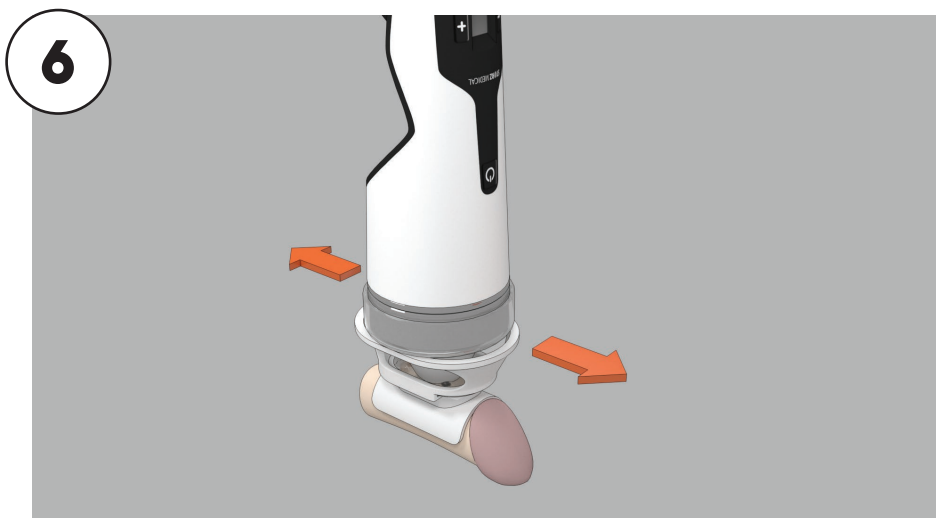

Move the Storz Medical ESWT device periodically to treat the area of the plaque.

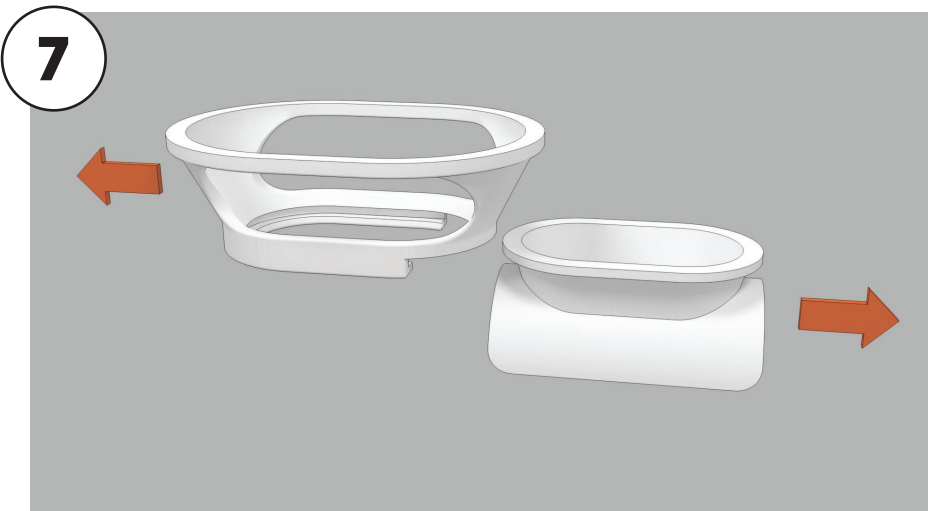

Disassemble the **Peyronie's Shockwave Guide** and **Contact Gel Funnel** for cleaning and sterilising.

For more information and to place an order please visit :
www.shockfocus.com