

# SMARTPALATE

*Software Installation Guide*



# WELCOME!



Congratulations on becoming a part of the CompleteSpeech family! The SmartPalate System Orientation is designed to prepare you to use your SmartPalate System effectively for speech therapy or accent training.

*Thank you,*

# SMARTPALATE

## Software Installation Guide



1. Uninstall your current version of the SmartPalate software, if previously installed.
2. Click [HERE](https://s3.amazonaws.com/completespeechsite/SmartPalate+Software/SmartPalate+Software.zip) to download the installer, or copy this link to download:  
<https://s3.amazonaws.com/completespeechsite/SmartPalate+Software/SmartPalate+Software.zip>  
Be sure to save the file to your computer.
3. Find the folder "SmartPalate+Software.zip" that you downloaded and saved on your computer. This will usually be in the Downloads folder.
4. Unzip the folder "SmartPalate+Software.zip" by right clicking and selecting "Extract All."
5. Within the extracted folder, find and open the folder "SmartPalate Software."
6. Double click "setup.exe" to run the installer.
7. Follow the on-screen prompts and finish the installation.
  - Note: the installation confirmation will not display automatically. Once the installer has finished, a blinking orange icon will appear in the task bar, usually at the bottom of your screen.
  - Click the this icon to pull up the final screen, then click OK.