


Collection & Shipping Guidelines for Blood Samples

START HERE

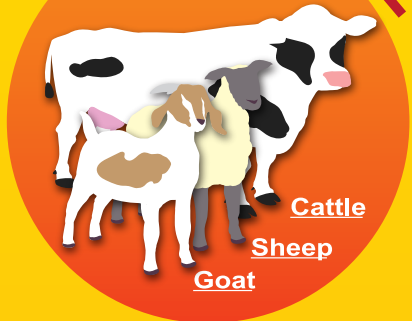

Red Top Tube or SST
(Whole blood or Serum)
NO ADDITIVES
DO NOT USE Purple Top Tubes


Label tubes with
Tube # & Animal ID



IF NOT SENDING RIGHT AWAY
1-2 Days Max in Fridge

1ml/cc of
Blood or Serum


Let Blood
Samples Sit for at
least 1 HOUR
Before Putting In
Fridge

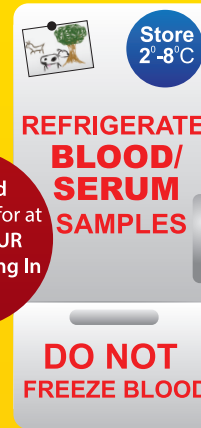

WHEN READY TO SHIP

Bundle Samples in
Max of 10 per Set

Cover Samples with
absorbent paper

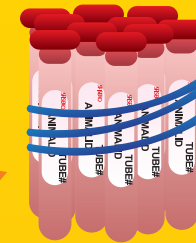

IF SENDING RIGHT AWAY

SHIPPING

- DO NOT send samples in an envelope. Samples must be sent in a box.
- NO Frozen Water or Dry Ice.
- Friday is the ideal time to ship samples.
- 3-5 day shipping for all samples except (BioPRYN) pregnancy. Pregnancy can be sent at room temperature and standard shipping.
- Email lab to provide tracking # & # of tests.
- BE AWARE OF TEST DAYS to avoid delays.

PAYMENT

- We accept Debit/Credit Cards, CHECKS, MONEY ORDERS and CASH. We do not advise sending cash by mail. Checks & money orders should be made to 'UBRL'

Strongly recommend to avoid USPS if possible.
UPS Ground and FedEx Ground is ideal.

3-5 Day shipping for
CL, CAE/OPPV, Johne's & Q-Fever.
Pregnancy (BioPRYN) can be sent standard shipping.




Frozen Ice Pack Required For
CL, CAE/OPPV, Johne's
& Q-Fever



If Sending
for Disease Testing

**NO Frozen Water
or Dry Ice**

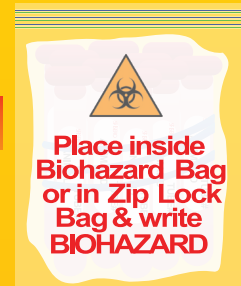

bioPRYN®
Blood Pregnancy
(No ice pack required
if only sending
pregnancy samples)

IF ONLY Sending Preg

IMPORTANT:
'EXEMPT ANIMAL SPECIMEN'
Must be written on box

Wrap in
Bubble Wrap
to protect
tubes.

DO NOT SHIP IN AN ENVELOPE!
(We receive broken tubes often from samples being sent in envelopes)

Place All Documents, Payment & ice pack inside its own zip lock bag to avoid being damaged



Invoice link will be emailed once samples arrive in lab

U UNIVERSAL
B BIOMEDICAL
R RESEARCH
L LABORATORY
Livestock Diagnostics



UBRL
1300 N. Fresno St.
Suite#220
Fresno, CA 93703

(559)498-0820

UBRL.ORG Lab@UBRL.ORG