


# D\_Core 2

DC-200A

## User's Manual

Thank you very much for purchasing D\_Core 2.

With this product, you can enjoy various types of massages aimed at recovery from fatigue and alleviation of fatigue. We hope that you will use this product as a good partner in your daily health management for many years.

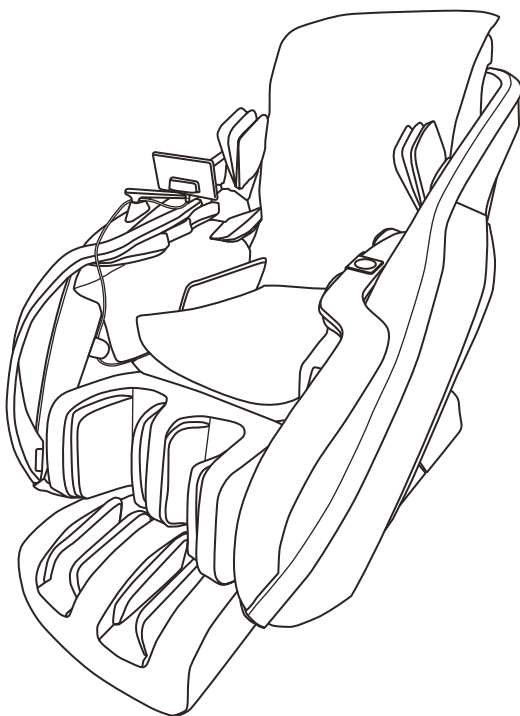

## Contents

■ Safety Precautions	1
■ Part Names and Functions	8
■ Before use	11
■ Chair Adjustments	15
■ Auto Programs	17
■ Manual Programs	23
■ Sofa Mode	26
■ Carry Mode	28
■ Language Selection	31
■ Bluetooth Speakers	33
■ USB Port	34
■ Troubleshooting	35
■ Inspection for Your Safety	37
■ FAQ	38
■ Maintenance and Storage Procedures	39
■ After-sales Service	40
■ Specifications	41
■ Distributor	Back cover



- ◎ Before use, thoroughly read this “Instruction Manual” for proper use.
- ◎ Keep this “Instruction Manual” close at hand for easy access at any time.
- ◎ Since this product is designed for home use, when it is used for purposes other than home use (business application, etc.), consult your dealer or distributor.







# Safety Precautions

Before use, thoroughly read these safety precautions for proper use. This Instruction Manual describes important items to prevent harm to users and others as well as property damage and to ensure safe and proper use of the product. Read the main text after thoroughly understanding the following (indications and symbols), and observe the instructions described. For part names, see pages 8 and 9.

## Indications




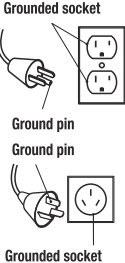
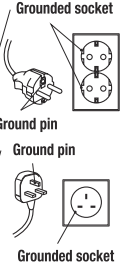
 <b>Warning</b>	Indicates a potentially hazardous situation that could result in death or serious injury if the indication is disregarded and handled incorrectly.
 <b>Caution</b>	Indicates a potentially hazardous situation that may result in minor or moderate injury or in property damage if the indication is disregarded and handled incorrectly.

## Symbols and their meanings

	The  symbol indicates prohibited actions. (The figure on the left-hand side indicates disassembly is prohibited.)
	The  symbol indicates compulsory actions and instructions. (The figure on the left-hand side instructs to unplug the power plug.)

Even after reading the instructions, keep this Instruction Manual in a safe place easily accessible to users at any time.



## Points to note on use location

 <b>Compulsory</b>	<ul style="list-style-type: none"> <li>● Use the product only after placing it in an area where the floor is horizontal. → In an unstable area, the main unit may overturn, causing an accident.</li> </ul>
 <b>Prohibited</b>	<ul style="list-style-type: none"> <li>● Do not use the product in a bathroom, sauna, or other high-humidity areas. → May cause an electric shock or malfunction.</li> </ul>
 <b>Ground</b>	<ul style="list-style-type: none"> <li>● Make sure to ground this product. → In the event of a malfunction or damage to the product, the risk of an electric shock can be reduced.</li> <li>● When in doubt about whether grounding is appropriate, contact a licensed electrician or serviceperson.</li> <li>● This product is used with a commonly-used AC circuit, and comes with a ground terminal as shown in the illustration on the right-hand side. If the shape of your wall socket does not match the plug, do not modify it on your own but consult a licensed electrician.</li> <li>● Do not use an adapter.</li> </ul> <div style="display: flex; justify-content: space-around; align-items: flex-end;"> <div style="text-align: center;">  </div> <div style="text-align: center;">  </div> </div>

## Points to note before use






### Warning

 <b>Prohibited</b>	<ul style="list-style-type: none"> <li>● <b>The following persons should not use the product.</b> <ul style="list-style-type: none"> <li>⊙ Persons who are prohibited from massages by doctors due to: thrombosis (embolism), severe aneurysm, acute varicosity, various dermatitis ailments and skin infections [including inflammation of subcutaneous tissue], etc.</li> <li>⊙ Persons suspected of having acute cervical sprain (whiplash)</li> <li>⊙ Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.</li> </ul> </li> <li>● <b>Do not use the product with power sources other than 120V AC.</b> <ul style="list-style-type: none"> <li>→ May cause a fire or electric shock.</li> </ul> </li> <li>● <b>Do not use the product with a transformer.</b> <ul style="list-style-type: none"> <li>→ May cause a malfunction or electric shock.</li> </ul> </li> <li>● <b>Do not use the product if the power cord or power plug is damaged or its insertion into a wall socket is loose.</b> <ul style="list-style-type: none"> <li>→ May cause an electric shock, short circuit, or ignition.</li> </ul> </li> </ul>
 <b>Compulsory</b>	<ul style="list-style-type: none"> <li>● <b>The following persons should consult a doctor before use.</b> <ul style="list-style-type: none"> <li>⊙ Persons who have heart trouble</li> <li>⊙ Persons who have wounds on parts of the body to be treated</li> <li>⊙ Persons who have sensory impairment disorders resulting from advanced peripheral circulatory disturbances such as diabetes, etc.</li> <li>⊙ Persons who have osteoporosis, and persons who have acute pain due to spinal fracture, sprain, muscle strain, etc.</li> <li>⊙ Persons who have body temperatures above 38 degrees C (duration of fever) for the duration of such time that acute inflammatory symptoms [malaise, chills, fluctuations in blood pressure, etc.] are strong; and when weak</li> <li>⊙ Persons who are unstable during the early stages of pregnancy or immediately after childbirth</li> <li>⊙ Persons who use pacemakers and other body-implantable electrical medical devices that are susceptible to electromagnetic interference</li> <li>⊙ Persons who have backbone (spine) abnormalities or lateral curvature of the spine</li> <li>⊙ Persons who intend to use the product on parts of the body having been medically treated previously or affected areas currently under treatment</li> <li>⊙ Persons who have malignant tumors</li> <li>⊙ Persons who need rest</li> <li>⊙ Persons who have visceral diseases and other acute symptoms</li> <li>⊙ Persons who are under medical treatment, and persons who feel physical abnormalities in particular</li> <li>⊙ Persons who have deformed joints due to rheumatism, gout, etc.</li> <li>⊙ Persons who experience dizziness, lightheadedness, and tinnitus symptoms</li> <li>⊙ Persons who have inflammation caused by sprain, muscle strain, etc.</li> <li>⊙ Persons who have or are suspected of having tenosynovitis</li> <li>⊙ Persons who are found to have a loss of temperature sensibility [heater-equipped models only]</li> </ul> </li> <li>● <b>Before starting a massage, make sure to check that the fabric of the main unit is not torn. In addition, make sure to check that the fabric of other parts is not torn. However small a tear may be, immediately stop using the product, unplug the power plug, and contact your distributor (printed on the back cover). Never remove the fabric of the main unit or use it in a torn state as this is very dangerous. → Using the product with the fabric in a torn state may result in an accident, injury, or electric shock.</b></li> <li>● <b>Even without a specific disease, elderly persons and persons who may have weak bones should consult a doctor before use.</b></li> <li>● <b>If there is no effect from use, consult a doctor or expert.</b></li> <li>● <b>Prevent children and pets from playing with this equipment.</b></li> <li>● <b>Since there is a danger that you may suffer low temperature burns, strictly observe the operating time.</b></li> <li>● <b>Burns will result from improper use. ● Read instructions before using.</b></li> <li>● <b>Do not wet - Do not use pins. ● Never remove cover.</b></li> <li>● <b>Temperatures sufficiently high to cause burns may occur regardless of setting. Do not use on an infant or invalid or on a sleeping or unconscious person. Skin in contact with the heated area of the appliance frequently to reduce the risk of blistering.</b></li> </ul>

# Safety Precautions



## Points to note before use

 <b>Caution</b>	
 <b>Prohibited</b>	<ul style="list-style-type: none"> <li>● <b>Do not use the product by methods other than those in this Instruction Manual.</b> In addition, do not use this equipment in combination with other therapeutic devices, an electric blanket, etc. → May adversely affect the body or cause injury.</li> <li>● <b>Do not move the product with any person on it.</b> → The main unit may overturn, causing an accident or injury.</li> <li>● <b>Do not sit in the chair while it is reclined.</b> → An impact may be applied to the footrest, causing a malfunction. In addition, the main unit may overturn, causing an accident or injury.</li> <li>● <b>Do not use this equipment with any other item than the included items.</b></li> <li>● <b>Do Not Crush – Avoid sharp folds.</b></li> </ul>
 <b>Compulsory</b>	<ul style="list-style-type: none"> <li>● <b>Before inserting the power plug into a wall socket, check that the main power switch is “OFF.”</b> → May cause a malfunction or electric shock.</li> <li>● <b>Make sure that the power plug and connector are completely inserted.</b> → May cause an electric shock, short circuit, or ignition.</li> <li>● <b>Before use, turn on all the switches in order by following this Instruction Manual, and check that the equipment functions normally.</b></li> <li>● <b>Start massage from a weak stimulation.</b> → If massage is started with a strong stimulation, injury may result. In particular, elderly persons and persons who may have weak bones should massage themselves with due caution.</li> <li>● <b>For your safety, check each mechanism position, and if any mechanism is not in the stowed position, turn on the main power switch, check that it is stowed properly, and then slowly sit down.</b></li> <li>● <b>Avoid such actions as stepping on, standing on, or sitting on the footrest.</b> → May cause an accident, injury, or malfunction due to overturning.</li> <li>● <b>When using the product again after not using it for a long time, make sure to thoroughly read this Instruction Manual again, and then check that the equipment functions normally before use.</b></li> </ul>



## Points to note during use

### Warning

 <b>Prohibited</b>	<ul style="list-style-type: none"> <li>● Do not wind the power cord around the main unit, or allow the main unit or footrest to press down on it. In addition, do not damage, break, rework, forcibly bend, pull, or twist the power cord. → The cord may be damaged, causing a fire or electric shock.</li> <li>● Do not spill water or the like on the main unit. → May cause an electric shock, short circuit, or ignition.</li> <li>● Do not sit in the chair when your body is wet, and do not operate with wet hands. → May cause an electric shock or malfunction.</li> <li>● Do not allow children, persons who cannot express their intentions for themselves, or persons with disabilities to use the product. In addition, do not allow children and pets to play around the main unit (behind the backrest, under the footrest, etc.) or get/sit on the main unit. → May cause an accident or injury.</li> <li>● Do not allow two or more persons to sit in the chair at the same time. In particular, do not use the chair while holding any small child (such as an infant). → May cause an accident or injury.</li> <li>● If the fabric is torn, do not use the product. Unplug the power plug from the wall socket. → May cause an accident, injury, or electric shock. For repair, consult your distributor (printed on the back cover).</li> <li>● Do not directly touch the LED parts while they are illuminated and immediately after turning off. → May cause an electric shock or malfunction.</li> <li>● Do not use the product while wearing a wristwatch, necklace, or other accessories. Never use pins or other metallic fasteners with this appliance. → May cause an accident or injury.</li> <li>● Keep long hair away from massager while in use. → May cause an accident or injury.</li> </ul>
 <b>Compulsory</b>	<ul style="list-style-type: none"> <li>● Before starting massage programs and when raising or lowering the reclining chair, make sure to check that there is no one (such as a child in particular) or any pet around the chair. In addition, be careful not to get your body or other objects caught behind the backrest and under the footrest. → May cause an accident or injury.</li> <li>● If your body or any objects are caught in the chair, press the reclining button to return it to a safe position.</li> <li>● Do not put any cloth, paper, plastic bag, or other flammable items onto the LED parts, or bring them into contact with each other.</li> <li>● Do not insert metals (hairpins, wire) or the like into the gaps of this equipment, etc. (such as LED parts and power cord in particular) → May cause an electric shock, short circuit, or ignition.</li> <li>● Never operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.</li> <li>● Never drop or insert any object into any opening.</li> </ul>

# Safety Precautions

## Points to note during use



### Caution



#### Prohibited

- Do not use the product for unclothed parts of the body.
- Do not use the product for any purpose other than massage and sitting.  
→ May cause an accident or injury.
- Do not put your hand in portions where the rollers go up and down, or movable portions of the reclining chair, etc. In addition, do not use the product while wearing hard items, such as a belt.  
→ May cause injury or a malfunction.
- Do not use the product for the head, chest, abdomen, and joints (such as elbows and knees).  
→ May cause you to feel sick or get injured.
- Do not allow the product to run unattended.  
→ May cause an accident or injury.
- Do not use the footrest for other parts of the body than legs.  
→ May cause injury.
- If one or both of your legs come out of the footrest while your legs are being massaged, do not try to force them back in.  
→ Forced insertion may cause a malfunction.
- Do not use the armrests for other parts of the body than arms.  
→ May cause injury.
- If one or both of your arms come out of the armrests while your arms are being massaged, do not try to force them back in.  
→ Forced insertion may cause a malfunction.
- Do not look directly at the LEDs of the armrests.  
→ May cause injury.




#### Compulsory

- In case of an emergency or if you feel any abnormality during operation, press the ALL STOP button to stop the operation.
- If you feel severe pain or any abnormality in your body during use, immediately stop using the product, and consult a doctor.
- If you feel that the massage stimulation is strong, weaken the stimulation by following this Instruction Manual. If you still feel that it is strong, stop using the product and consult your distributor (printed on the back cover).  
→ Continuous use under strong stimulation may result in injury.
- Be careful not to fall asleep during the massage.  
→ May cause injury.
- Only massage yourself within 30 minutes a day. Continuous use in the same place should be 5 minutes at most.  
→ Continuous use for a long time may result in excessive stimulation, having an adverse effect or causing injury.
- Make sure to check that there is no one (such as a child in particular) or any pet around the chair before use.  
→ May cause an accident or injury.
- Do not massage yourself with the back pad removed.
- Keep the cord away from heated surfaces.  
→ May result in a burns, fire, electric shock or injury.




## Points to note during use

### Caution

 <p>Unplug the power plug</p>	<ul style="list-style-type: none"> <li>● In the event of a power outage or if there is a possibility of a power outage, immediately stop using the product, turn “OFF” the main power switch, and unplug the power plug from the wall socket. → At the time of recovery from a power outage, an accident or injury may result.</li> <li>● If there is a possibility of a lightning strike, immediately stop using the product, turn “OFF” the main power switch, and unplug the power plug from the wall socket.</li> <li>● If no movement is made or you feel any abnormality, in order to prevent accidents, immediately stop using the product, turn “OFF” the main power switch, unplug the power plug from the wall socket, and ask your distributor (printed on the back cover) for inspection and repair. → May cause a fire due to an electric shock, current leakage, short circuit, etc.</li> <li>● Always unplug this appliance from the electrical outlet immediately after using and before cleaning.</li> </ul>
--	---

## Points to note after using and when not using the product



### Caution

 <p>Prohibited</p>	<ul style="list-style-type: none"> <li>● Use the power cord only for this equipment.</li> <li>● After using this equipment, make sure to raise the reclining chair completely.</li> <li>● Do not place unnecessary items on the product.</li> <li>● LED lights are not replaceable.</li> </ul>
 <p>Compulsory</p>	<ul style="list-style-type: none"> <li>● Keep the cord away from heated surfaces. → May result in a burns, fire, electric shock or injury.</li> <li>● After use, make sure to turn “OFF” the main power switch, and unplug the power plug from the wall socket. → May result in an accident or injury due to mischief made by children and pets, etc.</li> <li>● Store the power cord after untwisting it. → The power cord may deteriorate, causing an electric shock, current leakage, or fire.</li> </ul>
 <p>Unplug the power plug</p>	<ul style="list-style-type: none"> <li>● When unplugging the power plug from the wall socket, do not yank on the power cord, but make sure to do so holding the power plug. → May cause a fire due to an electric shock or short circuit.</li> <li>● When performing maintenance, make sure to unplug the power plug from the wall socket. In addition, do not plug in or unplug the power plug with wet hands.</li> <li>● When not using the product, turn all controls to the off position, make sure to unplug the power plug from the wall socket. → May result in insulation deterioration due to dust and moisture, causing an electric shock, current leakage, or fire.</li> </ul>

# Safety Precautions

## Other points to note

### Warning

 Prohibited	<ul style="list-style-type: none"><li>● Do not stand on the main unit. Do not exert a strong impact from above by sitting outside the seat, etc. → May cause an accident, injury, or malfunction due to overturning.</li></ul>
 Disassembly prohibited	<ul style="list-style-type: none"><li>● In the event of a malfunction of the equipment, do not repair it on your own but contact your dealer or distributor (printed on the back cover).</li><li>● If the power cord is damaged, which is dangerous, ask your distributor (printed on the back cover) for replacement of the power cord.</li><li>● Do not modify the equipment. → May cause a fire, electric shock or injury.</li></ul>


### Caution

 Compulsory	<ul style="list-style-type: none"><li>● When transferring or lending the product to another person, make sure to hand over this Instruction Manual together with the main unit.</li></ul>
---	---

## Request


- When disposing of the product and included items, inquire about proper disposal methods at the municipal office nearest you.

# Part Names and Functions

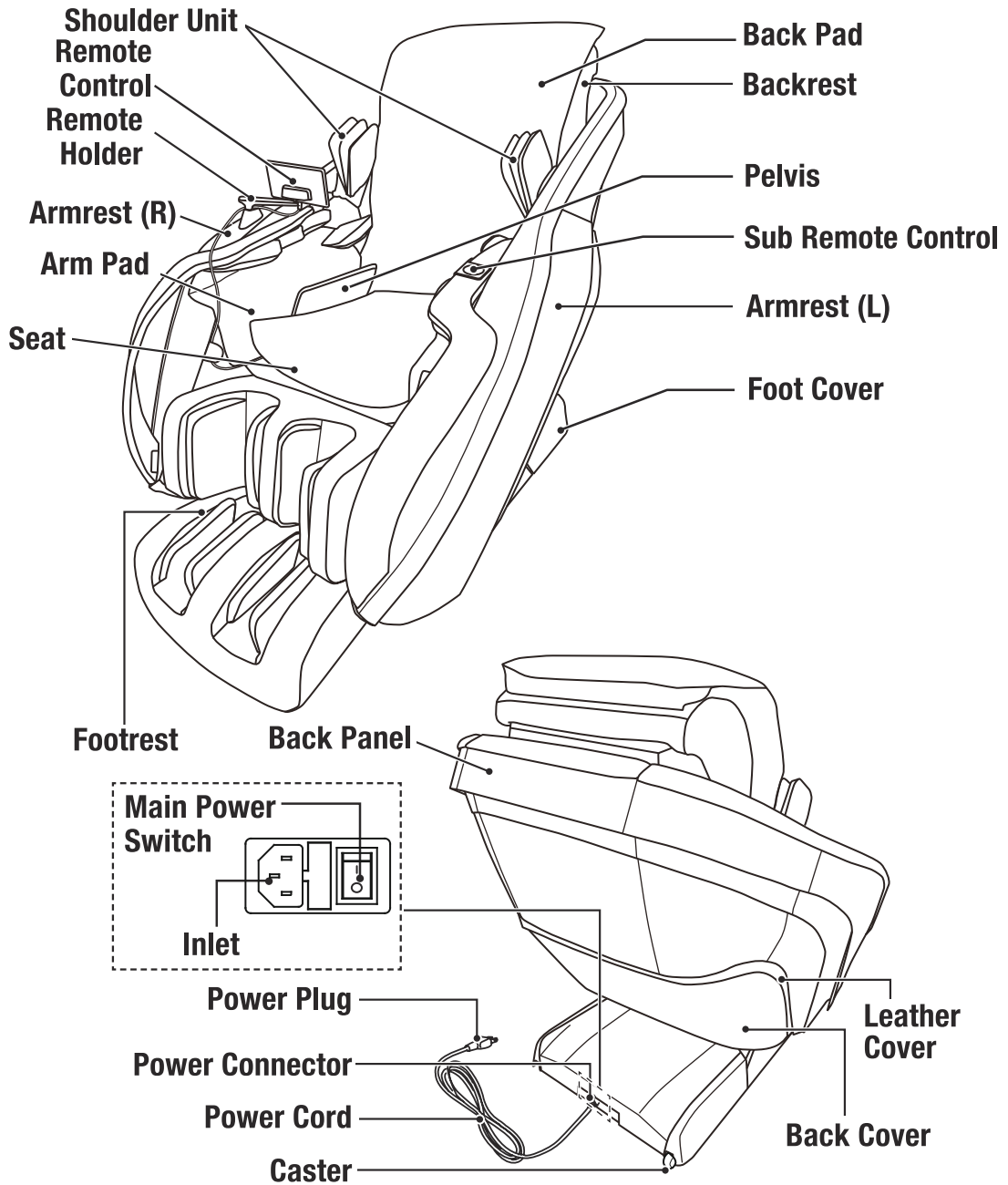


# Part Names and Functions

## Remote Control

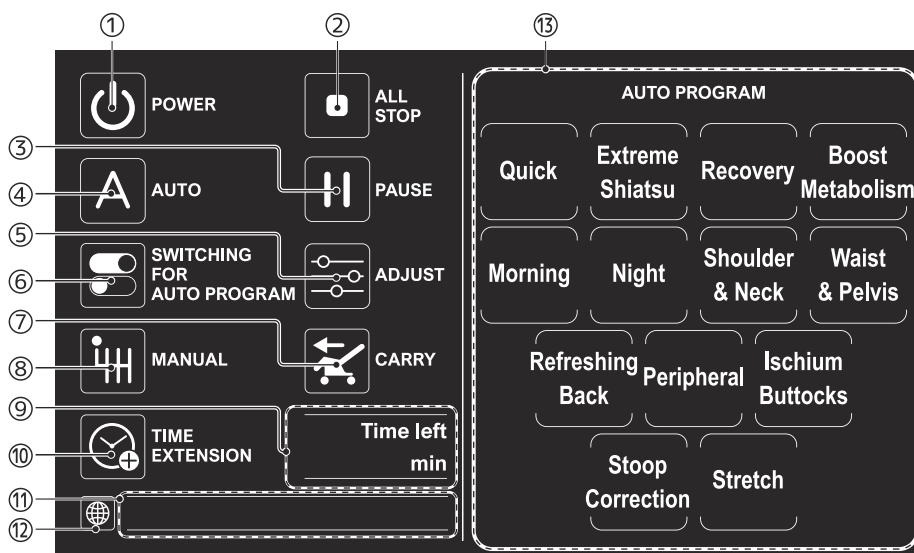

## Sub Remote Control

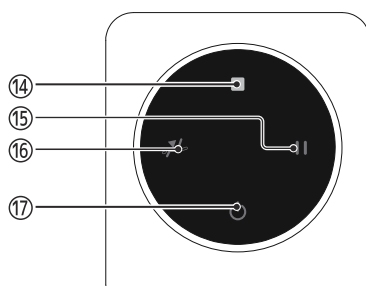

<b>Warning</b>	<ul style="list-style-type: none"> <li>Do not spill water or the like on the remote control.</li> <li>Do not operate with wet hands.</li> </ul>
<b>Caution</b>	<ul style="list-style-type: none"> <li>Do not sit on the remote control, drop or step on it, or give a strong pull to the cord.</li> <li>Do not press the buttons with a pointy or hard object. → May cause damage.</li> </ul>

①	POWER	Turns on or off the power to the remote control. If this button is pressed while the massage chair is running, the massage chair returns to the default position after the Massage stops. The remote control turn on screen light is always on.
②	ALL STOP	Please press this button when an emergency occurs or when you feel that the massage chair is running abnormally. The massage chair will stop running immediately.
③	PAUSE	Press this button if you want to pause the massage. The massage chair immediately stops running and returns to the default position.
④	AUTO	Pressing this button, the right side of the main remote control screen (⑬) turns into the auto program button, which allows you to select each program.
⑤	ADJUST	Pressing this button, the right side of the main remote control screen (⑬) turns into the adjustment button, which allows you to adjust the intensities of the back movement, the arms and legs, the heater on/off switch, the angle of the back/legs, and the roller position adjustment.
⑥	SWITCHING FOR AUTO PROGRAM	Pressing this button, the right side of the main remote control screen (⑬) turns into each function switch button, which can switch each part of the massage function, and this button is only effective in the automatic program state.
⑦	CARRY	If you want to carry the massage chair, please get up from the massage chair and press this button. The main remote control screen immediately jumps to the confirmation screen.
⑧	MANUAL	Pressing this button, the right side of the main remote control screen (⑬) turns into the manual program button which allows you to select basic and special massage.
⑨	Time Left	The time left is displayed after you select a massage program.
⑩	TIME EXTENSION	Press this button when you want to extend the massage time. The auto program will be extended to double the program time.
⑪	Language	Press this button when you want to change the language.
⑫	Status	The operation settings you have selected will be displayed here.
⑬	Function Screen	When you select each function, it will turn into a subsidiary menu for each program.

⑭	ALL STOP	Please press this button when an emergency occurs or when you feel that the massage chair is running abnormally. The massage chair will stop running immediately.
⑮	PAUSE	Press this button if you want to pause the massage. The massage chair immediately stops running and returns to the default position.
⑯	SOFA MODE	If you do not want to massage and just relax in the chair, please press this button. Then the massage chair stops running and reclines up automatically.
⑰	POWER	Turns on or off the power to the remote control. If this button is pressed while the massage chair is running, the massage chair returns to the default position after the Massage stops. The remote control turn on screen light is always on.

# Before Use

- This product is packed and delivered with the backrest raised and the shoulder unit lowered. Before use, make sure to raise the shoulder unit.
- When using the product after raising the room temperature in a very cold room, do not increase the room temperature suddenly but gradually. If this equipment is brought into a warm room when it is cold, wait for about one hour before use.  
→ Condensation may occur on the operating parts, and performance may not be fully demonstrated.

## About installation location


To prevent the chair from colliding with walls and other objects when reclined, leave a space of approximately 20cm (7.9 inches) or more behind the main unit and secure a space of approximately 70cm (28 inches) or more in front of the main unit.



### Caution

- Do not use the product in a bath room, sauna, or other high-humidity areas.
- Use the product only after placing it in an area where the floor is horizontal.
- Do not install the product in a place subjected to high temperatures, such as an area exposed to outdoors, direct sunlight or near a stove.
- Keep flammables away, including cigarettes and ashtrays, etc.
- Do not use the product in an area directly exposed to air from air conditioning equipment.

- In order to prevent floor damage and to lessen operating sounds, lay a carpet or the like under the main unit. (Do not lay an electric carpet under the main unit.)
- Since the product is heavy, be careful not to damage the floor surface when setting it down.
- Keep 1m (39 inches) or more away from a TV, radio, and other AV equipment. There may be noise interference.
- When a person weighing 100kg (220 lbs) sits in the chair, a load of approximately 3.9kg (8.6 lbs) is applied per square centimeter.

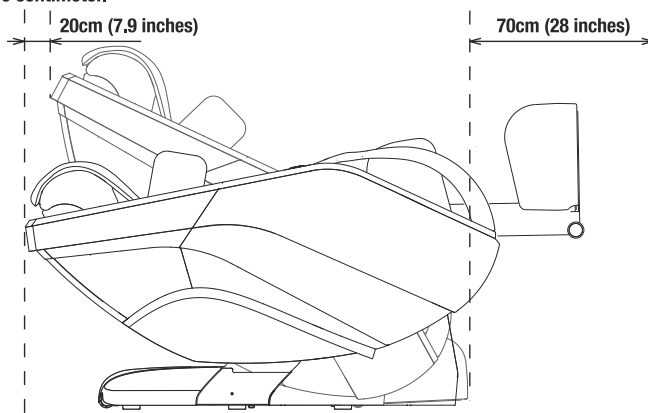

## Checking of the packaged items

Check that the packing box contains the main unit, footrest, armrests (L/R), and packaged items.

Main Unit	Footrest	Armrests	Packaged items			
						A set of assembly screws and tools (screw A: 4 screws, screw B: 2 screws, screw C: 2 screws, screw D: 1 screws, 1 L-wrench)



## Installation and assembly procedures


For this equipment, the main unit, footrest, and armrests (L/R) are packed separately.




- Be very careful not to get your hands and other objects caught during installation and assembly.
- Be careful not to step on the power cord with the foot cover etc., during installation.

- ① Please lift up the seat, and after opening the body shaft cover, lift the Footrest in the direction of the arrow and hang it on the body. Then close the body shaft cover and secure the Footrest to the body with screw A.

※Screw A with a small washer. Please be careful not to use screw B with a large washer.

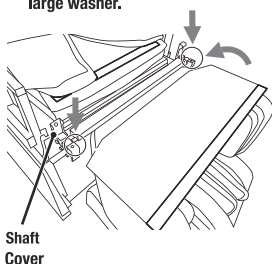


- ② Connect the air delivery hose and cable (Please connect hoses of the same color together. The cables must not be connected in reverse.) And attach the velcro showing the gray part in the picture.

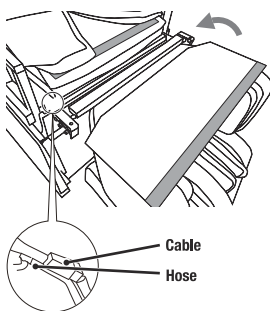


- ③ Please hang the back side of the Armrest (□ part in the picture) to the pegs on the back of the massage chair and place the center of the Armrest (○ part in the picture) on the frame of the massage chair seat.

※Take care to avoid pinching air hoses and cables.

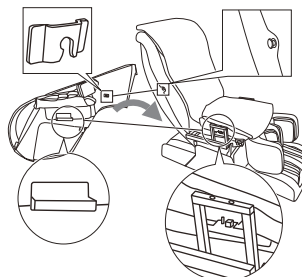


- ④ Secure the Armrest to the body with screw B.

※With the Armrest retainer in the stored position, pull back the shoulder.

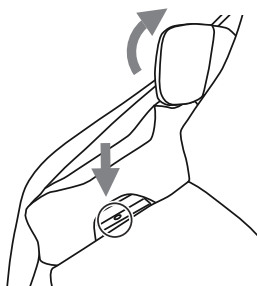

- ⑤ Connect the air delivery hose and cable. Please (connect the same color ones together.) Then turn the Arm Pad around and attach the Velcro.

※Please keep air hoses and cables behind bracket.

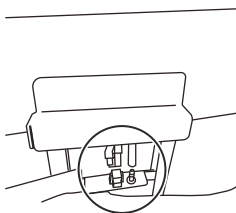


- ⑥ Install the armrest on the opposite side in a like manner by the same procedure as ③ to ⑤.

# Before Use


- ⑦ Hang the back panel on the hooks at the rear of the right and left armrests, push it in as far as it will go. Then, secure with screw C in the lower portion of the panel.

※Pay attention to the alignment of the seams on the left and right sides of the back cover when tightening the screws.

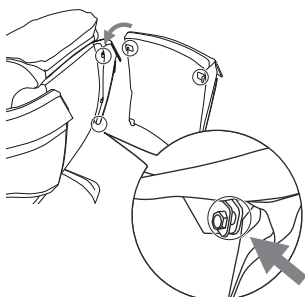


- ⑧ Hang the back cover on the hooks in front of the Armrests, and fix it with the velcro at the rear.

※Be careful not to allow the leather cover to enter inside the back cover.

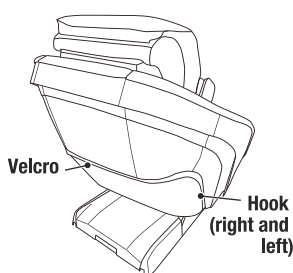


- ⑨ Lift the pelvis on both sides and attach the pelvis velcro to the back of the wrist pads.

※The pelvis is placed under the seat of the chair.

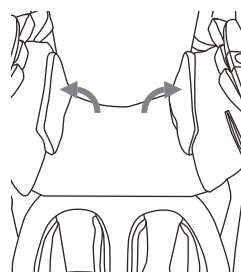


- ⑩ Place the remote holder on the armrest (R) and secure it with screw D. Then cover up the screw D position with the remote holder cover.

※Please note that the remote holder cover is packed in the same bag as the remote holder.

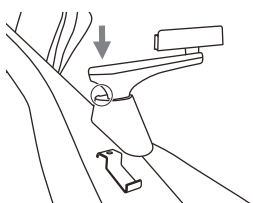


- ⑪ Hang the remote control on the remote control holder and push it all the way in to the bottom of the remote holder.

※Be careful to avoid pinching the remote control cable.

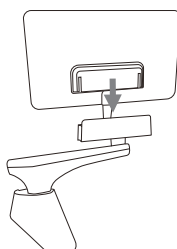


- ⑫ Pass the remote control cable through the loop on the front side of the armrest (R) and firmly press on the snap.

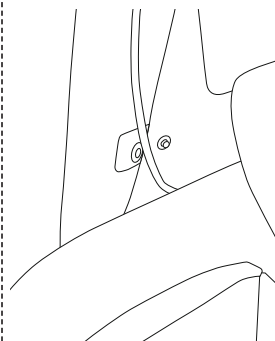

### Confirmation of fabrics

Unzip the top of the pillow cushion and lower the back cushion forward to confirm that the fabric of the massage chair is not broken. In addition, please confirm please check other parts to make sure that the fabric is not broken.

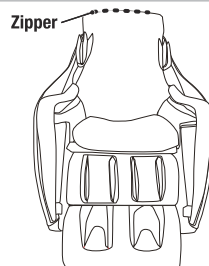


#### Warning

- Be sure to check the fabric in the massage area for cracks and breaks before massaging. Please check the other parts to make sure that there are no damages on the fabric. If you find any damage to the fabric, regardless of the size of the damage, unplug the product and contact the dealer from whom you purchased the product (listed on the back cover). Never remove the fabric from the massage area or use it when the fabric is broken, as this may pose a serious safety hazard.

### How to turn on the power

- ① Insert the power connector into the inlet.
- ② Insert the power plug into a wall socket. Securely connect the ground wire coming out from the power plug to the ground terminal of the power-supply wall socket. If the power-supply wall socket has no ground terminal, consult your dealer.  
\* Shapes of the plug and wall socket will vary from country to country.
- ③ Turn "ON" the main power switch at the rear of the main unit.
- ④ Hold the power button at the remote down until it turn "ON" .

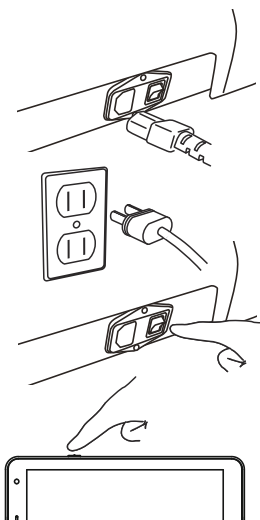

#### Warning

- Do not use the product with a transformer.  
→ May cause a malfunction or electric shock.
- Use the product only in the country of your purchase.  
→ May cause a malfunction or electric shock.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.  
→ May cause a malfunction or electric shock.



#### Caution

- Before inserting the power plug into the wall socket, check that the main power switch is "OFF."
- Securely insert the power plug as far as it will go.
- When using the product, turn on all the switches in order by following this Instruction Manual, and check that the equipment functions normally.

# Chair Adjustments

## How to sit in the chair

Normally, the mechanisms are stored in their stowed positions. For your safety, check each mechanism position, and if any mechanism is located outside its stowed position, do not sit down immediately but turn on the main power switch to return it to the stowed state, and then slowly sit down in the center of the chair.



### Caution

- Do not sit in the chair while it is reclined.  
→ May cause a malfunction as impact may be exerted on the backrest and footrest. In addition, the main unit may overturn, causing an accident or injury.
- Start massage from a weak stimulation.


When getting in/out of the chair, do not support yourself by placing your hands on the wooden handrails fitted on the armrests. Doing so may cause a malfunction. Get in/out of the chair while supporting yourself by placing your hands on the top surfaces of the armrests.




## How to use the buffing pad

If you feel that the massage is strong, place the included buffing pad on the back pad on the seat, place the sole pads on the footrest during the massage.

Sole Pads

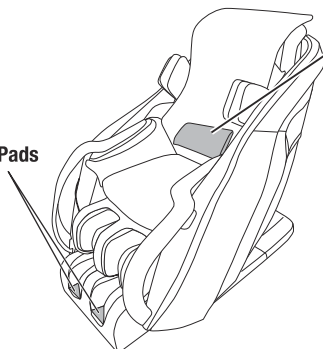

Buffing Pad

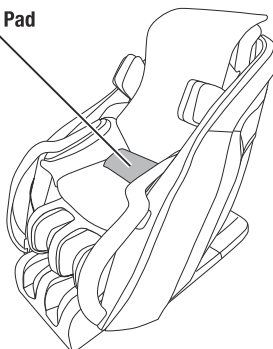

### Caution


- Check that the buffing pad is not curled or one-sided.  
→ May cause a malfunction or injury.

## How to use the armrests


As shown in the illustration below, place your arm so that elbows come between the armrest and backrest. In addition, adjust the angle of the shoulder unit to the desired angle.

Place elbows between the armrest and backrest.

Insert so that the palms face down.

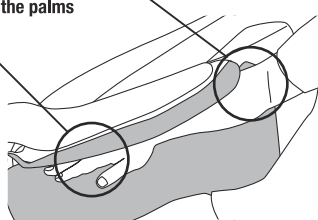

Adjust the shoulder unit to the desired angle.

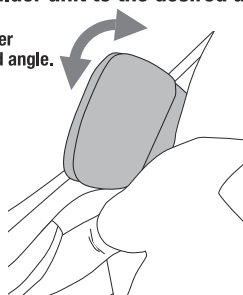

### Caution

- Do not use the armrests for other parts of the body than the arms.  
→ May cause injury.
- If one or both of your arms come out of the armrests while your arms are being massaged, do not try to force them back in.  
→ Forced insertion may cause a malfunction.
- Do not massage elbows. In addition, your palms should face down during the massage. Do not put your arms in other directions.  
→ May cause injury.

## How to use the reclining chair

By pressing the reclining buttons, the backrest and footrest can be adjusted.

### ■ For raising the reclining chair

The backrest can be raised by pressing the upper button of Back.  
At this time, the footrest is lowered in tandem.

### ■ For raising the footrest

The footrest can be raised by pressing the upper button of Leg.



Back



Leg



### ■ For lowering the reclining chair

The backrest can be lowered by pressing the lower button of Back.  
At this time, the footrest is raised in tandem.

### ■ For lowering the footrest

The footrest can be lowered by pressing the lower button of Leg.



### Warning

- Before starting a massage program and when lowering the reclining chair, make sure to check that there is no one (such as a child in particular) or any pet around the chair. In addition, be careful not to get your body or other objects caught behind the backrest and under the footrest.

# Auto Programs


## Programs and features

Programs can be easily selected from the preset menu items according to purpose/use.

Quick Program	(Approximately 10 minutes)	This program is recommended for people who do not have much time to massage. The entire body from the neck to the soles of the feet is massaged in a short time.
Extreme Shiatsu Program	(Approximately 30 minutes)	This program fully experienced the soothing, deep stimulation of the DC-200's sublimated extreme shiatsu function.
Recovery Program	(Approximately 20 minutes)	This program is the standard program for total body fatigue recovery that comes with all the features of the DC-200.
Boost Metabolism Program	(Approximately 20 minutes)	This program is a series of slow pushes and kneads on the entire body surface to accelerate blood flow and improve metabolism.
Morning Program	(Approximately 20 minutes)	Massages are performed along vein flows by sending blood flows from the body's extremities back to the heart by the centripetal method. When you want to refresh your body or awaken your body.
Night Program	(Approximately 20 minutes)	Massages are performed along arterial flows by promoting blood flows from the heart to the body's extremities by the centrifugal method. When you want to relax your body or before bedtime.
Shoulder & Neck Program	(Approximately 15 minutes)	This program intensively massages the base of the neck, the back of the neck, the upper parts of the shoulders, and the muscles around the shoulder blades.
Waist & Pelvis Program	(Approximately 15 minutes)	This program intensively massages the lower back, the hip, the pelvis, and the waist. This program is focused on improving stiffness, soreness and pain in the area.
Refreshing back Program	(Approximately 15 minutes)	This program focuses on improving the stiffness and soreness of the back muscles with the main techniques of Shiatsu, Kneading and Pushing.
Peripheral Program	(Approximately 15 minutes)	The program is a roller pushing back, with arms and legs massaged in the form of airbags to reach the peripheral circulation.
Ischium Buttocks Program	(Approximately 15 minutes)	This program is a hip • pelvis • ischium ( with thigh periphery) focused massage to the pelvis swinging from side to side and relaxing kneading around the hips.
Stoop Correction Program	(Approximately 15 minutes)	This program is shoulder • neck • shoulder blade peripheral kneading relaxation, neck back, chest forward, stretching back tendons and other functions to relieve modern people's stoop that based on using computer and cell phone.
Stretch Program	(Approximately 15 minutes)	This program is a neck-back, chest-forward, twisting, leg-stretching, and back-stretching movement that achieves muscle relaxation.

### Massage with auto program

- Equipped with 13 programs to relieve fatigue and improve body functions.
- Press the individual program buttons to select your preferred program and massage according to the preset menu.

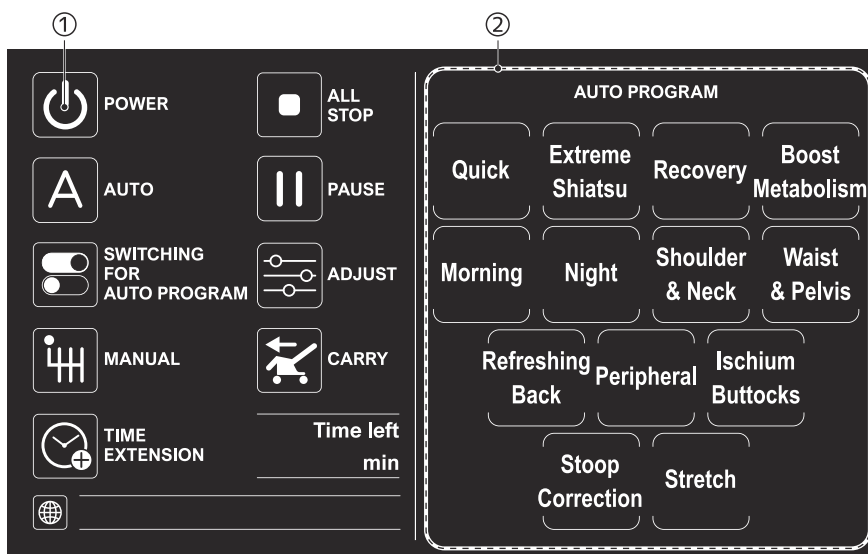

#### ① Press the POWER button.

If there is no selection entered after 90 minutes, the power turns off automatically.


#### ② Press the button corresponding to your desired program.

The reclining chair moves to the optimum angle for massages. After that, a search for shiatsu points is performed.

[Request]

Sit back in the seat and place your head on the backrest in a comfortable position. If your body is not seated, an accurate search for your shiatsu points cannot be performed.

# Auto Programs

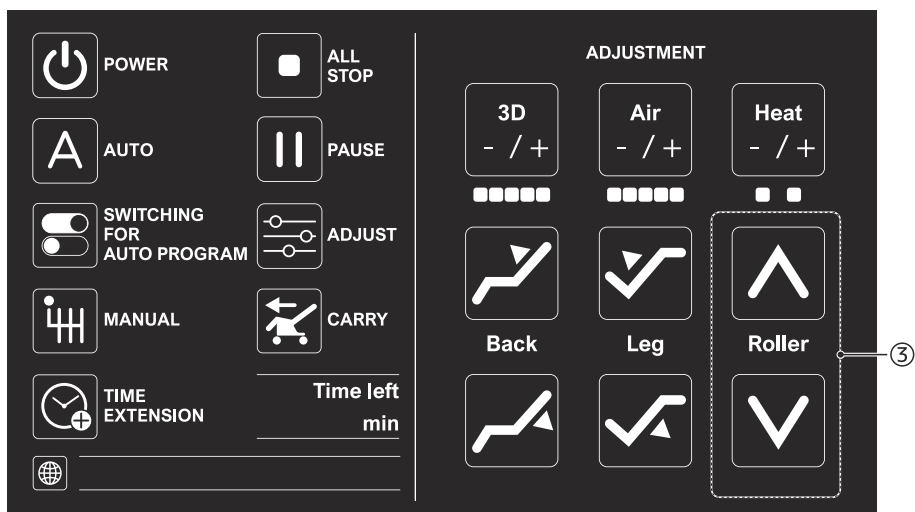


## ③ The shoulder position can be adjusted.

After shiatsu point search, the buzzer beeps. While the buzzer sounds, the shoulder position can be adjusted. If no adjustment is needed, leave it as is.

[Shoulder position adjustment]

If the shoulder positions are not aligned, adjust the shoulder positions while the buzzer beeps.

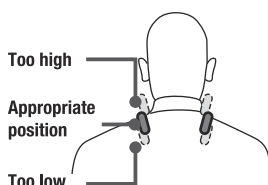

		While the button is pressed, the rollers keep moving upward.
Roller		
		While the button is pressed, the rollers keep moving downward.

## ④ The program starts.

Upon completion of shiatsu point search, the buzzer stops sounding, and the program starts. During massages, the mechanism intensity and the air intensity can be adjusted.

Massage intensity adjustment and heater temperature adjustment/cancellation			
<div>3D</div> <div>- / +</div> <div>■■■■■</div>	<div>Air</div> <div>- / +</div> <div>■■■■■</div>	<div>Heat</div> <div>- / +</div> <div>■ ■</div>	<p>The massage intensity and the heater temperature can be adjusted.</p> <p>Each time the 3D/Air button is pressed, massage intensity is switched. The intensity be weaker when you pressed “-”, be stronger when you pressed “+”. Each time the Heat button is pressed, switching is performed in the order of weak ⇒ strong ⇒ off. (When the program starts, it is set to off.)</p>



### ⑤ The program ends.

The rollers go back to their stowed positions, and the chair is into a standby state. After that, you can make other desired movements.

- After use, turn "OFF" the power of the remote control.

## Program change

Switching to another program can be performed during operation without waiting for the program to end. However, even in the middle of a program, massages end after 90 minutes.

### Auto program → Auto program


During the execution of an auto program, if another auto program button is pressed, the rollers go back to the once-detected shoulder position, and the shoulder position is reconfirmed, and then another program starts. While the shoulder position is reconfirmed, the shoulder position can be adjusted.

### Auto program → Manual program

During the execution of an auto program, if the MANUAL button is pressed, the auto program is suspended at the instant when the button is pressed, so that a manual program can be selected. Press the desired program button to start operation.

## Switches for each function

You can also switch each function on and off at will when running an auto program. To switch on/off, press the SWITCHING FOR AUTO PROGRAM button. The screen on the right side of the remote control screen will change to the function switch screen. Press each function button to switch it on or off at will (the default setting is ON when the auto program starts).

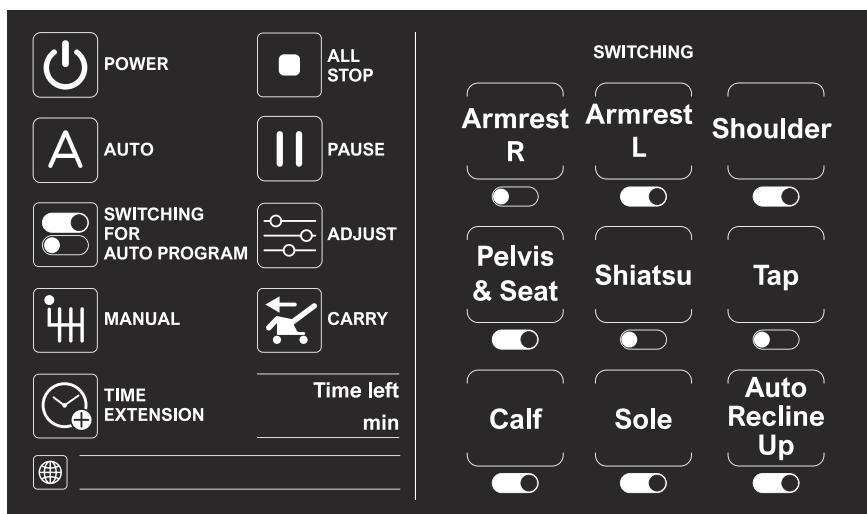

Please note that this setting is only saved until the mains power is switched off.

# Auto Programs

## Pause in the program

To put the program on pause, press the [PAUSE] button while it is running. Then the massage chair immediately stops massaging and reclines up to the default position. When the massage chair is paused, the remote control screen jumps to the pause announcement screen and you can select any action. Please press the [RESTART] button after confirming that it is safe to do so, and the program before pressing the pause button will start to continue running.

**The message is pausing in present.**

Please select the next action from below buttons.

Before select, please confirm the safety.



POWER OFF



ALL STOP



RESTART

## Program discontinuation

To discontinue the program, press the POWER button during operation. The program stops after stowing operation. **In case of an emergency or if you feel any abnormality** during operation, press the [ALL STOP] button. Operation stops instantly.

After safety confirmation, press the [RETURN] button. Stowing operation is performed and then a stop is made.

**The chair is emergency stopping in present.**

Please confirm the safety.

If you press the below "Return" button,  
the chair will return to its default position.



RETURN

In some auto programs, the reclining chair is automatically lowered when the programs are selected. In addition, in some programs, the footrest angle changes automatically. Therefore, check before use that there is no one or obstruction behind the chair. In addition, be careful that no one approaches during use.

### Extension of program time

You use this function to extend the program time. When any automatic program is running, pressing the time extension button extends the running time to twice the system time. For your safety, if the actual running time reaches 90 minutes the massage chair will automatically cut off the power. Therefore, extensions beyond 90 minutes are ineffective.

**Running time is over 90 minutes  
and the chair is returning to the default position.**

Please wait until it completes.



ALL STOP

#### Caution


- If you feel that the massage stimulation is strong, weaken the stimulation by following this Instruction Manual. If you still feel that it is strong, stop using the product and consult your distributor (printed on the back cover).  
→ Continuous use under strong stimulation may result in injury.
- Do not put your hand or foot in portions where the rollers go up and down, and movable portions of the reclining chair, etc.  
In addition, do not use the product while wearing hard items, such as a hair accessory, necklace, wristwatch, bracelet, ring, and artificial nails.
- If one or both of your legs come out of the footrest while your legs are being massaged, do not try to force them back in.
- Only massage yourself within 60 minutes a day. Continuous use in the same place should be 5 minutes at most.  
→ Continuous use for a long time may result in excessive stimulation, having an adverse effect or causing injury.

# Manual Programs

## Programs and features

Desired functions can also be combined. In addition, speed and intensity, as well as up-and-down and other massage movements can also be adjusted.

## How to operate and adjust each part

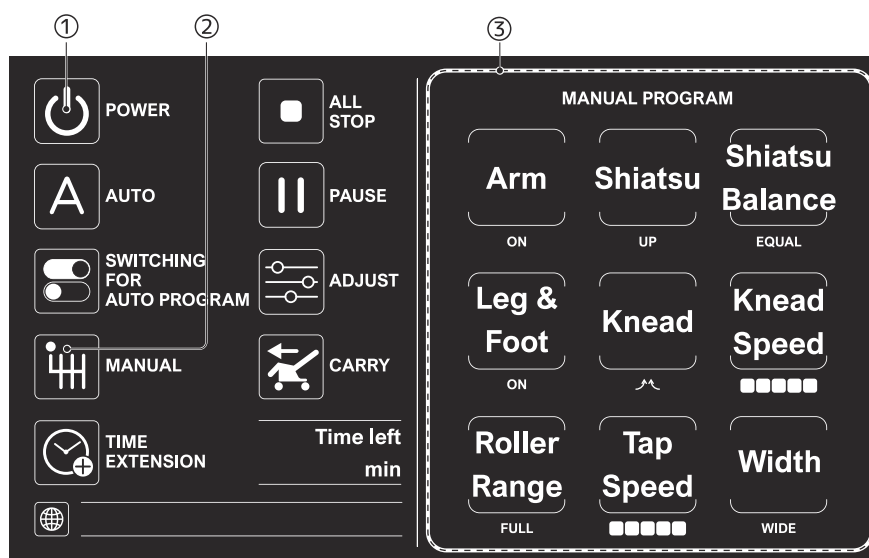

### ① Press the POWER button.

If there is no selection entered after 90 minutes, the power turns off automatically.

### ② Press the MANUAL button.

Pressing the Manual Program button transforms the right side of the remote control screen to the Manual Program screen, where you can select any manual program.





### ③ Press the button corresponding to your desired motions.

Select a combination of desired motions from Knead, Tap, Roller, Shiatsu, and Air massages.

### ④ The program starts.

During massages, the mechanism intensity and the air intensity can be adjusted, and the air operation can be canceled.

## Roller position adjustment

 Roller	[When Roller is not selected] The roller position can be adjusted to the desired massage position.		While the button is pressed, the rollers keep moving upward.
			While the button is pressed, the rollers keep moving downward.
		● When the rollers are at the uppermost position (lowermost position), if Up (Down) is selected, the warning sound will beep.	

## ⑤ The program ends.

The rollers go back to their stowed positions, and the chair is into a standby state. After that, you can make other desired movements.

- After use, turn "OFF" the power of the remote control.

Width adjustment		
<div>Width</div> <div>WIDE</div>		During width-adjustable basic motion, the roller width can be switched in the order of Tight ⇒ Mid ⇒ Wide. (When the program starts, it is set to Wide.)
Speed adjustment		
<div>Knead Speed</div> <div>■■■■■</div>	<div>Tap Speed</div> <div>■■■■■</div>	During speed-adjustable basic motion, the massage speed can be switched in 5 levels. (When the program starts, it is set so that 1st level operation is performed.)
Shiatsu balance adjustment		
<div>Shiatsu Balance</div> <div>EQUAL</div>		During shiatsu motion, the shiatsu balance can be switched in the order of Right ⇒ Left ⇒ Equal. (When the shiatsu motion starts, it is set to Equal.)
Massage intensity adjustment and heater temperature adjustment/cancellation		
<div>3D</div> <div>- / +</div> <div>■■■■■</div>	<div>Air</div> <div>- / +</div> <div>■■■■■</div>	<div>Heat</div> <div>- / +</div> <div>■ ■</div>

## Program change

Switching to another program can be performed during operation without waiting for the program to end. However, even in the middle of a program, massages end after 90 minutes.

### Manual program → Auto program

During the execution of a manual program, if the AUTO button is pressed, the manual program is suspended at the instant when the button is pressed, so that a auto program can be selected. Press the desired program button to start operation.

# Manual Programs

## Pause in the program

To put the program on pause, press the [PAUSE] button while it is running. Then the massage chair immediately stops massaging and reclines up to the default position. When the massage chair is paused, the remote control screen jumps to the pause announcement screen and you can select any action. Please press the [RESTART] button after confirming that it is safe to do so, and the program before pressing the pause button will start to continue running.

**The message is pausing in present.**

Please select the next action from below buttons.

Before select, please confirm the safety.



POWER OFF



ALL STOP



RESTART

## Program discontinuation

To discontinue the program, press the POWER button during operation. The program stops after stowing operation. **In case of an emergency or if you feel any abnormality** during operation, press the [ALL STOP] button. Operation stops instantly.

After safety confirmation, press the [RETURN] button. Stowing operation is performed and then a stop is made.

**The chair is emergency stopping in present.**

Please confirm the safety.

If you press the below “Return” button,  
the chair will return to its default position.



RETURN

In some auto programs, the reclining chair is automatically lowered when the programs are selected. In addition, in some programs, the footrest angle changes automatically. Therefore, check before use that there is no one or obstruction behind the chair. In addition, be careful that no one approaches during use.


### Caution

- If you feel that the massage stimulation is strong, weaken the stimulation by following this Instruction Manual. If you still feel that it is strong, stop using the product and consult your distributor (printed on the back cover).  
→ Continuous use under strong stimulation may result in injury.
- Do not put your hand or foot in portions where the rollers go up and down, and movable portions of the reclining chair, etc.  
In addition, do not use the product while wearing hard items, such as a hair accessory, necklace, wristwatch, bracelet, ring, and artificial nails.
- If one or both of your legs come out of the footrest while your legs are being massaged, do not try to force them back in.
- Only massage yourself within 60 minutes a day. Continuous use in the same place should be 5 minutes at most.  
→ Continuous use for a long time may result in excessive stimulation, having an adverse effect or causing injury.


## Sofa Mode

When this function is turned on, the massage movement moves towards the lumbar area and stops other massage functions. At the same time the back of the chair declines and the legs rise.

- ① Press the [  ] button on the sub remote control to turn on the sofa mode.

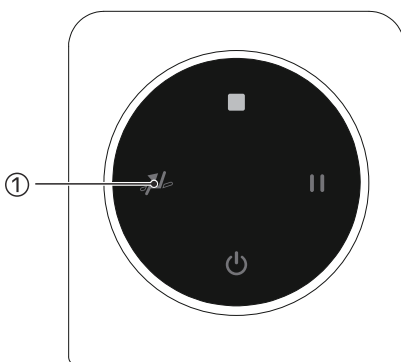

# Sofa Mode

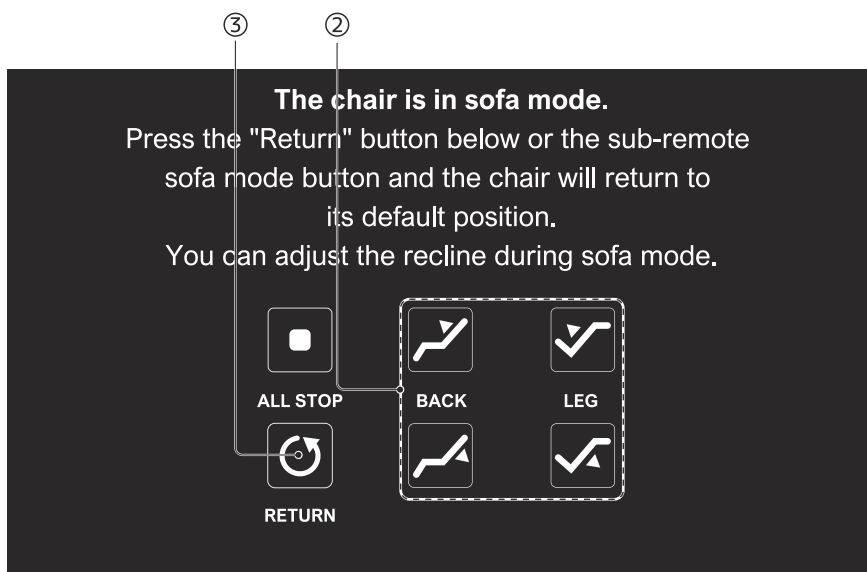

- ② To adjust the back or leg angle, press the buttons above and below the back or legs.  
To adjust the angle of the massage chair, press the buttons as described in "How to Adjust the Back of the Chair" on page 16 of this manual. You can adjust the massage chair to any angle.
- ③ To end sofa mode, press the RETURN button.  
Then the massage chair will reset to the default position. After completing the reset, you can turn on any other massage mode again.


## Caution

- If you want to use a cushion liner, please refer to page 15 of this manual for instructions.
- Sofa mode and other massage modes, cannot be used at the same time.
- When turning on other massage modes, please refer to the corresponding instructions.



# Carry Mode

Turn this function on to make it easier to carry the massage chair. Please note that you must stand up and walk away from the massage chair to turn this mode on.

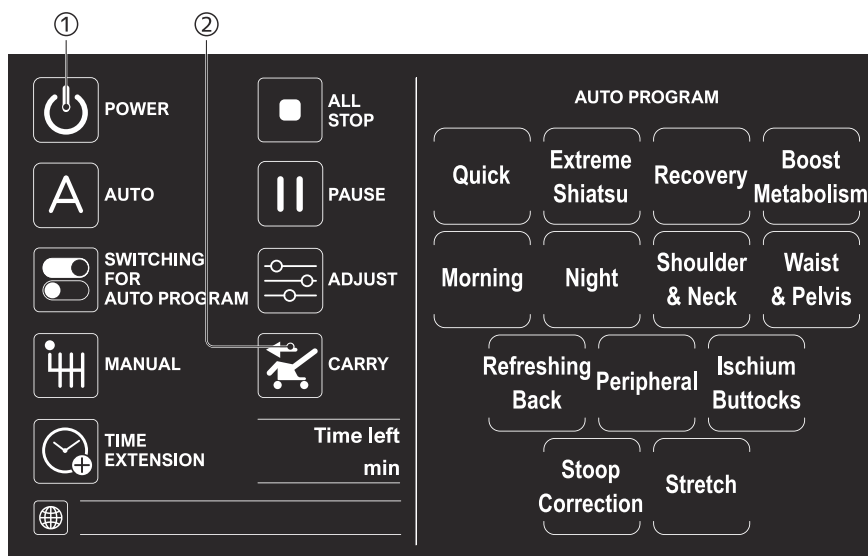


① **Press the POWER button.**

If there is no selection entered after 90 minutes, the power turns off automatically.

② **Press the CARRY button.**

When you press the Carry button, the remote control screen changes to a confirmation screen and you can choose whether to carry or not.

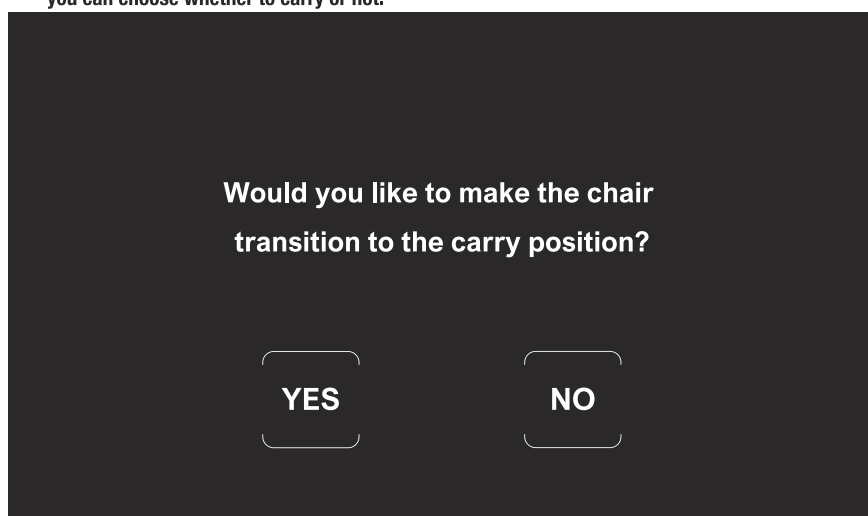

# Carry Mode

## ③ Press the YES button.

When you want to move the massage chair, press the YES button and the remote control screen changes to a waiting screen. If you want to cancel the transportation, press the CANCEL button, and the massage chair stops running immediately and returns to the default position.

The chair is in transition to the carry position.

Please wait until it completes.



ALL STOP



CANCEL

## ④ Turn off the main power switch and unplug the power cord from the outlet.

After the remote control screen displays the status shown below, please turn off the main power switch and unplug the power cord from the outlet to complete the carrying mode. If you want to cancel the handling, please press the CANCEL button, then the massage chair stops running immediately and returns to the default position.

The chair is in carry mode.

Please turn off the main power switch and  
remove the power code from inlet before carry.




CANCEL

### **Caution**

- Be sure to stand up and move away from the massage chair before using the carry mode.  
→ If used while seated, it may cause injury.
- Please make sure that there are no people or objects around the massage chair before you start moving.  
→ Collisions may cause injury or damage.

**Recline the chair, raise the footrest, lift the front side off the floor holding the undersides of the armrests, and move with the casters.**

**If it is difficult for one person to carry it, two persons should hold the undersides of the armrests at opposite sides to lift the front side off the floor and move with the casters.**

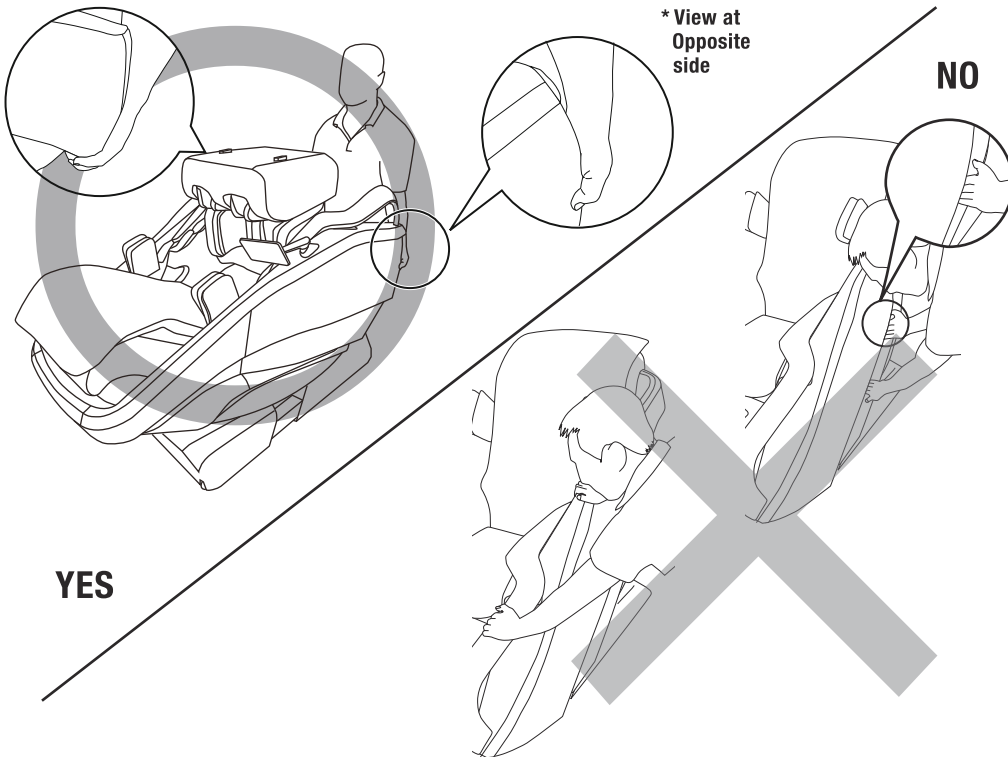

**If it cannot be moved by the usual method, remove the armrests and footrest by following the assembly procedure on pages 12 and 13 to move the product. If it still cannot be moved, contact your distributor printed on the back cover.**

- Since the product is heavy, wear gloves for transport so as not to drop the product or get injured when carrying in or moving the product.
- Since the product is heavy, be careful not to injure your lower back, etc., when carrying in or moving the product.
- Since the product comes with a remote control, be careful not to drop it when moving the product.

### **Caution**


- Be careful not to exert excessive force on the wooden parts.
- Do not move the product with a person on it.
- When moving the product, be very careful not to get your foot or power cord, etc., caught.
- Be careful not to damage the floor with the casters.
- Do not carry this appliance by supply cord or use cord as a handle.

# Language Selection

You can choose from 12 languages to be displayed on the remote control screen.

※The default setting is English.

## How to select the language

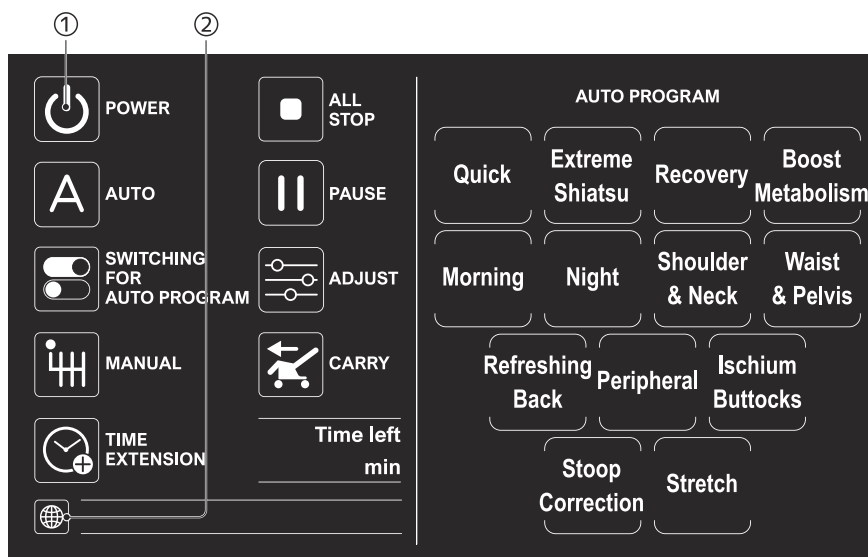


### ① Press the POWER button.

If there is no selection entered after 90 minutes, the power turns off automatically.

### ② Press the [ ] button.

Pressing this button transforms the remote control screen to the Language selection screen, where you can select 11 kinds of language.

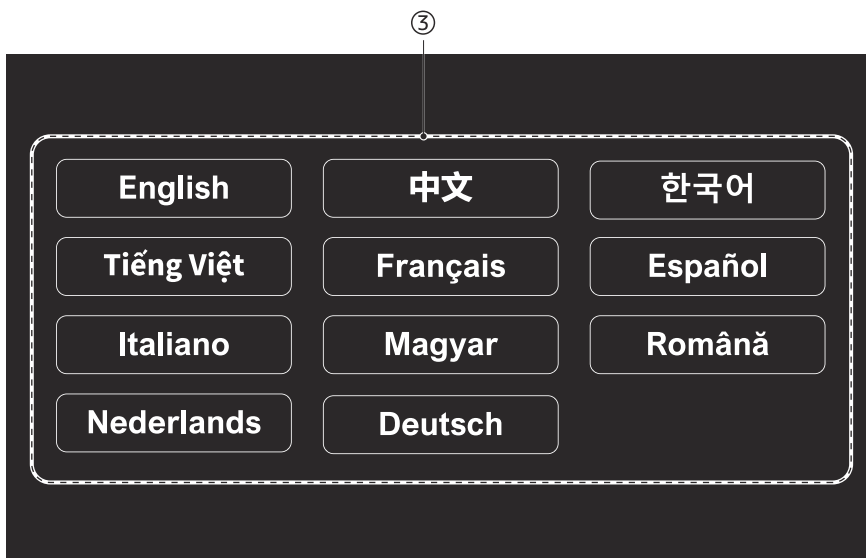


③ Press the button corresponding to your desired language.

Select a desired language from English, Mandarin Chinese, Korean, Vietnamese, French, Spanish, Italian, Hungarian, Romanian, Dutch, German, and Russian.

④ The screen turn around to the default page.

The screen of remote turn to default page, and the language also changes immediately.


- After use, turn "OFF" the power of the remote control.

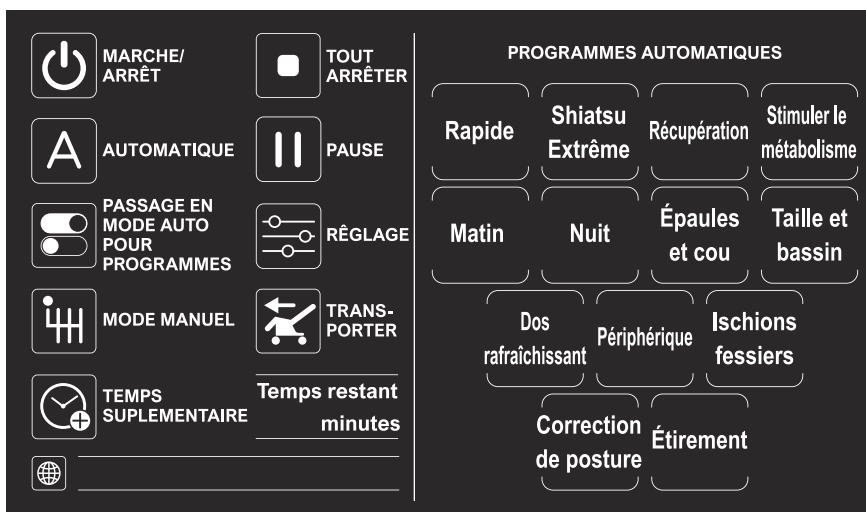

# Bluetooth Speakers

You can enjoy music by connecting to a Bluetooth speaker. Please follow the usage of your any music playing device to connect. The speaker is installed in the backrest and is energized by the massage chair without replacing or charging the battery.

Speaker Installation Position

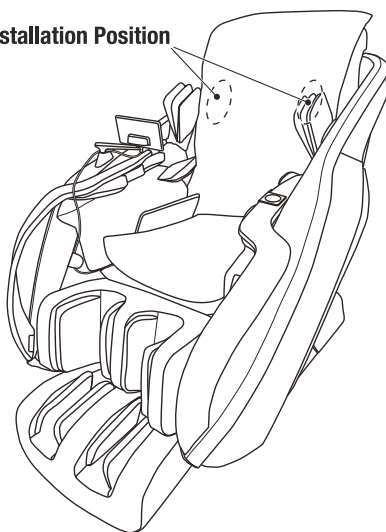

## On the first connection

Pairing operation is required. If the pairing is normal, it will be connected automatically from the second time. Connection information is saved for only one device.

※Multiple devices cannot be connected at the same time.

### ① Press the POWER button.

If there is no selection entered after 90 minutes, the power turns off automatically.

### ② Prepare music playing equipment.

Please turn your music playing device on to the Bluetooth settings screen in Settings.

### ③ Connect the device.

Select "D\_core" when it is displayed on your device. Connections are displayed on the device and it is ready to use.

#### 【Caution】

The volume of this speaker cannot be adjusted, so to avoid excessive volume playing, please turn down the volume of your music playing device beforehand.

## On second or subsequent connections

Turning on the main power supply of the massage chair or the power supply of the music playing device will automatically connect.

※Multiple devices cannot be connected at the same time. If you want to connect another device, please follow the order of the first connection as described above.

## Reset connection information

If you do not want to connect automatically, follow the instructions for your music player to cancel the connection to "D\_core".


### Caution

- When using it, be sure to read the instructions for your music playing device beforehand.
- Even in the unlikely event that the memory of the music playing device is damaged, we will not be held liable.
- Please turn down the volume of the device in advance.
- If there are electronic devices or transmitters in the area around the massage chair, noise interference may occur. In this case, keep them separate and use them.
- When used near a TV or radio antenna, it may cause noise interference to the TV or radio. In this case, keep them separate and use them.

## USB Port

This massage chair has a USB port on the sub remote of the left wooden armrest. You can connect any type A USB charging cable to charge your smartphone, etc.

USB port

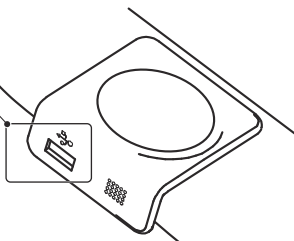


### Caution

- When using it, be sure to read the manuals for your connected device beforehand.
- Even in the unlikely event that the memory of the music playing device is damaged, we will not be held liable.

# Troubleshooting

## Structural status

Check the following items, and if there still are abnormalities, immediately stop using the product to prevent accidents, unplug the power plug from the wall socket, and consult your dealer or distributor (printed on the back cover).

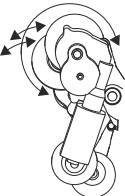Problem	Cause & Solution	
The right and left mechanism rollers are out of alignment.	This occurs because of the adoption of the alternate shiatsu mechanism and alternate tapping mechanism, and it is not a malfunction. 	This occurs because of the structure, and it is not a malfunction.
The rollers stop after moving to the shoulder sections.	It is designed so that the rollers move to the shoulder sections to prevent them from interfering with your body at the end of a massage by the auto timer function.	
During kneading motion, the rollers sometimes skip.	This occurs because of getting caught at the boundaries between the places where the mechanisms are subjected to high load and other places where the mechanisms are not subject to a high load when a person is sitting in the chair, as well as individual differences, such as weight and body shape.	
The rollers stop during a massage.	When excessive force is exerted on the rollers, movements may stop. Slightly lift your body or lightly raise the reclining chair. (Extra caution should be taken when a person weighing 100kg (220 lbs) or more uses the product.)	
A basic motion or special motion does not start.	Turn "OFF" the main power switch at the rear of the main unit, wait for about 5 seconds, and then turn on the main power switch again.	After you perform inspection, if there still are abnormalities, immediately stop using the product.
Automatic shiatsu point search fails. (The rollers are not aligned with the height of the shoulders.)	Sit back in the seat and position a pillow behind your head. Immediately after shiatsu point search, the shoulder position can be adjusted by the shoulder position adjustment button.	
Even when pressing any button on the remote control, it does not work properly. Or, the power cannot be turned on.	Check the following items: ① Check to see if the power plug is securely inserted into the wall socket. ② Check to see if the main power switch at the rear of the main unit is "ON." ③ Check for simultaneous pressing of two or more buttons.	
The power cord and/or plug is abnormally hot.	Immediately stop using the product.	

- Persons other than servicepersons should not perform disassembly or repair.
- Consult your dealer or distributor (printed on the back cover).




### Operating sound

The below sounds will be heard during operation, but they will not affect performance or other functions.

#### Operating sounds

#### The following sounds will be heard during operation.

Sound during tapping motion	Rattling sound
Sound of the rollers rubbing against the fabric during kneading motion	Squeaking sound Grinding sound
Sound of the rollers moving up and down, sound during kneading/shiatsu motion	Whirring sound
Sound during air massages	Tapping sound
Sound of the air pump	Booming sound
Sound of air discharge	Hissing sound
Sound of the reclining chair being raised and lowered	Whirring sound

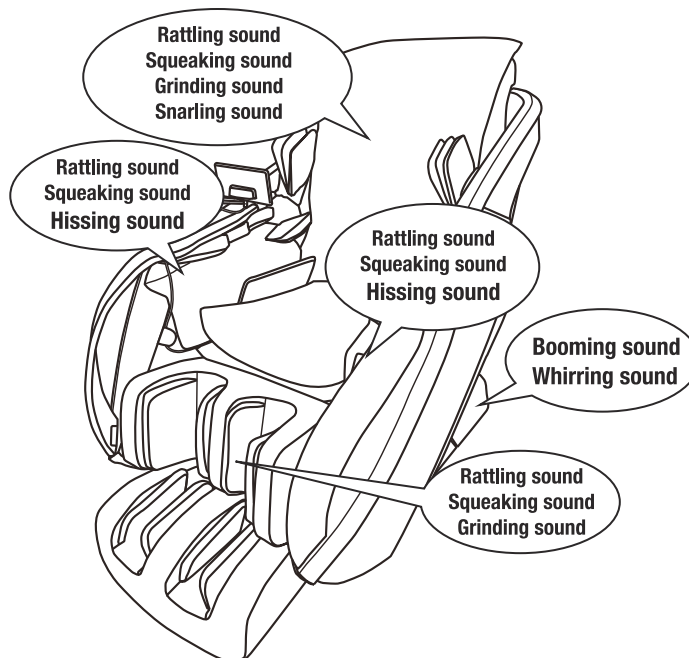

# Inspection for Your Safety

**For long-term use, have the product periodically inspected.**

## **Do you have any of these symptoms?**

- The product sometimes does not operate even if the switch is turned on.
- The power cord is “cracked” or “flawed.”
- The power cord and/or power plug becomes abnormally hot.
- There is a burning odor or abnormal sound or vibration during operation.
- The timer sometimes does not function automatically.
- There are other abnormalities or malfunctions.

If you find any of these symptoms as shown on the left-hand side, stop using the product to prevent malfunctions and accidents, unplug the power plug from the wall socket, and make sure to ask your dealer or distributor (printed on the back cover) for inspection.



## **For safe use**

The fabric on the treating parts is consumable, and its useful life is one year. (Operating conditions: 60 minutes a day, use on a daily basis)

For safe use, it is recommended to have the product periodically inspected by your dealer or distributor (printed on the back cover) before the useful life expires.

However, if the operating conditions are different from above, the useful life will change.

After about 5 years, it is recommended to have the entire chair inspected for your safety.

For costs related to inspection and repair, consult your dealer or distributor (printed on the back cover).

## FAQ

**Q** Is it OK if I place the chair on an electric carpet?

**A** Do not place the chair on an electric carpet as this may cause a fire.

Since the chair is heavy, the heating element of the electric carpet may be damaged, and ignition may occur.

**Q** Is it OK if I install the chair on flooring or floor heating?

**A** There is no problem.

However, since the floor may be damaged, it is recommended to lay a carpet down and install the chair on it. (The same applies to tatami mats or the like.)

**Q** How can I make the massage stronger?

**A** Try the following methods.

- Sit back in the seat so as to apply your body weight to the backrest.
- Set the intensity to "5th level" with the intensity adjustment button.

A stronger massage can be made by the above method.

**Q** How can I make the massage weaker?

**A** Try the following methods.

- Set the intensity to "1st level" with the intensity adjustment button.
- Place the buffer pad on the region to be treated.

A weaker massage can be made by the above method.

**Q** Water spilled on the product by accident. What should I do?

**A** An electric shock may occur. Immediately stop using the product, and follow the procedures given below.

- Turn "OFF" the power switch.
- Unplug the power plug from the wall socket.

After carrying out the above, consult your dealer or distributor (printed on the back cover).

**Q** When a program ends, does the reclining chair move automatically?

**A** At the end of a program, the reclining chair does not move automatically for your safety, but when the power is turned "OFF," the reclining chair is raised automatically.

**Q** Can a person less than 150cm (4.92ft) or taller than 180cm (5.91ft) in height use the chair?

**A** Yes.

**Q** Is it OK to use this product if I am a heavy person?

**A** Water spilled on the product by accident. 120kg (264 lbs) or less should use the chair.

# Maintenance and Storage Procedures

## Maintenance procedure



### Caution

- For maintenance, make sure to unplug the power plug from wall socket. In addition, do not plug in or unplug the power plug with wet hands.

### [Maintenance of sewn products]

Wipe off dirty adherents with a soft cloth wrung out of lukewarm water in which a small amount of neutral detergent has been dissolved.



### Caution

- If the product is in contact with vinyl, interior wallpaper, etc., for a long time, discoloration may result.

### [Maintenance of plastic]

Wipe with a soft cloth that is first wrung with lukewarm water in which a small amount of neutral detergent has been dissolved, and then wipe thoroughly with a dry cloth so that no detergent remains.

- Do not use alcohol or other solvents as well as polishing powder for maintenance as this may result in flaws, discoloration, or cracking.

What is appropriate	What is not appropriate

### [Maintenance of the wooden panel, wooden handrails, and remote control]

Wipe dirt off the wooden panel, wooden handrails, and remote control with a dry cloth.

- Never wipe with a wet towel or the like.

→ May result in a malfunction.

## Storage procedure

- If the product is not being used, cover it to prevent adhesion of dust. In addition, perform maintenance for the next use, and store it in a place with little dampness together with the included items.
- In order to prevent your pet(s) from biting the cord, keep your pet away from this equipment during storage.



### Caution

- Do not install the product in a place subjected to high temperatures, such as an area exposed to direct sunlight or near a stove.

## 40




# Specifications

Product Name	DC-200A
Power Supply	AC120V
Rated Power Consumption	250W
Rated Frequency	60Hz
Rated Time	60 min.
Kneading Speed	6 times/min. at minimum, 19 times/min. at maximum
Tapping Speed	170 times/min. at minimum, 410 times/min. at maximum
Shiatsu Speed	[Back] 19 times/min. at maximum [Arm] 18 times/min. at maximum [Leg] 15 times/min. at maximum [Sole] 18 times/min. at maximum
Roller Up-and-down Movement Speed	2.5 cm / sec
Massage Range	90 cm
3D Range of Rollers	8 cm
Air Pressure	Approx. 30 kPa
Heater	45°C at maximum
Auto Programs	13 types
Manual Programs	6 types
Program Timer	Approx.10~30 min.
Anti-overuse of timers	Approx. 90 min. (In continuous operation)
Reclining Angle	Approx. 113° to approx. 145° from the horizontal Stepless (electric reclining)
Main Unit Dimensions	Width approx. 94cm x depth approx. 149 (184)cm x height approx. 126 (94)cm (Width approx. 37 inches x depth approx. 59 (72) inches x height approx. 50 (37) inches) Regarding depth and height, figures in parentheses apply to a state in which the backrest is fully reclined.
Mass	Approx. 128kg (282 lbs)
Exterior Fabric	Artificial leather
Included Items	Power cord, buffing pad, Sole pads
Packing	2 packages/unit



# Memo

---

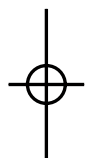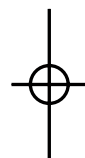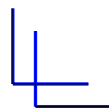

Made in Japan

Manufactured by NIN Co., Ltd.

Postal Code: 107-0061

#505 Mezon-Aoyama

2-7-26 Kitaaoyama, Minato-Ku, Tokyo, Japan

Date of revise: April, 19, 2024

Date of issue: March, 15, 2024

3BC00700510101

