



Fiber Handpieces Cleaning & Sterilization

Cleaning Procedure

Fiber Handpieces should be cleaned immediately after use due to occasional build-up of tissue, blood & debris.

- Clean all external surfaces with a solution of detergent & water. Use a small wire or small tube brush to clean the lumen.
- Rinse thoroughly- use an air gun to remove liquids in the lumen.
- Remove the rear sealing cap by unscrewing and set aside. Remove the rubber bushing or fiber gripper with a pointed instrument. Clean the seal housing and seal. Ultrasonic cleaning with a non-corrosive solution is recommended.
- After cleaning, Place a small amount of silicone lubricant onto the screw threads of the rear sealing cap. Stow rear sealing cap and fiber gripper in the designated area in tray. Do not assemble.

Sterilization Procedure

CYCLE:	PRE-VAC	PRE-VAC
TEMPERATURE:	270° F (132°C)	273° F (134°C)
MINIMUM STERILIZATION TIME:	4 Minutes	4 Minutes
MINIMUM DRYING TIME:	30 Minutes	8 Minutes