



Model: Stack-Mini (Stack)

7/25/2025

Your Scleral & Hybrid (Hard) Lens Insertion Tool

Congratulations on acquiring Stack-Mini, which utilizes previously patented and successfully used technology to enhance the insertion of Scleral and Hybrid Hard Lenses.

Our Scleral Lens users have found that having light shine up through the Vented Cup provides a precise focal point, allowing them to insert larger lenses while reducing trapped air bubbles, misalignments, and hassle. --- Your Stack-Mini also uses our exclusive Cushioned Pad and Soft Light to help the lens insertion process go quickly and easily.

The following will help you get the most out of your Scleral Lens Insertions. Some customers have instant success, while others may take a few hits and misses. Use the Stack for at least 10 days from the date of delivery. If, after 10 days, you're not completely satisfied with our product, you'll have an additional 20 days to contact us and initiate your refund. Just like with many first-time used products, there's a bit of a learning curve. Give Stack and yourself a fair chance. We won't penalize you if Stack doesn't meet your needs, and we will issue you a full refund when Stack is returned. We will pay for the return postage as well.

To Start.

- 01: Thoroughly wash and dry your hands.
- 02: Unscrew the bottom of Stack and remove the plastic battery insulating chip. (Save chip)
- 03: Screw the bottom back onto its other half and tighten until the O-ring seals and the light stays on.
- 04: Place and level the Scleral or Hybrid Lens on top of the Vented Cup.
- 05: Fill the lens with enough solution to rise above the edge of the lens. Some spillovers are okay.
- 06: If needed, use both hands to hold your eyelids completely open.
- 07: Position your head over Stack until the light shines directly into the eye receiving the lens.
- 08: Slowly lower your head until your eye reaches the solution, and the lens pushes into place.
- 09: Close your lid and pull away to secure the lens onto your eye.
- 10: Check vision by looking for trapped air bubbles. If you have bubbles, reinstall the lens.
- 11: To insert the other lens, repeat steps 04 through 10.

After both lenses are inserted, and while the light is still on, rinse Stack under warm running tap water to prevent Saline Solution buildup. Then dry and rotate the switch bottom counterclockwise until the light remains off.

If you plan to travel with Stack, place the dime-size round plastic insulation chip between the battery and contact. This keeps Stack from accidentally turning on during transit.

Stack-Mini, Stack-Jr, and Stack-SL1 use globally sourced components. However, all remanufacturing, designing, assembly, testing, and distribution are performed in America!


Stack has a full TWO-year replacement warranty!

If it fails, we will send you a replacement at no charge when the original product is returned to our office. --- The warranty is void if the Stack has been damaged or has missing parts. Normal wear is acceptable. You can go to www.GetStack2See.com and start the warranty replacement process.

Stack can use one or two CR2032 Lithium Batteries. -- Two give a brighter light.

When installing batteries, polarity is essential. The CR2032 battery has a wider side with etched battery information. The etched side touches the + mark on the base of the battery compartment, and the smaller side faces the coiled contact spring.

When returning Stack for a refund or replacement, remove and keep the batteries.

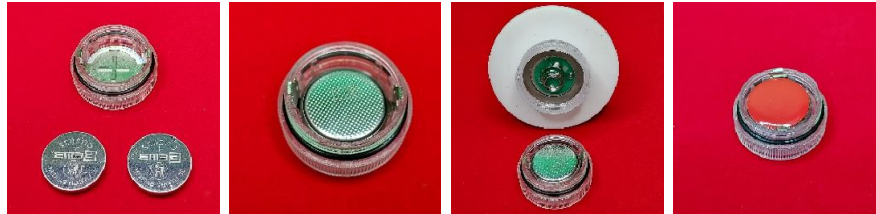

⚠ WARNING	
<ul style="list-style-type: none">• INGESTION HAZARD: DEATH or serious injury can occur if ingested.• A swallowed button cell or coin battery can cause Internal Chemical Burns in as little as 2 hours.• KEEP new and used batteries OUT OF REACH OF CHILDREN• Seek immediate medical attention if a battery is suspected to be swallowed or inserted inside any part of the body.	



U.S.M.C. Veteran & Family Owned Small Business in Menifee, California.	

John Baker: Direct Line... 951-515-9862

See Details & Videos at: www.GetStack2See.com

GetStack2See – 28257 Hearthside Dr. Menifee, CA 92584