



Stack-Mini



Stack-Jr



Stack-SL1

All Stack units turn **on** and **off** in the same way. The **RED plastic chip**, located in the round battery compartment at the bottom of the Stack, prevents the unit from turning on during transit.

#1: The first time you use Stack, unscrew the bottom half, remove and store the plastic RED chip inside of Stack's container/jar.

#2: Reattach the bottom half and tighten until the O-ring seals and the light stays on.

#3: After inserting lenses, rinse Stack with tap water to remove the saline solution.

#4: After rinsing/using, loosen the bottom until the light stays off.

Twist on - Twist off!

Please call me if you need help or have questions.

John Baker: Direct Line... 951-515-9862

See Details & Videos at: www.GetStack2See.com

GetStack2See – 28257 Hearthside Dr. Menifee, CA 92584