



**** We are changing our name from TheGreenBullsEye to GetStack2See. We believe our new name is more specific to what our product can do for you.**

Model: Stack-SL1

Your Scleral Lens Insertion System

Congratulations on your eye care investment in acquiring STACK-SL1. This product helps secure peace of mind by dramatically improving your lens-eye

alignment, giving you a more precise and comfortable fit when inserting your lenses.

Our Scleral Lens users have found that having light shine up through the Vented Cup gives them a clear point to focus on, allowing them to insert large lenses while reducing trapped air bubbles and misalignments.

When removed from the box, your silicone table mat will have wrinkles, which will fade away over time. To speed up the process, you can place the mat in hot tap water and, when warm, lay it out on a flat surface to cool.

Follow these instructions to ensure a hassle-free lens insertion.

The 'Stack' product family is designed to help you insert lenses, **NOT REMOVE THEM**. Your eye care professional will have specific information about lens care and upkeep, including the required wet solutions.

The following will help you get the most out of your Scleral Lens Insertions. Some customers have instant success, while others may take a day or two. This is especially true when transitioning from a regular contact lens to a large Scleral Lens. Give it time. You have 30 days to initiate a return refund when purchasing through Amazon. Use Stack for at least 10 days. The Stack success rate is over 90%. Of course, not every product will work for every person. Give it a fair try.

To start.


- 01: Wash and thoroughly rinse your hands, then dry using a Lint-Free towel.
- 02: Rotate the switch on the bottom of the white flexible shaft clockwise until the focal point light stays on and the watertight seal is compressed. (Remove plastic insulation chip, first)
- 03: Three black Foam pads can be added to the bottom of your Stack for added stability and to prevent sliding. Use if desired.
- 04: Install and level the Vented Cup on the top of Stack's Shoulder.
- 05: Put the Lens on top of the Vented Cup while keeping everything level.
- 06: Fill the lens with enough solution to rise above the edge of the lens. It's okay if there's a spillover.
- 07: If needed, use both hands to hold your eye thoroughly open.
- 08: Position your head over 'Stack' until the focal light shines directly into the eye receiving the lens.

- 09: Slowly lower your head until your eye reaches the solution, and the lens pushes into place.
- 10: Check vision. If it's not clear, look for air bubbles under the lens. If you have bubbles, reinstall the lens. (There could be a slight learning curve to get it right)
- 11: To insert the other lens, repeat steps 04 through 10.
- 12: When completed, remove and rinse the tan-colored Vented Cup with 70% isopropyl alcohol. Then, AFTER you wipe down the 'Stack' with a Lint Free towel, rotate the switch dial counterclockwise until the light remains off. --- It's recommended that the 'Stack' assembly be rinsed under clean running water at least weekly to prevent the saline solution from building up. MAKE SURE the O-ring seal is compressed whenever you use or rinse your 'Stack.'
- 13: If you plan to travel with 'Stack,' place the dime-size round plastic insulation piece between the battery and contact. This keeps 'Stack' from accidentally turning on during transit.

Stack-SL1 is guaranteed to function correctly for two years. If it fails, we will send you a replacement at no charge when the original product is returned to our office. --- The guarantee and warranty are VOID if your 'Stack' has been physically damaged, misused, or missing parts. Normal wear is acceptable. Your Stack-SL1 operates from two CR2032 Lithium Batteries (3.0V) and will provide output light for months.

When installing batteries, polarity is essential. The CR2032 battery has a wider side with etched battery information. The etched side touches the + mark on the battery compartment's base, and the smaller side faces up toward the contact spring. Stack works using one or two batteries. Two batteries give a brighter light.

When traveling, insert the plastic chip into the battery compartment to prevent an accidental power-on condition. (save the chip)

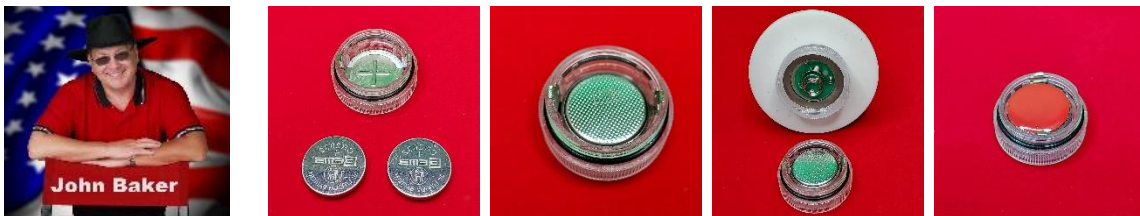

REMOVE AND PROPERLY DISPOSE OF THE BATTERIES BEFORE YOU MAIL STACK.

Text John Baker at 951-515-9862 if you have any questions.

