

Drape Unfolding Instructions



REF 202455

QTY 1

LOT



STERILE EO




Manufactured for Variamed, LLC • Chattanooga, TN

Licensed under US Patent # 6,019,102

US Patent # 8459265 B2

CAUTION: Federal (US) law restricts this device to sale by or on the order of a physician.

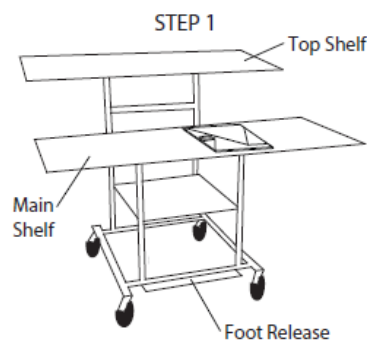


Using the foot release, push main shelf back to innermost setting. Center drape on main shelf with **label facing up** and **arrow #1** pointing left.



Pull arrow tab #1 **left** and over the left edge of the table. Pull arrow tab #2 **right** and over the right edge of the table.



Pull arrow tab #3 **up** and towards back of table, covering top shelf.

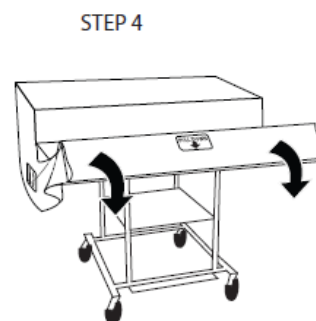

Pull arrow tab #4 **down** and forward, covering front of main shelf.



Using the foot release, pull main shelf forward to desired working position.