



Soft Tissue Trimmer – **NEW PRODUCT!**

USE WITHOUT WATER SPRAY

Use at speeds of 300,000 RPM - 450,000 RPM

Run the Tissue Trimmer for 30 to 45 seconds before use to allow heat generation at the tip.

Tissue Trimmers generate enough heat to cause tissue coagulation, which allow gingival modeling almost entirely without bleeding or damage to the tooth. It can be used as an alternative to a scalpel or electro-surgery.

Some Applications include:

- Crown lengthening
- Recovering Implants
- Contouring of gingiva for cosmetics or inflamed gingival soft tissue
- Widening the sulcus for impressions
- Exposing deep cervical caries or external resorption
- Operculectomy
- Exposing partially erupted wisdom or fractured teeth
- Excising granulation tissue

