

FreeStyle Comfort MDD Product Update Overview



Visual Changes

■ Product Label

■ Menu Button


- Press to access Settings Menu Screen:
 - Status Screen
 - Airplane Mode On/Off Screen
 - Breath Sensitivity Adjustment Screen
 - autoSAT On/Off Screen
- If no activity within 20 seconds of pressing Menu Button, will revert to main screen

■ autoSAT Indicator

- autoSAT On/Off Feature


■ Bolus Delivery Indicator

- “Blinks” with each dose delivered

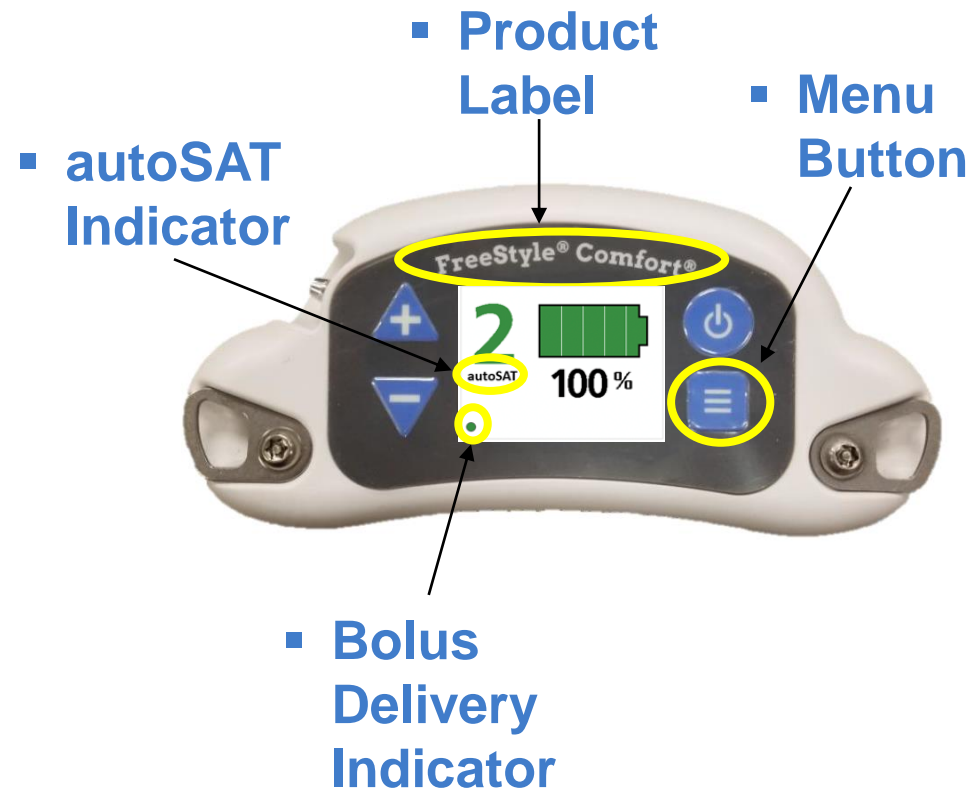

Settings Menu: Status Screen

- @ Main Screen, press the Menu Button Once
- Hour Meter
 - Displays the total number of hours on the machine (Not Adjustable)
- Battery Discharge Cycle Meter
 - Displays total number of discharge cycles on battery currently attached to the Comfort
 - **Estimated** Service Life of FreeStyle Comfort Battery is 300 charge/discharge cycles
- Exit Symbol
 - Press Menu Button next to Exit Symbol at any time to return to main menu
- Firmware Version
 - Displays Firmware Version of FreeStyle Comfort
- Menu Scroll Symbol
 - Press “-” button to scroll to next Setting Menu Screen

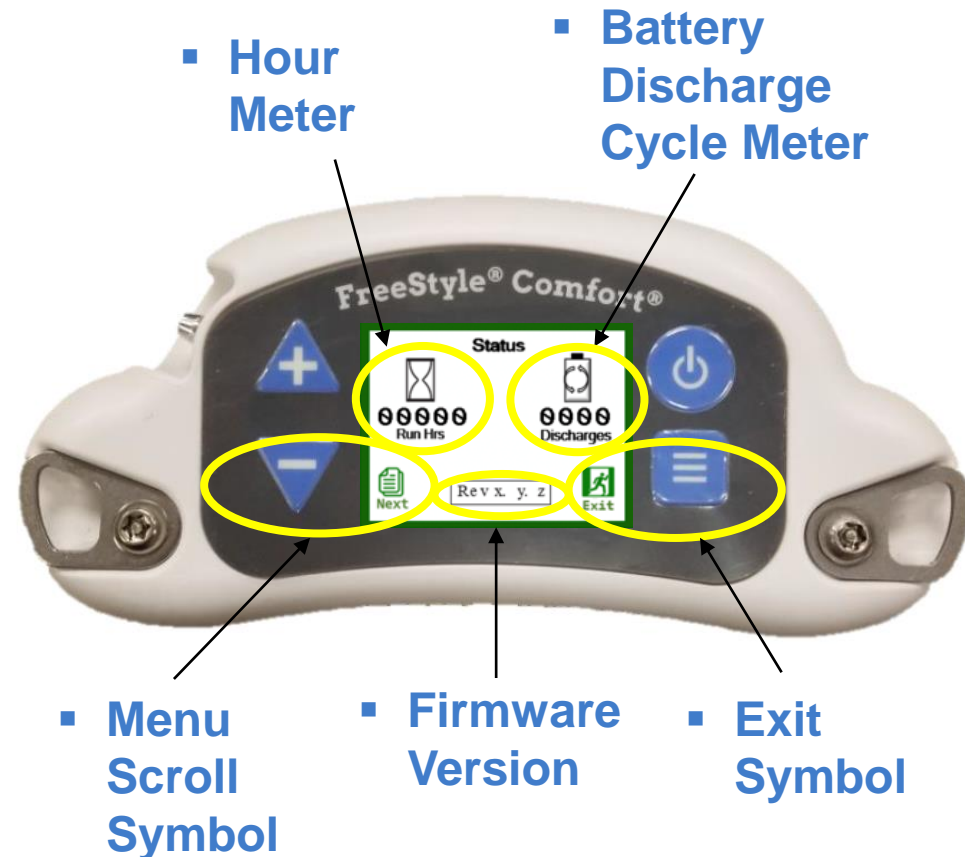

Settings Menu: Airplane Mode

- From Status Screen, Press “-” to scroll to Airplane Mode Screen
- On Airplane Mode Screen, Press “-” to scroll to the next screen
- Press **Menu Button** next to **Exit Symbol** at any time to return to main menu
- Press “+” to toggle Airplane Mode On/Off
 - After changing selection, text color of “On” or “Off” will illuminate blue
- Press **Menu Button** at any time to Cancel any setting change
- Press “-” to Save the Selection
 - NOTE: If Save is not selected within 20 seconds, the selection will revert to original setting and will be illuminated in black text
 - After Saved, “On” or “Off” will illuminate black



▪ Menu Scroll Symbol

- Change Selection Symbol



▪ Menu Scroll Symbol

▪ Exit Symbol



▪ Save Symbol

▪ Cancel Symbol

Settings Menu: Breath Sensitivity

- From Airplane Mode Screen, Press “-” to scroll to Airplane Mode Screen
- On Breath Sensitivity Screen, Press “-” to scroll to the next screen
- Press **Menu Button** next to **Exit Symbol** at any time to return to main menu
- Press “+” to adjust breath sensitivity setting
 - Continue pressing “+” to obtain desired setting.
 - After changing selection, text color of setting number will illuminate **blue**
 - **NOTE:** Setting 3 is the Default Factory Setting
 - **NOTE:** Setting 1 is the **LEAST** sensitive, Setting 5 is the **MOST** Sensitive
- Press **Menu Button** at any time to Cancel any setting change
- Press “-” to Save the Selection
 - **NOTE:** If Save is not selected within 20 seconds, the selection will revert to original setting and will be illuminated in black text
- After Saved, text color of setting number will illuminate black



▪ Menu Scroll Symbol

- Change Selection Symbol



▪ Menu Scroll Symbol

▪ Exit Symbol



▪ Save Symbol

▪ Cancel Symbol

Settings Menu: autoSAT On/Off Screen

- From Breath Sensitivity Screen, Press “-” to scroll to autoSAT Screen
- On autoSAT Screen, Press “-” to scroll to the next screen (Status Screen)
- Press **Menu Button** next to **Exit Symbol** at any time to return to main menu
- Press “+” to toggle autoSAT Mode On/Off
 - After changing selection, text color of “On” or “Off” will illuminate blue
 - **NOTE:** autoSAT ships from factory in “Off” mode
- Press **Menu Button** at any time to Cancel any setting change
- Press “-” to Save the Selection
 - **NOTE:** If Save is not selected within 20 seconds, the selection will revert to original setting and will be illuminated in black text
 - After Saved, text color of “On” or “Off” will illuminate black



▪ Menu Scroll Symbol

- Change Selection Symbol



▪ Menu Scroll Symbol

▪ Exit Symbol Symbol



▪ Save Symbol

▪ Cancel Symbol