

Obtaining a DNA Sample from Toenail


You will need some quick stop (blood clotting powder) and toenail clippers.

You will want to clip a toenail, but I usually clip about half of the nail off. Definitely in the darker part so you draw blood. Hold her toenail to the circle on the card and squeeze her toe to make the blood come out. If it is not bleeding well, you may have to cut more off. Try to fill the whole circle if possible.

Apply the quick stop powder to get her toe to stop bleeding. Let card air dry for 10-15 minutes. Place back in baggie. Fill out rest of submission form, and include payment (write credit card number on form, or include check for \$19.50).

I usually place submission form and blood card in a weather proof envelope. :) Make sure to check the box as to how you want your results sent! I do email... they are very fast!

Mail to:
Animal Genetics, INC
1336 Timberlane Road
Tallahassee, FL 32312

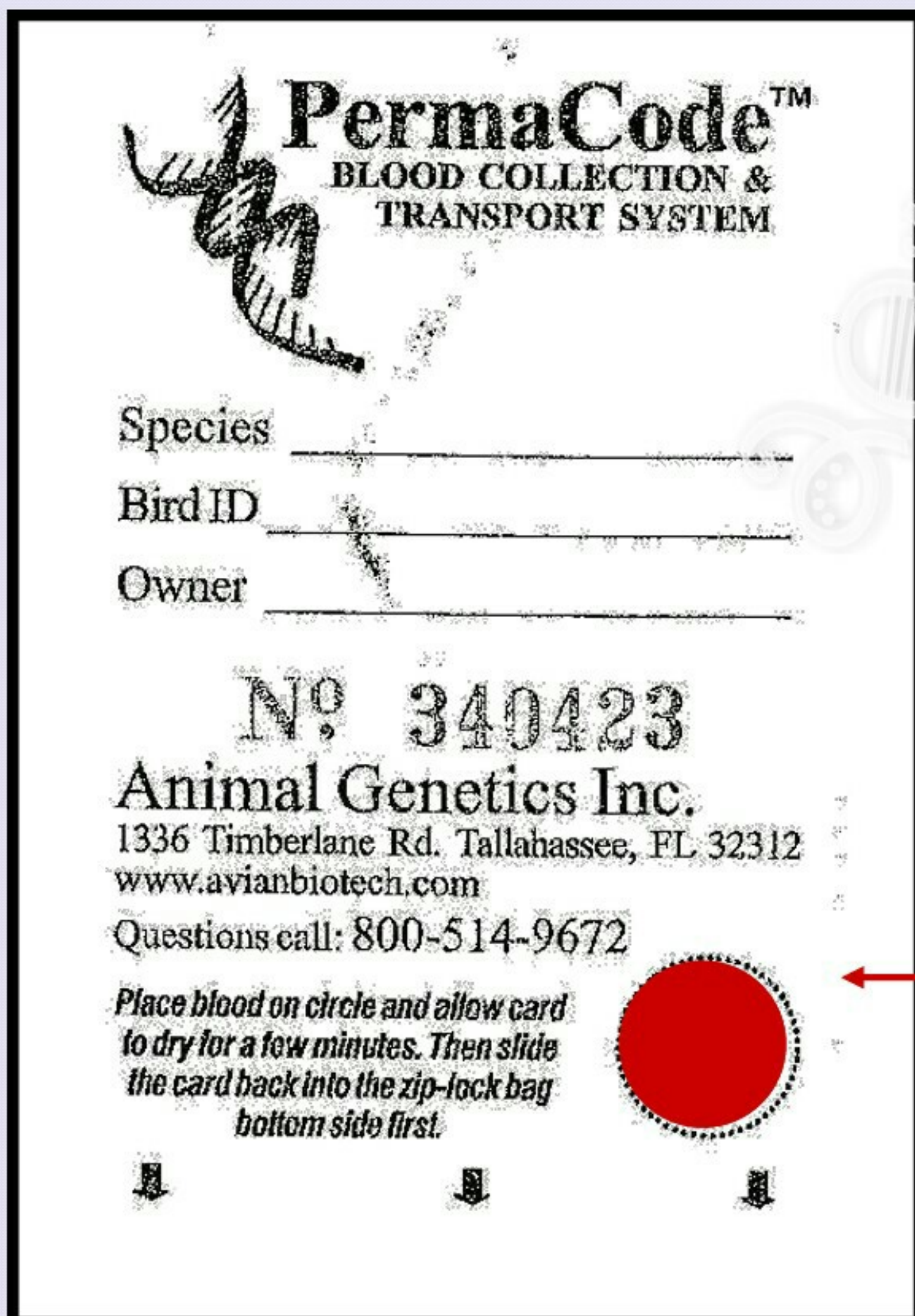


PermaCode™
BLOOD COLLECTION &
TRANSPORT SYSTEM

Species _____
Bird ID _____
Owner _____

Nº 340423
Animal Genetics Inc.
1336 Timberlane Rd. Tallahassee, FL 32312
www.avianbiotech.com
Questions call: 800-514-9672

Place blood on circle and allow card to dry for a few minutes. Then slide the card back into the zip-lock bag bottom side first.

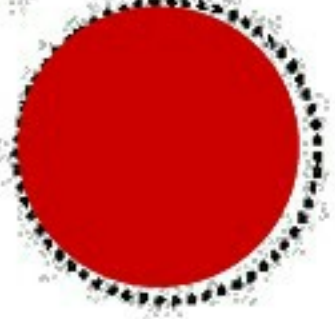

**Press toe
to
this circle!**