"Kacey Products Providing Better Animal Care"

Micropot 900 The ONLY Vet meter with 20-900mg/dl linear range! This highly accurate, capillary action loading meter is designed for Canine and Feline testing in the clinic or at home with results in 5 seconds!

When you need to know if the insulin is reducing the glucose level and when monitoring is essential for the sliding scale requirements for insulin stabilization of the animals.

- Fast: One Step 5 Second Test •
- **Easy:** Simply touch the test strip to the blood site
- Accurate: Linear range from 20~900mg/dl (controls available)
- **Ergonomic:** It fits in your hand (3.7"x2"x 0.6") at just 1.7oz



1. Insert the Glucose Test Strip and check code number.



2. Prepare for a drop of capillary blood from the animal's ear.



KACE

 \odot

108

Authorized Dealer:

3. Apply capillary blood sample until the meter beeps.

4. Wait 5 seconds for results to be displayed. Contact us or your local KACEY dealer for more information about the Kacey Microdot 900 meter.

CE

FOR VETERINARY USE ONLY

Micro Dot



"Check Code" chip Lithium Battery 3V

Glucose Meter Lancing Device

10 Sterile Lancets

10 Capillary action

- (CR2032)
- Carry case

test strips

Kit Includes:

User quide



MicroDot is a trademark of Kacey Inc. © 2016 All Rights Reserved.