

Kacey® Ethylene Glycol & Oxalate Veterinary Combo Strips

“The only ONE step, TEN minute test for Canine and Feline use!”

We're proud to introduce the new Antifreeze Combo Pak the only combination test to effectively track **Stage 1** EG in the blood up to 9-10 hours AND its conversion into the insoluble crystals forming **Stage 2** Calcium Oxalate 9+ hours later. **A critical test for survivability when initial exposure is unknown.** Only 2 drops of plasma with results in 10 minutes!

**“Timing Is Everything,
When Treating Ethylene Glycol
Poisoning**



EG/OX Test Specifications:

Feline sensitivity @ 10mg/dl

Canine sensitivity @ 10mg/dl

Stage 1 Ethylene Glycol 9 –10hrs

Stage 2 Oxalate 9+ hrs.- 30Hours


Only 2 drops of plasma needed


Detects EG 30 min. after ingestion

Results in 10 minutes

Two color comparison chart

Minimum 12-14 month Exp, Date
at the time of shipping

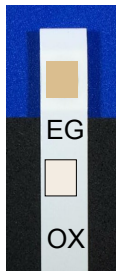



Ethylene Glycol (Stage-1)

Neg 0

10

20

50

75 mg/dL

Canine/Feline Lvl of Detection

Feline I.D.

Canine I.D.

10µl of plasma needed - Place on pad - Window of opportunity approx. 8-10 hrs. - EG can be detected approximately 30 min after ingestion - Results in 10 minutes

Veti-Stx™

- Two Parameter EG & OX
- For Canine & Feline
- In Vitro Use Only
- Store in Refrigerator
- Veterinary Use Only
- Requires Plasma

(See insert for more details)

Part#: 30310

Qty: 5

Lot #:

Exp #

Oxalate (Stage-2)

NEG

0.25

0.5

1

1.5 mg/dL

10µl of plasma needed - Place on pad, compare to chart - Window of opportunity **POST** 8-10 hrs. -to approx. 24-36 hrs. Test with Ethylene Glycol concurrently - Results in 10 minutes

1854A Hendersonville Rd, Suite 112
Asheville, NC 28803
Ph 828-685-3569

www.kaceydiagnostics.com

Kacey Inc. © 2019 All Rights Reserved.

KACEY® EG & Ox Combo strips packaged in either 5 strip #30310 or 10 strip #30332 bottles for your convenience. Convenient 10 ul pipet and pipet tips are also available. Strips can be read manually or on the NEW Kacey SDR (Smart Diagnostic Reader).

***The antidote for Stage 1 EG poisoning (4MP)
is available from Kacey Diagnostics.**

Contact:



**1854 A Hendersonville Rd
Asheville, NC 28803
Ph: 1(828)685-3569
Fax: 1(828)685-3571
www.kaceydiagnostics.com**

“Kacey Products Providing Better Animal Care”