
# Instructions For Use (IFU)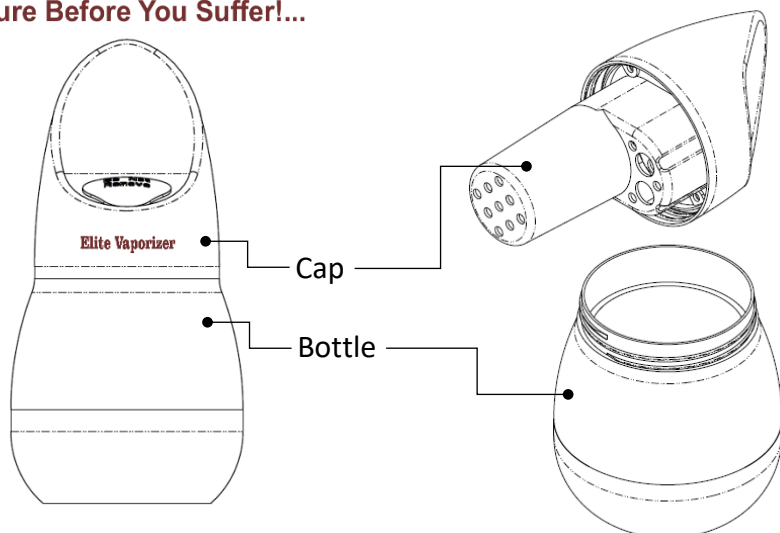

## USEFUL FOR

- Common cold
- Decongestion
- Throat infection
- Bronchitis
- Sinusitis
- Asthma
- Laryngitis
- Aayurvedic treatment
- Aroma treatment

## CHARACTERISTICS FEATURES

- Double walled plastic body
- Light weight attractive body
- Cable 1.5 m length
- Shock proof
- Specialized Lock system preventing the leakage of hot water and burn

## HOW TO USE

- Open the cap by anticlockwise motion.
- Fill half of unit with water and close the cap properly
- Water should be more than minimum level marked on the container
- Plug into the socket and turn on the switch
- After few minutes vapors starts to come out
- Bring the nose close to the vaporizer and inhale steam
- Switch off once finish inhaling

- Use Regular Drinking water only
- Do not use RO Water
- Remove plug from socket when filling or refilling the container
- It is recommended that do not use vaporizer more than 5 min in single use

## HISTORY

- Inhalation of plain or medicated steam has considered as one of the best treatment for cough, loosening the lung congestion, and throat irritation .
- Steam inhalation relieves the inflammation of mucous membranes in acute cold and sinusitis.
- Irritation in bronchitis and whooping cough moistening can be achieved by steam inhalation
- Steam inhalation provides the antiseptic action on the respiratory tract
- Steam inhaler is extremely effective and an efficient equipment for home and clinic use .
- It produces controlled amount of steam continuously & u uniformly which makes it easy and convenient to use.

## SAFETY TIPS

- While filled with hot water keep it straight do not tilt
- Keep it away from children
- Avoid touching the plug with wet hands
- Handle gently with care
- Avoid use of very hard water
- As it produces uncontrolled steam

### Manufactured By:

**PSP TechnoCADD Ancillary Private Limited,**  
Aurangabad, Maharashtra

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