


# Do it Yourself

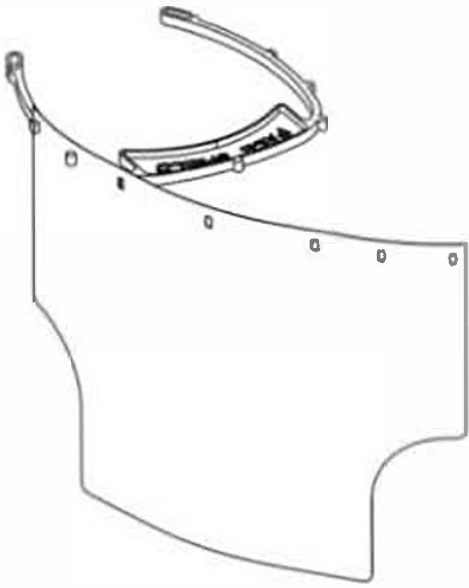

## STEP 4

Align the next consecutive hole - latch and gently push the transparent shield all the way up to the base of the latch



## STEP 2

Align any extreme perforated hole on transparent face shield with the extreme L-latch of the forehead clip



## FACE SHEILD IS READY TO BE USED


Repeat the procedure for proceeding latches until other extreme L - shaped latch is mounted.

In case if you need additional fit, extreme ends of the forehead clips are capable of adapting a rubber band/ elastic.



## STEP 3

Gently slide in the L - Shaped latch through the first hole

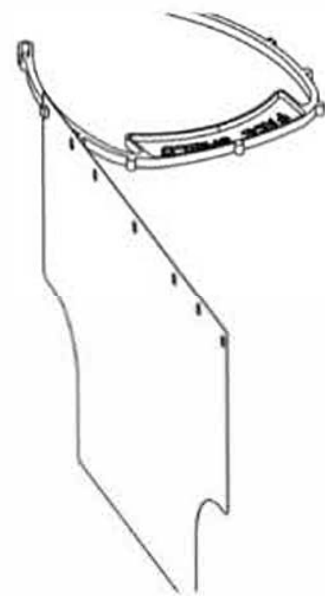

## STEP 1

Orient transparent shield with perforated holes upward and forehead clip with face shield embossing upwards, as shown in picture