

GLUCO-CARE[®]

SAFETY PEN NEEDLES



Safe

Clear safety shield automatically locks after use to protect the needle

Simple

Color indicator becomes invisible after engagement of safety mechanism

Ergonomic Design

Ergonomic main housing with ribs for comfortable grip

Universal

Compatible fit with most pen injectors

SAFE

- Automated shielding mechanism ensuring that needle is covered before and after use
- Single-use mechanism preventing sharps injuries and reuse of the device
- All safety pen needles are ETO-sterilized and protected with cover and sterility peel-off tab

SIMPLE

- Safety mechanism engagement indicator via "Click" sound and disappearance of color coded inner shield after fully inserting the needle into the skin
- Optimally designed, clear shield for precise placement on injection site with improved visibility

GENTLE

- Design with concealed needle helps to relieve patient's anxiety
- Polished, silicone-coated, ultra-thin needles are ideally positioned during skin penetration thanks to precise tracks for the safety mechanism elements enhancing patients comfort
- Ribbed housing provides a comfortable grip for easy attachment and removal from the pen

UNIVERSAL

- Universal design for compatible use with most injection pens available on the market
- Tight fit prevents leakages of insulin



UNIVERSAL STANDARDS


- Rigorously tested per international standards for fit, functionality and reliable performance

Compatible

- Tight fit prevents leakages and medication loss.
- Universal design for compatibility with most injection pens available on the market that meet the requirements of ISO 11608-1.

INSTRUCTIONS FOR USE


- Intuitive handling with simple steps. Insulin delivery has never been easier...

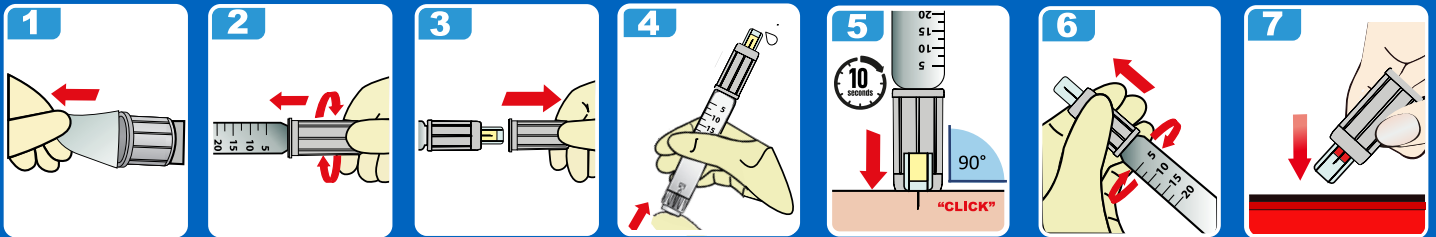

CAUTION:


Do not use if the protective seal or the outer cover have been previously removed or damaged.

www.gluco-care.com/IFU

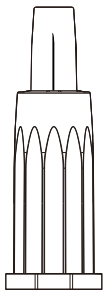


BEFORE USE

Needle tip is covered by the transparent shield to prevent accidental needle stick.

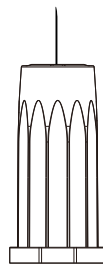

DURING INJECTION

Needle tip is exposed when the shield is depressed during injection.

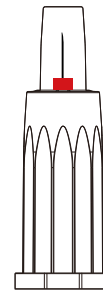

AFTER INJECTION

Shield is permanently locked (red mark as activation indicator) to reduce the risk of accidental needle sticks.

PRODUCT SELECTION CHART

REF	PRODUCT	COLOR	PACKAGING
3111	GLUCO-CARE® Safety Pen Needles 31G x 5mm		100 pcs / box
3112	GLUCO-CARE® Safety Pen Needles 30G x 8mm		100 pcs / box

www.gluco-care.com

MediVena[®]
inspired by innovation

MediVena

1 Glenlake Parkway, Ste 700, Atlanta,
GA 30328, USA

+1 404 514 2586 info@medivena.com

CUSTOMER SERVICE
TOLL FREE: 888 270 6457

MediVena Sp. z o.o.

Al. Jana Pawła II 27,
00-867 Warsaw, POLAND

+48 695 995 700 info@medivena.com