

MicroFlex Stand


Assembly

Instructions



Capital Medical Resources LLC
4002 North Hampton Dr
Powell, OH. 43065
O. 614.389.1334
M. 614.657.7780
F. 614.467.3923
E. info@capitalmedicalresources.com
W. www.capitalmedicalresources.com

Assembly

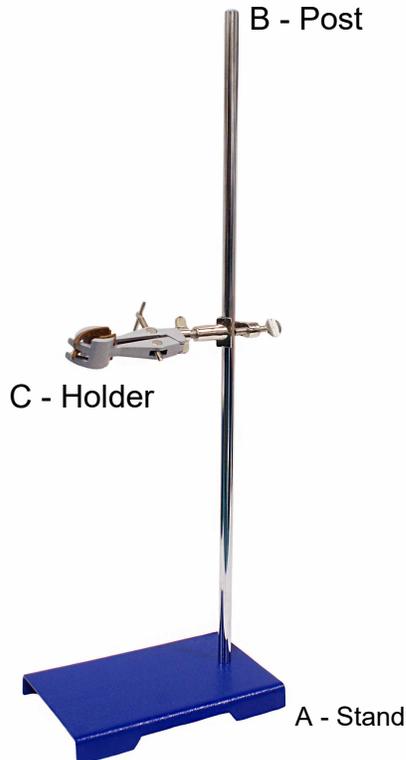

The MicroFlex stand comes in 3 pieces.

- A. Base (blue)
- B. Post
- C. Adjustable Holder

Step 1: Remove the black cap on the post (B) to expose the threads on the post.

Step 2: Screw the post into the base (A)

Step 3: Loosen the back nut on the holder (C) enough that it will slide over the post (B). Tighten to secure in place. You may loosen and tighten to adjust the height of the holder to the desired position.

Step 4: Loosen the T-bar to open and close the grasping portion of the holder. Place the body of the MicroFlex scope in the holder and tighten the T-bar to secure.

Important: Position the scope so that you do not impede the focus mechanism on the scope.

Step 5: Adjust the angle of the holder by loosening the middle nut (between the t-bar and back nut) and tighten to secure when you have the angle at the desired position.

To clean, simply wipe down with a medical grade antiseptic wipe.



Back nut



T-bar



Middle nut



Leave focus mechanism accessible