

SARS-CoV-2 Antibody Seroconversion Panel
MRNDx PANEL ID: MRNCOV-504, Donor ID: 774449

All Plasma was collected from a 37 year old Hispanic male donor. The donor tested PCR positive for SARS-CoV-2 on 10/27/2020.

No Symptoms. Patient was not Hospitalized.

Panel ID	Abbott Architect SARS-CoV-2 IgM (Cut-off ≥ 1.0)	Abbott Architect SARS-CoV-2 IgG (Cut-off ≥ 1.4)	Chembio DPP SARS-CoV-2 IgM/IgG Rapid IgM (Cut-off ≥ 24)	Chembio DPP SARS-CoV-2 IgM/IgG Rapid IgG (Cut-off ≥ 24)	Ortho Eci SARS-CoV-2 IgG (R&D) (Cut-off ≥ 1.0)	DATE OF LAST COVID SYMPTOMS	DAYS SINCE LAST SYMPTOMS	DOD
MRNCOV-504-01	0.36	0.71	3	3	0.01	10/27/2020	6	11/2/2020
MRNCOV-504-02	1.7	0.67	19	3	0.02	10/27/2020	9	11/5/2020
MRNCOV-504-03	15.73	4.97	187	143	7.77	10/27/2020	16	11/12/2020
MRNCOV-504-04	21.92	5.65	218	124	6.41	10/27/2020	21	11/17/2020
MRNCOV-504-05	17.64	5.84	196	162	5.31	10/27/2020	27	11/23/2020
MRNCOV-504-06	22.6	6.17	164	117	6.19	10/27/2020	34	11/30/2020
MRNCOV-504-07	18.69	6.07	152	168	6.1	10/27/2020	37	12/3/2020
MRNCOV-504-08	18.8	6.28	153	166	6.28	10/27/2020	42	12/8/2020
MRNCOV-504-09	13.01	5.63	85	106	5.22	10/27/2020	48	12/14/2020
MRNCOV-504-10	12.4	5.41	109	81	5.07	10/27/2020	52	12/18/2020

Abbott Architect SARS-CoV-2 IgM - Kit Lot#25047FN00 - EXP:06/06/2021, Abbott Architect SARS-CoV-2 IgG - Kit Lot#25159FN00 - EXP:06/29/2021

CE - Chembio DPP SARS-CoV-2 IgM/IgG Rapid Test - Kit Lot#2CV120420/A - EXP:10/28/2022, Ortho Eci SARS-CoV-2 IgG (R&D) - Kit Lot#0335 - EXP:06/06/2021

All plasma was collected at a U.S. Licensed Plasmapheresis Center.

The Plasma was tested and found Negative for Anti-HCV and Anti-HIV 1/2 by FDA approved tests.

CAUTION: HUMAN BLOOD

All materials listed above are considered to be potentially infectious and should be handled with universal precautions.

FOR RESEARCH USE ONLY. Not for use in Diagnostic Procedures.