




Single Patient Use Disposable Pads

Instructions and Safety Guidelines

- Adjustable, one size fits most
- Comfortable and lightweight
- Latex-free
- Individual pads work in conjunction with the ThermaZone Continuous Thermal Therapy unit.

BEFORE YOU BEGIN:

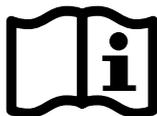

Read the Instruction and Safety Manual included with your ThermaZone unit.




This is not a substitute for the ThermaZone Instruction and Safety Manual.

Use the ThermaZone unit and Single Patient Use Pads for intended use only.
Please consult the ThermaZone Instruction Manual for Warranty information.

THERMA[®] ZONE





CONTINUOUS THERMAL THERAPY

Connecting the Tubes and Pad

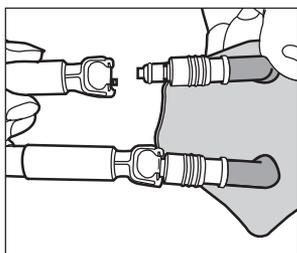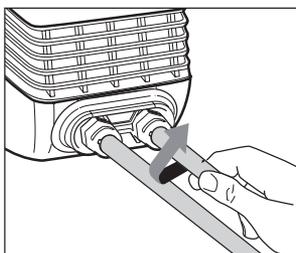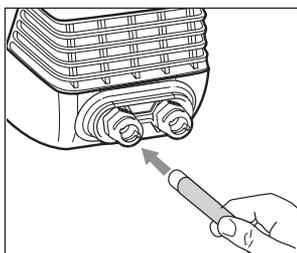




Back Relief Pad

MODEL # 003-28

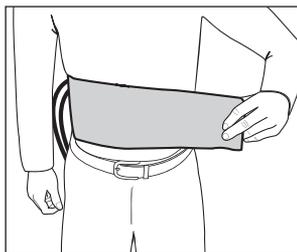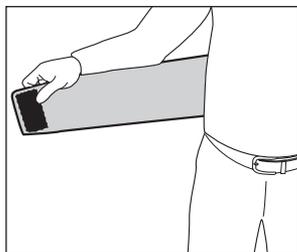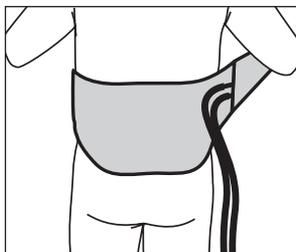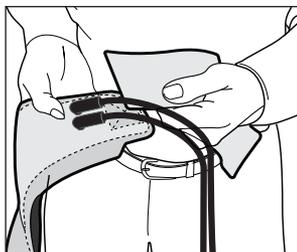




Knee Relief Pad

MODEL # 003-27

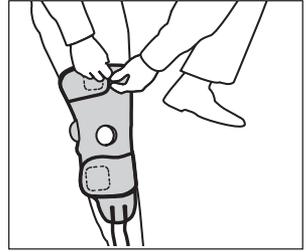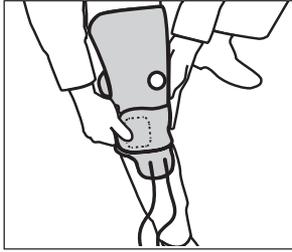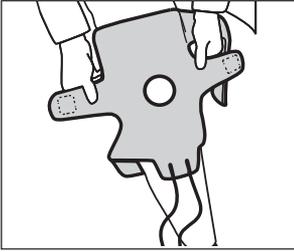






Ankle Relief Pad

MODEL # 003-29

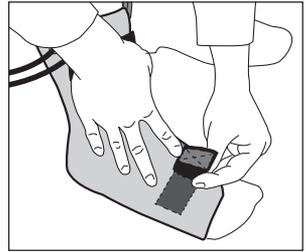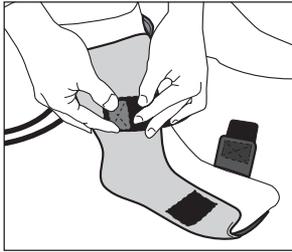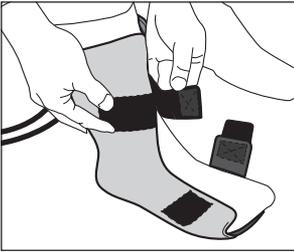

Large Shoulder Relief Pad

MODEL # 003-30

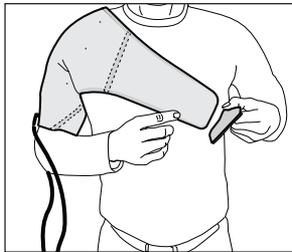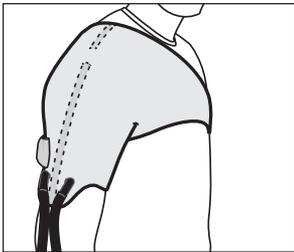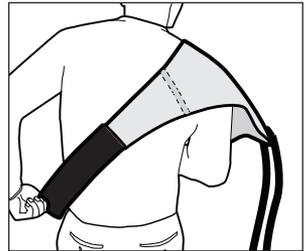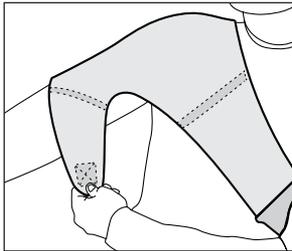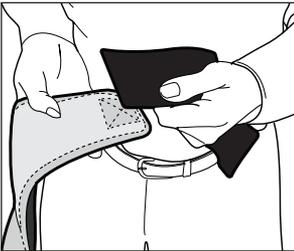

WARNING:

DO NOT use on more than one patient. Single Patient Use pads are designed for use by a single patient for the length of the individual patient treatment.

DO NOT apply to open wounds or compromised skin. Single Patient Use Pads can be applied directly to intact skin or over a thin barrier.

Single Patient Use Pads are NOT sterilized. DO NOT place Single Patient Use Pads in an autoclave or attempt to sterilize.

ALWAYS follow the protocol provided by the attending physician.

Due to individual differences in sensitivity and susceptibility to cold or heat, the patient's skin should be frequently observed. Prolonged exposure to cold or heat has a potential to cause injury to tissue. Follow the instructions of the attending physician for length, frequency, and duration of treatment.

Medical supervision is required for patients with decreased skin sensitivity, children, elderly, or incapacitated patients, as well as those with poor circulation, vein ligation, recent skin grafts, or extremely low blood pressure.

DO NOT use cold therapy on patients with Raynaud's disease, poor peripheral circulation, diabetes, or hypersensitivity to cold.

DO NOT apply Single Patient Use Pads so tightly as to restrict blood or fluid flow. Dressings used under the pad should be applied lightly.

To avoid adverse effects to the skin, shut off the ThermaZone unit if patient begins to feel uncomfortable.

Discontinue use if dizziness, severe pain, or severe skin reaction occur.

Disposable pads are designed for post-operative recovery for up to 3 weeks of use.

Single Patient Use Pads should be used only with the ThermaZone unit and in a medical facility or clinical environment with direct healthcare provider supervision. If a healthcare practitioner determines it is appropriate for a patient to continue therapy at home, the healthcare provider must provide patient with adequate and appropriate instructions for use at home.

DO NOT use Single Patient Use Pads near an open flame or while smoking.

DO NOT operate ThermaZone unit in an oxygen-rich environment or where oxygen is being administered.

DO NOT operate ThermaZone unit when intended user is connected to life support equipment, cardiac device, and/or electrophysiology device.

STORAGE AND DISPOSAL

DO NOT store the unit or the Single Patient Use Pads in a location where the temperature is below 60°F (16°C) or above 85°F (29.4°C).

AVOID EXPOSURE to sunlight and high heat conditions for extended periods. Keep away from flames.

Dispose of the ThermaZone unit and Single Patient Use Pads in a manner that is consistent with your federal, state, and local laws and regulations.