"Hey OrCam" Voice commands.

To use voice commands start by saying "Hey OrCam" first, followed by one of the commands below.

General.

- -Speed up/Speed Down
- -Volume up/Volume Down
- -Tell Time
- -Tell Date
- -Tell Battery Status
- -Tell Build
- -Connect to Bluetooth Device
- -Main menu

Facial Learning.

-Start Face learning

Smart Reading

- -Start Smart Reading
 - Find (pancakes for menu)
 - Read everything
 - Start from
 - Restart (to take new pic)

EXIT (remember to always exit smart reading).