



OrCam Read

Quick Instruction Guide


Basic Instructions

To Power On/Off: At the back end of the device, press the rectangular button for 2 seconds.

Volume Up/Navigate Forward: Press the “plus sign” (the second button behind the camera) to turn the volume up. Use the same button to fast forward when the device is reading.

Volume Down/Navigate Backward: Press the “minus sign” (the third button behind the camera) to turn the volume down. Use the same button to rewind/repeat text when the device is reading.

Trigger Button: The first round button behind the camera is the trigger button. Press once to: read captured text, stop reading, or wake up from suspend mode. Press twice quickly while reading to pause the device and twice quickly again to resume reading from where the device paused.

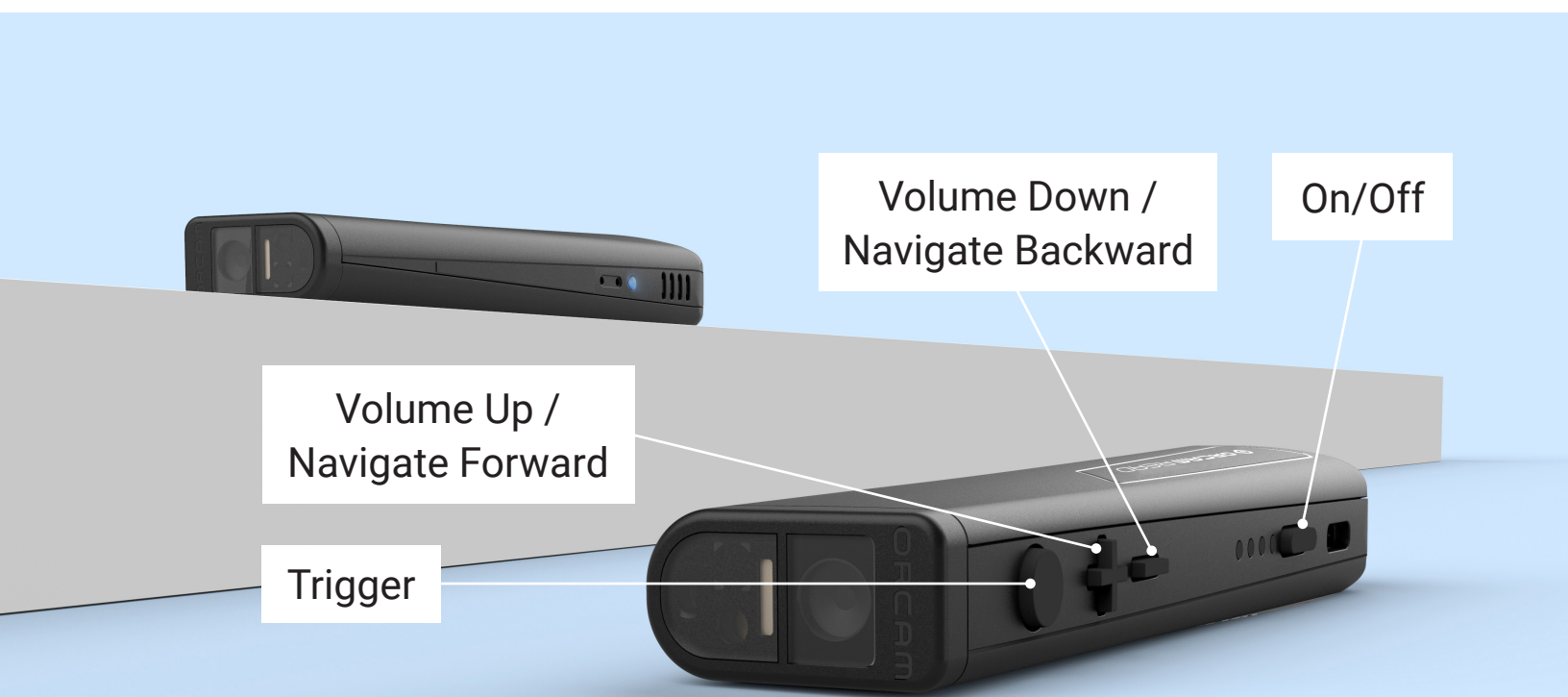

How To Read and Recognize Barcodes and Money

OrCam Read is able to read text and recognize preloaded barcodes and money notes.

1. Hold the device directly facing the text, barcode or money note at a distance about 10 inches away.
2. Press the trigger button. As you press, you can adjust the laser to capture the desired text, barcode or money note.
3. Release the trigger button. You will hear a camera shutter sound. OrCam Read will begin to read the text or recognize the barcode or money note.

Note: Do not hold the trigger button for more than 10 seconds or the device will time out and you will need to press again.



Smart Reading

Use the Smart Reading feature to easily find specific information you are looking for. You can ask the OrCam Read to read specific information or keywords on items like bills, menus, and business cards using Smart Reading voice commands.

- Hold the device directly facing the text at about 10 inches away.
- Use the voice command “Hey OrCam, Start Smart Reading” or double-press the trigger button and say “Smart Reading” after the beep. The device will start making camera shutter sounds.
- The device will say “Ready” and you can begin Smart Reading using the available smart reading vocal commands listed below.
- If you wish to stop the feedback and say a new command while the device is reading, press the trigger button once and the device will return to “Always Listening” mode.
- To exit this feature, either say “Exit,” or press the trigger button for 5 seconds and then release. OrCam Read will say “Exiting” as it exits the Smart Reading feature.

Smart Reading Voice Commands

- Start Smart Reading
 - Find “X” (e.g. “Find Dessert”)
 - Start from “X”
 - Jump to “X”
 - Read Everything
 - Restart (will capture a new picture)
 - Previous
 - Next
 - Repeat the Last Sentence
 - Read the Phone Numbers
 - Read the Dates
 - Read the Amounts
 - Read the Headlines
 - Read the Article Number “X” (can be a number 1-10)
 - Read the Article About “X”
 - Speak “Louder,” “Softer,” “Faster,” or “Slower”
- Exit

Voice Commands

Start by saying “Hey OrCam” followed by one of the commands below:





General Commands

- Speed Up/Speed Down
- Volume Up/Volume Down
- Tell Time
- Tell Date
- Tell Battery Status
- Connect to Bluetooth Device
- Main Menu
- Read Everything
- Go to Sleep
- Change Voice
- Change Laser


Stand Mode

- Start Stand Mode
- End Stand Mode



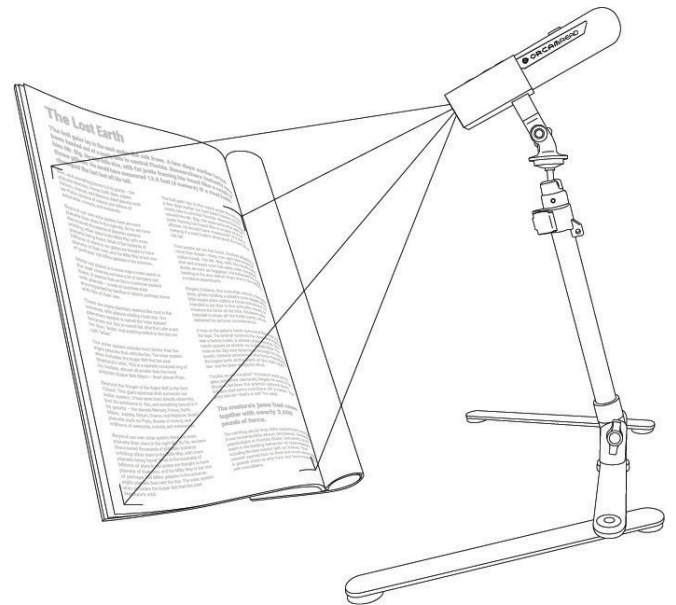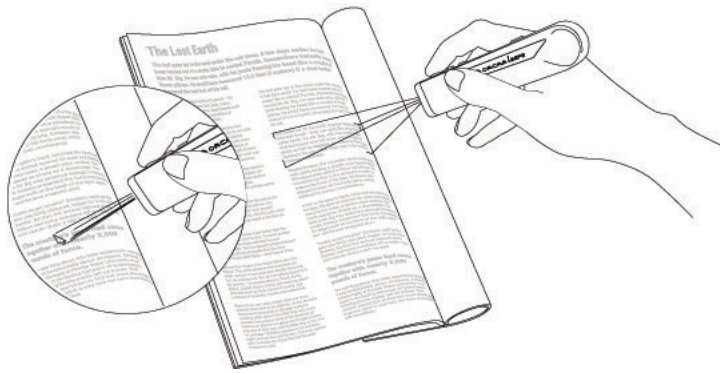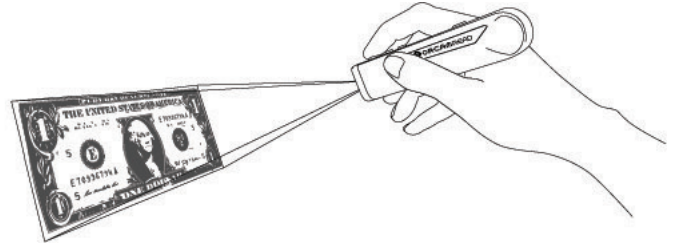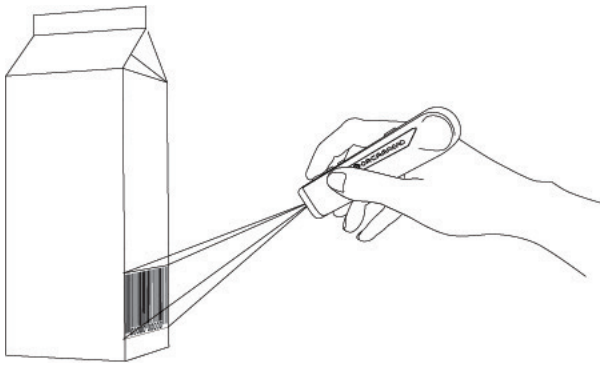

"Read the
Phone Numbers"

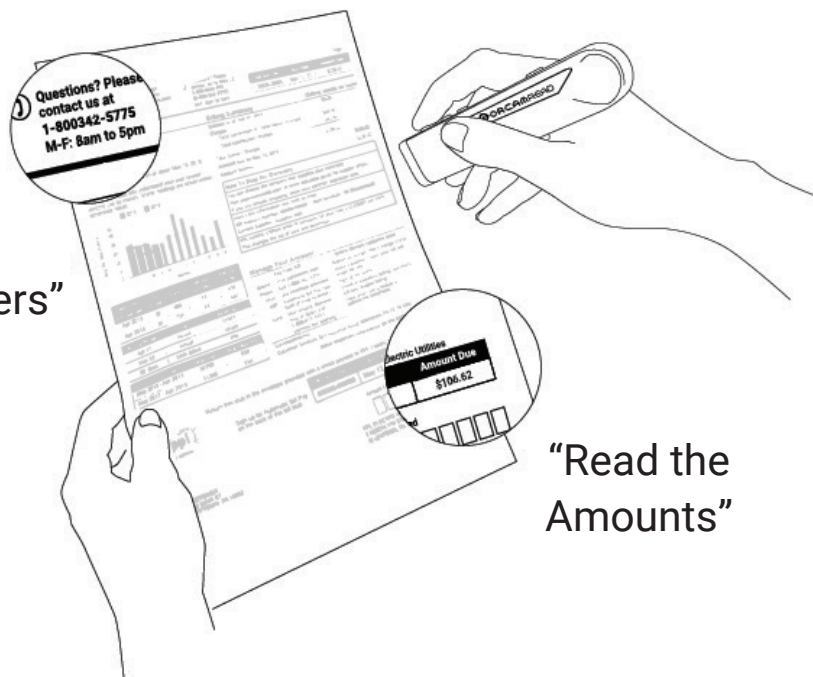

"Read the
Amounts"

Corinne Nero, Senior Area Sales Manager

630-642-9810

corinne.nero@orcaml.com

OrCam Devices for Low Vision



OrCam Read



OrCam MyEye



OrCam Read 3



Email: usorcaml@orcaml.com