



OrCam Read 5

Quick Instruction Guide

Congratulations on your purchase of the OrCam Read 5!

The device has unique features while offline, connected to your phone's Bluetooth and when connected to a smart screen/computer.

Basic Instructions to Read Text Offline

To Power On/Off: At the back end of the device, press the rectangular button for 2 seconds.

To Control Volume: While device is not reading, you can press the + to increase volume or – button to reduce volume.

To Read: Press and hold the round trigger button for a red laser box to surround the text you want read. Once text is centered, release trigger and text will be read. Press the + button to skip forward, or the – button to skip backward.

To Use Handheld AI Assistant

Download OrCam App available on iOS and Android.

Turn device on holding Power button at end of device.

Open app and tap “Let's Start” to pair. This is only necessary in initial pairing of phone. Device will be recognized by the app on future uses.

Place the round trigger button for a red laser box to surround the text you want read. Once text is centered, release trigger and text will be read. Press the + button to skip forward, or the – button to skip ahead.

Once you have captured text and the app is running, you can push and hold the trigger button and ask the Read 5 additional questions pertaining to the text in front of you.

For example, you can ask Read 5 to summarize the text, find a specific phrase in the text, find an amount due, or even ask for additional information on the topic on the text captured. Push down the trigger button while asking a question and release the trigger button for answer. The device is equipped to respond to questions spoken while the trigger button is held down; releasing the trigger will enable the device to respond.

OrCam App

While device is turned on, you will have full functionality of App.


Locate device tutorials on the Home page.

On the lower middle area of the App, press Gear symbol – you can increase/decrease Volume by sliding bar to the right/left.

You can increase/decrease the reading rate by moving Reading Rate Bar to the right/left.

You can locate the device as long as the device is powered on by pressing the Find Me icon. Your Read 5 device will beep loudly and flash its light to help you locate your device.

You can connect to speakers, wireless headsets, or other Bluetooth devices by pressing “Connectivity Settings” and locating your preferred source.

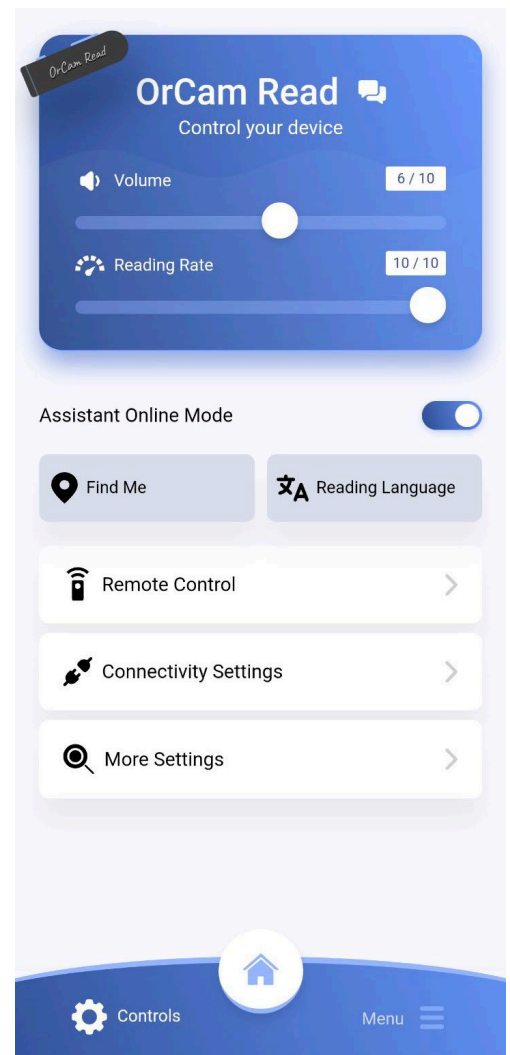


Remote Control Feature

You can also use the Remote Control feature on the OrCam App to increase or decrease volume and reading rate.

You can click on the capture icon to capture an image with your Read 5 so the device can process it directly. You will not be able to save or upload an image from the app; this can only be done while using the Magnification Feature discussed below.

The “Start Listening” button activates the AI assistant by enabling the device to listen for a question from the user.

Finally, the bottom bar allows the user to skip back, play, skip ahead or stop while it is reading text.

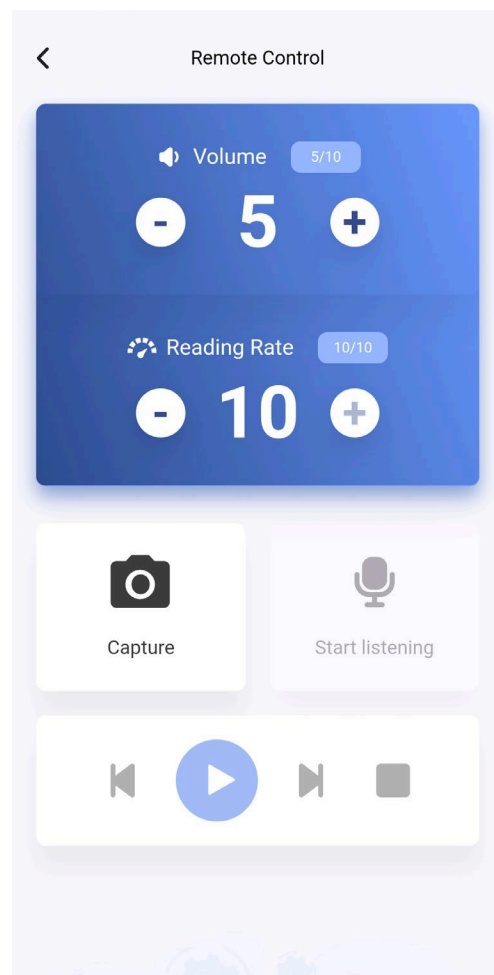

Connecting to Magnification Feature

Connectivity Options: Connect your Read 5 to a computer screen to use magnification features. This can be done via the provided USB-C cord, requiring no Wi-Fi connection, or it can be done wirelessly via Wi-Fi.

To connect with USB-C cord:

- Power on the Read 5 device
- Connect the Read 5 to your computer using a USB Type-C cord
- Place the Read 5 on the supplied stand
- On your computer, visit cast.orcaml.com
- Press “Start”

- Create an account using your email address
- Click “Connect via USB” to connect your computer to the Read 5 device

To connect without USB-C via Wi-Fi:

- Make sure Read 5 device and computer are connected to the same Wi-Fi
 - The Read device can be connected via the OrCam Read app or QR code by going to the website orcam.com/en-us/wifiset
- Visit cast.orcam.com on your computer
 - Press “Start”
 - Create an account using your email address
 - Follow instructions on screen to connect your device

Operating the Magnifier and AI assistant

Place a book or document underneath the Read 5 camera or upload a file from your computer. Use the Read 5 stand for simple hands-free document reading.

Live stream mode

Make sure the document or book is placed underneath the device camera so all text is visible.

- **AI assistant** – Ask any question related to the scanned text and OrCam will provide an answer.
 - Press the star button located in the center of the button bar
If requested, allow access to your camera and microphone. These permissions are required for the web application to work properly.
 - The Read 5 will snap a picture of the text, process it and then play a summary of the text

- You can now ask any questions regarding the text or any general questions, for example:
 - *Can you explain the first paragraph?"*
 - *"When is the due date?"*
 - *"Who is this letter addressed to?"*
- If you want the magnifier to read the entire text aloud instead of summarizing, press the "Audio Book" icon to switch to text reading mode.

- **Using the left side toolbar**



Change connection type: switch between live capture and upload file



Toggle between the photo or text mode



Download a JPEG, copy text or download MP3 File



Change the colors of your reading content



Rotate the image



Adjust brightness of the image



Increase magnification



Decrease magnification

*If using access software, tab will move through each button or use hotkeys below for easy access

Hot Key Commands

Instead of your mouse, you can use the hot keys on your computer as shortcuts to navigate the magnifier.

Key	Action
1	Download
2	Copy Text
3	Download MP3 File
Escape	Back
Backspace	Back
t	Change Contrast
r	Rotate
c	Contrast
Ctrl+-	Zoom Out
Ctrl+=	Zoom In
-	Zoom Out
=	Zoom In
a	Change Footer Note
Enter	Hey OrCam Assistant
P	Stop Hey OrCam Assistant
Ctrl + r	Change Voice Rate
?	Show Help Dialog

Troubleshooting

If having trouble connecting to your device from your phone's hotspot, please check your devices settings to confirm that the "Maximize Compatibility" option is turned on.

Should you receive an error issue while using the handheld AI application, please use "Reset Installation Retries" under General Settings.

If you have any additional issues: shut down the app, hold down the OrCam App Icon, agree to delete and redownload the application from the App Store. Typically most user issues will be corrected directly.

OrCam Assistive Technology Devices: Independence Wherever You Go



OrCam **Read**



OrCam **MyEye**



OrCam **Read 3**



OrCam **Read 5**



Email: usorcam@orcam.com