

Serres Suction Bags and Canisters - Instructions

- Connect vacuum source to the gray angled connector.
- Place bag in canister.
- Turn vacuum on and lightly press middle of lid.
- Allow bag to completely inflate
- Cover white angled patient connector with finger while pressing down in the middle/not from the sides to fully seat lid into place.
- Connect patient tube to the white angled connector.
- Turn off the vacuum source if the procedure isn't immediately started.
- When the suction bag is full disconnect the patient tube/white angle connector and plug the top of the lid.
- Do not turn off the vacuum until you have plugged the bag.
- Turn off vacuum.
- Lift bag from handle and dispose.

Instruction Video (especially at 50 seconds to 1:25):

<https://www.youtube.com/watch?v=kml-ltFWkcE>