

# THE NEXT EVOLUTION IN CRYOSURGICAL TECHNOLOGY.

CryoProbe sets a new standard for the cryogenic treatment of benign skin lesions, offering the ultimate in control, convenience, and cost-effectiveness – as well as an outstanding patient experience.


CryoProbe XP+ comes equipped with two extended applicator tips providing a better vantage point for treating in the cavities and hard-to-reach areas.

Designed for intensive and varied use in removing common benign lesions, the CryoProbe XP+ goes a step further by offering improved access and accuracy.




## Key features

- Ergonomic design for superior control
- Leading-edge microtechnology for controlled gas flow
- Takes easy-to-load 8g or 16g N<sub>2</sub>O cartridges
- Aluminum body with applicators in medical-grade materials
- Provided with 6 applicators
- Suitable for intensive and varied use
- Certified **CE1639** **FDA** **MDSAP**

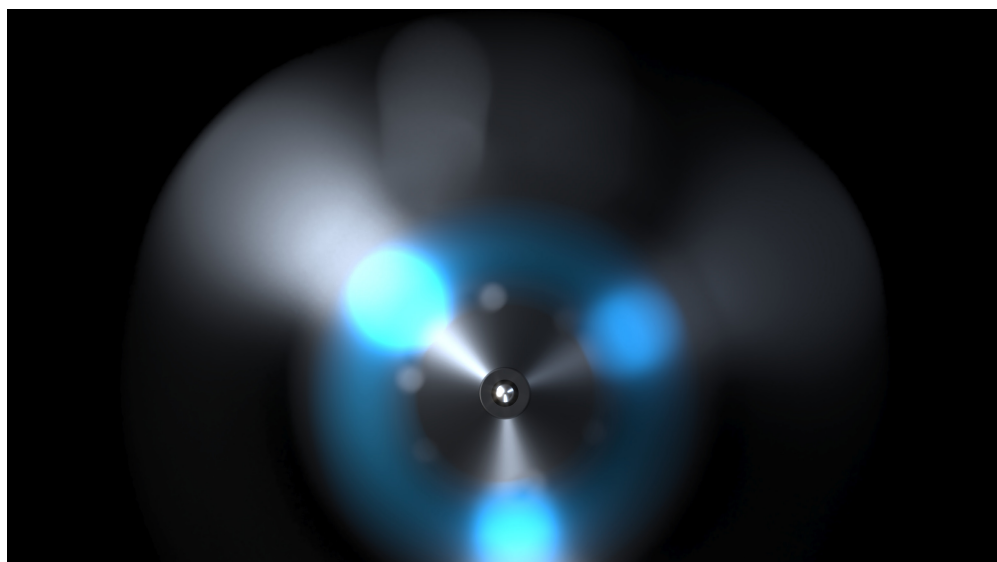





## APPLICATORS

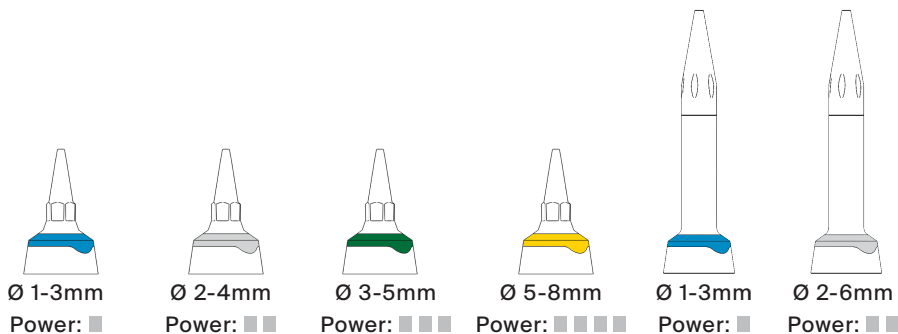

### Continuous flow time with a 16g cartridge

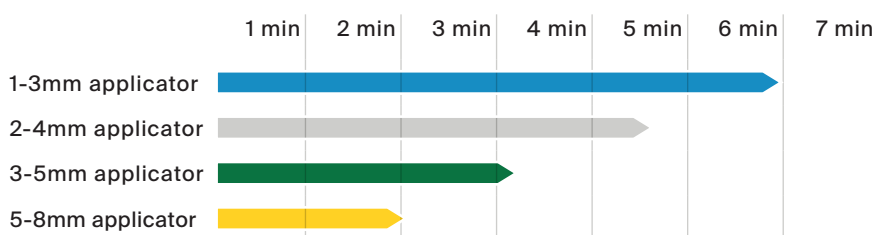

### Continuous flow time with an 8g cartridge

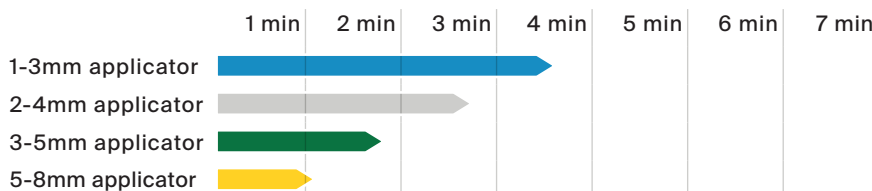

## PRODUCT INFORMATION

### Tutorial

Scan or click:



### Device

The CryoProbe XP+ is suitable for all treatment situations, whether more delicate or stronger freezing power is required. The device can be used with 8g or 16g cartridges, and with the complete range of applicators, making it a highly versatile instrument.

### Procedure

No special preparation is required. Apply the jet of nitrous oxide for 2 to 45 seconds on each lesion, depending on the size and depth of the lesion. For larger lesions, use circular or back-and-forth movements to treat all the tissue.

### What kind of lesions?

Acrochordon, actinic keratosis and cheilitis, capillary hemangioma, filiform warts, flat warts, vulgar warts, plantar warts, anogenital warts, granuloma annulare, molluscum contagiosum and seborrheic keratosis.

## CONTENTS OF THE STARTER SET

### Device

CryoProbe XP+

### Applicators

1-3mm applicator  
2-4mm applicator  
3-5mm applicator  
5-8mm applicator  
1-3mm applicator long  
2-4mm applicator long

### Cartridges

6x 16g cartridges

### Accessories

Metal carrying case  
Forceps  
Converter for 8g cartridge  
Device stand

### Documentation

Treatment overview  
User manual

## BASICS

### Dimensions

Device: 22.6 x 3.3 x 3.3 cm  
Carrying case: 28.8 x 22.8 x 6.7 cm  
Starter set: 38 x 30 x 7.5 cm

### HS codes

90189084 (device)  
28112930 (cartridges)

### Weight

Device with a full cartridge: 300g  
Starter set: 2,487g

