

Pro-Nox Nitrous Oxide - *CAREstream America*

Safely Manage Pain and Anxiety with Close to 100% Satisfaction

- Fixed at 50% Nitrous Oxide/50% Oxygen
 - Delivers a Very Calming Non-Addictive Analgesia - Minimize Medications
 - On Demand & Self-Administered - Empowers Patient with Their Own Pain Control
 - Very Safe and Economical - Seamlessly Integrate into Your Practice
 - Quick and Natural with Full Recovery in 10 Minutes
 - Improve Patient Retention/Practice Growth with a Better Experience for Everyone
 - The Pro-Nox Nitrous Oxide System Will Differentiate Your Practice
1. Airgas is an option for tanks/**MEDICAL-GRADE** gas. Their phone number is (877) 717-4540 x6. You would need **E-tanks** of nitrous and oxygen. **They must have a post-valve with no regulator and no handle (CGA-870 Post Valve - Pin Indexing - 2 Holes - Bare/Empty Post)**. You might start with 2 nitrous tanks and 3 oxygen. Oxygen lasts for up to 20 patients (3-3.5 hours) per tank and nitrous is up to 40 patients (7-7.5 hours) per tank. Some other options for gas are Roberts Oxygen, Dalco Medical 800-989-3252 (in DE and parts of PA) or a local company with **MEDICAL-GRADE** gas. Some other possible sources of gas are:
AGL Gas (973) 478-5000 SJ Welding (856) 778-4440 Seaboard (732) 493-8100
AAA (973) 566-0003 Vineland (856) 691-9659 PA Welding (215) 329-7600
All Gas (973) 786-2121 Pleasantville (609) 641-5952 Mac Medical Gas (215) 443-0782
Wilson Products (610) 253-9609 Daily Gas (570) 344-1303
 2. The mouthpiece/tubing/filtration is \$14.99 each.
 3. Replace the Bodok Seal (on regulator) as needed. **Discard the seal that comes with each tank.** Be careful to not lose the Bodok Seal when changing tanks.
 4. Attach the magenta exhaust tube and point to the bottom of the floor.
 5. **Open the nitrous tank valve first** (one full rotation or more). Opening the oxygen tank valve first will trigger alarm/visual indicator will flash (detecting low nitrous). Close the oxygen tank first when finished for the day.
 6. Patient should initially take 5-9 slow deep breaths in and out **prior to the start of the procedure** with a good seal around the mouthpiece.
 7. Consider the optional 90-degree elbow (at the mouthpiece) when patient is lying down.
 8. **Replace a tank when the first alert sounds/visual indicator flashes.** Always close the oxygen tank first regardless of which one you're replacing and always check to make sure that the Bodok Seal is properly seated in the regulator. Unscrew the regulator slowly to let any remaining gas to slowly release.
Do not use the plastic washer that comes with the tank. Only use the Pro-Nox brass/rubber washer (Bodok Seal).

Cory Steiner
eortho@aol.com
eortho.com

(800) 366-8051
(215) 791-0674 cell
(215) 689-1558 fax