



THE MOST COMPACT CRYOSURGERY DEVICE

CryoProbe sets a new standard for the cryogenic treatment of benign skin lesions, offering the ultimate in control, convenience, and cost-effectiveness – as well as an outstanding patient experience.

This high-performance, ergonomic cryotherapy pen dispenses a fine jet of nitrous oxide under high pressure, allowing you to work with millimeter precision.

The CryoProbe XPS is our most compact model, and is suitable for intensive and varied use.



Key features


- Ergonomic design for superior control
- Leading-edge microtechnology for controlled gas flow
- Takes easy-to-load 8g N₂O cartridges
- Aluminum body with applicators in medical-grade materials
- Provided with 2 applicators
- Suitable for intensive and varied use

• Certified **CE1639** **FDA** **MDSAP**






APPLICATORS

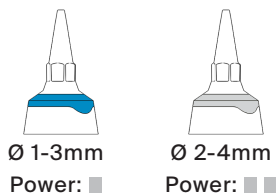

Continuous flow time with an 8g cartridge

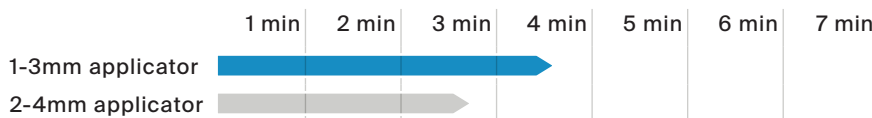

PRODUCT INFORMATION

Tutorial

Scan or click:



Device

The CryoProbe XPS is suitable for all treatment situations where delicate freezing power is required.

Procedure

No special preparation is required. Apply the jet of nitrous oxide for 2 to 45 seconds on each lesion, depending on the size and depth of the lesion. For larger lesions, use circular or back-and-forth movements to treat all the tissue.

What kind of lesions?

Acrochordon, actinic keratosis and cheilitis, capillary hemangioma, filiform warts, flat warts, vulgar warts, plantar warts, anogenital warts, granuloma annulare, molluscum contagiosum and seborrheic keratosis.



CONTENTS OF THE STARTER SET

Device

CryoProbe XPS

Applicators

1-3mm applicator

2-4mm applicator

Cartridges

12x 8g cartridges

Accessories

Metal carrying case

Forceps

Device stand

Documentation

Treatment overview

User manual

BASICS

Dimensions

Device: 20 x 2.75 x 2.75 cm

Carrying case: 28.8 x 22.8 x 6.7 cm

Starter set: 38 x 30 x 7.5 cm

HS codes

90189084 (device)

28112930 (cartridges)

Weight

Device with a full cartridge: 200g

Starter set: 2,356g

