




How to load Liocan Intraocular Lens in the supplied Injector Set

1. Fill with viscoelastic the tip of the injector and wet the bottom of the loading chamber.

2. Take the IOL from the holder, from the rounded side where it is being held (picture on the right). The IOL has two notches (shown in red).



3. Place the IOL in the loading chamber without rotating the forceps (as it was in the holder). See picture below.

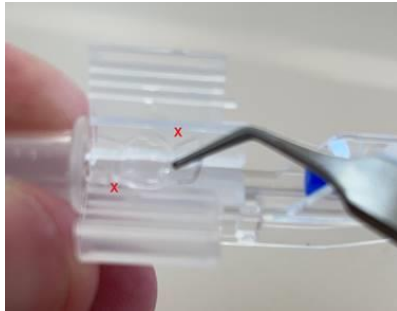

4. Using a forceps with elongated tips, gently push down on the IOL centering it between the rails of the injector (picture below).



5. Close the cartridge and add a little viscoelastic at the end of the loading chamber.
6. Make sure the IOL hasn't been jammed by bringing the plunger to the cartridge/IOL and pushing slightly; it should go very smoothly, no stop should be felt, otherwise open the cartridge and center the IOL again.
7. Inject slowly without stopping, keeping the cartridge's wings at 12 o'clock position. Do not rotate your hands.

This is an addendum to the Instructions For Use supplied with Liocan Intraocular Lens. Carefully read the instructions prior to use the device. Failure to do so may result in patient complications.