


Tools & Equipment

TIGO-02
(Quick Connect Handle)



TIGO-14
(Tapered Awl)

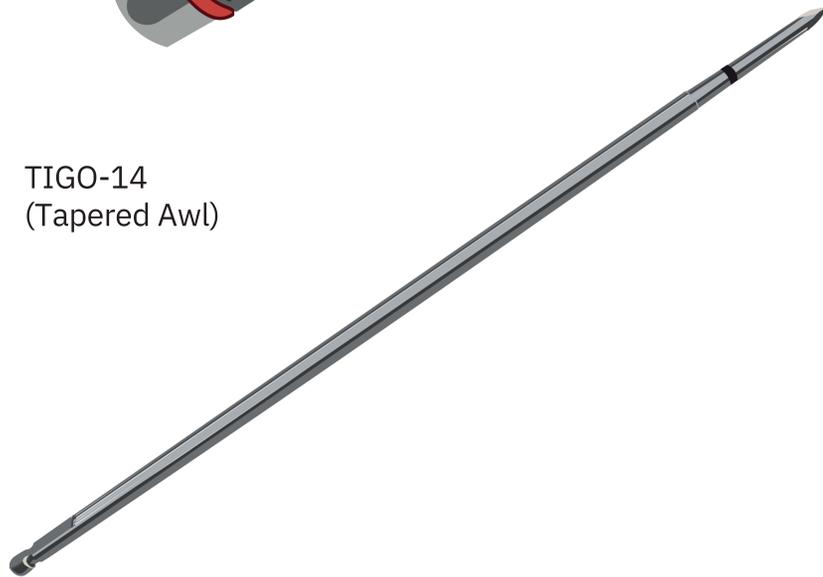

TIGO-16 (Blunt Medial
Anchor Inserter)



Medial Anchor
(Available in various sizes &
suture combinations)



MIX.
MATCH.
MAXIMIZE.™

GREENREPAIR®



(410) 544-2833

office@tigonmedical.com

303 Najoles Rd, Suite 104
Millersville, MD 21108

www.tigonmedical.com
www.greenrepair.net

MST4, ISSUE 1
EFF. DATE: 15DEC2023

Medial Anchor


Surgical Technique

Use:

Reattachment of soft tissue to the bone.

Primary Surgery:

Rotator Cuff Repair

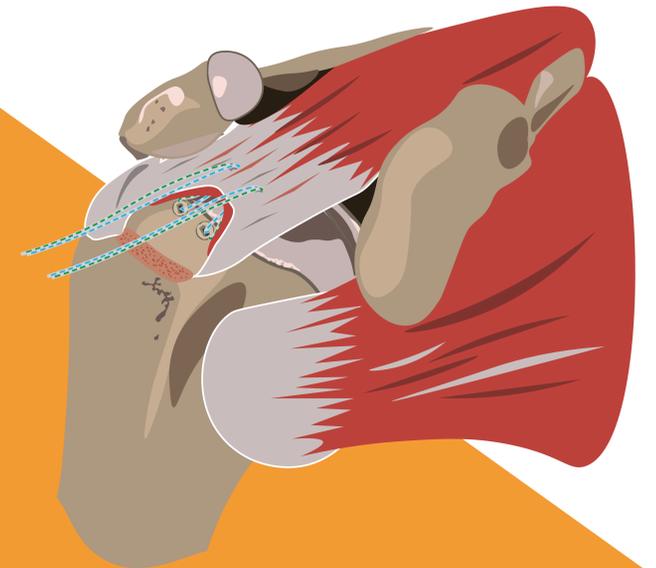


MANTICORE



- 1** Connect the Tapered Awl (TIGO-14) to the Quick Connect (TIGO-02)




- 3** If anchor is not provided with suture: Insert desired suture through the hole on the medial anchor.

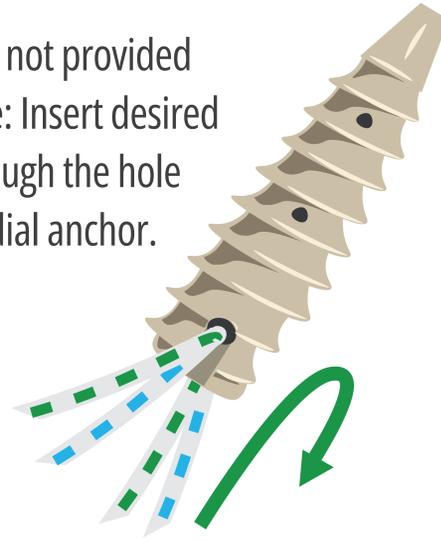


- 4** Remove the Tapered Awl from the Quick Connect and attach the Inserter



- 5** Pull the suture to the side and place anchor onto the inserter

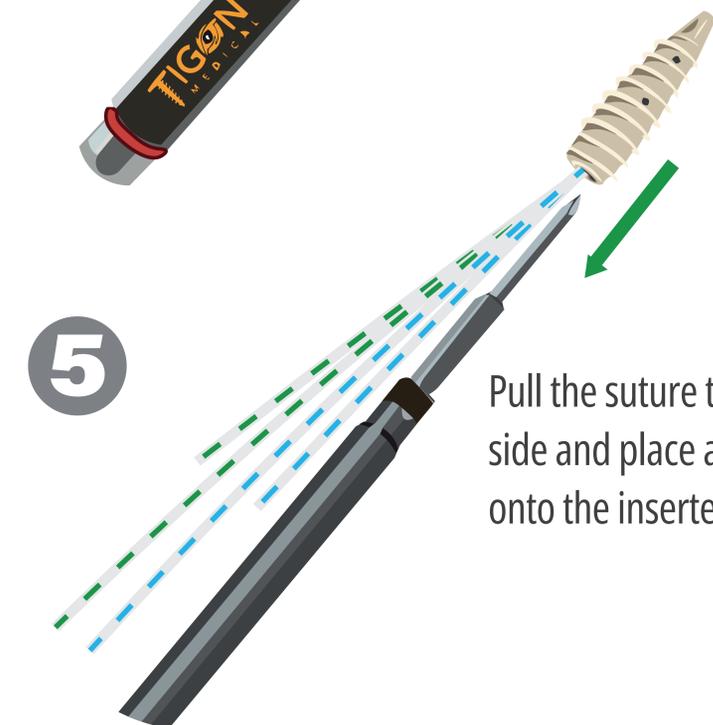

- 6** Place anchor into the pilot hole and twist clockwise until the laser line on the inserter is flush with the bone. To countersink 2mm, bury the laser line.

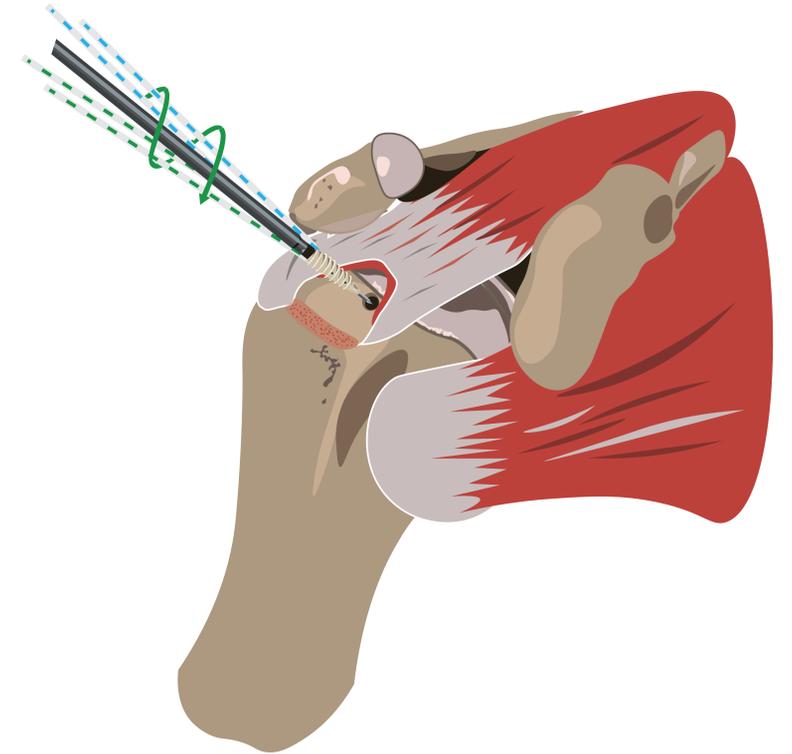

- 2** Align the Tapered Awl (TIGO-14) to the desired position. Mallet the awl to the black laser line to create a pilot hole for the anchor.

