


# Tools & Equipment


TIGO-02  
(Quick Connect Handle)




TIGO-14  
(Tapered Awl)

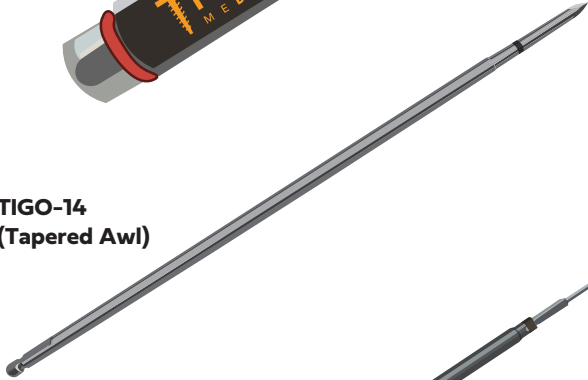


TIGO-05 (Spiked Medial  
Anchor Inserter)

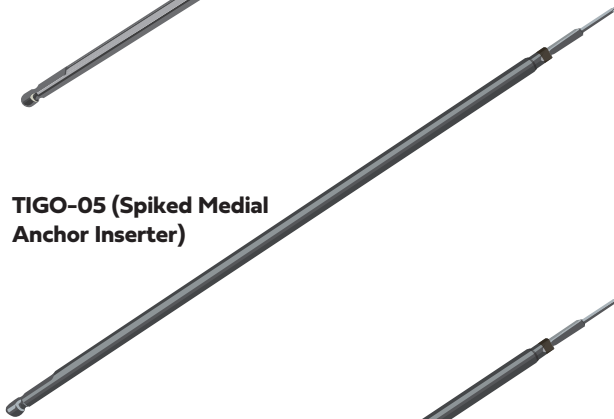

TIGO-16 (Blunt Medial  
Anchor Inserter)

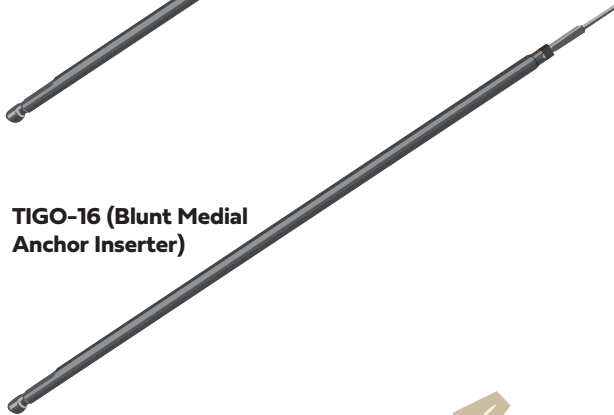

TIG55  
(5.5mm Medial Anchor)

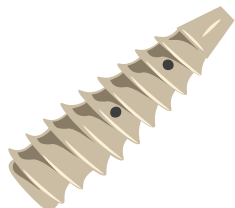

MIX.  
MATCH.  
MAXIMIZE.™

GREENREPAIR®



(410) 544-2833

[office@tigonmedical.com](mailto:office@tigonmedical.com)

303 Najoles Rd Suite 104  
Millersville, MD 21108

[www.tigonmedical.com](http://www.tigonmedical.com)

MST4, REV F

# Medial Anchor


## Surgical Technique — TIG55

**Use:**

Reattachment of soft tissue to the bone.

**Primary Surgery:**

Rotator Cuff Repair.

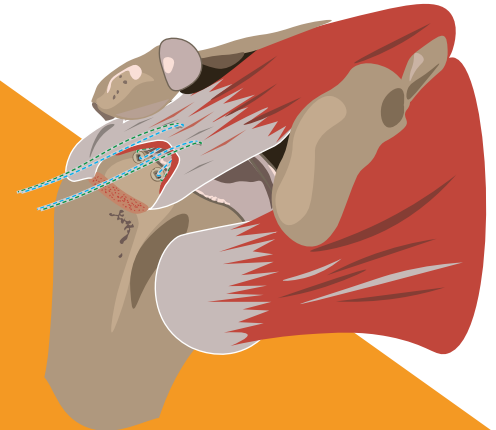


MANTICORE




**1** Connect the Tapered Awl (TIGO-14) to the Quick Connect (TIGO-02).



**3** Insert desired suture through the hole on the 5.5 mm Medial Anchor (TIG55).

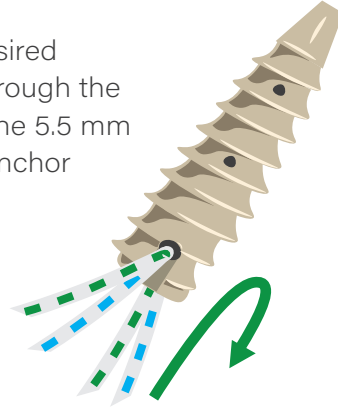


**4** Remove the Tapered Awl from the Quick Connect and attach the TIG55 Inserter (TIGO-05 or TIGO-16).

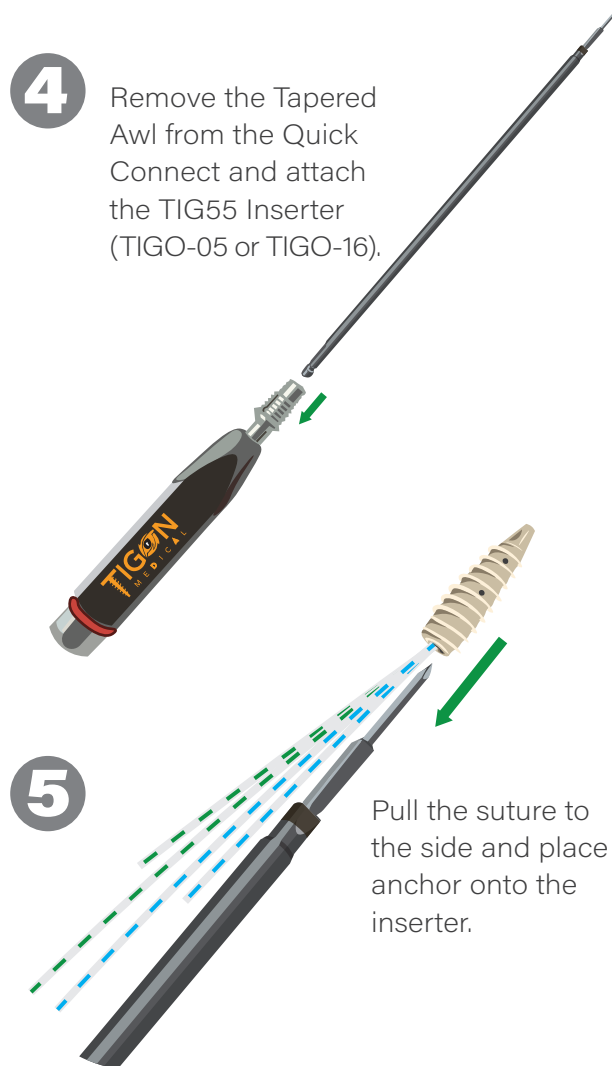

**5** Pull the suture to the side and place anchor onto the inserter.

**6** Place anchor into the pilot hole and twist clockwise until the laser line on the inserter is flush with the bone. To countersink 2 mm, bury the laser line.

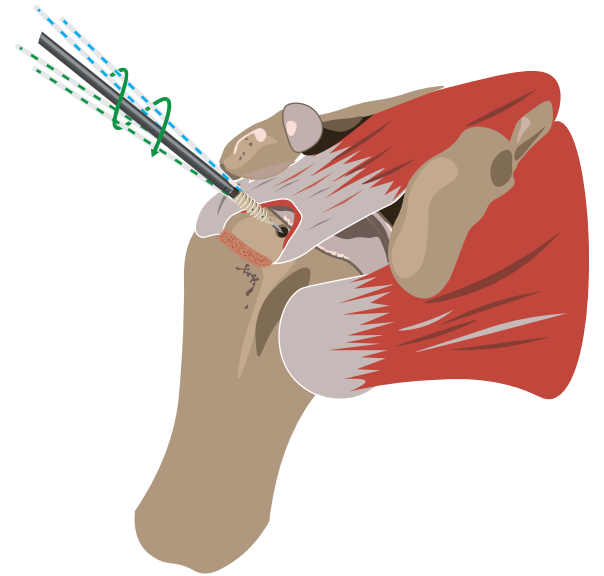

Repeat steps 1-6 for the second medial anchor.



**2** Align the Tapered Awl (TIGO-14) to the desired position. Mallet the awl to the black laser line to create a pilot hole for the TIG55 anchor.

