



All on X case Stage 0

Stage 0: Pre-Op records

- Absolutely CRUCIAL for accurate immediate load
- Can be done digitally with IO scans or also with impressions
- Proposed VDO for the case must be determined at this stage
- Records sent to lab technician for Pre-Op plan

Stage 0 Records must:

- Determine the VDO
- Determine the proposed implant positions
- Determine how the immediate load appliance will be fabricated and converted during surgery
- Determine the amount of bone reduction needed
- Determine the type of Prosthesis to be restored
- Determine the ideal smile for the patient

8 Items

1. Radiographs (scan with teeth apart for normal dentition) or (scan in bite with Dentures (Dentures will need scan markers for CBCT & IO scans

Photos: (all with **teeth closed** & **eyes open** looking at the camera) **no glasses** on



normal smile ☐



super big smile ☐



retracted smile ☐

2. Photos

3. VDO

4. IO scans (upper, lower, bite)

5. Facial Scans

6. Type of Prosthetic Planned (FP1, FP2, FP3)

7. Alignment Strategy

8. Lab Slip

If your stage 0 records are off, your case will be off

Stage 1

Who is providing parts & type of temp for surgery

Nighthawk Dental, LLC
4825 Higbee Ave NW Ste 104
Canton, Ohio 44718
330-691-6736

nighthawkdental.com
caddesign@nighthawkdental.com

