


CURRENT ADIFAB2 TESTING PROCESS

Although this illustration depicts the measurement process with the Accufluor, you can also use other fluorometers where the time per measurement depends on the fluorometer specifications.

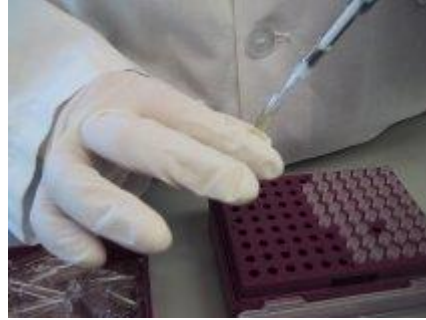

1. Fill cuvette with HEPES: 8 seconds 2. Add sample, mix: 14 seconds



3. Insert cuvette, press READ: 12 seconds 4. Add ADIFAB2, mix: 16 seconds



5. Insert cuvette, press ENTER: 6 seconds 6. FFAu value displays: 12 seconds

TOTAL TIME: 1 MINUTE & 8 SECONDS