

TRYXpro

Thank you for ordering the wheelchair from Tryxpro. Below are some helpful tips regarding the chair and service.

If you have any problems or questions please contact us through our website tryxpro.com

If for some odd reason the manual is not included in the box you can download it from our website under "Product Support."

If your chair has a small un-used wire at the bottom of the seat just ignore it. It was made for some radio feature.

Your new wheelchair might be a little hard to unfold at first but over time after a few uses it will loosen up. There is no locking mechanism to unfold just simply place your hand on the step board and the other on the handlebar of the backrest. Or some people might find it easier to place the foot on the stepboard (footrest) and pull up the handlebar with your hand. Once it's unfolded you can lock the latch for safety purpose.

If the battery is not yet mounted in the chair please slide it into the lever securely. Connect the battery wire to the chair connector. The connectors go in only one way to prevent short circuit.

To power it up press the on/off button for a couple seconds.

Before you operate you must charge the battery till the light turns green. The charging port is beneath the joystick.

Your one year warranty starts from the date of your payment receipt via email.

Enjoy your wheelchair!

From Tryxpro team