

Surgical Hydrogel Patch (Sterile)

Please read the product manual before use and follow the instructions.

Product Description

The Motif Protect Surgical Hydrogel Patch is developed with medical-grade hydrophilic polyurethane gel, which is comfortable, moisturizing, non-allergenic and anti-adhesive. The patch provides moderate self-adhesion and adequate tension. Adequate tension is needed to maintain a moist wound environment and promote wound protection.

Intended Use

Applicable to Caesarean Section wound care.

Composition

Polyurethane gel, polyurethane film, medical grade pressure sensitive adhesive, release paper.

Instructions for use

1. The Gel size should be larger than 1 cm around the wound.
2. Trim any excess hair around the wound area, to ensure proper adhesion and improved comfort.
3. Clean the wound in accordance with normal procedures.
4. Dry the surrounding skin thoroughly.

Specification


Gel size	Film size
3x17 cm	6x21 cm

How do I apply the wound dressing correctly?

Step 1: Carefully remove the dressing from sterile packaging, and remove the release paper.

Step 2: Check the dressing alignment with the wound, and then with two fingers gently push outwards from the center of the dressing.

Step 3: Remove the upper release paper, and gently press the dressing at the center and the four corners to ensure adherence.

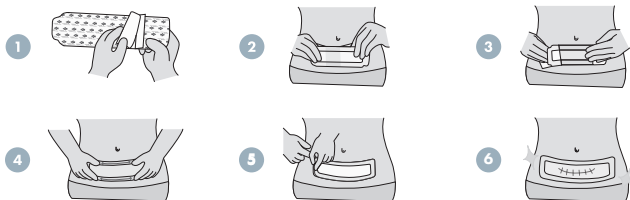

How do I remove the wound dressing correctly?

When removing, gently press the dressing with one hand, and straighten out with a hand to extend straightly so that the film is separated from the skin surface. In this way, the dressing can be slowly removed while pressing. Please use sterile saline solution to help removing when necessary.

Contraindications

1. Do not use and discontinue use on signs of sensitivity, irritation, or allergy from the dressing or its materials.
2. Do not use with alcohol or povidone-iodine at the same time.

Precautions

1. This product is for external use only. Please read the product manual before use.
2. Intended for one time use per person. Do not reuse. Reuse may spread contamination, and increase risk of infection.
3. If the dressing is damaged, peeled or contaminated, change the dressing immediately.
4. If the user has a weakened immune system, diabetes, or a wound infection, please change the dressing frequently.
5. The product is sterilized. Do not use if inner package is damaged or opened prior to use.
6. This product can be used continuously. There will be differences between individual users due to different physical conditions.
7. When using Motif Surgical Hydrogel Patch, it is possible to take a bath, but the edges of the dressing need to be pressurized and tightly attached to the skin to prevent water penetration and moisture permeability.
8. If swallowed inadvertently, it may cause obstruction of the digestive tract and need to be treated immediately.
9. It is not suitable to this product when the wound originally had the following symptoms of infection, such as: redness, swelling or discharge and pain, fever.
10. If the wound is red, hot, swollen, painful, allergic or uncomfortable when using this dressing, stop using it immediately and seek medical advice.
11. When the edge of the product is not sticky due to friction or curling, it is normal.
12. This product must be stored in a cool dry place, avoid sunlight.

Timing of use

Precautions

If there is presence of diabetes, cancer, or autoimmune disease, the wound may recover more slowly. Please make sure that the wound is completely healed before moving on to the Stage3(Scar Care Dressing).

Timing of use

- a. It is recommended to change one piece per week.
- b. The outer layer is a waterproof material, which can be used while showering, doing light yoga, or other sports. However, avoid strenuous exercise or sweating heavily.



take a shower



light yoga



avoid swimming



avoid going to
a hot tub

- c. If you sweat heavily after exercising, please change the dressing immediately. Too much moisture will reduce the adhesiveness of the dressing.



Manufactured For:

Motif Medical, LLC
Dover, DE 19901
Made in Taiwan