

Scar Care Dressing (Non-Sterile)

Please read the product manual before use and follow the instructions.

Product Description

The Motif Restore Scar Care Dressing is a polyurethane gel dressing specifically developed for the purpose of C-Section scar prevention. It reduces and softens the scar and surrounding skin. With moderate adhesiveness, the tension on the scars and surrounding skin could be relieved. In addition, the soft sensation from the dressing can reduce discomfort when continuously used, preventing further scarring and the softening and flattening of scars. This product is recommended to be used continuously for at least 4-6 months or until the effect is satisfactory.

Intended Use

Caesarean section.

Composition

Polyurethane gel, polyurethane film, medical grade pressure sensitive adhesive, release paper.

Instructions for use

1. The Gel size should be larger than 1 cm around the wound.
2. Trim any excess hair around the wound area, to ensure proper adhesion. and improved comfort.
3. Clean the wound in accordance with normal procedures.
4. Dry the surrounding skin thoroughly.

Specification

Gel size	Film size
3x17 cm	6x21 cm

How do I apply the wound dressing correctly?

Step 1: Carefully remove the dressing from packaging, and remove the release paper.

Step 2: Check the dressing alignment with the wound, and then with two fingers gently push outwards from the center of the dressing.

Step 3: Remove the upper release paper, and gently press the dressing at the center and the four corners to ensure adherence.



How do I remove the wound dressing correctly?

When removing, gently press the dressing with one hand, and straighten out with a hand to extend straightly so that the film is separated from the skin surface. In this way, the dressing can be slowly removed while pressing. Please use sterile saline solution to help removing when necessary.

Contraindications

1. Do not use and discontinue use on signs of sensitivity, irritation, or allergy from the dressing or its materials.
2. Do not use this product for wounds that have not completely healed.
3. Do not use with alcohol and povidone-iodine at the same time.

Precautions

1. This product is for external use only. Please read the product manual before use.
2. Intended for one time use per person. Do not reuse. Reuse may spread contamination, and increase risk of infection.
3. If the dressing is damaged, peeled or contaminated, change the dressing immediately.
4. Check the product packaging before use and do not use it if it is damaged.
5. Before use, please make sure that the skin is clean, dry, and no ointment or lotion is applied, so as not to affect the suitability.
6. It is recommended to use for a full 12 hours a day. If it is well-adjusted and then add time until it can be used throughout the day. The normal metabolism of the skin will make Scar Care Dressing gradually lose the stickiness. This product can be used continuously. It can be used for up to five to seven days. During the period, it has different usage condition according to the individual's constitution.
7. If swallowed inadvertently, it may cause obstruction of the digestive tract and need to be treated immediately.
8. If the wound is red, hot, swollen, painful, allergic or uncomfortable when using this dressing, stop using it immediately and seek medical advice.
9. When the edge of the product is not sticky due to friction or curling, it is normal.
10. This product must be stored in a cool dry place, avoid sunlight.

Timing of use

Precautions

If there is presence of diabetes, cancer, or autoimmune disease, the wound may recover more slowly. Please make sure that there is no discharge or clear liquid (exudate) on the wound.

Timing of use

- a. It is recommended to change one piece per week.
- b. The outer layer is a waterproof material, which can be used while showering, doing light yoga, or other sports. However, avoid strenuous exercise or sweating heavily.



take a shower



light yoga



avoid swimming



avoid going to
a hot tub

- c. If you sweat heavily after exercising, please change the dressing immediately. Too much moisture will reduce the adhesiveness of the dressing.



Manufactured For:

Motif Medical, LLC
Dover, DE 19901
Made in Taiwan