



How To Put It On

Locked-On Cast

Removable Splint

STEP 1

Place thumb half of cast on.
Place other half on, and hold together to test fit and make any adjustments* needed before securing to arm. (*See back.)



STEP 1

Pinch toggle and move to leave a small loop at the end of each bungee as shown.
Slide arm into splint.



STEP 2

Once the fit is comfortable, feed the end of the cable tie through the hole in the slot side as shown.



STEP 2

Place finger in bungee loop, pinch toggle and slide toward the cast tightly as shown.



STEP 3

Bend the cable tie and feed it back through the second hole.
Feed the end through the head of the cable tie as shown.



STEP 3

Wrap loop across slot and secure onto hook.
Press toggle back against cast.
See video at:
<https://youtu.be/VxelS1Ke34M>



STEP 4

Pull end of cable tie with a rocking motion to tighten. (You may use pliers if needed.) The cable tie head should recede into the slot.
Ensure there is no movement between the cast halves.
Clip off the excess tail of the cable tie.

STEP 5

To remove the cast, clip the cable ties off with nail clippers.
Additional 4" cable ties may be purchased at any local hardware store.



ActivArmor Devices can be converted from a fixed device to a removable device with the included bungee kit.

See here for instructions:



https://www.youtube.com/watch?v=hUzJ35_uvVw