



PRODUCT INFORMATION GUIDE

To Put On: Open clamshell of device and slide onto limb.

Removable Option (Bungees): Tighten the cord by placing your finger into the loop and pulling it tight while holding the toggle clamp open. Once the clamp is snug, wrap the loop back over the top, and secure onto the hook on the opposite side to hold the loop secured. To remove, simply unhook, pinch the toggle clamp and slide back. <https://youtu.be/VxelS1Ke34M>

Non-Removable Option (Zip Ties): Feed the end into the slot, and bend up and back through. Feed the end through the head of the zip tie and pull tight until head of zip tie hides inside slot. Clip end off close to the zip tie head.

To Remove: Use the needle nose clippers provided in your tool kit to cut the zip tie. In an emergency, slide a cutting tool down the seam of the cast, being careful not to cut the patient's skin.

REMOVAL VIDEO:

<https://youtu.be/DKTH528AV9c>



ADJUSTMENTS VIDEO:

<https://youtu.be/TBxdiddULPY>



SPACERS: Spacers are provided with each device. If tightness or swelling occurs with the non-adjustable closure option, spacers may be placed between the halves and the zip ties threaded through.

PLASTAZOTE: Adhesive-backed padding is also provided to tighten or pad the device as needed. Cut into small strips and remove adhesive backing and place inside the device as needed. They can be sanded with fine-grit sandpaper to smooth or make slight changes to the fit.

Use Indication: ActivArmor Canada Inc. devices are labelled for external, non-sterile use to provide support, stability, and immobilization for extremities.

All consumers are required to sign a waiver, either electronically for online orders, or in their provider's office, choosing to use ActivArmor's products or upon the advice and oversight of their physician, and at their own risk. Whether you have signed the waiver or not, by accepting this device, the consumer agrees to these terms and conditions. The consumer acknowledges that ActivArmor Canada Inc. has not and will not be providing the consumer any medical advice concerning their injury or health. The consumer agrees to have their physician check and approve the device, its use, positioning and fit, before using as medical treatment. The consumer accepts full responsibility for any damage or harmful results that may occur as a result of products sold by ActivArmor Canada Inc., including ActivArmor custom orthoses. Should the consumer have any health questions, pain or negative healing outcome as a result of the injury, it is their responsibility to contact their physician immediately. Should you have any questions, please contact ActivArmor Canada Inc. at 1-873-500-5228 (Toll Free)

The consumer indemnifies and holds harmless ActivArmor Canada Inc., and ActivArmor, LLC, should any damage or injury occur, resulting from or related to the performance of ActivArmor Canada Inc. or ActivArmor, LLC staff or products. The consumer relinquishes the right to sue or receive any compensation as a result of the design, manufacturing, fit, or marketing defects, or should damage arise from the use of said product, whether the product is used properly or improperly. The consumer acknowledges that all products created, manufactured, or distributed by ActivArmor Canada Inc., or ActivArmor LLC, are offered on the condition that their users accept full responsibility for any adverse outcomes or ill effects the product may cause. Due to the custom designed and fabricated nature of the product, the consumer acknowledges that there are no refunds.

ActivArmor devices have not been tested for acute injuries such as bone fractures, and **are not intended to provide bone-level strength to the extremity.**

ActivArmor™ devices may cause muscle atrophy with extended use. Ask your physician for the appropriate use intensities, times and duration for your condition. Children using ActivArmor™ may need to be supervised by adults familiar with this Product Information Guide and doctor instructions.

ActivArmor™ devices are fit to body image scans and will no longer fit in cases of changes in body shape or weight of the user. If a device no longer fits, it should be recycled or disposed of, and users should contact their doctor to be fitted for a new device. **Washing Instructions:** ActivArmor™ does not need to be removed for swimming or bathing and can be cleaned with mild soap and water. If water stays trapped under the device, use a hair dryer to dry the skin underneath.

ActivArmor™ devices should be stored at room temperature and not be left in extreme heat, cold, or exposed to hot direct sunlight for extended periods of time.

Warnings: Do not expose devices to acetone or open flame. The devices are flammable. If swelling occurs, loosen or remove the device immediately and ask your ActivArmor providing clinic about spacers that can be used to bi-valve the splint. Discontinue use in the event of reaction or breakage. Use only under adult supervision. Clips can be a choking hazard. The devices are not intended to be as durable or protective as a fiberglass cast. Do not use for impact resistance. Do not use the device for leverage for your whole body or to move objects heavier than 15 lbs (6.8 kg). Provide children using ActivArmor™ with adult supervision that is aware of this product information guide and doctor instructions. ActivArmor™ devices are to be used at the sole risk of the consumer, and are not labeled for use with injuries requiring physician oversight or for acute injuries. ActivArmor™ is not responsible for injury or adverse healing outcomes from the device use or misuse.

Warranty Info: ActivArmor devices made in Canada are warranted by ActivArmor Canada Inc. for 12 Months from receipt for breakage while under recommendation use listed and not warranted against misuse or issues requiring a device redesign.

Should you have any questions, please contact ActivArmor Canada Inc. at 1-873-500-5228 (Toll free)