

# Fentanyl Drug Test

## Test Procedure



### 1 What do you need



**DRUGS**



**PLASTIC  
BOTTLE CAP**



**10MG OF SUBSTANCE  
(RED AREA of teaspoon)**




**FENTANYL TEST  
STRIP**

### 2 Crush and shake

0.1



0.2

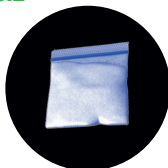

=



### 3 Measure and dilute

**METH OR  
ECSTASY**

=

**10MG**



+

**1 tablespoon of water**



**ALL  
OTHER  
DRUGS**

=

**50MG**



+

**1 tablespoon of water**



## 4 Stir and test

0.1



0.2



0.3




## 5 Result Interpretation

**FENTANYL**



**NO FENTANYL**

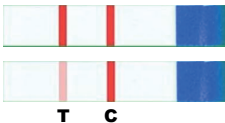

**INVALID**



**NOTE:** The intensity of color in the test region (T) may vary. If the control line (C) is still present, any shade of color in the test region should be considered negative.

## 6 Video Instructions



Follow us on Instagram  
and

**JLB SOURCING**

[www.jlbsourcing.com](http://www.jlbsourcing.com)

**YOUR FEEDBACK MATTERS!**



**amazon**  
★★★★★

