


INVALID RESULTS

If the control line (C) is not visible the test is invalid, even if any test line is visible. Re-test with a new swab and new test device.

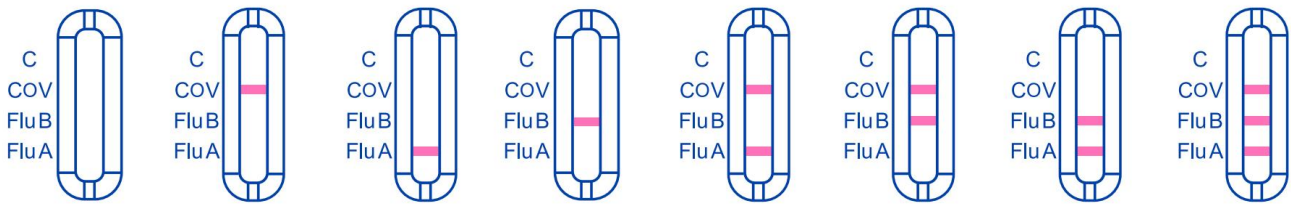


NEGATIVE RESULTS

If the control line (C) is visible, but no other lines appear the test is negative.

To increase the chance that the negative result for COVID-19 is accurate, you should test again in 48 hours.

If you still have COVID-19, Flu B, Flu A, or symptoms, you should seek follow-up care with your healthcare provider.

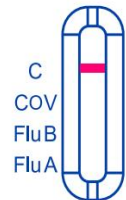

POSITIVE RESULTS

If the control line (C) is visible and one or more lines appear(s) for any of the viruses, the test is positive for that or those viruses.

NOTE: It is possible to have more than one positive test line, which could indicate a co-infection with influenza A, B, and/or SARS-CoV-2. If more than one positive test line is observed, retest with a new sample and new test kit. If you continue to have a “dual positive” result, you should contact your healthcare provider to be tested with a molecular assay to confirm your results.

