



KONICA MINOLTA

REGIUS Σ II



Giving Shape to Ideas

COMPACT

REGIUS Σ II

■ Konica Minolta's Ultra-Compact CR

The REGIUS Σ II is Konica Minolta's smallest and lightest CR. The desktop CR has a footprint of only 0.31m² and weighs a mere 28kg.

0.31m²
& 28kg



■ Environmentally friendly

Energy consumption has been reduced to 100VA- about the same as a household light bulb. That's one tenth the power of the smallest film processor.

■ Powerful

The powerful REGIUS Σ II achieves high throughput of 60plates/hour (maximum). It provides smooth and stress-free workflow.

HIGH QUALITY

■ High Resolution

The REGIUS Σ II supports pixel sizes of 87.5 μ m and 175 μ m for all cassette sizes, achieving a maximum resolution of 4020x4892 pixels (14x17 at 87.5 μ m).




87.5 μ m/
175 μ m



■ Integral Processing

Integral Processing - the result of Konica Minolta's vast imaging experience - is a robust algorithm that produces the optimum diagnostic image automatically. This patented technique greatly simplifies x-ray image acquisition while delivering incredibly consistent image quality and virtually eliminating retakes.

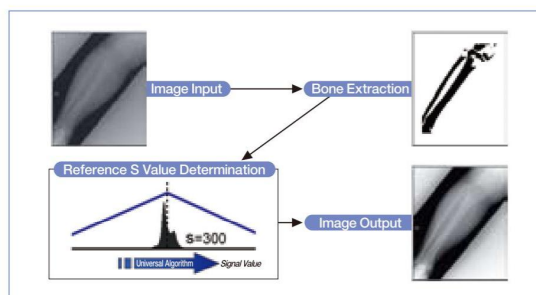

SMART CONSOLE

ImagePilot

■ All-in-One!

The ImagePilot software combines CR image acquisition and PACS functionality. It is a complete workflow solution for clinics, imaging centers, and small hospitals. From one GUI, users can acquire, review, distribute, and archive patient images.



■ Easy Control

ImagePilot supports a multi-touch display for the easiest operation. The more than 100 functions and features are literally at the tips of your fingers.



From patient registration to long term archive, Konica Minolta provides complete workflow.



I. Register

Manual or automated patient registration from anywhere in the office. Select the patient from the worklist or input patient information. No need to specify body part or exam type.

■ Patient data management

The ImagePilot server can manage more than 200,000 patients. With any registered patient, users can acquire new images or review prior exams.

■ Seamless Link with EMR and RIS

Automatically receive patient information from any EMR or RIS that supports DICOM or HL7.



■ Useful Registration

Use an existing PC* (i.e. at the reception area) for patient registration.

*Please confirm the specification for the client PC of ImagePilot.



II. Digitize

Just insert an exposed cassette into the REGIUS Σ II CR reader. The ImagePilot then automatically processes the image for optimized diagnostic viewing.

■ Quick Preview

The fully automated image processing produces consistent, optimized images that display within 39 seconds.

■ Print composition

ImagePilot supports a print composer mode. Use this mode from any client station to print multiple images on one sheet of film or paper.

■ Easy recovery

If the plate jam occurs, it can be removed easily by opening the top cover or the backside cover.



III. View

Images can be viewed at up to 20 ImagePilot client workstations throughout the office. Each client supports all the same ImagePilot functions - the same GUI at every seat.

■ Multifunctional Viewing


ImagePilot supports over 100 features and functions.

- Annotation
Line/Arrow/Rectangle/Text/
Shutter/Marker, etc
- Scaling
Cardio Index / Distance/
Angle/Rectangular CT/
Pelvis Analysis, etc
- Image Tool
Window Level / Zoom / Pan
Magnifier/REGIUS Processing/
Cropping, etc
- Report
- Other Modality Image



■ Web Viewing

At your home and trip destination, you can see the images of ImagePilot via VPN.

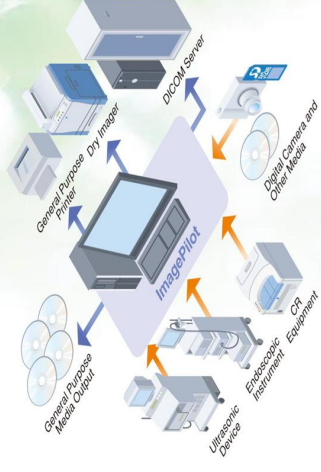

IV. Archive

With storage capacities up to 2TB, ImagePilot can archive up to 300,000 images on our expandable servers. And this can be expanded even further with add-on storage.

*For available configurations, please contact your salesperson.

■ DICOM Image Communication

ImagePilot supports DICOM Store for sending patient images to a remote PACS or for receiving images from other PACS systems or modalities. ImagePilot also supports DICOM query/retrieve for greater utility.




■ NAS/DVD Backup

Patient images and other data can be securely backed up using either the NAS or DVD backup tools.

■ PDI Output

Patient images can be written to CD or DVD in PDI format (With DICOM software). The images can be viewed by any PC.

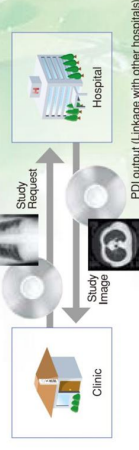

REGIUS SIGMA2 Specifications

■ Cassette size

14"×17" / 14"×14" / 11"×14" / 10"×12" / 8"×10" /
18×24cm / 24×30cm / 15×30cm

■ Sampling Pitch

2Types: 87.5μm / 175μm

■ Maximum Resolution

4,020×4,892 (14"×17", 87.5μm)

■ Digital Gradation Level

4096 levels (12bit)

■ Processing Capability

Up to 60 plates / hour (14"×17" / 175μm)

■ Slots

Insertion / ejection slot × 1

■ Outer Dimensions/Weight

W510×D610×H355mm / Approximately 28kg

■ Power Supply Conditions

100VA max during operation: 20VA when not in use.

■ Operating Conditions

Temperature : 10-30°C

Humidity : 15-80%RH (No Condensation)

■ Readable Cassette

REGIUS CASSETTE RC-300

14"×17", 14"×14", 11"×14", 10"×12", 8"×10",
24×30cm, 18×24cm, 15×30cm



■ Readable Plate

REGIUS PLATE FP-1S

14"×17", 14"×14", 11"×14", 10"×12", 8"×10", 24×30cm, 18×24cm, 15×30cm



REGIUS SIGMA2 Outer Dimensions

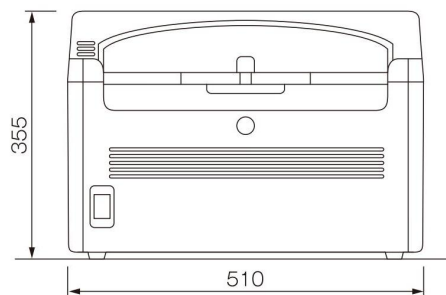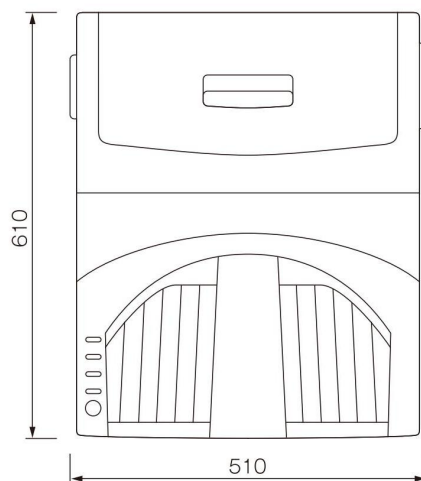

Unit : mm 1/10 scaled

Brand Name : REGIUS ΣII
Type : DIRECT DIGITIZER
Model : REGIUS SIGMA 2

★Specification are subject to change without prior notice.



- Lowest Weight CR (approx. 28kg)
- Power consumption is less than one tenth of prior models.
- Smallest carbon footprint – CO₂ emissions reduced by more than 64%
(Calculated at lifecycle, compared to our conventional CR)



KONICA MINOLTA

KONICA MINOLTA, INC.

1 Sakura-machi, Hino-shi, Tokyo, 191-8511, Japan

Marketed & Serviced by:

KONICA MINOLTA HEALTHCARE INDIA PVT. LTD.

Office No 1121, 2nd Floor, Building No 11, Solitaire Corporate Park,
Chakala, Andheri East, Mumbai 400093

Sales Enquiry in West Bengal:

SynerMED Technologies LLP

183, S K Deb Road, Kolkata - 700048

Phone: 9681374490 / 9073481182

Email: synermedtechnologies@gmail.com

Website: www.synermed.co.in