




# FENTANYL TEST STRIP QUICK GUIDE

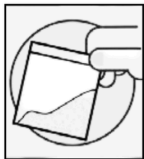

Crush and mix the substance well to distribute any fentanyl that might be inside.

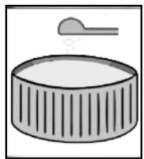

Add **10mg** into a teaspoon (5 ml) of water. A plastic bottle cap holds just over a teaspoon.

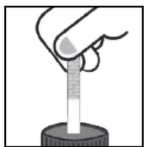

Hold the end labeled FEN and dip the strip in the water for 15 seconds. Do not insert beyond the dotted area.




Lay the strip on a flat surface for 3 minutes.




Any second line, no matter how faint, is a **NEGATIVE** result




# XYLAZINE TEST STRIP QUICK GUIDE

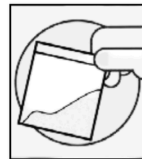

Crush and mix the substance well to distribute any xylazine that might be inside.

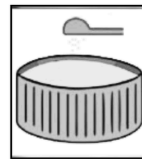

Add **10mg** into a teaspoon (5 ml) of water. A plastic bottle cap holds just over a teaspoon.

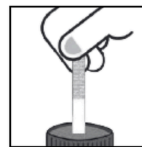

Hold the end labeled FEN and dip the strip in the water for 15 seconds. Do not insert beyond the dotted area.



Lay the strip on a flat surface for 3 minutes.



Any second line, no matter how faint, is a **NEGATIVE** result

