

DRUG TEST CUP PROCEDURE CARD




Security Seal Label

Cup Body


Peel-Off Label

Cup Panel Label

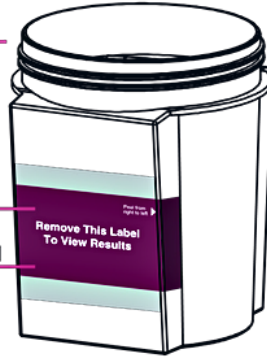

Cap

Temperature Indicator Strip

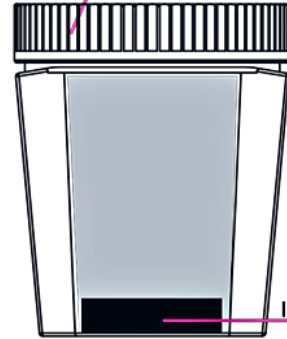

- 1** Collect urine specimen.
If desired, cap can be held by operator, urine specimen between minimum 20 mL to maximum 60 mL (Marked on the cup).



- 2** Once urine sample has been collected, screw cap firmly until you hear a click. Operator checks cap for tightness.
Start timer



- 3** Remove peel-off label.




- 4** After 2 minutes check urine temperature by reading green mark on temperature indicator strip. If applicable, read adulteration strip pads against enclosed color card.



	Normal	Abnormal
DOB		
SG		
pH		
NET		
GLUT		
CREA		


Adulteration Color Card
For Guidance Only

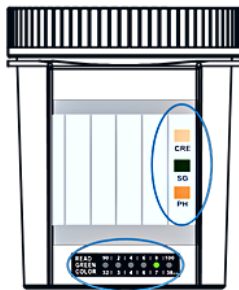

PASS: Proceed to Step 5

FAIL: Repeat test

- 5** Read test results at 5 minutes. Refer to diagram below for result interpretation.

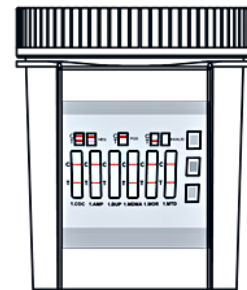


C
T or
NEGATIVE

C
T
POSITIVE

C
T or
INVALID

- 6** If there is presumptive positive result, operator attaches the security seal label over cap as shown, with donor's name, initials and test date.

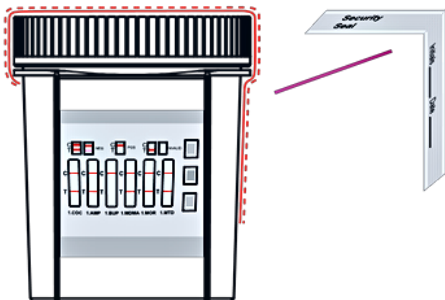

Important: Do not cover test result window with the seal.

- 7** Presumptive positive test results must be confirmed by another test method. Send cup and urine specimen with security seal label intact to a toxicology laboratory by GCMS or LC-MS/MS for confirmation.

