USER INFORMATION

CE 2777

DCK GRUP İÇ DIŞ TİCARET MEDİKAL LTD.ŞTİ

MACUN MAH. BATI BULV.
ATB İŞ MERKEZİ E BLOK 1/108 YENİMAHALLE
ANKARA 06374 TÜRKİYE

+90 312 387 00 17

info@dckgrup.com
www.dckgrup.com.tr

MASTERFUL NİTRİL TEK KULLANIMLIK ELDİVEN KAT III KULLANIM TALİMATI

KULLANIM:

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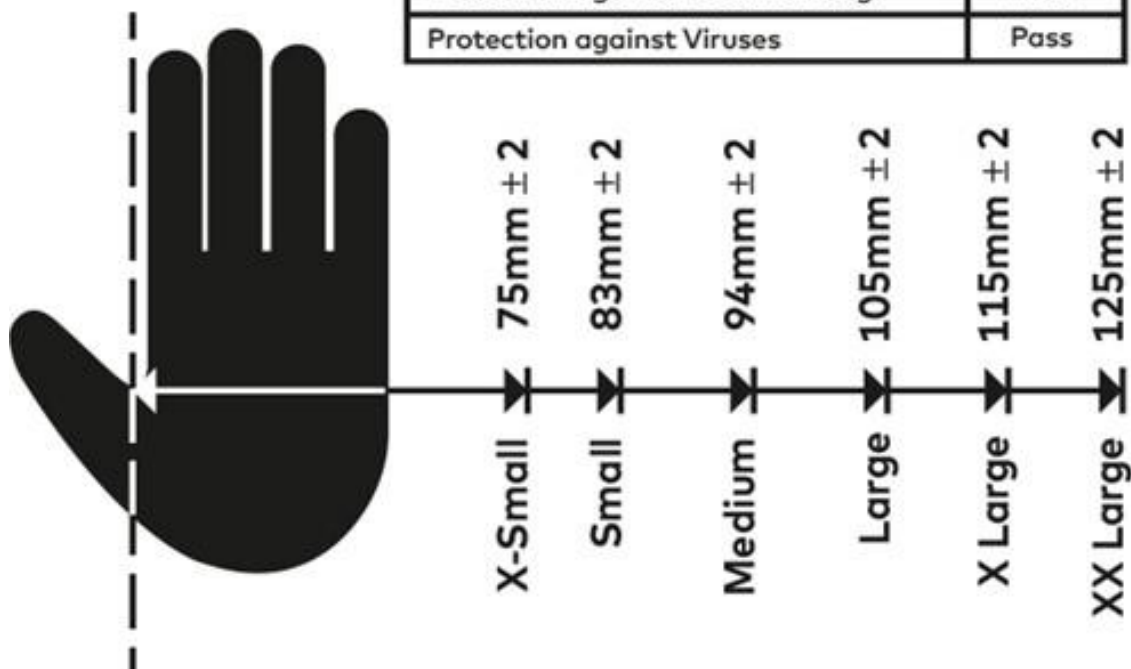
EN ISO 374-4:2019

Degradation result indicate the change in puncture resistance of the glove after exposure to challenge chemicals.

EN ISO 374-5:2016	Level
Protection against Bacteria Fungi	Pass
Protection against Viruses	Pass

CAUTION: Primary Material-Nitrile Rubber. Does not contain Natural Rubber Latex.Components

used in making gloves may cause allergic reactions in some users.

Good quality nitrile gloves provide an excellent biological barrier.

Store in cool, dry place, avoid excessive heat (40°C, 104°F). Open box should be shielded from exposure to direct sun or fluorescent lighting.

Conformity assessment based on Regulation (EU) 2016/425 and SATRA Technology Europe Limited,
Bracetown Business Park, Clonee, D15YN2P, Ireland
Instruction for donning and doffing

This information does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals.

“The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 400 mm - where the cuff is tested also) and relates only to the chemical tested. It can be different if the chemical is used in a mixture.”

“It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation.”

“When used, protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves”

“Before usage, inspect the gloves for any defect or imperfections.”

Storage and transport: When not in use, store the product in a well-ventilated area away from extremes of temperature

Glove performance quoted is based on laboratory data and may not reflect the actual duration of protection in the workplace due to other factors influencing the performance such as temperature, abrasion, degradation etc.)

The glove does not contain any substances that are known to cause allergies.

The Gloves have no mechanical protection offered.

For single use only, do not littering.

Check for damage before use, do not use damaged gloves

How to put the gloves on

- a. Take out glove from the box.
- b. Hold the glove by the cuff with the same hand that you use to pull it out from the box and put the other hand into the glove without touching the working surface.
- c. Stretch the glove on the glove wearing hand.
- d. Take another glove from the box with the glove wearing hand.
- e. Put the glove on the other hand and smooth it out.

How to take the gloves off

- a. Grasp the outside edge of the one cuff with gloved hand, avoid touching the wrist.
- b. Pull the glove off, turning it inside out and hold it with your gloved hand.
- c. Slide the un-gloved thumb under the cuff of the remaining glove.
- d. Peel the glove off inside out over the previously removed glove and bagging both gloves.
- e. Remove gloves should be discarded with appropriate waste disposal container after use according to local disposal regulations.

Caution statement

The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only and relates only to the chemical tested. This information does not reflect the actual duration of the protection in the workplace and the differentiation between mixtures and pure chemicals.

It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation.

When used, protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves. Before usage, inspect the gloves for any defect or imperfections. Discard when the gloves are damaged.

The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen.

DCK GRUP İÇ DIŞ TİCARET MEDİKAL LTD.ŞTİ

MACUN MAH. BATI BULV. ATB İŞ
MERKEZİ E BLOK 1/108 YENİMAHALLE ANKARA 06374
TÜRKİYE

+90 312 387 00 17

info@dckgrup.com www.dckgrup.com.tr