

Advanced Wound Care



Skin Health

A holistic approach to healing.



Discover evidence-based products to help **treat and manage hard-to-heal wounds**

Ensure your facility has the tools and expertise to tackle chronic wounds with the broadest range of advanced wound care products in the industry and a comprehensive package of educational resources.

Together we help you implement best practices to positively affect the quality and efficiency of care and optimize patient outcomes.

Our latest innovations

PluroGel ® concentrated surfactant technology	p. 7
IoPlex ® with I-Plexomer™ technology —the world's only controlled release iodine foam dressing	p. 13
Optifoam ® Gentle Ag+ Post-Op with Smart Stretch™ technology	p. 23
OptiView ™ with HydroCore® technology	p. 31
Opticell ® with Chytoform technology	p. 34
AccuWrap —designed with Canadian clinicians in mind	p. 49
PureWash ™ Hypochlorous Acid Wound Cleanser	p. 59

Empower clinicians with customized support

Medline's clinical and educational resources provide you with support you need to raise the standard of care.

- **Skin Health Product Specialists**
Help you optimally leverage the support Medline offers as you strive to enhance quality of care.
- **Educational Packaging**
Ensure that caregivers have complete step-by-step instructions for use at their fingertips.
- **Medline University**
Expand staff expertise with unlimited access to over 200 free continuing education courses for healthcare professionals. Visit: MedlineUniversity.com.
- **The NE1 Wound Assessment Tool**
Helps increase accuracy, consistency and transparency in wound assessment.
- **Skin Health Solutions**
Minimize care variation across your facility with a customized approach to standardizing product, practice and education.
- **Wound and Skin Care Guidelines**
Clinical support to help you determine and communicate your treatment guidelines.
- **Online Product Application Videos**
Complete library of instructional application videos helps train new staff on proper product application. Visit: www.medline.com/awcvideos.
- **Advanced Wound and Skin Care Pocket Reference Guide**
This compact booklet contains a wealth of clinical and product information for use as a reference tool for caregivers.
- **Skin Health Digital Platform**
Mobile app makes it easier for caregivers to do the right thing. Encourage staff to download our Skin Health app with QR code.



DIMES

A systematic approach to wound healing

You face diverse and complex challenges. That's why we offer a wide variety of products and programs that help clinicians deliver world-class care tailored to any given clinical context.

Throughout this catalogue, you'll find evidence-based products*, information and resources, all organized around the DIMES system of wound bed preparation and treatment.



*Contact Medline Industries, LP. for data related to the products mentioned herein.

D

Debridement

The removal of nonviable tissue by creating an optimal moist wound environment.

PluroGel®	7
TheraHoney®	10

I

Infection/Inflammation

Addressing bioburden and inflammation in the wound.

IoPlex®	13	Optifoam® Ag+ Post Op	22
Opticell® Ag+	16	Optifoam® Gentle Ag+ Post Op	23
Maxorb® Ag+	19	PrimaSeal® Post Op Ag+	24
Optifoam® Ag+	20	SilvaSorb®	25
Optifoam® Gentle Ag+	21		

M

Moisture Balance

Achieving and maintaining moisture balance in and around the wound.

Optifoam® Gentle	27	OptiLock®	36
Optifoam®	30	Qwick™	37
OptiView	31	Exuderm®	39
Opticell®	34	Hydrogel Dressings	40
Maxorb®	35		

E

Edge/Environment

Treating stalled wounds where epithelium fails to migrate.

Puracol® Collagen Dressings	42
-----------------------------	----

S

Support Products, Services and Education

Complementary products to enhance outcomes and round out your wound healing toolkit.

Marathon®	45	Gentac® Silicone Island Dressing	56
Sureprep®	48	Cover Dressings	57
AccuWrap™	49	Dressing Retention Tapes	58
TwoFlex®	51	Wound Cleansers	59
ThreeFlex® and FourFlex®	52	OptiClose™	60
Unna-Z Stretch	53	NE1® Wound Assessment Tool	63
Medigrip™ and Medigrip LF	54	Education and Support	64
Versatel® and Versatel One	55	Skin Health Digital Platform	65

DIMES

Debridement

The first step in wound healing is the removal of necrotic tissue that can impede the growth and proliferation of healthy granulating tissue.

Sphere-shaped micelles trap wound debris for subsequent removal

PluroGel® Burn and Wound Dressings

A unique approach to wound management

PluroGel is a concentrated surfactant gel containing Poloxamer 188 (P-188). This unique copolymer aids the burn injury debridement process by helping to define the areas that may require surgery and sometimes avoiding the need for excision.¹

What is P-188?

P-188 is a multi-block copolymer surfactant with an inverse thermosensitivity. It is soluble in aqueous solutions at cool temperatures and gels at warmer temperatures. It has low toxicity, it is less irritating to skin and it is completely compatible with other therapeutic agents.²

Promotes patient comfort

- Facilitates the removal of necrotic tissue and slough
- Softens and loosens debris to prepare the wound bed for healing
- Maintains optimal moist wound healing environment that can help protect healthy tissue and soften wound debris
- Indicated for partial and full thickness wounds, dry to moderately draining wounds, and 1st and 2nd degree burns



Prepares the wound for healing



Full-thickness dehisced surgical wound

A 55-year-old female with a history of poorly controlled diabetes presented with a dehisced groin incision following a femoral endarterectomy. The wound bed was covered with yellow slough and draining copious amounts of clear fluid. Sharps debridement was performed, and then the wound was packed daily using concentrated surfactant-based gel (CSG) dressing, PluroGel.³

89% reduction
in wound size¹



Week 1
Wound measures 12 x 10 x 2 cm



Week 2
Wound measures 5 x 4 x 2 cm



Week 3
Wound measures 4.5 x 3 x 0.5 cm

Rapid response to undermining wounds*

54 year old patient Thrombophilia fell during the night and developed a large hematoma on left anterior shin.⁴



Day 1



Day 14



Day 27



Day 55

Application of PluroGel Burn and Wound Dressing daily resulted in rapid response—a dramatic reduction in wound size and resolution of undermining. Wound healed in 8 weeks.

* Medline owns the rights to the photos in this case study.

About PluroGel

- 100% water-soluble, bio-compatible, cell-friendly
- Softer wound debris rinses off more easily at dressing change
- Helps to maintain a moist microenvironment
- Remains on the wound bed for better protection

Recommended Uses

- Light to moderately draining wounds

Recommended Secondary Dressings

- Optifoam® Gentle
- Optifoam Thin (dry wounds)
- Petrolatum based dressings (burns)

Indications

Over-the-counter:

- Minor cuts, minor abrasions, minor lacerations and minor burns

Under the supervision of a healthcare provider:

- Partial and full-thickness wounds
- First and second degree burns

Change Frequency

- PluroGel should be changed daily or as needed or directed by a physician to keep wound moist
- PluroGel can remain in the wound bed for up to 3 days
- Be sure to apply 5 mm thick if changing every 3 days

Ordering information

Item No.	Description	Pkg.
PGL007	7 g (0.25 oz), Tube	50/cs
PGL020	20 g (0.70 oz), Tube	35/cs
PGL050	50 g (1.75 oz), Jar	24/cs
PGL050TUBE	50 g (1.75 oz), Tube	12/cs
PGL400	400 g (14.1 oz), Jar	6/cs



TheraHoney Honey Wound Dressings

Create an optimal wound healing environment.

TheraHoney products contain 100% sterile, medical-grade Manuka honey with no fillers or additives. Honey is well-tolerated by patients and helps promote healing in hard-to-treat wounds. Its high sugar content and low water activity result in osmotic pressure that promotes autolytic debridement of necrotic tissue and creates a moist wound healing environment.

Explore the 3 key benefits of Manuka honey

Promotes natural debridement

Honey's high sugar levels help clear away necrotic tissue, maintain a moist wound environment for healing and reduce wound odour.

Decreases wound pH

High wound pH can be a barrier to healing. The use of Manuka honey, with a pH of 3.5–4.5, may help lower wound pH.

Reduces wound odour underneath

Wounds can smell for a variety of reasons. The use of honey is known to reduce odours stemming from the wound bed.



About TheraHoney

- Made of 100% medical-grade honey (*Leptospermum scoparium*)⁵
- A moist wound environment allows autolytic debridement of necrotic tissue.⁶⁻⁸
- May help reduce odour.^{6,8}
- Should be paired with an absorbent dressing⁹
- Compatible with silver dressings and wound cleanser surfactants¹⁰

Recommended Uses

- All drainage levels
- As a primary dressing

Recommended Secondary Dressings

- Optifoam® Gentle
- OptiLock®
- Qwick™

Indications

Over-the-counter:

- Minor abrasions, minor lacerations, and minor cuts
- Minor scalds and minor burns

Under the supervision of a healthcare provider:

- Leg ulcers
- Pressure injuries
- First- and second-degree burns
- Diabetic foot ulcers
- Surgical and trauma wounds

Change Frequency

- TheraHoney change frequency may depend on amount of drainage but may be left in place for up to 7 days

Contraindications

- Third-degree burns
- Individuals with a known sensitivity to honey or bee venom



TheraHoney draws fluid from the wound bed, encouraging autolytic debridement. **Use Qwick Superabsorbent Wound Dressings in conjunction with TheraHoney** for optimal absorbency.

See page 37 for more info.

Ordering information

Item No.	Description	Pkg.
MNK1344	Foam Flex, 4" x 4" (10.2 x 10.2 cm)	10/bx
MNK0005	Gel, 30 mL (0.5 oz.)	10/bx
MNK0015	Gel, 44 mL (1.5 oz.)	12/bx
MNK0077	Sheet, 4" x 5" (10.2 x 12.7 cm)	10/bx
MNK0082	HD Sheet, 2" x 2" (5.1 x 5.1 cm)	10/bx
MNK0087	HD Sheet, 4" x 5" (10.2 x 12.7 cm)	10/bx
MNK0089	HD Rope, 1" x 12" (2.5 x 30.5 cm)	10/bx

TheraHoney HD sheet has 2x more honey than the TheraHoney sheet





DIMES

Infection/Inflammation

Potentially harmful microorganisms contaminate nearly all wounds. Actively managing bioburden and infection risk is key to effective wound management.

Antimicrobial silver and iodine dressings manage bioburden to help prevent delayed healing due to infection.

IoPlex® Iodophor Foam Dressings

Let iodine take biofilm to task.

More than 90% of chronic wounds contain biofilm, which stalls wound healing and contributes to chronicity. Biofilm is very difficult to remove because it firmly adheres to surrounding tissue and it is highly tolerant to antibiotics. Not even debridement adequately manages biofilm.¹¹

Enter iodine. Long known as a powerful antiseptic, numerous in-vitro studies demonstrate its superiority to other antibacterial agents—including silver—at controlling biofilm.¹²⁻¹⁵

Unlock healing potential with IoPlex

IoPlex with I-Plexomer™ technology is the world's only controlled release iodine foam dressing.

A proprietary controlled-release system allows for regulated and sustained infection management through the slow release of iodine within the wound dressing.

What Is Biofilm?

Biofilms are bacterial structures physically attached to a surface and characterized by significant tolerance to antibiotics and biocides. Their existence is argued as the single most important cause of delayed wound healing.¹⁶

Why Iodine?

Iodine has been used in wound care since the the 1860s. Although quite cytotoxic in its native state, the iodine of today is bonded to a carrier molecule for a gentler release that maintains efficacy against microorganisms.¹⁷



Reduces bacterial burden within the wound dressing




Effectively removes exudate and debris



Sustained release over 24 to 72 hours

Colour changes from black to white as the iodine is released

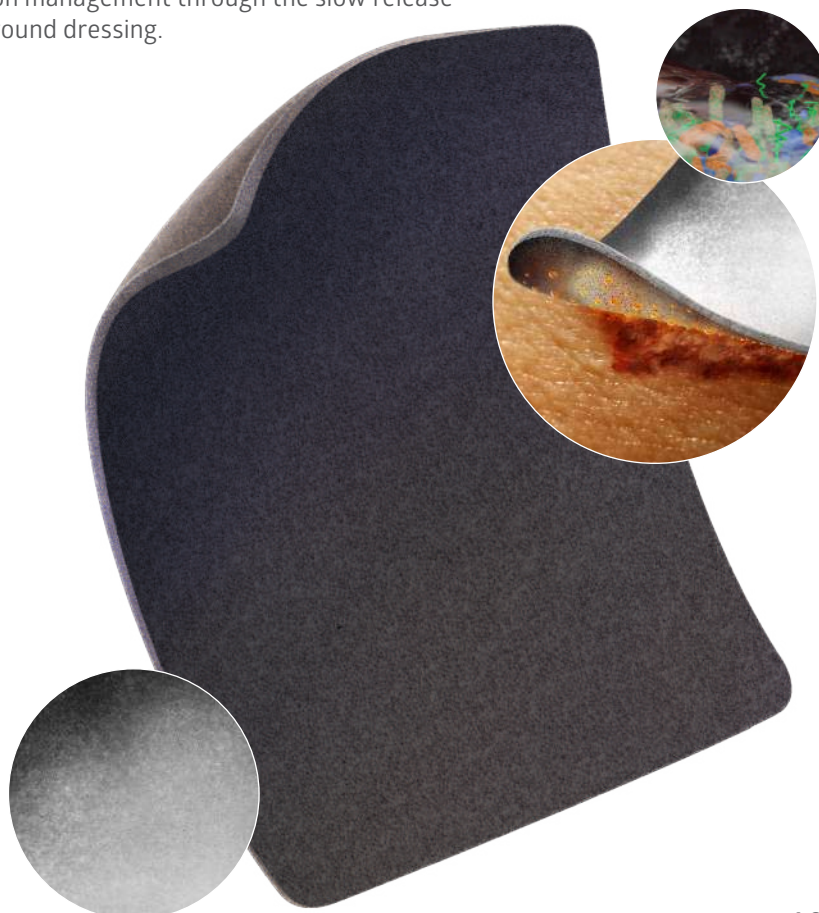

PVA foam has exceptional capillary action to remove excess wound exudate.



Highly absorbent



Can be cut to wound shape and stacked




Easy to apply and remove

IoPlex® Iodophor Foam Dressings, continued

IoPlex vs Hydrofera Blue

IoPlex killed more *P. aeruginosa* biofilm than Hydrofera Blue at 24 hours *in vitro*¹⁸

Dead biofilm Living biofilm

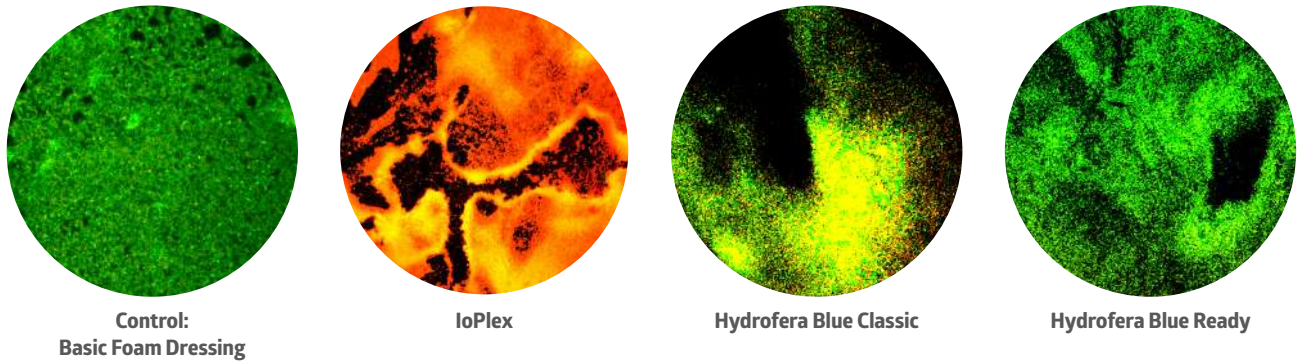


Laser microscopy. Dressing images at 40x magnification after 24 hours of biofilm exposure

IoPlex manages biofilm *in-vitro*

In-vitro testing showed that IoPlex had a greater than 4 log reduction against *S. aureus* and *P. aeruginosa* biofilm strains.¹⁸ Clinical significance of these findings have not been determined.

IoPlex demonstrated a 4 log or greater kill against MRSA in **5 min** and *P. aeruginosa* in **30 min**¹⁸

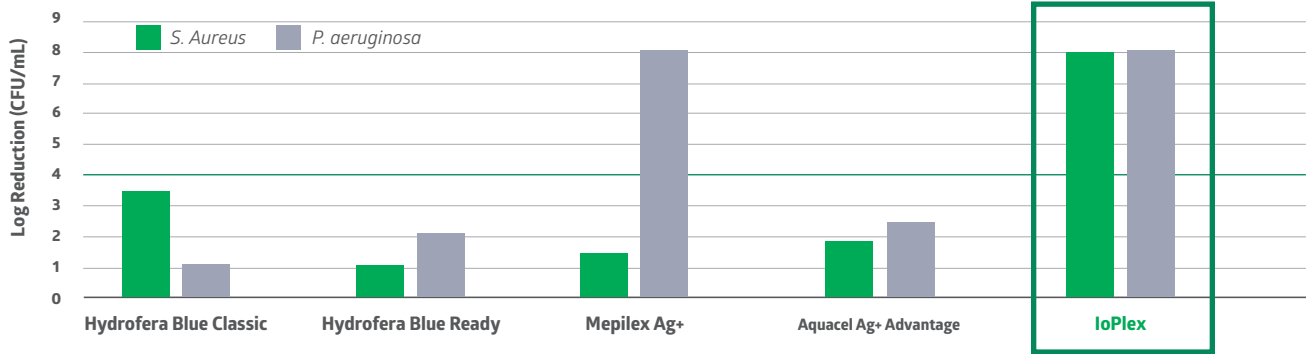

About IoPlex

- Proprietary I-Plexomer® technology delivers controlled-release of iodine
- Regulated and sustained infection management
- Iodophor foam dressing
- Changes colour
- Highly conformable and stackable

Recommended Uses

- All wound depths
- All drainage levels
- As a primary dressing

Recommended Secondary Dressings

- Optifoam® Gentle
- Bordered gauze
- Stratasorb® Composite
- Gentac Silicone Island Dressing

Indications

- Cleaning wet ulcers and wounds, including diabetic ulcers, pressure ulcers, arterial ulcers and venous stasis ulcers
- Infected traumatic wounds
- Infected surgical wounds
- Infected burns

Change Frequency

- Should be changed three times a week or when at least half of the IoPlex has changed in colour from black to yellow/off white

Contraindications

- Individuals with a history of Grave's disease, Hashimoto's thyroiditis or goiter
- Pregnant or lactating woman

Ordering information

Item No.	Description	Pkg.
MSC5322EP	2" x 2" (5.1 x 5.1 cm)	10/bx, 60/cs
MSC5345EP	4" x 5" (10.2 x 12.7 cm)	8/bx, 40/cs
MSC5369EP	6" x 9" (15.2 x 22.8 cm)	5/bx, 30/cs



Opticell® Ag+ Antibacterial Chitosan-Based Gelling Fiber Dressings

Expect more from your gelling fiber

Opticell Ag+ utilizes Chytoform™, our chitosan-based gelling fiber technology, to deliver excellent benefits to wounds. Opticell Ag+ is highly absorbent and can be moistened to address dry or lightly draining wounds.



Reinforced ribbon for extra strength

An optimal healing environment


Opticell Ag+ is highly conformable and retains its shape and size in the wound better than other gelling fibers. This quality ensures that Opticell Ag+ remains in optimal and complete contact with the wound even as it absorbs moisture.

Opticell Ag+ with Chytoform technology

The smooth, thin and conformable profile provides for intimate contact with the wound.

Competition (stitched gelling fiber)

The ridges in the dressing may lead to an uneven wound contact surface.

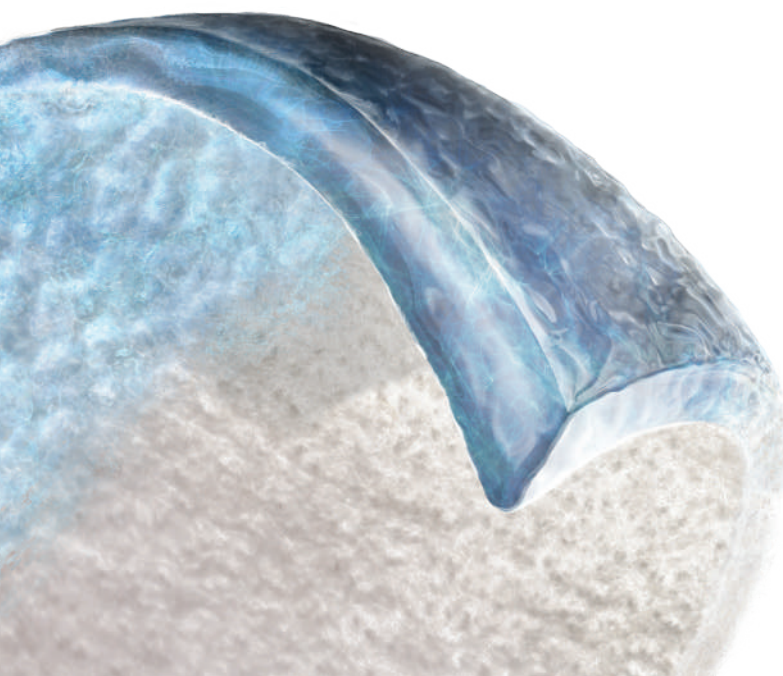


What Is Chytoform?

Chytoform is the chitosan-based gelling fiber technology inside Opticell. Chitosan is derived from shellfish, bringing the science of the ocean to wound care.



Opticell's exceptional absorption capacity helps reduce dressing change frequency and limit maceration

Absorption Study Results¹⁹

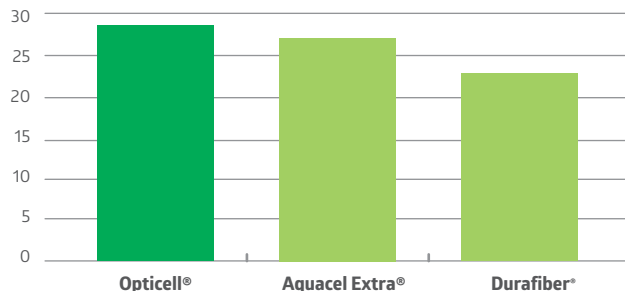

Average Fluid Absorbed (g/100cm²)

Based on *in vitro* testing, Opticell had the highest average absorbency of the three dressings tested

In vitro antibacterial efficacy of Opticell Ag+

Opticell Ag+ has been shown to effectively kill microorganisms for up to 7 days, including:¹⁹

- Methicillin-resistant *Staphylococcus aureus* (MRSA) ATCC 33591—gram positive bacteria
- *Escherichia coli* ATCC 8739—gram negative bacteria
- *Pseudomonas aeruginosa* ATCC 9027—gram negative bacteria
- *Candida albicans* ATCC 10231—yeast
- Vancomycin-resistant *Enterococcus faecium* (VRE) ATCC 51575—gram positive bacteria
- *Staphylococcus aureus* ATCC 6538—gram positive bacteria

Manage patient discomfort at the donor site

When moistened with fluid and exudate, the absorbent fibers transform into a gel



The gelling action enables effective management of drainage and removal of dead tissue

After harvesting the donor skin, Opticell is placed over the donor site

Opticell Ag+ Antibacterial Chitosan-Based Gelling Fiber Dressings, continued

About Opticell Ag+

- Chytoform™ gelling fiber technology
- Antibacterial ionic silver
- Highly conformable
- No lateral wicking—limits maceration
- Highly absorbent¹⁹
- Maintains a moist wound-healing environment
- Surface area memory (SAM)—retains its size

Recommended Uses

- All wound depths
- All drainage levels
- As a primary dressing

Recommended Secondary Dressings

- Optifoam® Gentle
- Bordered gauze
- Stratasorb® Composite
- Gentac® Silicone Island Dressing

Indications

- Partial- and full-thickness wounds
- Venous stasis ulcers
- Pressure injuries/ulcers
- First- and second-degree burns
- Diabetic foot ulcers
- Surgical wounds
- Trauma wounds
- Donor sites
- Arterial ulcers and leg ulcers of mixed etiology

Change Frequency

- May be left in place for up to 7 days
- Dressing change frequency will depend on the amount of drainage

Contraindications

- Individuals with a sensitivity to silver or chitosan, which is derived from shellfish
- Third-degree burns



Ordering information


Item No.	Description	Pkg.
MSC9822EP	Sheet, 2" x 2" (5.1 x 5.1 cm)	10/bx, 100/cs
MSC9845EP	Sheet, 4" x 5" (10.2 x 12.7 cm)	10/bx, 50/cs
MSC9866EP	Sheet, 6" x 6" (15.2 x 15.2 cm)	5/bx, 50/cs
MSC98812EP	Sheet, 8" x 12" (20.3 x 30.5 cm)	5/bx, 50/cs
MSC9818R	Ribbon, 0.75" x 18" (1.9 x 45.7 cm)	5/bx, 50/cs
MSC93918R	Ribbon, 0.39" x 18" (1 x 45.7 cm)	5/bx, 50/cs

Maxorb® Ag+ Antibacterial Calcium Alginate Dressings

More absorbency, more strength.

Antimicrobial Efficacy Vs. SilverCel²⁰

Maxorb Extra Ag demonstrates a greater efficacy in preventing bacterial proliferation than Silvercel. A higher zone of inhibition value is preferable.

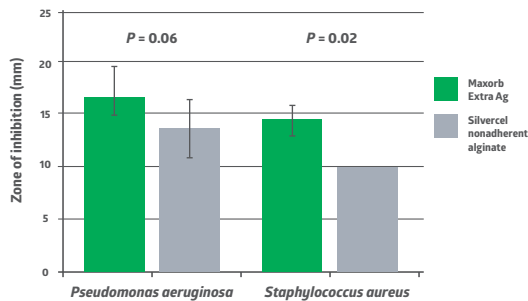

Infection/Inflammation

Ordering information

Maxorb II Ag+

Item No.	Description	Pkg.
MSC9922EP	Sheet, 2" x 2" (5.1 x 5.1 cm)	10/bx, 100/cs
MSC9945EP	Sheet, 4" x 4.75" (10.2 x 12.1 cm)	10/bx, 50/cs
MSC9948EP	Sheet, 4" x 8" (10.2 x 20.3 cm)	5/bx, 50/cs
MSC9966EP	Sheet, 6" x 6" (15.2 x 15.2 cm)	5/bx, 50/cs
MSC99812EP	Sheet, 8" x 12" (20.3 x 30.5 cm)	5/bx, 50/cs
MSC9912EP	Reinforced Ribbon, 1" x 12" (2.5 x 30.5 cm)	5/bx, 50/cs
MSC9918EP	Reinforced Ribbon, 1" x 18" (2.5 x 45.7 cm)	5/bx, 50/cs

Maxorb Extra Ag+

Item No.	Description	Pkg.
MSC9422EP	Sheet, 2" x 2" (5.1 x 5.1 cm)	10/bx, 100/cs
MSC9445EP	Sheet, 4" x 4.75" (10.2 x 12.1 cm)	10/bx, 50/cs
MSC9448EP	Sheet, 4" x 8" (10.2 x 20.3 cm)	5/bx, 50/cs
MSC9466EP	Sheet, 6" x 6" (15.2 x 15.2 cm)	5/bx, 50/cs
MSC94812EP	Sheet, 8" x 12" (20.3 x 30.5 cm)	5/bx, 50/cs
MSC9412EP	Reinforced Ribbon, 1" x 12" (2.5 x 30.5 cm)	5/bx, 20/cs

Maxorb ES Ag+

Item No.	Description	Pkg.
MSC1918EP	Reinforced Ribbon, 0.75" x 18" (1.9 x 45.7 cm)	5/bx, 50/cs

About Maxorb Ag+

- Maxorb II is a 100% calcium alginate dressing
- Maxorb Extra is a blend of CMC and calcium alginate fibers
- Helps manage bacterial burden within the dressing^{19,20}
- Controlled-release ionic silver
- Easy dressing changes
- Highly absorbent²¹
- Reinforced silver ribbon for easy removal
- Ribbon is ideal for tunneling or undermining wounds
- Cost-effective²³

Indications

- Pressure injuries^{22,24}
- Leg ulcers
- First- and second-degree burns²⁵
- Moderate to heavily draining partial- and full-thickness wounds
- Diabetic foot ulcers
- Surgical wounds
- Graft and donor sites
- Trauma wounds

Change Frequency

- Maxorb II Ag+ may be left in place for up to 7 days
- Maxorb Extra Ag+ may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Recommended Uses

- All wound depths
- Moderate to heavy drainage levels
- As a primary dressing

Contraindications

- Third-degree burns
- Dry or lightly draining wounds
- Patients with a known sensitivity to alginate or silver
- To control heavy bleeding
- As a surgical implant

Recommended Secondary Dressings

- Optifoam® Gentle
- Stratasorb® Composite
- Bordered gauze
- Gentac Silicone Island Dressing

Optifoam® Ag+ Antibacterial Foam Dressings

Treat and prevent.



About Optifoam Ag+

- Ionic silver provides an antibacterial barrier²⁸
- Continuous antibacterial protection²⁸
- Absorbent²⁸
- Conformable²⁸
- Can manage repeated bacteria introduction
- Non-staining
- Moisture vapor transmission rate (MVTR) adjusts to fluid level
- Helps create an ideal healing environment
- Waterproof, low friction and shear outer layer

Indications

- Pressure injuries
- Partial- and full-thickness wounds
- Surgical wounds
- Wounds with colonization
- Ideal for managing bioburden in post-operative wounds.
- Leg ulcers
- Diabetic foot ulcers
- Donor sites
- Lacerations and abrasions
- First- and second-degree burns
- Under compression bandages

Recommended Uses

- All wound depths*
- All wound drainage levels
- Primary or secondary dressing

*As a secondary dressing

Recommended Secondary Dressings**

- Medfix™ Tape
- Gentac™ Tape
- Medigrip™ Tubular Bandage

**Only non-bordered dressings require a secondary dressing

Change Frequency

- Optifoam Ag+ may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Contraindications


- Third-degree burns
- Lesions with active vasculitis
- Individuals with a known sensitivity to silver

Clinical highlight

Comparative Antibacterial Effect Study²⁸

Optifoam Ag+ has the least number of surviving colony forming units at four hours.

4 hrs at 37°C exposure to 4 sq cm of each dressing 106–107 Colony Forming Units (CFUs) initial population.

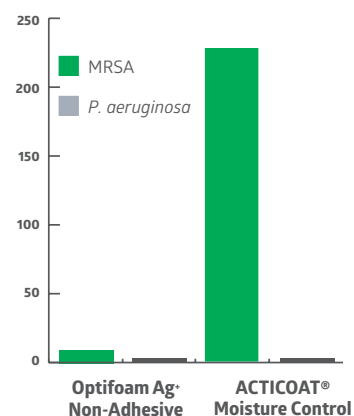

Ordering information

Item No.	Description	Pkg.
MSC9604EP	Adhesive, 4" x 4" (10.2 x 10.2 cm), Pad: 2.5" x 2.5" (6.4 x 6.4 cm)	10/bx, 100/cs
MSC9606EP	Adhesive, Sacrum, 6" x 5.5" (15.2 x 14.2 cm)	10/bx, 100/cs
MSC9614EP	Non-adhesive; 4" x 4" (10.2 x 10.2 cm)	10/bx, 100/cs

Optifoam® Gentle Ag+ Antibacterial Silicone Foam Dressings

Treat and prevent.



About Optifoam Gentle Ag+

- Silicone dressing provides gentle adhesion
- Highly conformable²⁹
- Moisture vapor transmission rate (MVTR) adjusts to fluid level
- Ionic silver provides an antibacterial barrier²⁹
- Continuous antibacterial barrier protection²⁹
- Helps maintain a moist wound-healing environment
- Waterproof outer layer
- Highly absorbent²⁹
- Non-staining
- Low friction and shear outer layer

Recommended Uses

- All wound depths*
- All wound drainage levels
- As a primary or secondary dressing

*May be used as a secondary dressing for deep wounds

Recommended Secondary Dressings**

- Gentac™ Tape
- Elastic Net
- Medigrip™ Tubular Bandage

**Only non-bordered dressings require a secondary dressing

Indications

- Pressure injuries
- Partial- and shallow full-thickness wounds
- Surgical wounds
- Wounds with colonization
- Leg ulcers
- Diabetic foot ulcers
- Donor sites
- Lacerations and abrasions
- Skin tears
- First- and second-degree burns

Change Frequency

- Optifoam Gentle Ag+ may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Contraindications

- Third-degree burns
- Lesions with active vasculitis
- Individuals with a known sensitivity to silver
- As a surgical implant

Ordering information

Antibacterial Silicone-Faced and Bordered Foam

Item No.	Description	Pkg.
MSC9644EP	4" x 4" (10.2 x 10.2 cm); Pad: 2.5" x 2.5" (6.4 x 6.4 cm)	10/bx, 100/cs
MSC9666EP	6" x 6" (15.2 x 15.2 cm); Pad: 4.5" x 4.5" (11.4 x 11.4 cm)	10/bx, 100/cs

Antibacterial Non-Bordered Silicone-Faced Foam

Item No.	Description	Pkg.
MSC9544EP	4" x 4" (10.2 x 10.2 cm)	10/bx, 100/cs
MSC9566EP	6" x 6" (15.2 x 15.2 cm)	10/bx, 100/cs
MSC9588EP	8" x 8" (20.3 x 20.3 cm)	10/bx, 100/cs

Antibacterial Silicone-Bordered Foam

Item No.	Description	Pkg.
MSC9744EP	4" x 4" (10.2 x 10.2 cm); Pad: 2.5" x 2.5" (6.4 x 6.4 cm)	10/bx, 100/cs
MSC9766EP	6" x 6" (15.2 x 15.2 cm); Pad: 4.5" x 4.5" (11.4 x 11.4 cm)	10/bx, 100/cs

Optifoam® Ag+ Post-Op Antibacterial Post-Op Dressings

Help reduce risk of surgical site infection.

About Optifoam Ag+ Post-Op

- Ionic silver provides antibacterial barrier³⁰ over incision sites
- Continuous antibacterial protection³⁰
- Thin, conformable adhesive border
- Can manage repeated bacteria introduction
- Non-staining
- Waterproof backing
- Moisture vapor transmission rate (MVTR) adjusts to fluid level
- Helps maintain a moist wound-healing environment

Recommended Uses

- Incision sites
- All wound drainage levels
- As a primary or secondary dressing

Indications

- Post-operative wounds

Change Frequency

- Optifoam Ag+ Post-Op may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Contraindications

- Third-degree burns
- Lesions with active vasculitis
- Individuals with a known sensitivity to silver



Ordering information

Antibacterial Acrylic-Bordered Post-Op Dressings

Item No.	Description	Pkg.
MSC9636	3.5" x 6" (8.9 x 15.2 cm); Pad: 1.5" x 4" (3.8 x 10.2 cm)	10/bx, 100/cs
MSC96310	3.5" x 10" (8.9 x 25.4 cm); Pad: 1.5" x 8" (3.8 x 20.3 cm)	10/bx, 100/cs
MSC96314	3.5" x 14" (8.9 x 35.6 cm); Pad: 1.5" x 12" (3.8 x 30.5 cm)	10/bx, 100/cs

Optifoam® Gentle Ag+ Post-Op Antibacterial Post-Op Dressings


Help reduce risk of surgical site infection.

Infection/Inflammation

Clinical highlight

Direct Kill Antibacterial Study³¹

In the direct kill assay, Optifoam Gentle Ag+ reduced the microbial numbers below the limit of detection and reduced viability by a log of four within 24 hours against *S. aureus*, *P. aeruginosa* and *C. albicans*.

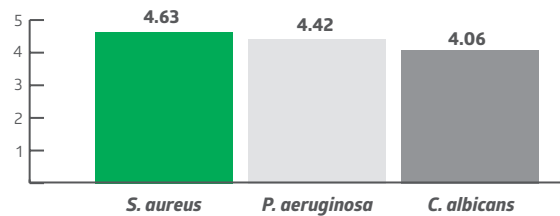

About Optifoam Gentle Ag+ Post-Op

- Ionic silver provides antibacterial barrier³⁰ over incision sites
- Continuous antibacterial protection³⁰
- Thin, conformable adhesive border
- Can manage repeated bacteria introduction
- Non-staining
- Waterproof backing
- Moisture vapor transmission rate (MVTR) adjusts to fluid level
- Helps maintain a moist wound-healing environment
- Silicone adhesive border

Recommended Uses

- Incision sites
- All wound drainage levels
- As a primary or secondary dressing

Indications

- Post-operative wounds

Change Frequency

- Optifoam Gentle Ag+ Post-Op may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Contraindications

- Third-degree burns
- Lesions with active vasculitis
- Individuals with a known sensitivity to silver

Ordering information

Antibacterial Silicone-Faced and Bordered

Item No.	Description	Pkg.
MSC9746	4" x 6" (10.2 x 15.2 cm); Pad: 2" x 4" (5.1 x 10.2 cm)	10/bx, 100/cs
MSC9748	4" x 8" (10.2 x 20.3 cm); Pad: 2" x 6" (5.1 x 15.2 cm)	10/bx, 100/cs
MSC97410	4" x 10" (10.2 x 25.4 cm); Pad: 2" x 8" (5.1 x 20.3 cm)	10/bx, 100/cs
MSC97412	4" x 12" (10.2 x 30.5 cm); Pad: 2" x 10" (5.1 x 25.4 cm)	10/bx, 100/cs
MSC97414	4" x 14" (10.2 x 35.6 cm); Pad: 2" x 12" (5.1 x 30.5 cm)	10/bx, 100/cs

Antibacterial Silicone-Bordered

Item No.	Description	Pkg.
MSC97162EP	1.6" x 2" (4.1 x 5.1 cm); Pad: 0.8" x 1.2" (2 x 3 cm)	10/bx, 70/cs
MSC9736	3.5" x 6" (8.9 x 15.2 cm); Pad: 1.5" x 4" (3.8 x 10.2 cm)	10/bx, 100/cs
MSC97310	3.5" x 10" (8.9 x 25.4 cm); Pad: 1.5" x 8" (3.8 x 20.3 cm)	10/bx, 100/cs
MSC97314	3.5" x 14" (8.9 x 35.6 cm); Pad: 1.5" x 12" (3.8 x 30.5 cm)	10/bx, 100/cs

PrimaSeal® Post-Op Ag+ Antibacterial Hydrocolloid Border Dressings

Effective post-operative protection.

Infection/Inflammation



Clinical highlight

About PrimaSeal Post-Op Ag+

- Ionic silver provides antibacterial barrier³² over incision sites
- Continuous antibacterial effect³³
- Can manage repeated bacteria introduction³²
- Non-staining
- Hydrocolloid adhesive border
- Highly breathable film to help maintain moisture balance within incision site

Recommended Uses

- Incision sites
- All wound drainage levels
- As a primary or secondary dressing

Indications

- Post-operative wounds

Change Frequency

- PrimaSeal Post-Op Ag+ may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Contraindications

- Individuals with a known sensitivity to dressing components

Comparative Antibacterial Effect Study³²

PrimaSeal Post-Op Ag+ achieves a faster 4 log kill than a competitive surgical dressing

Number of hours required to achieve 4 log kill within first 6 hours of microbe exposure³²

	<i>Pseudomonas aeruginosa</i>	<i>E. Coli</i>	<i>Klebsiella pneumoniae</i>
Competitive Surgical Dressing	4 hours	Not seen	6 hours
PrimaSeal	2 hours	4 hours	2 hours

Ordering information


Item No.	Description	Pkg.
MSCPS34	3.5" x 4" (9 x 10 cm), Pad: 2" x 2" (5.1 x 6.3 cm)	10/bx, 100/cs
MSCPS36	3.5" x 6" (9 x 15 cm), Pad: 2" x 4.4" (5.1 x 11 cm)	10/bx, 100/cs
MSCPS310	3.5" x 10" (9 x 25 cm), Pad: 2" x 8.4" (5.1 x 21 cm)	10/bx, 100/cs
MSCPS312	3.5" x 12" (9 x 30 cm), Pad: 2" x 10" (5.1 x 26 cm)	10/bx, 100/cs
MSCPS314	3.5" x 14" (9 x 35 cm), Pad: 2" x 12" (5.1 x 30 cm)	10/bx, 100/cs

SilvaSorb® Antimicrobial Hydrogel Dressings

Sustained antimicrobial protection.



Survival Curve with SilvaSorb³⁴

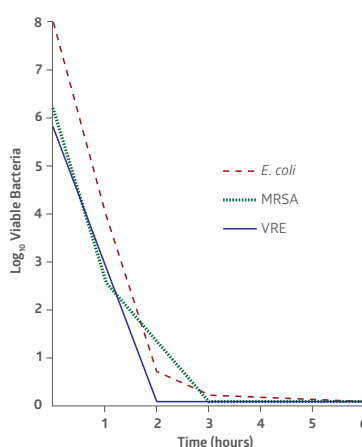

Clinical highlights

SilvaSorb has powerful antimicrobial activity (*in vitro*), 6–8 log reduction within 4 hours.

About SilvaSorb

- Helps manage bacterial burden^{34,36}
- Ionic silver
- Continuous antimicrobial protection^{34,37}
- Non-staining³⁴
- Advanced fluid management³⁴
- Extended wear time^{37,38,39}

Recommended Uses

- All wound depths
- Light to moderately draining wounds
- As a primary dressing

Recommended Secondary Dressings

- Stratasorb Composite
- Bordered Gauze
- Optifoam Gentle
- Gentac Silicone Island Dressing

Indications

- Pressure injuries
- Partial- and full-thickness wounds
- Leg ulcers
- Diabetic foot ulcers
- Graft wounds and donor sites
- Skin tears
- Surgical wounds
- Lacerations and abrasions
- First- and second-degree burns

Change Frequency

- Amorphous gel may be left in place for up to 3 days
- Dressing change frequency will depend on amount of drainage

Contraindications

- Individuals with a known sensitivity to silver

Pressure Ulcer Case Study³⁵

A 93 year-old wheelchair-bound male with no significant nutritional deficits developed a chronic trochanteric pressure ulcer. The ulcer was treated with various wound care therapies for four months without progress. The wound bed was friable and therefore presumptive for high bioburden. It was elected to start an antimicrobial dressing in conjunction with off-loading the affected hip. Closure was attained within four weeks.




Before



After 4 weeks

Ordering information

Item No.	Description	Pkg.
MSC93025EP	0.25 oz (7.08 g), Tube	25/bx
MSC9301EP	1.5 oz (42.5 g), Tube	12/cs
MSC9303EP	3 oz (85 g), Tube	12/cs
MSC9308EP	8 oz (226.7 g), Tube	6/cs

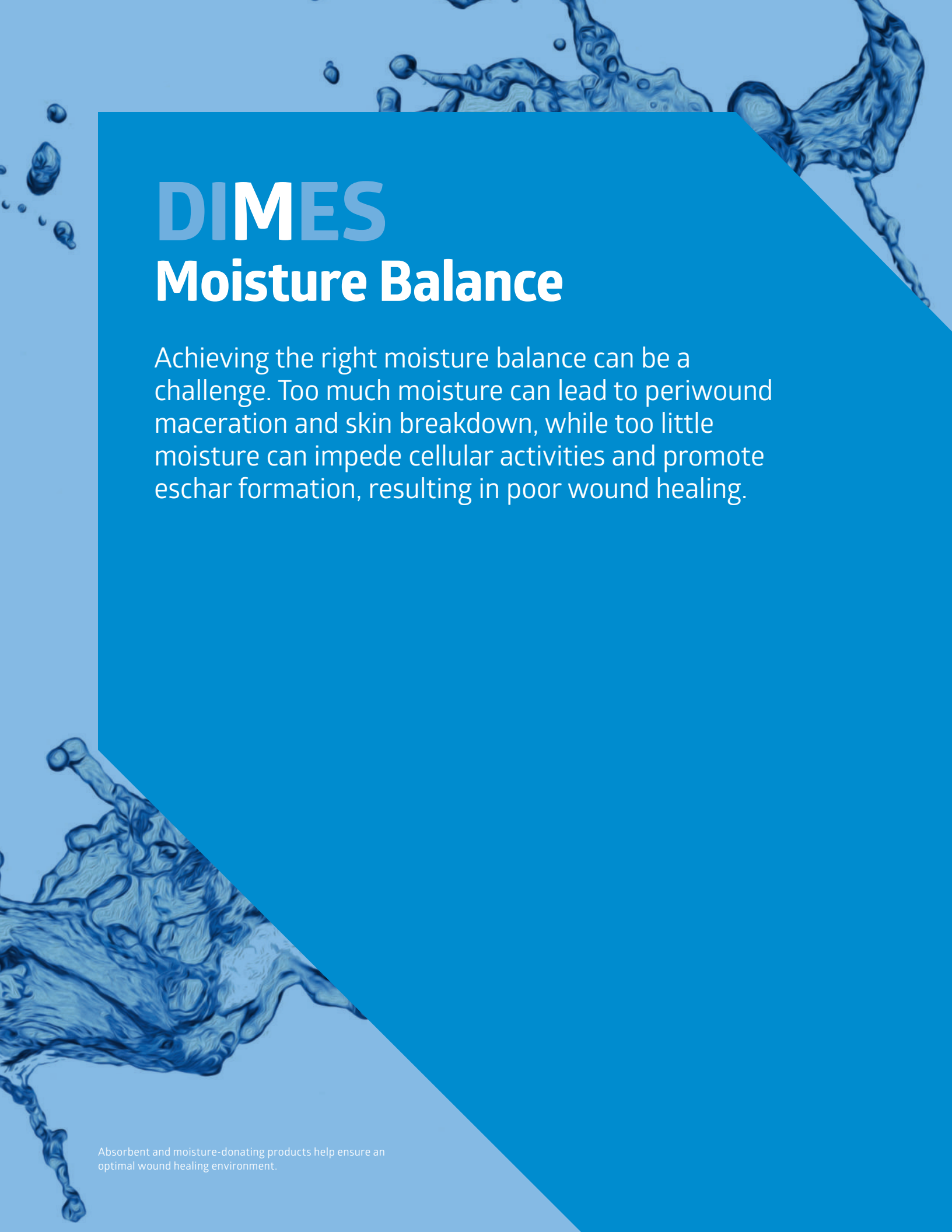The background of the slide features a dynamic water splash in shades of blue, with droplets and splashes visible at the top and bottom edges. A large, solid blue diagonal shape cuts across the center of the page, serving as a backdrop for the text.

DIMES

Moisture Balance

Achieving the right moisture balance can be a challenge. Too much moisture can lead to periwound maceration and skin breakdown, while too little moisture can impede cellular activities and promote eschar formation, resulting in poor wound healing.

Absorbent and moisture-donating products help ensure an optimal wound healing environment.

Optifoam® Gentle Absorbent Foam Dressings with Silicone Adhesive

Every wound deserves gentle care.


Optifoam Gentle helps ease the discomfort and trauma of dressing changes. Featuring a specially-formulated silicone adhesive that helps keep it in place, Optifoam's superabsorbent core absorbs and retains fluid, helps manage bacterial burden within the dressing*, and helps protect against shear and friction.

Treat and prevent with Optifoam Gentle

Foam dressings can be used as part of a pressure injury prevention protocol.

Studies in the evaluation of the use of a foam dressing with silicone border on patients at risk for hospital acquired pressure injuries (HAPI) resulted in a significant reduction in pressure ulcers over a 3-month period with lower monthly incidence and enhanced prevention.⁴⁰

More choices, more gentle care

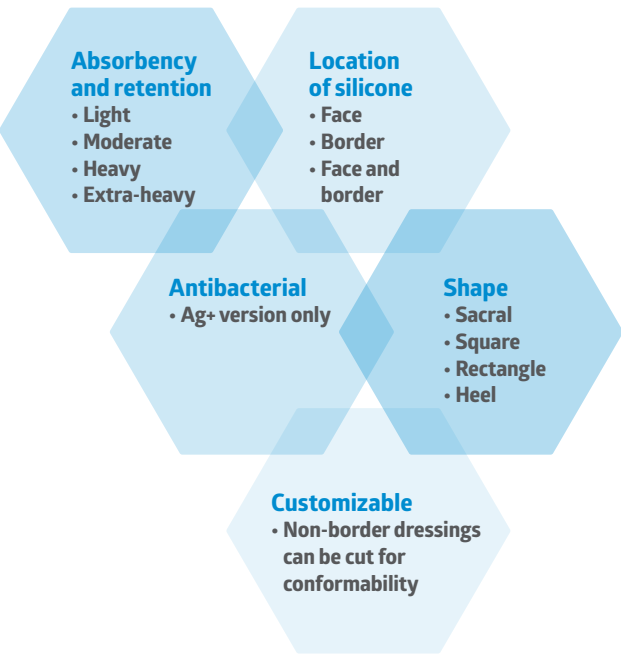

*Available on Ag dressings only

Optifoam® Gentle Absorbent Foam Dressings with Silicone Adhesive, continued

About Optifoam Gentle

- Silicone adhesive provides gentle adhesion
- Highly conformable⁴¹ dressing can be lifted and reapplied
- Moisture vapor transmission rate (MVTR) adjusts to fluid level
- Highly absorbent⁴¹
- Helps create ideal healing environment
- Waterproof outer layer protects wound and keeps dirt and germs out
- Low friction and shear outer layer

Recommended Uses

- As a primary or secondary dressing
- All wound depths*
 - *May be used as a secondary dressing for deep wounds
- All wound drainage levels:

Product	Drainage Level
Optifoam Gentle Lite	Low
Optifoam Gentle Lite Non-bordered	Low
Optifoam Gentle	Low to Moderate
Optifoam Gentle Non-bordered	Low to Moderate
Optifoam Gentle EX	Moderate to Heavy
Optifoam Gentle Flex	Moderate to Heavy
Optifoam Gentle LQ	Heavy

Recommended Secondary Dressings[†]

- Gentac Tape
- Elastic net
- Medigrip Tubular Bandage

^{††}Applies to Optifoam Silicone-Faced Foam only

Indications

- Pressure injuries
- Partial- and full-thickness wounds
- Leg ulcers
- Donor sites
- Lacerations and abrasions
- Skin tears
- First- and second-degree burns

Change Frequency

- Optifoam Gentle may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Contraindications

- Third-degree burns



Ordering information

Optifoam Gentle LQ

Silicone-faced foam and border dressings with Liquitrap core.

Item No.	Description	Pkg.
MSC2333EP	3" x 3" (7.6 x 7.6 cm); Pad: 1.75" x 1.75" (4.4 x 4.4 cm)	10/bx, 100/cs
MSC2344EP	4" x 4" (10.2 x 10.2 cm); Pad: 2.5" x 2.5" (6.4 x 6.4 cm)	10/bx, 100/cs
MSC2366EP	6" x 6" (15.2 x 15.2 cm); Pad: 4.5" x 4.5" (11.4 x 11.4 cm)	10/bx, 100/cs
MSC2377EP	Sacrum: 7" x 7" (17.8 x 17.8 cm)	5/bx, 40/cs
MSC2399EP	Sacrum: 9" x 9" (22.8 x 22.8 cm)	10/bx, 100/cs

Optifoam Gentle Flex

Silicone-faced foam and border dressings with superabsorbent core.

Item No.	Description	Pkg.
MSCFL33EP	3" x 3" (7.6 x 7.6 cm); Pad: 1.75" x 1.75" (4.4 x 4.4 cm)	10/bx, 100/cs
MSCFL44EP	4" x 4" (10.2 x 10.2 cm); Pad: 2.5" x 2.5" (6.4 x 6.4 cm)	10/bx, 100/cs
MSCFL66EP	6" x 6" (15.2 x 15.2 cm); Pad: 4.5" x 4.5" (11.4 x 11.4 cm)	10/bx, 100/cs
MSCFL77EP	Sacrum: 7" x 7" (17.8 x 17.8 cm)	5/bx, 40/cs
MSCFL99EP	Sacrum: 9" x 9" (22.8 x 22.8 cm)	5/bx, 25/cs

Optifoam Gentle Heel

Silicone-faced foam and border dressings with superabsorbent core.

Item No.	Description	Pkg.
MSC1200B	Bordered Heel: 9" x 9" (22.8 x 22.8 cm)	10/bx, 50/cs

Optifoam Gentle EX

Silicone-faced foam and border dressings, superabsorbent core.

Item No.	Description	Pkg.
MSCEX33EP	3" x 3" (7.6 x 7.6 cm)	10/bx, 100/cs
MSCEX44EP	4" x 4" (10.2 x 10.2 cm)	10/bx, 100/cs
MSCEX55EP	5" x 5" (12.5 x 12.5 cm)	10/bx, 100/cs
MSCEX66EP	6" x 6" (15.2 x 15.2 cm)	10/bx, 100/cs
MSCEX88EP	8" x 8" (20.2 x 20.2 cm)	5/bx, 50/cs
MSCEX77EP	Sacrum: 7" x 7" (17.8 x 17.8 cm)	5/bx, 40/cs
MSCEX99EP	Sacrum: 9" x 9" (22.8 x 22.8 cm)	5/bx, 25/cs

Optifoam Gentle Post-Op

Silicone-faced and bordered post-operative foam dressings with superabsorbent core.

Item No.	Description	Pkg.
MSC2148	4" x 8" (10.2 x 20.3 cm); Pad: 2" x 6" (5.1 x 15.2 cm)	10/bx, 100/cs
MSC21410	4" x 10" (10.2 x 25.4 cm); Pad: 2" x 8" (5.1 x 20.3 cm)	10/bx, 100/cs
MSC21412	4" x 12" (10.2 x 30.5 cm); Pad: 2" x 10" (5.1 x 25.4 cm)	10/bx, 100/cs

Optifoam Gentle

Silicone-bordered foam dressings.

Item No.	Description	Pkg.
MSC20162EP	1.6" x 2" (4 x 5 cm); Pad: 0.8" x 1.2" (2 x 3 cm)	10/bx, 70/cs
MSC2033EP	3" x 3" (7.6 x 7.6 cm); Pad: 1.5" x 1.5" (3.8 x 3.8 cm)	10/bx, 100/cs
MSC2044EP	4" x 4" (10.2 x 10.2 cm); Pad: 2.5" x 2.5" (6.4 x 6.4 cm)	10/bx, 100/cs
MSC2066EP	6" x 6" (15.2 x 15.2 cm); Pad: 4.5" x 4.5" (11.4 x 11.4 cm)	10/bx, 100/cs
MSC2077EP	Sacrum: 7" x 7" (17.8 x 17.8 cm)	5/bx, 50/cs
MSC20109EP	Sacrum: 10" x 9" (25.4 x 22.9 cm)	5/bx, 50/cs

Optifoam Gentle Non-bordered

Non-bordered silicone-faced foam dressings.

Item No.	Description	Pkg.
MSC2244EP	4" x 4" (10.2 x 10.2 cm)	10/bx, 100/cs
MSC2266EP	6" x 6" (15.2 x 15.2 cm)	10/bx, 100/cs
MSC2288EP	8" x 8" (20.3 x 20.3 cm)	5/bx, 50/cs

Optifoam Gentle Lite

Lite silicone-faced and bordered foam dressings.

Item No.	Description	Pkg.
MSC28162B	1.6" x 2" (4 x 5 cm)	10/bx, 100/cs
MSC2833B	3" x 3" (7.6 x 7.6 cm)	10/bx, 100/cs
MSC2844B	4" x 4" (10.2 x 10.2 cm)	10/bx, 100/cs
MSC2866B	6" x 6" (15.2 x 15.2 cm)	10/bx, 100/cs

Optifoam Gentle Lite


Lite non-bordered silicone-faced foam dressing.

Item No.	Description	Pkg.
MSC2944NB	4" x 4" (10.2 x 10.2 cm)	10/bx, 100/cs
MSC2966NB	6" x 6" (15.2 x 15.2 cm)	10/bx, 100/cs

Optifoam® Foam Dressings


Versatile protection.

Coefficient of Friction of Film Backing⁴²

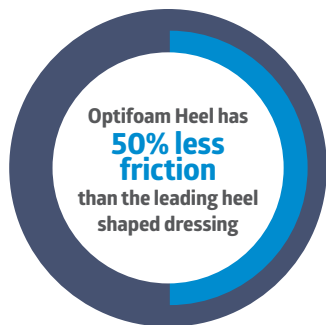

■ Optifoam Heel
■ Alleevyn® Heel

Clinical highlights

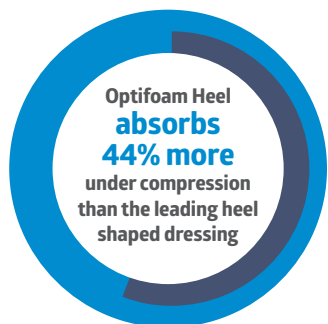

About Optifoam

- Moisture vapor transmission rate (MVTR) adjusts to fluid level
- Absorbent
- Helps create an ideal healing environment
- Waterproof outer layer protects wound and keeps dirt and germs out (except Optifoam Basic)

Recommended Uses

- All wound depths*
- All wound drainage levels
- As a primary or secondary dressing

*May be used as a secondary dressing for deep wounds

Recommended Secondary Dressings**

- Medfix™ Tape
- Gentac™ EZ Tape
- Medigrip™ Tubular Bandage

**Applies only to Optifoam Non-Adhesive, Optifoam Basic and Optifoam Heel

Indications

- Pressure injuries
- Partial- and full-thickness wounds
- Leg ulcers
- Donor sites
- Lacerations and abrasions
- Skin tears
- First- and second-degree burns

Change Frequency

- May be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Contraindications

- Third-degree burns
- Lesions with active vasculitis



Ordering information

Optifoam Adhesive

Adhesive bordered foam dressings.

Item No.	Description	Pkg.
MSC1044EP	4" x 4" (10.2 x 10.2 cm), Pad: 2.5" x 2.5" (6.4 x 6.4 cm)	10/bx, 100/cs
MSC1066EP	6 x 6" (15.2 x 15.2 cm), Pad: 4.5 x 4.5" (11.4 x 11.4 cm)	10/bx, 100/cs
MSC1065EP	Sacrum: 6" x 5.5" (15.5 x 14.2 cm)	10/bx, 100/cs

Optifoam Non-Adhesive

Non-bordered foam dressings.

Item No.	Description	Pkg.
MSC1244EP	4" x 4" (10.2 x 10.2 cm)	10/bx, 100/cs
MSC1266EP	6" x 6" (15.2 x 15.2 cm)	10/bx, 100/cs

Optifoam Heel

Heel-shaped, non-adhesive foam dressings.

Item No.	Description	Pkg.
MSC1200EP	Heel Shaped	5/bx, 40/cs

Optifoam Basic

Foam dressings without polyurethane layer.

Item No.	Description	Pkg.
MSC1133	3" x 3" (7.6 x 7.6 cm)	10/bx, 100/cs
MSC1133F	3" x 3" (7.6 x 7.6 cm) with Fenestration	10/bx, 100/cs
MSC1145	4" x 5" (10.2 x 12.7 cm)	10/bx, 100/cs

Optifoam Site

Circular site dressings with radial slit and starburst opening

Item No.	Description	Pkg.
MSC1104	4" (10.2 cm) Adhesive Dressing, 2" Pad	30/bg, 120/cs

Optifoam Thin

Thin, adhesive-faced dressings.


Item No.	Description	Pkg.
MSC1523EP	2" x 3" (5.1 x 7.6 cm)	10/bx, 100/cs
MSC1544EP	4" x 4" (10.2 x 10.2 cm)	10/bx, 100/cs

OptiView[®] Transparent Dressing with HydroCore Technology[™]

The solution is clear with OptiView.

Transparent HydroCore[™] island takes the guesswork out of skin assessments, leading to less dressing changes and longer wear time.

OptiView[™] is indicated for partial and full thickness wounds with low drainage and can be used as part of pressure injury prevention protocol. The unique design lets you spend less time changing dressings, and more time focusing on patient care.

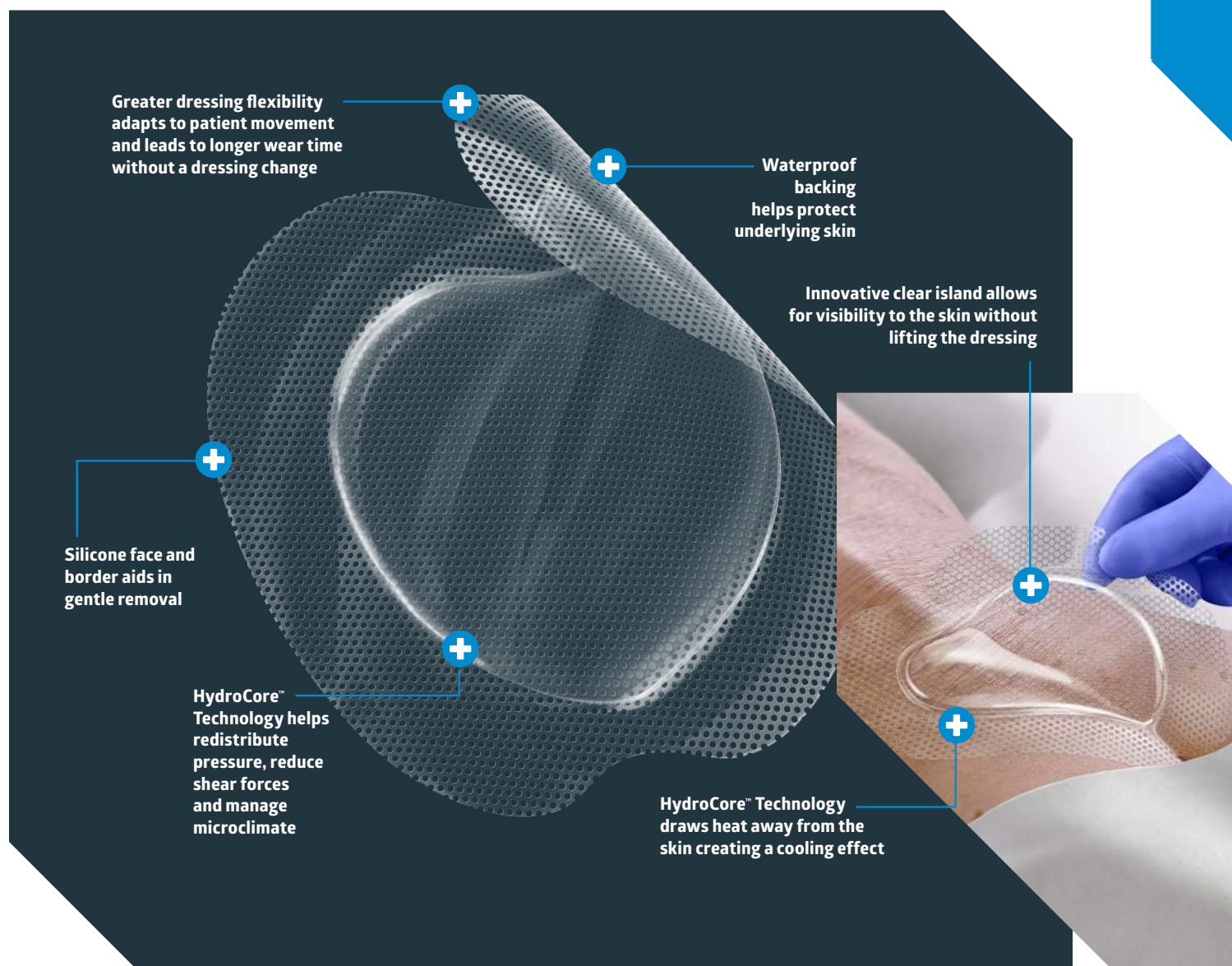

OptiView reduces peak pressure by as much as 62%⁴³

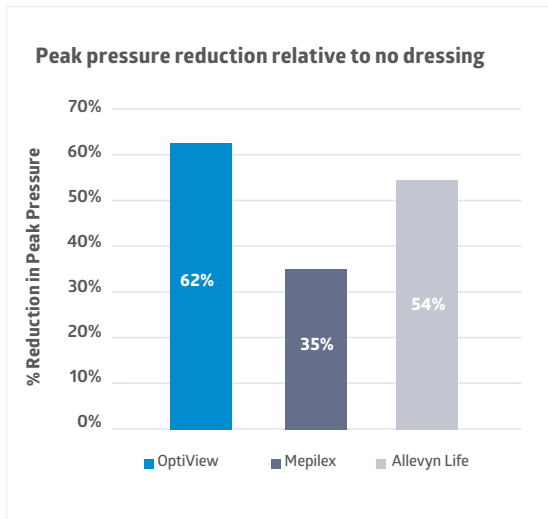


Figure 1. Peak pressure reduction with each dressing compared to no dressing.

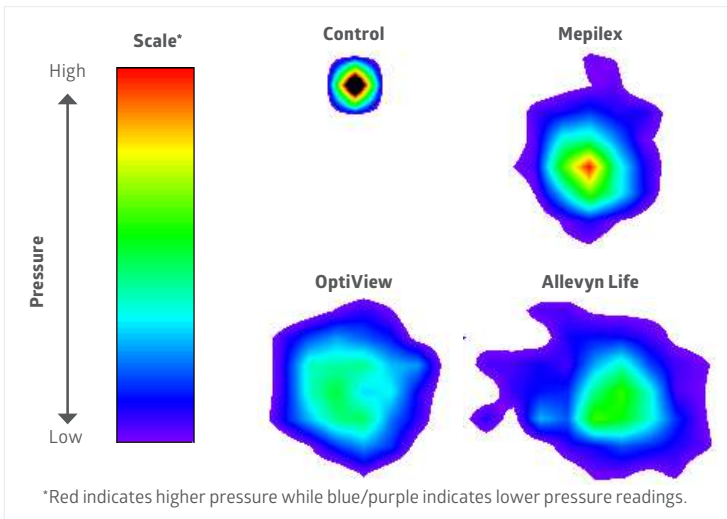


Figure 2. Pressure map for each dressing over the 5 minute period of testing.

About OptiView

- Designed to be used as part of a pressure injury prevention protocol and is ideal for partial and full-thickness wounds with low drainage
- Effectively mitigates the key forces of pressure injury development: shear and microclimate (equivalent to current standard of care)
- Clear HydroCore island allows for visibility to skin without lifting the dressing
- Greater dressing flexibility helps improve conformability and adapts to patient movement
- HydroCore Technology draws heat away from skin, helping to create a cooling effect
- HydroCore center cushions and helps protect underlying skin while silicone face and border aid in gentle removal

Recommended Uses

- Low drainage levels
- Pressure injury prevention
- As a primary dressing

Change Frequency

- OptiView can be left in place for up to 7 days depending on amount of drainage/moisture

Contraindication

- Third-degree burns

OptiView helps keep skin cooler⁴³

Clinical highlights

Moisture Balance

Figure 3

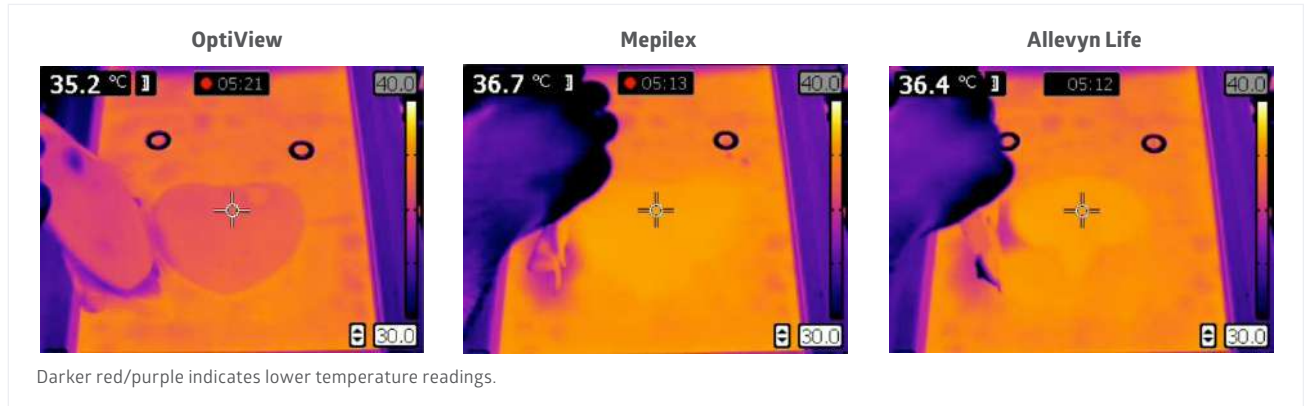


Figure 4

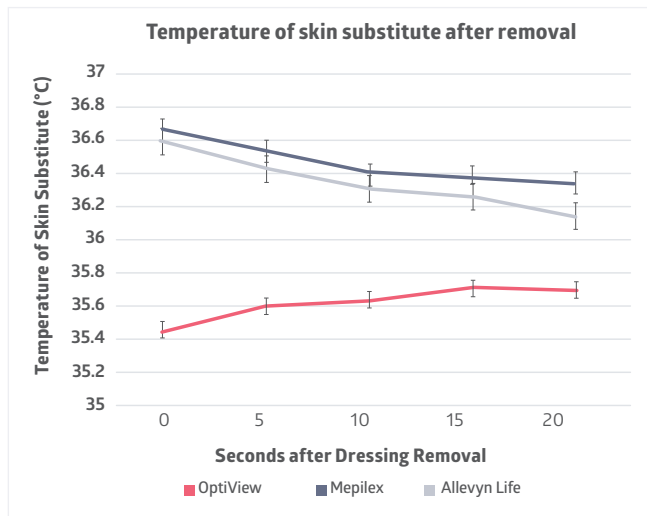


Figure 3 (above). The images show the heat index of the skin substitute after application, with temperature scale between 30.0°C (dark purple) and 40.0°C (bright yellow/white).

Figure 4 (left). The graph shows the temperature readings over time after application. Data are presented as the mean ± standard deviation. OptiView has p<0.05 compared to Mepilex and AllevynLife.

Ordering information

Item No.	Description	Pkg.
MSCOVM3EP	Multisite, 3" x 3" (7.5 x 7.5 cm)	100/cs
MSCOVM6EP	Multisite, 6" x 6" (15.2 x 15.2 cm)	50/cs
MSCOV44EP	OptiView Bordered Square, 4" x 4" (10 x 10 cm)	100/cs
MSCOV77EP	Sacrum, 7" x 7" (17.8 x 17.8 cm)	40/cs
MSCOV99EP	Sacrum, 9" x 9" (22.8 x 22.8 cm)	25/cs
MSCOV1200B	Heel, 9" x 9" (22.8 x 22.8 cm)	25/cs



Opticell® Chitosan-Based Gelling Fiber Dressings

Highly conformable.



Moisture Balance

About Opticell

- Chytoform™ gelling fiber technology
- Controls minor bleeding
- Highly conformable
- No lateral wicking—limits maceration
- Highly absorbent⁴⁵
- Maintains a moist wound-healing environment
- Surface area memory (SAM)—retains its size
- Gentle removal from the wound
- Designed to minimize pain upon removal

Indications

- Partial- and full-thickness wounds
- Venous stasis ulcers
- Pressure injuries
- First- and second-degree burns
- Diabetic foot ulcers
- Surgical wounds
- Trauma wounds
- Donor sites
- Arterial ulcers and leg ulcers of mixed etiology
- Oncology wounds

Recommended Uses

- All wound depths
- All drainage levels
- As a primary dressing

Recommended Secondary Dressings

- Optifoam® Gentle
- Bordered Gauze
- Stratasorb® Composite
- Gentac Silicone Island Dressing

Change Frequency


- Opticell® may be left in place for up to 7 days
- Opticell may remain in place up to 14 days in the context of burn treatment
- Dressing change frequency will depend on the amount of drainag

Contraindications


- Individuals with a sensitivity to chitosan which is derived from shellfish
- Third-degree burns

Surface Area Memory (SAM) Results⁴⁴

Retains its shape and size to maintain complete wound coverage.

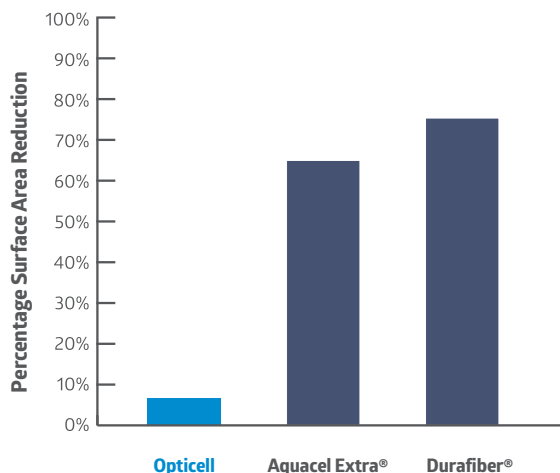

Wet Tensile Strength⁴⁵

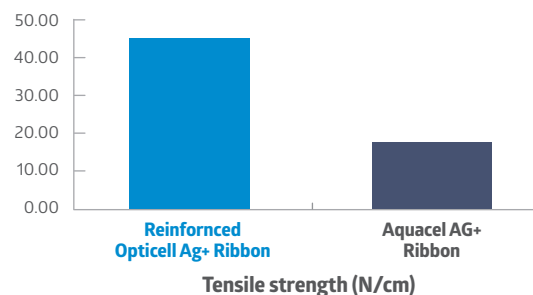


Ordering information

Item No.	Description	Pkg.
MSC7822EP	Gelling Fiber Sheet 2" x 2" (5.1 x 5.1 cm)	10/bx, 100/cs
MSC7844EP	Gelling Fiber Sheet 4.25" x 4.25" (10.8 x 10.8 cm)	10/bx, 50/cs
MSC7866EP	Gelling Fiber Sheet 6" x 6" (15.2 x 15.2 cm)	5/bx, 50/cs
MSC7818R	Gelling Fiber Ribbon 0.75" x 18" (1.9 x 45.7 cm)	5/bx, 50/cs

Maxorb® Calcium Alginate Dressings

Superior absorption.

Maxorb II Absorbency Comparison⁴⁶

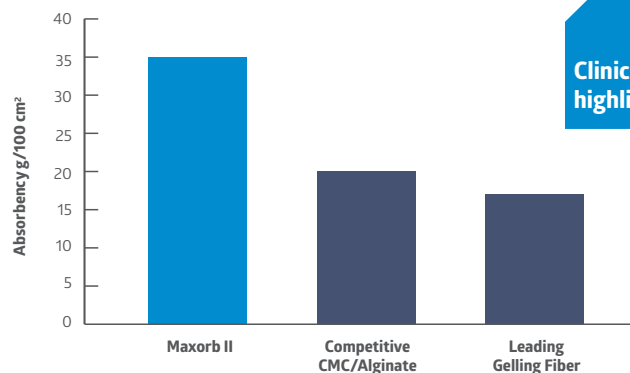

Clinical highlight



Moisture Balance

About Maxorb

- Maxorb II is a 100% calcium alginate dressing
- Maxorb Extra is a blend of CMC and calcium alginate fibers
- Maxorb ES is reinforced for easy removal from tunneling wounds
- Superior fluid handling⁴⁶
- High wet strength—removes in one piece
- Fluid does not wick laterally
- Improved gelling capability

Indications

- Pressure injuries
- Partial- and full-thickness wounds
- Leg ulcers
- Diabetic ulcers
- Surgical wounds
- Donor sites
- Lacerations and abrasions
- First- and second-degree burns

Change Frequency

- May be left in place for up to 7 days
- Dressing change frequency will depend on the amount of drainage

Contraindications

- Individuals with a known sensitivity to alginates
- Third-degree burns
- To control heavy bleeding
- As a surgical sponge
- Dry or lightly draining wounds

Recommended Uses

- All wound depths
- Moderate/heavy drainage
- As a primary dressing

Recommended Secondary Dressings

- Optifoam® Gentle
- Bordered gauze
- Stratasorb® Composite
- Gentac Silicone Island Dressing

Ordering information

Maxorb II

Item No.	Description	Pkg.
MSC7322EP	Sheet 2" x 2" (5.1 x 5.1 cm)	10/bx, 100/cs
MSC7344EP	Sheet 4" x 4" (10 x 10 cm)	10/bx, 50/cs
MSC7348EP	Sheet 4" x 8" (10.2 x 20.3 cm)	5/bx, 50/cs
MSC7366EP	Sheet 6" x 6" (15.2 x 15.2 cm)	5/bx, 50/cs
MSC7312EP	Reinforced Ribbon 1" x 12" (2.5 x 30.5 cm)	10/bx, 50/cs
MSC7318EP	Reinforced Ribbon 1" x 18" (2.5 x 45.7 cm)	10/bx, 50/cs

Maxorb Extra

Item No.	Description	Pkg.
MSC7022EP	Sheet 2" x 2" (5.1 x 5.1 cm)	10/bx, 100/cs
MSC7044EP	Sheet 4" x 4" (10.2 x 10.2 cm)	10/bx, 50/cs
MSC7048EP	Sheet 4" x 8" (10.2 x 20.3 cm)	5/bx, 50/cs
MSC7012EP	CMC/Alginate Ribbon 1" x 12" (2.5 x 30.5 cm), Rope	5/bx, 20/cs
MSC7112EP	CMC/Alginate Ribbon 1" x 12" (2.5 x 30.5 cm), Flat Rope	5/bx, 20/cs

Maxorb ES

Item No.	Description	Pkg.
MSC7918EP	Reinforced Ribbon 0.75" x 18" (1.9 x 45.7 cm)	5/bx, 50/cs

OptiLock® Superabsorbent Dressings

For heavily exuding wounds.

Clinical highlights

Moisture Balance



About OptiLock

- Superabsorbent polymer core
- Locks in drainage under compression
- Adjusts absorption to the amount of drainage
- Protects skin from maceration
- Non-adherent wound contact layer

Recommended Uses

- All wound depths*
- Moderate to heavy drainage
- As a primary dressing

*May be used as a secondary dressing for deep wounds

Recommended Secondary Dressings

- Medfix™ Tape
- Elastic net
- Gentac® EZ Tape
- TwoFlex
- Accuwrap

Indications

- Pressure injuries
- Partial- and full-thickness wounds
- Leg ulcers
- Lacerations and abrasions
- Wounds under compression

Change Frequency

- OptiLock may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Contraindications


- Third-degree burns
- Individuals with a known sensitivity to the product itself or its components

OptiLock Absorbs and Retains More Fluid⁴⁷

OptiLock's remarkable absorption and fluid retention capabilities raise the bar for gentle wound care.

Super-Absorbent Free Swell Absorption Study⁴⁸

Compared to all traditional super absorbent dressings, OptiLock absorbed the most and significantly more than ConvaMax and KerraMax.

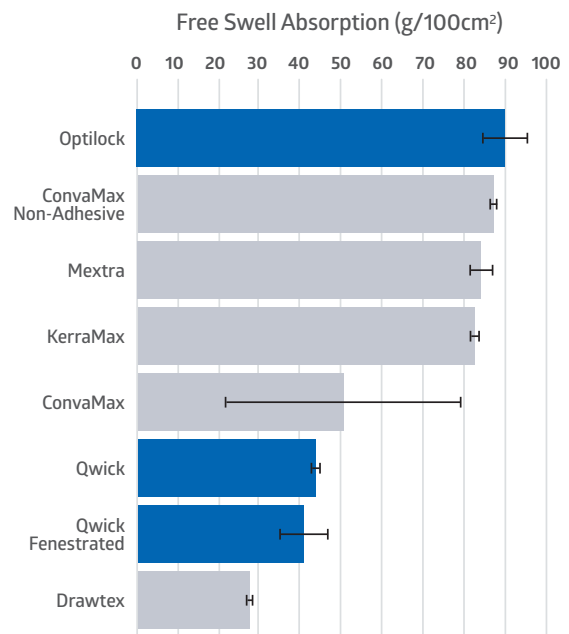

Ordering information

Item No.	Description	Pkg.
MSC6433EP	3" x 3" (7.5 x 7.5 cm)	10/bx, 100/cs
MSC6444EP	4" x 4" (10.2 x 10.2 cm)	10/bx, 100/cs
MSC6455EP	5" x 5.5" (12.7 x 14 cm)	10/bx, 100/cs
MSC64610EP	6.5" x 10" (16.5 x 25.4 cm)	10/bx, 50/cs
MSC64812EP	8" x 12" (20.3 x 30.5 cm)	10/bx, 80/cs


OptiLock is ideal for highly draining venous leg ulcers because of its gentle contact layer and fluid-locking feature. Even under high compression bandages, the fluid is retained in the dressing. **Use OptiLock in conjunction with AccuWrap compression bandage system.** To learn more, see pg. 50.

Qwick™ Superabsorbent Aquaconductive™ Dressings

Quick wicking for draining wounds.

Aquaconductive Technology


Aquaconductive Technology is the mechanism by which the three layers of Qwick pull exudate away from or out of the wound to help create an optimal wound healing environment.

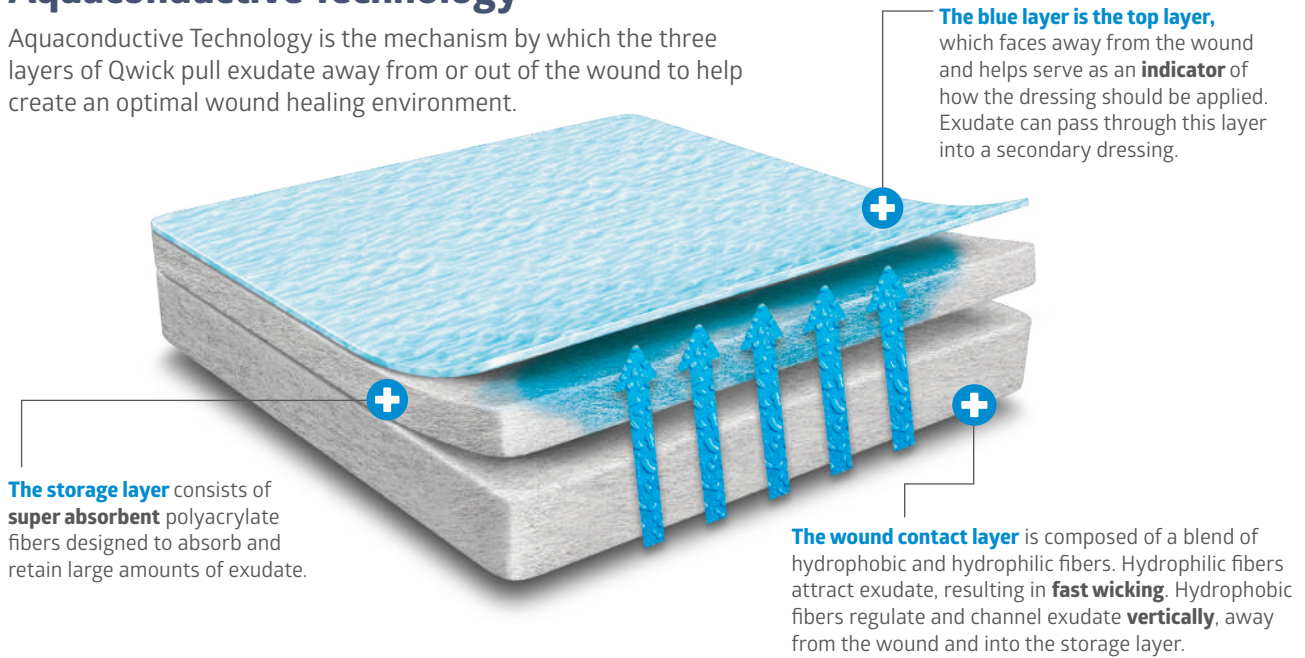

Qwick outperforms competitive product⁴⁹




Average Absorbency (g/100 cm²)

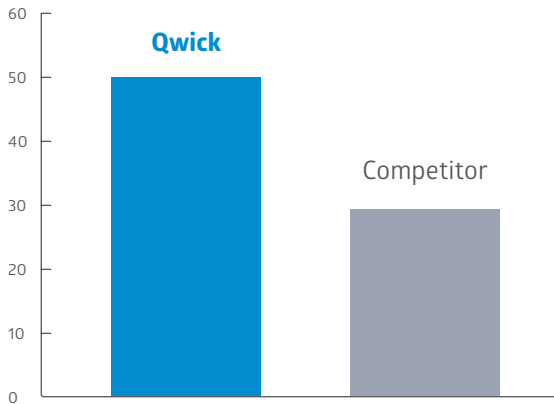

Qwick dressing is nearly twice as absorbent as the competitor's dressing.

Average Fluid Retention Under Compression (g/100 cm²)

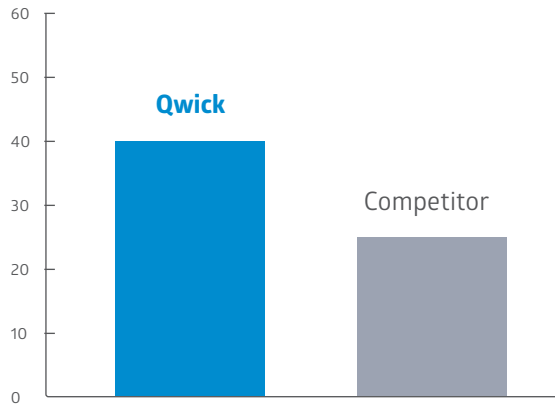

Qwick can retain nearly 60% more fluid than the competitor's dressing under pressure.

Qwick™ Superabsorbent Aquaconductive™ Dressings, continued

About Qwick

- Multilayer construction wicks and retains fluid to help protect the skin from maceration
- Can be cut to fit
- Flexible
- Superabsorbent
- Wicking
- Featuring Aquaconductive™ Technology
- Can be used as a transfer layer

Recommended Uses

- All wound depths
- Moderate to heavy drainage
- As a primary dressing

Recommended Secondary Dressings

- Bordered gauze
- Gentac® Silicone Island Dressing
- Optifoam® Gentle
- Elastic net

Indications

- Pressure injuries
- Partial- and full-thickness wounds
- Leg ulcers
- Lacerations and abrasions
- Wounds under compression

Change Frequency

- Qwick may be left in place for up to 7 days
- Dressing change frequency will depend on the amount of drainage

Contraindication

- Third-degree burns

Ordering information

Item No.	Description	Pkg.
MSC5822	2" x 2" (5.1 x 5.1 cm)	10/bx, 50/cs
MSC5844	4.25" x 4" (10.8 x 10.2 cm)	10/bx, 50/cs
MSC5844F	Fenestrated, 4.25" x 4" (10.8 x 10.2 cm)	10/bx, 50/cs
MSC5868	6.125" x 8" (15.6 x 20.3 cm)	10/bx, 50/cs



Exuderm® Hydrocolloid Dressings


Keep wounds in a moist healing environment.

Moisture Balance

Water Absorption⁵⁰

Percent of absorption at 24 hours.

Clinical highlight

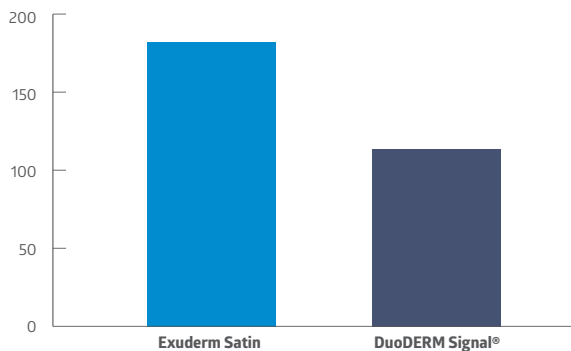

Ordering information

Exuderm Satin Hydrocolloid

Hydrocolloid wound dressings with satin-finish backing.

Item No.	Description	Pkg.
MSC5422	2" x 2" (5.1 x 5.1 cm)	20/bx
MSC5444	4" x 4" (10.2 x 10.2 cm)	10/bx
MSC5466	6" x 6" (15.2 x 15.2 cm)	5/bx
MSC5488	8" x 8" (20.3 x 20.3 cm)	5/bx
MSC5470	Sacrum: 4" x 3.6" (10.2 x 9.1 cm)	10/bx
MSC5475	Sacrum: 6" x 6.5" (16.3 x 16.5 cm)	5/bx

Exuderm LP Low Profile Hydrocolloid

Thin hydrocolloid dressings.

Item No.	Description	Pkg.
MSC5122	2" x 2" (5.1 x 5.1 cm)	20/bx
MSC5133	3" x 3" (7.5 x 7.5 cm)	20/bx
MSC5124	2" x 4" (5.1 x 10.2 cm)	20/bx
MSC5100	4" x 4" (10.2 x 10.2 cm)	10/bx
MSC5146	4" x 6" (10.2 x 15.2 cm)	10/bx
MSC5125	6" x 6" (15.2 x 15.2 cm)	5/bx

About Exuderm

- Manages drainage⁵⁰ to help maintain a moist wound healing environment
- Longer wear time
- Protective, occlusive barrier
- Satin-finish backing is low friction for longer wear time

Recommended Uses

- All wound depths*
- All drainage levels
- As a primary dressing

*May be used as a secondary dressing for deep wounds

Indications

- Pressure injuries
- Partial- and full-thickness wounds
- Leg ulcers
- Donor sites
- Wounds with light to moderate drainage
- Lacerations and abrasions
- First- and second-degree burns

Change Frequency

- Exuderm dressings can be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Contraindication

- Third-degree burns

Skintegrity® Hydrogels and Hydrogel Products

Fill irregularly-shaped wounds.

Cytotoxicity Test for Skintegrity Hydrogel⁵¹

Clinical highlight

Using Murine L929 Embryo Fibroblasts
Agar Overlay Neutral Red Assay

Test Material	IDRCC #278 (Wound Gel)	Negative Control	Positive Control
Concentration	100%	N/A	100%
Exposure (Time)	21 Hours	21 Hours	21 Hours
Zone of Inhibition (mm)	0	0	0
Comments/Observations (Grade and Reactivity)	0 / None	0 / None	Entire disk was clear/cells dead

Skintegrity Hydrogel is not harmful to tissue.



About hydrogels

- Donate moisture
- Rinse easily from the wound
- Skintegrity 1-oz. bellows bottle reduces waste and eases application

Recommended Uses

- All wound depths
- No/minimal drainage
- As a primary dressing

Recommended Secondary Dressings

- Bordered gauze
- Stratascorb® Composite
- Gentac® Silicone Island Dressing

Indications

- Pressure injuries
- Partial- and full-thickness wounds
- Leg ulcers
- Surgical wounds
- Lacerations, abrasions and skin tears
- First- and second-degree burns

Change Frequency

- Skintegrity may be left in place for up to 3 days
- Dressing change frequency will depend on amount of drainage

Contraindications

- Patients with a known sensitivity to components of the gel
- Heavily draining wounds

Ordering information

Skintegrity® Eco Hydrogel

Item No.	Description	Pkg.
MSC6201	1-oz. (29.5 mL), Bellows Bottle	30/cs
MSC6204	4-oz. (118 mL), Tube	12/cs
MSCHS44	4" x 4" (5.1 x 5.1 cm), Wound Dressing Sheet	10bx, 60/cs

DIMES

Edge/Environment

If the wound edge isn't migrating after appropriate wound bed preparation (debridement, infection/inflammation, moisture balance) and healing is stalled, advanced therapies should be considered.

Puracol® Collagen Wound Dressing

Break the non-healing cycle with native collagen.

Venous ulcers, pressure injuries, diabetic ulcers and surgical incisions are all wounds that may be likely to stall. What can you do when this happens?

One approach is adding collagen. As one of the body's primary natural resources and a component within the dermis, collagen plays a key role in each phase of wound healing.



Gentle processing makes the difference

Puracol's exclusive manufacturing process preserves the collagen's natural, triple helix structure. The result? A dressing that more closely resembles our body's native collagen.

This high quality collagen dressing jump starts wound healing and stays in contact with the wound bed for a longer period of time.⁵²

Collagen with a higher level of nativity⁵³

This *in vitro* study shows that Puracol Plus and Puracol Plus Ag+ have the highest quantity of collagen and native collagen per dressing. This is achieved through our unique manufacturing process.

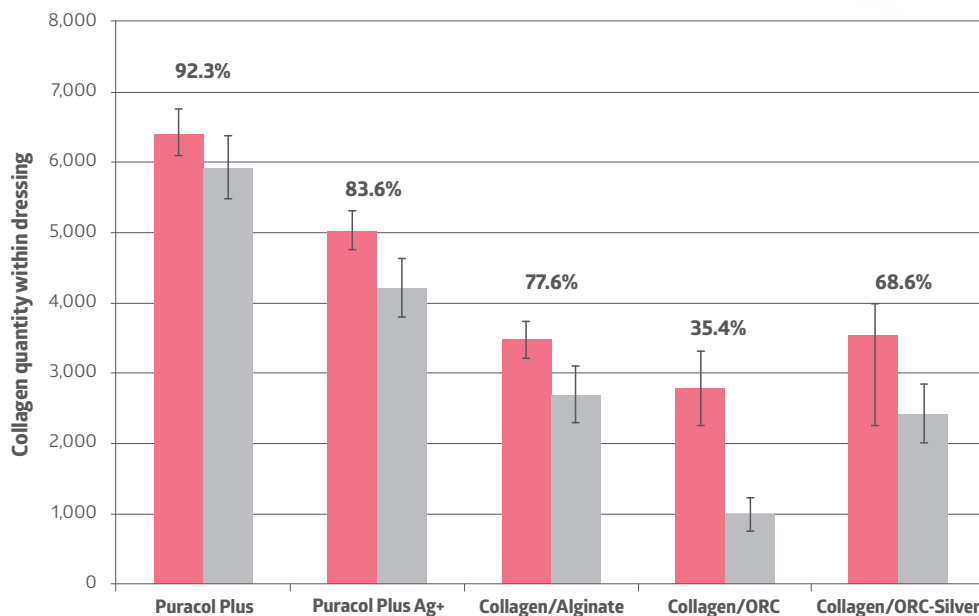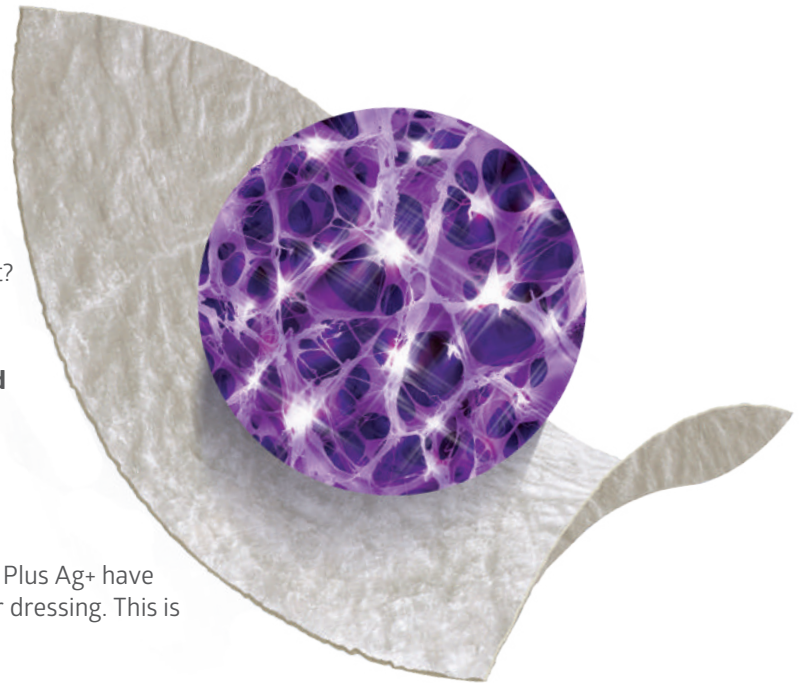

Clinical highlight

■ Collagen Quantity
■ Native Collagen Quantity

Evaluation of a Bovine 100% Native Collagen for the Treatment of Chronic Wounds⁵⁴

After managing the wounds with a bovine-derived, 100% native, Type I collagen, 83% achieved wound closure within 90 days (15 out of 18 patients).⁵⁴



Clinical highlight

About Puracol

Puracol Plus

- 100% collagen with a high degree of nativity^{53,55}
- High gel integrity⁵⁶
- Helps promote a natural wound environment conducive to wound healing
- Biodegradable
- Can be used in combination with negative pressure wound therapy (NPWT)⁵⁷

Recommended Uses

- All wound depths
- All drainage levels
- As a primary dressing

Recommended Secondary Dressings

- Optifoam® Gentle
- Stratasorb® Composite
- Bordered gauze
- Gentac® Silicone Island Dressing
- Optifoam® Adhesive

Indications

- Pressure, venous, diabetic ulcers
- Partial- and full-thickness wounds
- Ulcers caused by mixed vascular etiologies
- Donor sites and other surface wounds
- Abrasions
- Traumatic wounds healing by secondary intention
- Dehisced surgical wounds

Contraindications

- Active vasculitis or patients with known sensitivity to collagen
- Third-degree burns



MSCCA8622

Ordering information

Puracol Plus

Item No.	Description	Pkg.
MSCCA8622	2" x 2.25" (5.1 x 5.7 cm)	10/bx, 50/cs



DIMES

Support Products

Compression systems, skin protectants, cover dressings, surgical glue, tapes, and wound cleansers often play a critical role in the context of chronic wound care.

Compression products support healthy venous blood flow to reduce edema and promote healing.

Marathon® No-Sting Cyanoacrylate Skin Protectant

Innovating skin protection with Cyanoacrylate Technology.

Marathon No-Sting Cyanoacrylate Skin Protectant provides a long-lasting, robust barrier that defends damaged or intact skin from breakdown caused by moisture, friction, shear and adhesive stripping.

This technology bonds to the skin through the polymerization process, becoming an additional layer of skin that will wear off as the epidermis naturally renews.

No-sting



Breathable




Long lasting

Tested and proven⁵⁸

An independent study tested 12 subjects ages 60+ and compared how bare skin, skin with an application of Marathon, and skin with an application of Cavilon® resisted exposure to a corrosive fluid (synthetic urine).

Results:

Areas where Marathon was applied showed better resistance after each of the five urine and wash-off cycles compared to the areas where Cavilon or no product at all were applied.

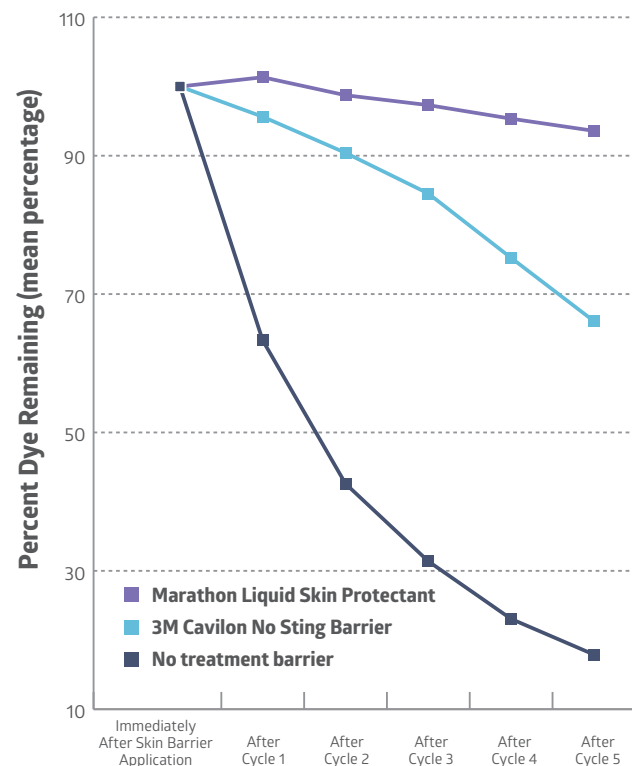

Clinical highlight

Percentage of retained dye after all five urine and wash-off cycles (mean percentage)⁵⁸

Comparison	Percentage
Marathon	94%
Cavilon	66%
Skin with No Treatment	18%

Comparison	P value
Cavilon vs Marathon	<0.05
Cavilon vs No Treatment	>0.05
Marathon vs No Treatment	<0.001

Marathon® No-Sting Cyanoacrylate Skin Protectant, continued

Clinical highlight

Marathon XL⁵⁹

vs.

Solvent-Based Cyanoacrylate Skin Protectant⁵⁹


100% cyanoacrylate	5–10% cyanoacrylate
5x thicker and more robust	Less adherent to skin
Reduces external force transmitted to tissue	More external force is transmitted to tissue
A higher threshold of strain must be applied before tearing	A lower threshold of strain must be applied before tearing
Non-sticky film	Sticky film
Violet tint for easy visualization	Transparent

Cyanoacrylate-based Barrier



A >20 µm layer of cyanoacrylate-based barrier can be clearly seen at x200 magnification. There are NO visible gaps between the skin and Marathon, since it bonded directly to the skin.⁵⁹

Solvent-based barrier

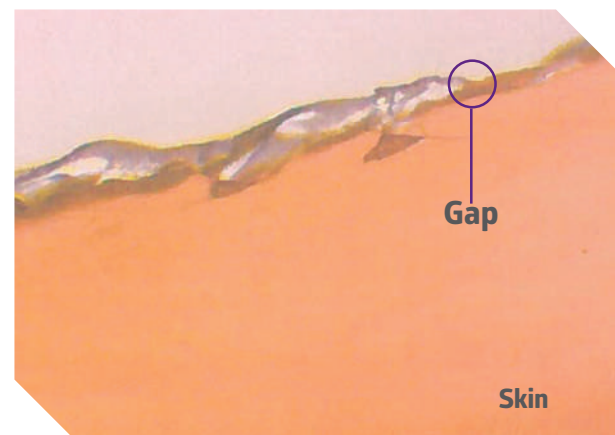

The approximate <5 µm layer of a solvent-based barrier at x200 magnification is seen in this image. The gap between the skin and the solvent-based barrier is visible.⁵⁹

Marathon® No-Sting Cyanoacrylate Skin Protectant, continued

About Marathon

- Robust, flexible and long-lasting
- Non-stinging; contains no solvents or activators
- Protects from the effects of friction and shear
- Protects from moisture- associated skin damage caused by urine, exudate, perspiration, and other body fluids
- Can be used on intact or damaged skin
- Fast drying
- Breathable

Recommended Uses

- Protection from moisture- associated skin damage
- Protection from friction and shear
- Protection from adhesive trauma

Indications

Protects intact or damaged skin from:

- Body fluids
- Moisture
- Friction and shear
- Adhesive stripping

Change Frequency

- Up to 3 days, reapply as needed

Contraindications

Do not apply directly to:

- Deep, open, bleeding, or chronic wounds
- Second- or third-degree burns
- Infected areas

Marathon provides robust, breathable protection which facilitates wound closure. **Use Marathon in conjunction with Opticell Ag+ to treat or prevent maceration of the periwound.** To learn more, see pg. 16.



Ordering information

Item No.	Description	Pkg.
MSC093001	0.5 g ampoule	5/bx
MSC093005	0.5 g ampoule	10/bx
MSC093001XL	1.5 g ampoule	5/bx




Sureprep® Skin Protectants

Protection against friction and incontinence.

Transepidermal Water Loss (TEWL)⁶⁰

On day 4 and day 5, subjects using Sureprep No-Sting experienced significantly less water loss than subjects using 3M Cavilon™ No-Sting.

Clinical highlight

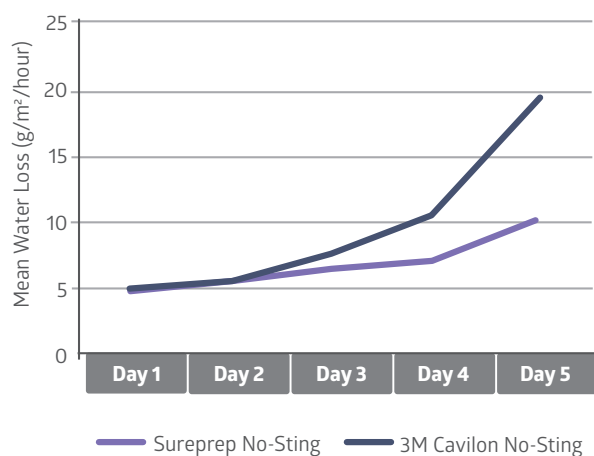

Ordering information

Sureprep No-Sting

Water-based no-sting skin protectant.

Item No.	Description	Pkg.
MSC1505	Protective Wipes, 1 mL	50/bx, 500/cs
MSC1510	Wand Applicator, 1 mL	25/bx, 125/cs
MSC1513	Wand Applicator, 3 mL	25/bx, 100/cs

Sureprep Rapid Dry

Fast-drying no-sting skin protectant.

Item No.	Description	Pkg.
MSC1605	Protective Wipes, 1 mL	25/bx, 100/cs
MSC1610	Wand Applicator, 1 mL	25/bx, 100/cs
MSC1613	Wand Applicator, 3 mL	25/bx, 100/cs
MSC1528	Spray, 28 mL	12/cs

Sureprep Wipes

Skin protectant for intact skin; contains alcohol.

Item No.	Description	Pkg.
MSC1500	Protective Wipes, 1 mL	50/bx, 1,000/cs

About Sureprep

- Protects from adhesive stripping^{61,62}
- Safe for delicate skin⁶³
- Outperformed 3M Cavilon® in controlled study⁶⁰
- Fast drying⁶⁴
- Vapor permeable
- Creates a waterproof barrier on periwound skin
- Protection from friction and body fluids
- Transparent

Recommended Uses

- Protection from moisture associated skin damage
- Protection from corrosive fluids
- Protection from adhesive trauma

Indications

To be applied to intact or damaged skin in order to provide a primary barrier against:

- Bodily fluids
- Adhesive stripping

Change Frequency

- Up to 72 hours or with every dressing change

Contraindications

- On infected areas of skin
- Near the eyes
- As the only covering in situations that require additional dressing protection from bacterial contamination/penetration, e.g., intravenous therapy catheter sites and full- or partial-thickness wounds

AccuWrap™ 2-Layer Compression Bandage System

Manage venous disease with accurate compression.

Venous leg ulcers are caused by an obstruction of the venous system or a defective venous valve. The condition begins with leg swelling and skin inflammation, which can lead to a venous ulcer. These wounds heal slowly and 70% return after healing. Compression therapy is a primary strategy to treat recurrence.^{65,66}

Put the comfort in compression

AccuWrap is a two-layer system that delivers therapeutic compression to manage venous disease and associated edema for up to 7 days. AccuWrap is more comfortable and breathable under clothing, making it easier for patients to wear throughout treatment.

AccuWrap promotes patient education

Research shows that when patients understand why they're receiving compression therapy, they are more likely to be compliant.⁶⁶

Layer 1

Short stretch padding layer

100% polyester

Comfortable and breathable

Slight cohesive texture helps it stay in place

Layer 2

Woven short stretch compression layer

Sturdy, breathable woven fabric

Accuracy indicators confirm ideal compression level

Thinner than other leading brands⁶⁹

AccuWrap™ 2-Layer Compression Bandage System, continued

About AccuWrap

- Low-profile system easily fits under clothing and footwear
- Absorbent padding bandage designed to conform to bony prominences
- Stocking included to ease movement
- Extra-long kit available for larger legs
- Easy and consistent application method
- AccuWrap and AccuWrap XL are designed to provide 30-40 mmHg of therapeutic compression⁶⁷
- AccuWrap Lite is designed to provide 20-30 mmHg of therapeutic compression⁶⁷

Recommended Use

- Venous leg ulcers

Recommended Dressings

- Qwick™
- Opticell® Ag+
- OptiLock®
- IoPlex®

Indication

- To deliver therapeutic compression to manage venous disease and associated edema

Change Frequency

- AccuWrap compression system may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Contraindications

- AccuWrap and AccuWrap XL are contraindicated for a patient with severe arterial occlusive disease with an ABI of less than 0.8
- AccuWrap Lite is contraindicated for a patient with severe arterial occlusive disease with an ABI of less than 0.5

Ordering information

Item No.	Description	Pkg.
MSC7800	AccuWrap Kit Padding: 4" x 4.4 yd. (10 cm x 4 m) Cohesive: 4" x 6 yd. (10 cm x 5.5 m) stretched	8 kits/cs
MSC7800XL	AccuWrap XL Kit Padding: 4" x 5.5 yd. (10 cm x 5 m) Cohesive: 4" x 7 yd. (10 cm x 7.5 m) stretched	8 kits/cs
MSC7802	AccuWrap Lite Kit Padding: 4" x 4.4 yd. (10 cm x 4 m) Cohesive: 4" x 6 yd. (10 cm x 5.5 m) stretched	8 kits/cs




TwoFlex™ 2-Layer Compression Bandage System

Conformable under layer promotes greater patient compliance.

Layer 1

Foam padding layer

- Soft and conformable, which helps with comfort and promotes patient compliance
- Foam bandage absorbs moisture
- Flexible to allow ankle motion and to wrap the heel
- Can be torn by hand

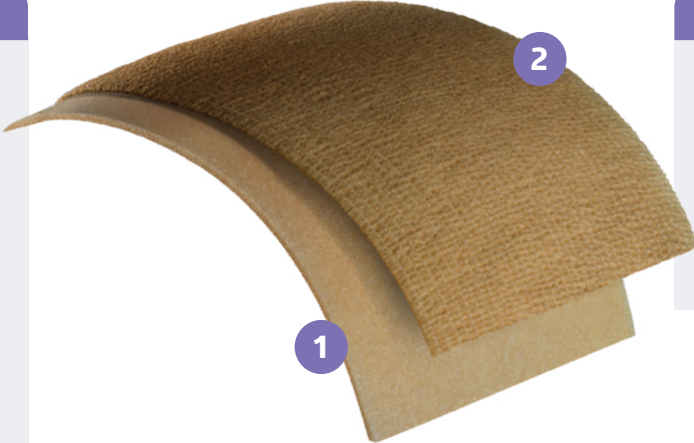

Layer 2

Cohesive compression layer

- Short stretch bandage
- Cohesive texture to help it stay in place
- Can be torn by hand

About TwoFlex

- Low-profile system easily fits under clothing and footwear
- Stocking included to ease movement
- Both layers are easily torn by hand
- Patient information card included in every kit
- Extra-long kit available for larger legs
- Easy and consistent application method

Recommended Use

- Venous leg ulcers

Recommended Dressings

- Qwick™
- Opticell Ag+®
- OptiLock®
- IoPlex®

Indication

- To deliver therapeutic compression to manage venous disease and associated edema

Change Frequency

- TwoFlex compression system may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Contraindications

- TwoFlex and TwoFlex XL are contraindicated for a patient with severe arterial occlusive disease with an ABI of less than 0.8
- TwoFlex Lite is contraindicated for a patient with severe arterial occlusive disease with an ABI of less than 0.5



Ordering information

Item No.	Description	Pkg.
MSC6800	TwoFlex Kit Padding: 4" x 3.4 yd. (10 cm x 3.1 m) unstretched Cohesive: 4" x 6.5 yd. (10 cm x 5.9 m) unstretched	8 kits/cs
MSC6800XL	TwoFlex XL Kit Padding: 4" x 5.5 yd. (10 cm x 5 m) unstretched Cohesive: 4" x 7 yd. (10 cm x 7.5 m) unstretched	8 kits/cs
MSC6802	TwoFlex Lite Kit Padding: 4" x 3.4 yd. (10 cm x 3.1 m) unstretched Cohesive: 4" x 6.5 yd. (10 cm x 5.9 m) unstretched	8 kits/cs

ThreeFlex® and FourFlex® Multilayer Compression Bandage Systems

Effective therapeutic compression.

ABI Chart⁶⁸

To determine the Ankle Brachial Index (ABI), divide the ankle systolic pressure by the brachial systolic pressure.

$$\frac{\text{Ankle Pressure}}{\text{Brachial Pressure}} = \text{ABI}$$

Interpretation of the Ankle Brachial Index

Greater than 1.3	Abnormally high range (more studies are needed)
0.8 to 1.3	Normal range
0.50 to 0.80	Indicates mild to moderate arterial disease, compression should only be used under direct medical supervision
Below 0.50	Severe arterial insufficiency, compression is contraindicated

Clinical highlight



Ordering information

ThreeFlex Three-Layer Compression System

For lighter compression or for mixed etiology.

Item No.	Description	Pkg.
MSC4300	Padding: 4" x 4 yd. (10 cm x 3.6 m)	8 kits/cs
	Light Conforming: 4" x 4.9 yd. (10 cm x 4.5 m) Stretched	
	Cohesive: 4" x 6.9 yd. (10 cm x 6.3 m) Stretched	

FourFlex Four-Layer Compression System

For the management of chronic venous insufficiency.

Item No.	Description	Pkg.
MSC4400	FourFlex Kit Padding: 4" x 4 yd. (10 cm x 3.6 m)	8 kits/cs
	Light Conforming: 4" x 4.9 yd. (10 cm x 4.5 m) Stretched	
	Compression: 4" x 9.5 yd. (10 cm x 8.7 m) Stretched Cohesive: 4" x 6.9 yd. (10 cm x 6.3 m) Stretched	
MSC4400XL	FourFlex XL Kit Padding: 4" x 5 yd. (10 cm x 4.6 m), 25% Longer	8 kits/cs
	Conforming: 4" x 6.3 yd. (10 cm x 5.7 m) Stretched, 29% Longer	
	Compression: 4" x 12.3 yd. (10 cm x 11.2 m) Stretched, 29% Longer Cohesive: 4" x 8.9 yd. (10 cm x 8.1 m) Stretched, 29% Longer	

About ThreeFlex and FourFlex

- Effective therapeutic compression
- Extended wear time
- Absorbs drainage
- Educational packaging
- FourFlex XL is 25% longer for larger legs
- FourFlex and FourFlex XL delivers 30–40 mmHg of compression⁷⁰
- ThreeFlex delivers 20–30 mmHg of compression⁷⁰

Indication

- To deliver therapeutic compression to manage venous disease and associated edema

Change Frequency

- Multilayer compression bandages may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Recommended Use

- Venous leg ulcers

Recommended Dressings

- Qwick™
- Opticell Ag+®
- OptiLock®
- IoPlex®

Contraindication

- Patients with severe arterial disease

Unna-Z™ Stretch Unna Boot Bandages

Elasticity for enhanced conformability.

About Unna-Z

- Provides semi-rigid support for conditions requiring mild compression
- Impregnated with zinc oxide paste
- Improved knitted design
- Inner plastic core for easier application
- Unna-Z Stretch provides greater elasticity

Recommended Use

- Venous leg ulcers

Recommended Dressings

- Qwick™
- Opticell Ag+®
- OptiLock®
- IoPlex®

Recommended Covering Wrap

- Medline cohesive wraps
- Bulkee® Gauze Wrap

Indication

- Venous leg ulcers

Change Frequency

- Unna-Z may be left in place for up to 7 days, depending on drainage

Contraindications

- Patients with a known sensitivity to components (zinc)

Ordering information

Unna-Z Stretch

Elastic zinc impregnated bandage.

Item No.	Description	Pkg.
NONUNNAS130	3" x 10 yd Stretched (7.6 cm x 9.1 m), Elastic	12/cs
NONUNNAS140	4" x 10 yd Stretched (10.2 cm x 9.1 m), Elastic	12/cs



Unna-Z Zinc Oxide Compression Bandages

Item No.	Description	Pkg.
NONUNNA13	3" x 10 yd (7.5 cm x 9.1 m)	12/cs
NONUNNA14	4" x 10 yd (10 cm x 9.1 m)	12/cs



Medigrip™ and Medigrip™ LF Elasticated Tubular Bandages

Provides excellent support.



About Medigrip

- Provides excellent support for joints
- Easy to apply and reapply
- Wide range of applications
- Good for securing dressings
- Can be used as mild compression when doubled
- Also comes in latex-free version

Recommended Uses

- Dressing retention
- Light compression

Recommended Dressings

- Qwick™
- Opticell Ag+®
- OptiLock®
- IoPlex®

Indications

- Edema
- Treatment of chronic venous insufficiency
- Dislocations
- Sprains
- As a retention dressing

Change Frequency

- Medigrip may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Ordering information

Item numbers with "LF" are not made with natural rubber latex (Medigrip LF).

Medigrip™ Tubular Bandage

11 yd. (10 m) long

Standard SKU	Latex Free (LF) SKU	Size	Width	Application	Pkg.
MSC9500	MSC9500LF	A	1.75" (4.5 cm)	Small Extremities	1 ea
MSC9501	MSC9501LF	B	2.5" (6.3 cm)	Hands and Small Arms	1 ea
MSC9502	MSC9502LF	C	2.625" (6.8 cm)	Medium Arms, Small Ankles	1 ea
MSC9503	MSC9503LF*	D	3" (7.5 cm)	Large Arms, Medium Ankles, Small Knees	1 ea
MSC9504*	MSC9504LF*	E	3.5" (8.75 cm)	Large Ankles, Medium Knees, Small Thighs	1 ea
MSC9505*	MSC9505LF*	F	4" (10 cm)	Large Knees, Medium Thighs	1 ea
MSC9506*	MSC9506LF*	G	4.75" (12 cm)	Large Thighs	1 ea
MSC9507	MSC9507LF	J	6.75" (17.1 cm)	Small Trunks	1 ea
MSC9508	MSC9508LF	K	8.25" (20.1 cm)	Medium/Large Trunks	1 ea

*Available 1.1 yd. long at 30/cs. Add "YD" at end of SKU.

Medigrip sizing chart

Limb Measurement*	Compression Level for Double Layer		
	Low (5-10 mmHg)	Medium (10-20 mmHg)	High (20-30 mmHg)
5"-6" (13-15 cm)	B	-	-
6"-7" (15-18 cm)	B	-	-
7"-8" (18-20 cm)	C	B	-
8"-9" (20-23 cm)	D	B	-
9"-10" (23-25 cm)	E	C	-
10"-12" (25-30 cm)	E	D	-
12"-15" (30-38 cm)	F	E	B
15"-18" (38-46 cm)	G	E	C
18"-23" (46-58 cm)	G	E	D
23"-28" (58-71 cm)	J	F	E
28"-38" (71-97 cm)	K	G	-

Medigrip LF sizing chart

Limb Measurement*	Compression Level for Double Layer		
	Low (5-10 mmHg)	Medium (10-20 mmHg)	High (20-30 mmHg)
5"-5.5" (13-14 cm)	A	-	-
5.5"-6" (14-15 cm)	B	A	-
6"-7" (15-18 cm)	C	B	A
7"-8" (18-20 cm)	D	B	A
8"-9" (20-23 cm)	E	C	B
9"-10" (23-25 cm)	F	D	B
10"-12" (25-30 cm)	F	E	C
12"-15" (30-38 cm)	G	F	D
15"-18" (38-46 cm)	G	F	E
18"-23" (46-58 cm)	J	G	F
23"-28" (58-71 cm)	J	G	-
28"-38" (71-97 cm)	K	J	-

*For full arm coverage, measure the largest part of the forearm. For full leg or below the knee coverage, measure the widest point of the calf. For the hand, measure around metacarpophalangeal joint.


Versatel® and Versatel® One Contact Layer Dressings

Reduce trauma from secondary dressing wound adherence.

Clinical highlight



Fluid Transfer Study⁶⁹

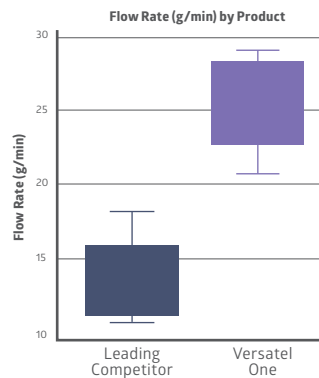

A fluid transfer lab study found that Versatel One has a significantly higher average flow rate than that of the leading competitor.

About Versatel

- Silicone-based atraumatic adhesive
- Flexible and pliable to conform to body contours and improve comfort
- Versatel One features one-sided silicone contact for easy handling
- Reduces potential trauma from secondary dressing wound adherence
- Channels allow fluid to easily transfer to an absorbent dressing
- Translucent for easy wound visualization

Indications

- Dry to heavily draining abrasions
- Partial- and full-thickness wounds
- Venous ulcers or pressure injuries
- Skin tears
- First- and second-degree burns
- Blisters, cuts and lacerations
- Surgical and trauma wounds

Change Frequency

- Versatel may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

Contraindications


- Third-degree burns
- Individuals with a known sensitivity to silicone
- Not for surgical implantation

Recommended Use

- To prevent secondary dressing adhesion to the wound

Recommended Secondary Dressings

- OptiLock®
- Qwick™
- Maxorb® II

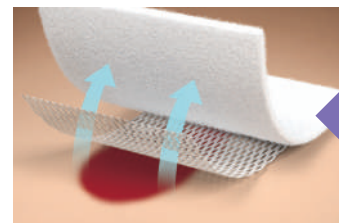

Versatel is designed to allow wound fluid to easily transfer into a secondary dressing.⁶⁹

Ordering information

Versatel One

One-sided silicone contact layer dressing for easier handling.

Item No.	Description	Pkg.
MSC1823EP	2" x 3" (5.1 x 7.6 cm)	10/bx, 50/cs
MSC1834EP	3" x 4" (7.6 x 10.2 cm)	10/bx, 50/cs
MSC1845EP	4" x 5" (10.2 x 12.7 cm)	10/bx, 50/cs
MSC1847EP	4" x 7" (10.2 x 17.8 cm)	10/bx, 50/cs
MSC18812EP	8" x 12" (20.3 x 30.5 cm)	5/bx, 25/cs

Versatel

Two-sided silicone contact layer dressing.

Item No.	Description	Pkg.
MSC1723EP	2" x 3" (5.1 x 7.6 cm)	10/bx, 50/cs
MSC1734EP	3" x 4" (7.6 x 10.2 cm)	10/bx, 50/cs
MSC1747EP	4" x 7" (10.2 x 17.8 cm)	10/bx, 50/cs
MSC17812EP	8" x 12" (20.3 x 30.5 cm)	5/bx, 25/cs

Gentac® Silicone Island Dressing

Easy on and off.



About Gentac

- Silicone adhesive for gentle removal
- Water-resistant backing
- Low profile design increases flexibility and comfort
- Can be lifted and reapplied

Recommended Uses

- All wound depths
- Minimal drainage levels
- Primary dressing
- Secondary dressing

Indications

- Partial- and full-thickness wounds
- Surgical and drain sites

Change Frequency

- Can be left in place for up to 7 days
- Change the dressing as indicated by the amount of drainage or as frequently as the primary dressing indicates

Contraindications

- Third-degree burns
- Patients with a known sensitivity to components of the dressing

Ordering information

Item No.	Description	Pkg.
MSC3322	2" x 2" (5.1 x 5.1 cm); Pad: 1" x 1" (2.5 x 2.5 cm)	15/bx, 150/cs
MSC3344	3" x 4" (10.2 x 10.2 cm); Pad: 2.5" x 2.5" (6.4 x 6.4 cm)	15/bx, 150/cs
MSC3366	6" x 6" (15.2 x 15.2 cm); Pad: 4.5" x 4.5" (11.4 x 11.4 cm)	15/bx, 150/cs
MSC3348	4" x 8" (10.2 x 20.3 cm); Pad: 2" x 6" (5.1 x 15.2 cm)	15/bx, 150/cs
MSC33410	4" x 10" (10.2 x 25.4 cm); Pad: 2" x 8" (5.1 x 20.3 cm)	15/bx, 150/cs
MSC33414	4" x 14" (10.2 x 35.6 cm); Pad: 2" x 12" (5.1 x 30.5 cm)	15/bx, 150/cs

Cover Dressings

Easy-to-use and convenient secondary dressings.

About Cover Dressings

- Non-adherent deluxe soaker pad
- Non-woven adhesive border
- Waterproof backing (Stratasorb)
- Ideal for incision sites

Change Frequency

- Can be left in place for up to 7 days
- Change the dressing as indicated by the amount of drainage or as frequently as the primary dressing indicates

Recommended Uses

- All wound depths
- All drainage levels
- Primary dressing
- Secondary dressing

Contraindications

- Third-degree burns
- Patients with a known sensitivity to components of the dressing

Indications

- Pressure injuries
- Partial- and full-thickness wounds
- Incision sites

Ordering information

Stratasorb® Composite

Waterproof, convenient secondary dressing.

Item No.	Description	Pkg.
MSC3044	4" x 4" (10.2 x 10.2 cm); Pad: 2.5" x 2" (6.4 x 5.1 cm)	10/bx, 100/cs
MSC3066	6" x 6" (15.2 x 15.2 cm); Pad: 4" x 4" (10.2 x 10.2 cm)	10/bx, 100/cs
MSC3068	6" x 7.5" (15.2 x 19.1 cm); Pad: 4" x 6" (10.2 x 15.2 cm)	10/bx, 100/cs
MSC30410	4" x 10" (10.2 x 25.4 cm); Pad: 2" x 8" (5.1 x 20.3 cm)	10/bx, 100/cs
MSC30414	4" x 14" (10.2 x 35.6 cm); Pad: 2" x 12" (5.1 x 30.5 cm)	10/bx, 100/cs

Bordered Gauze

Water-resistant, easy-to-use secondary dressing.

Item No.	Description	Pkg.
MSC3222	2" x 2" (5.1 x 5.1 cm), Pad: 1" x 1" (2.5 x 2.5 cm)	15/bx, 150/cs
MSC3244	4" x 4" (10.2 x 10.2 cm), Pad: 2.5" x 2.5" (6.4 x 6.4 cm)	15/bx, 150/cs
MSC3245	4" x 5" (10.2 x 12.7 cm), Pad: 2" x 2.5" (5.1 x 6.4 cm)	15/bx, 150/cs
MSC3248	4" x 8" (10.2 x 20.3 cm), Pad: 2" x 6" (5.1 x 15.2 cm)	15/bx, 150/cs
MSC3266	6" x 6" (15.2 x 15.2 cm), Pad: 4" x 4" (10.2 x 10.2 cm)	15/bx, 150/cs
MSC32410	4" x 10" (10.2 x 25.4 cm), Pad: 2" x 8" (5.1 x 20.3 cm)	15/bx, 150/cs
MSC32414	4" x 14" (10.2 x 35.6 cm), Pad: 2" x 12" (5.1 x 30.5 cm)	15/bx, 150/cs
MSC3236	3" x 6" (7.6 x 15.2 cm), Pad: 1" x 4" (2.5 x 10.2 cm)	15/bx, 150/cs



Dressing Retention Tapes

Non-woven, silicone and zinc oxide options.

About dressing retention tapes

MedFix

- MedFix tapes are water resistant
- MedFix has a printed s-curve release liner

StickRight

- StickRight is linerless and perforated
- StickRight is hypoallergenic and has low-sensitivity that is gentle on skin

Gentac EZ

- Gentac EZ uses gentle and repositionable silicone adhesive
- Gentac EZ is easily torn to size

Pinc

- Pinc utilizes a zinc oxide adhesive
- Pinc tape is waterproof and adheres well even in moist conditions

Recommended Use

- Dressing and device retention

Indications

- To secure primary dressings
- To secure gastrostomy tubes and other feeding tubes

Change Frequency

- Dressing change frequency will depend on the primary dressing and amount of drainage

Contraindication

- Contraindicated as a primary dressing

Ordering information

MedFix Tapes

Non-woven tapes with S-curve liner.

Item No.	Description	Pkg.
MSC4002	2" x 11 yd. (5.1 cm x 10 m)	1 ea
MSC4004	4" x 11 yd. (10.2 cm x 10 m)	1 ea
MSC4006	6" x 11 yd. (15.2 cm x 10 m)	1 ea

StickRight™ Tapes

Linerless non-woven tapes with perforations for easy tearing.

Item No.	Description	Pkg.
MSC4201EP	1" x 10 yds. (2.5 cm x 9.1 m)	12/bx
MSC4202EP	2" x 10 yds. (5.1 cm x 9.1 m)	12/bx
MSC4203EP	3" x 10 yds. (7.6 cm x 9.1 m)	12/bx
MSC4204EP	4" x 10 yds. (10.2 cm x 9.1 m)	12/bx
MSC4206EP	6" x 10 yds. (15.2 cm x 9.1 m)	12/bx
MSC4222EP	2" x 2 yds. (5.1 cm x 1.8 m)	12/bx
MSC4224EP	2" x 4 yds. (5.1 cm x 3.7 m)	12/bx
MSC4226EP	2" x 6 yds. (5.1 cm x 5.5 m)	12/bx

Gentac EZ Silicone Tape

Silicone tapes.

Item No.	Description	Pkg.
MSC1411	1" x 1.5 yd. (2.54 cm x 1.37 m)	500/cs
MSC1421	2" x 1.5 yd. (5.1 cm x 1.37 m)	250/cs
MSC1415	1" x 5.5 yd. (2.54 cm x 5 m)	120/cs
MSC1425	2" x 5.5 yd. (5.1 cm x 5 m)	60/cs

Pinc™ Tapes

Zinc oxide adhesive tapes.

Item No.	Description	Pkg.
OMAM55CS	0.5" x 5 yd. (1.3 cm x 4.57 m)	36/cs
OMAM111CS	1" x 5 yd. (2.54 cm x 4.57 m)	36/cs
OMAM222CS	2" x 5 yd. (5.1 cm x 4.57 m)	36/cs



MedFix



StickRight



Gentac EZ



Pinc

Wound Cleansers

A variety of cleansers for all types of wounds.

About wound cleansers

- Adjustable trigger, PSI of 8.6 at 3" - PSI is within Wounds International⁷¹ guidelines
- PureWash contains hypochlorous acid (HOCl), which is a key preservative that inhibits microbial contamination within the solution
- PureWash has a neutral pH, non-cytotoxic, non-irritating, and non-sensitizing

Recommended Use

- Cleansing all types of wounds

Indications

To clean a wide variety of wounds including:

- Pressure injuries
- Partial- and full-thickness wounds
- Infected and non-infected wounds

Change Frequency

- With every dressing change

Contraindication

- Patients with a known sensitivity to ingredients in the wound cleanser



Ordering information

Skintegrity®

Surfactant-based wound cleanser with spray and stream nozzle.

Item No.	Description	Pkg.
MSC6008EP	Spray Bottle, 8 oz. (236 mL)	6/cs
MSC6016EP	Spray Bottle, 16 oz. (473 mL)	6/cs
MSC6001	Squeeze Bottle, 1 oz. (30 mL)	30/cs



PureWash™

Hypochlorous Acid Wound Cleanser.

Item No.	Description	Pkg.
MSC9004	Pour Bottle with Flip Top Cap, 4 oz. (118 mL)	12/cs
MSC9008	Bottle with Trigger Sprayer, 8 oz. (236 mL)	12/cs

Cell-friendly solution: PureWash is gentler on fibroblast cells²

Fibroblasts are essential for wound healing and tissue regeneration. They migrate to the injury site, help close wounds by contracting edges and produce collagen and extracellular matrix to rebuild tissue. Fibroblasts also promote the formation of new blood vessels and regulate immune responses. Preserving fibroblast viability ensures proper vascularization, immune balance and structural repair.

In vitro cytotoxicity assays were conducted with PureWash and leading competitor, Brand V, following modified fibroblast MEM cytotoxicity protocols in accordance with ISO 10993-5 and USP <87> standards. Confluent L-929 monolayers were exposed to each test solution for 15 seconds, followed by MEM replacement and 1-hour incubation. The exposure cycle was repeated 4 times. Cellular morphology was graded using light microscopy after each incubation.



Light microscopy images (200X) of L-929 fibroblast monolayers following repeated exposure to PureWash and Brand V test solutions. Cells were exposed to each solution for 15 seconds followed by 1-hour incubation, repeated over four cycles. After the initial exposure, PureWash showed slight reactivity (Grade 1: high cell density, few rounded cells, no extensive detachment) (A), while Brand V showed mild reactivity (Grade 2: increased cell rounding and early detachment) (B). By the fourth exposure, PureWash remained at Grade 1 (C), whereas Brand V progressed to moderate reactivity (Grade 3: mostly rounded cells, cell lysis, sparse distribution, and detachment) (D).

OptiClose™ Topical Skin Adhesives

Innovating wound closure with cyanoacrylate technology.


Designed with doctors to meet today's needs

Now more than ever, efficient and effective options are needed to answer all of today's wound closure needs. We offer a comprehensive selection of **innovative wound closure alternatives designed to increase efficiency, promote healing, and elevate traditional wound care.**

Ideal for use in the office and the OR, these topical applications provide a smarter, faster way to work—while keeping patient comfort and cosmetic outcome in mind.


Shorter wounds and incisions such as:

- Thyroid
- Pacemaker
- Catheter
- Laparoscopic
- Hernia repair
- Excisions
- Lacerations

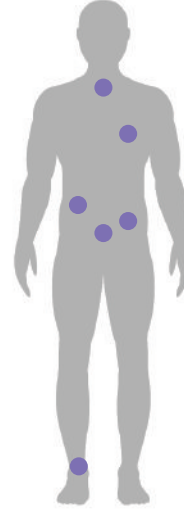


Longer wounds and incisions such as:

- Shoulders
- C-sections
- Hips
- Knees
- Ankles

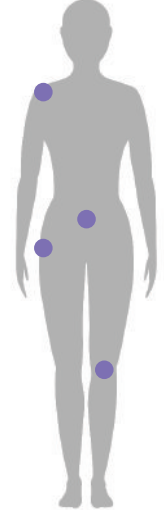

OptiClose Rapid

- **Accelerant-coated tip** speeds up the adhesive drying process and helps reduce patient turnover time
- **Porous foam tip** helps control the application of the adhesive without dripping
- **Winged applicator** protects fingers from the risk of glass shards and makes it easy to activate, express and control with one hand⁵
- **Highlighter shape** allows for accurate application of broad or thin lines

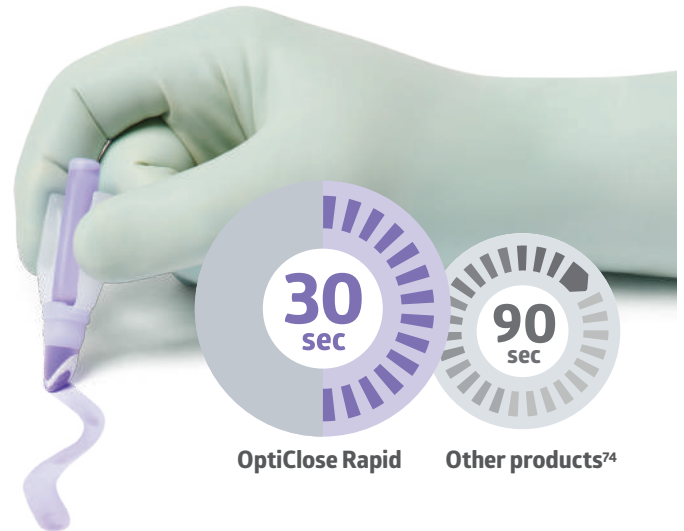


OptiClose Rapid

Other products⁷⁴

Why clinicians choose OptiClose Rapid

Benefits and features	OptiClose Rapid	Leading competitor
Adhesive set time	30 seconds	90 seconds ⁷²
Fill volume	0.8 mL	0.7 mL
Risk of glass puncture	No	Yes
Force needed to express	45 N	63 N
Formula	2-octyl cyanoacrylate	2-octyl cyanoacrylate
Microbial barrier	Yes	Yes
Heat of polymerization	9°C	15°C

Greater wound closure strength⁷³

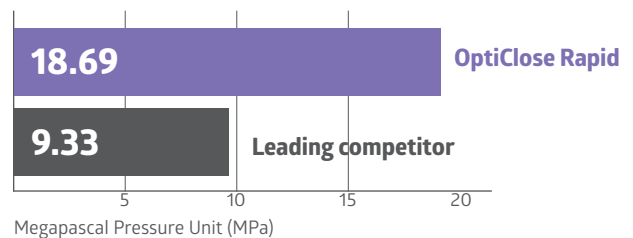


Stronger and more durable to help prevent dehiscence.

OptiClose Essential

- **Low exothermic reaction** means patients who aren't under general anesthesia do not feel a burning sensation
- **Extended application time** for use on multiple incisions without risk of clogging
- **High viscosity** allows for more control during application
- **Versatile foam tip** allows for broad and narrow strokes
- **Elevated ridges** on applicator intuitively position user finger tips for enhanced grip and safety

Delivers more adhesive⁷⁴

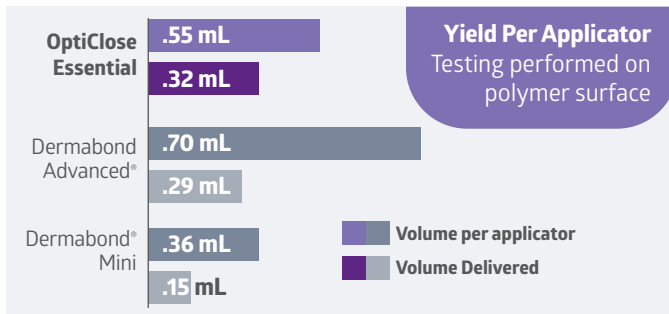

Yields more adhesive per applicator than competitors



OptiClose Secure

- **Mesh strip redistributes tension** away from the wound to the surrounding healthy surface area
- **Significantly greater skin holding strength** than skin staples or sutures
- **Controlled adhesive application** that dries quickly on skin
- **Effective microbial barrier**
- **Water resistant** and shower safe
- **Doesn't stick to post-op dressings** like competitors do
- **Lasts for up to 14 days** on the skin



Dries faster than competitors⁷²



How OptiClose Secure compares

	OptiClose Secure	Leading competitor
Adhesive set time	102 seconds	207 seconds ⁷²
One-hand activation	Yes	No
Activation method	Winged applicator	Pinch and twist activation
Applicator tip	Wide foam tip	Rubber nozzle
Clear mesh alignment guide	Yes	No
Accelerant	In applicator (will dry on or off mesh)	On mesh (will run if adhesive isn't in contact with mesh)
Formula	2-octyl cyanoacrylate	2-octyl cyanoacrylate
Mesh length	22, 44 and 66 cm	22, 42 and 60 cm

About OptiClose 2-octyl cyanoacrylates

- Provide an effective microbial barrier for infection prevention
- Makes a strong, flexible film that provides a durable, long-lasting wound closure
- No need for removal; sloughs off naturally
- Faster than competitors and the traditional methods of wound closure
- Ideal alternative to sutures and staples

Recommended Uses

- OptiClose Rapid and Essential for shorter wounds and incisions
- OptiClose Secure for longer wounds and incisions

Indications

- Acute lacerations and surgical incisions with easily approximated skin edges

Contraindications

- Infected wounds
- Animal bites
- Allergies to cyanoacrylates

Wear Time

- OptiClose Rapid and Essential: 5–10 days
- OptiClose Secure: up to 14 days

Ordering information

OptiClose Rapid

Item No.	Description	Pkg.
MSC091050	0.8 mL	6/bx, 36/cs

OptiClose Essential

Item No.	Description	Pkg.
MSC091040	0.55 mL	12/bx, 72/cs

OptiClose Secure


Item No.	Description	Pkg.
MSC091060	22 cm	2/bx, 12/cs
MSC091070	44 cm	2/bx, 12/cs
MSC091080	66 cm	2/bx, 12/cs



MSC091050



MSC091040

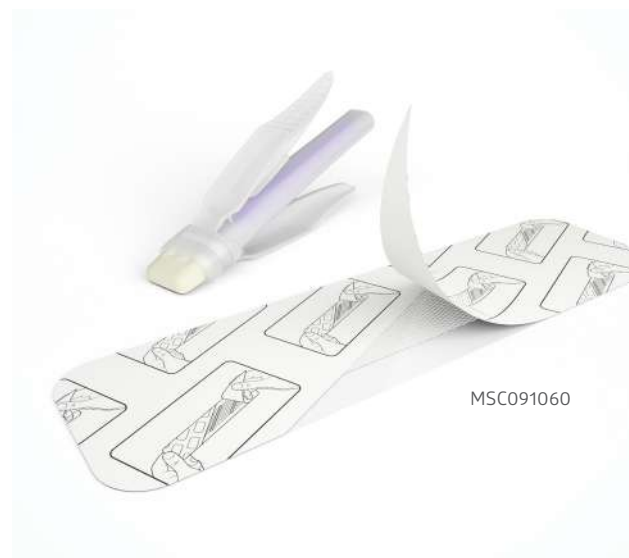


MSC091060

NE1® Wound Assessment Tool


Measure and assess all types of wounds.

About the NE1 Wound Assessment Tool

- One time use tool to assist with wound evaluation upon admission, throughout healing and at discharge
- Easy-to-use colour-matching technique
- Reduces errors and promotes accurate wound assessment
- Standardizes wound documentation
- Free online education

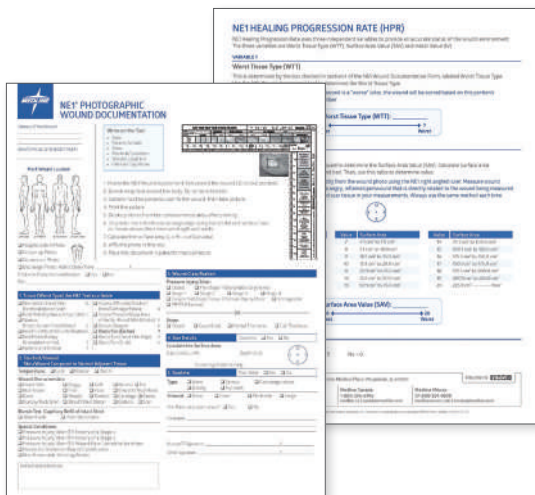


This comprehensive documentation form simplifies the assessment and documentation process. A PDF can be found at www.MedlineNE1.com.

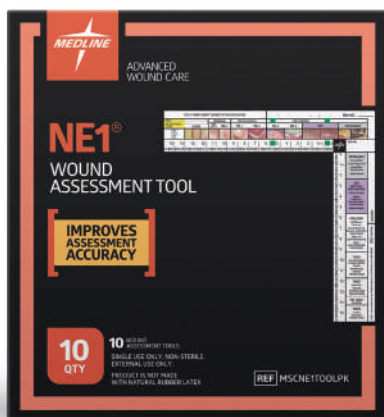

Wound Assessment Study⁷⁴

The study showed a statistically significant improvement in wound assessment. Overall, the clinicians more than doubled their accuracy with the help of the NE1 Wound Assessment Tool.

Clinical highlight

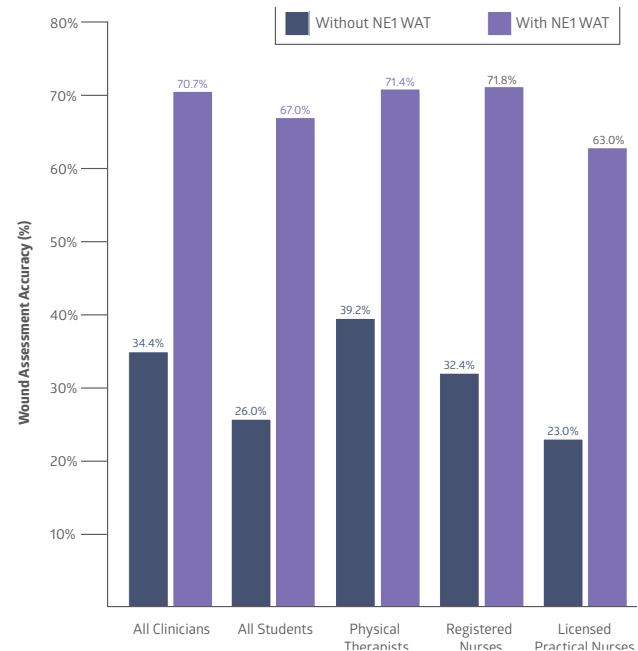

Ordering information

Item No.	Description	Pkg.
MSCNE1TOOL	Wound Assessment Tool	100/bx
MSCNE1TOOLPK	Wound Assessment Tool	10/pk

Educational packaging

Many nurses find themselves in situations on a daily basis where valuable time is wasted searching for the right dressing. Often the wrong dressing is used or the dressing is not used properly.

By listening to you, Medline has found a way to improve this process and ensure that nurses have the information they need. It is called Educational Packaging. The package focuses on patient safety and correct product application. Each package serves as a 2-Minute Course on Wound Care™.



Medline University on HealthStream®

Get instant online access to exclusive clinical education courses covering skin and wound care, including:

- 100+ courses and 20 CE to reinforce best practices
- Skin Health resource center: articles, webinars and patient education
- Clinical guides for caregivers, patients and HME staff to enhance care quality
- Product guides with over 40 competency assessments to ensure proper use of our products

Visit www.medlineuniversity.com for more information.




Skin Health Digital Platform

From prevention to treatment, empower staff to confidently care for skin.

Advance wound healing with our digital platform and improve patient outcomes

Research shows—many caregivers feel they don't have the tools or training they need to provide proper skin and wound care. Make it easier for staff to assess, treat and monitor wounds, and deliver evidence-based care to patients, with our Skin Health digital platform.

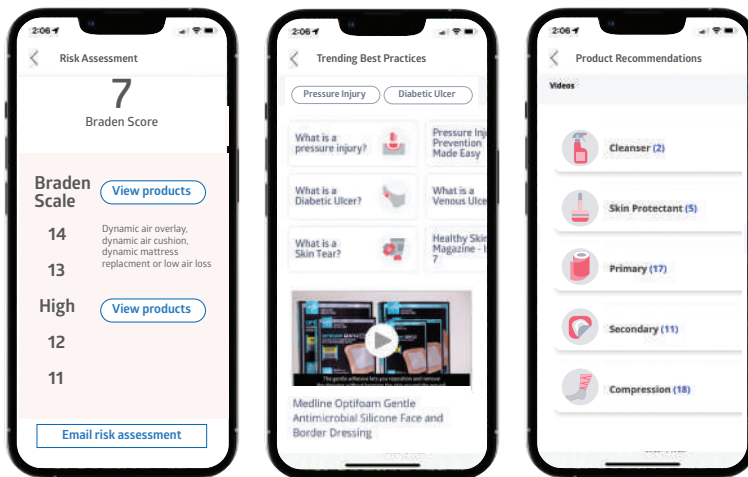

Risk assessment

Predict pressure ulcer risk using Braden or Norton scales.

Education and training

Trending best practices to keep staff up to date, including detailed product information and usage videos.

Product selector

Select the ideal products to help treat patients in a specific clinical context.


Download the app to learn about skin and wound care, repositioning and offloading, nutrition and more—all from your smartphone.



Skin Health Education Program

Become a Skin Health Partner and gain access to education, tools and resources on Skin and Wound Care.

Built on Best Practice Recommendations from Wounds Canada and provincial nursing guidelines, the Skin Health program reinforces evidence-based practice.

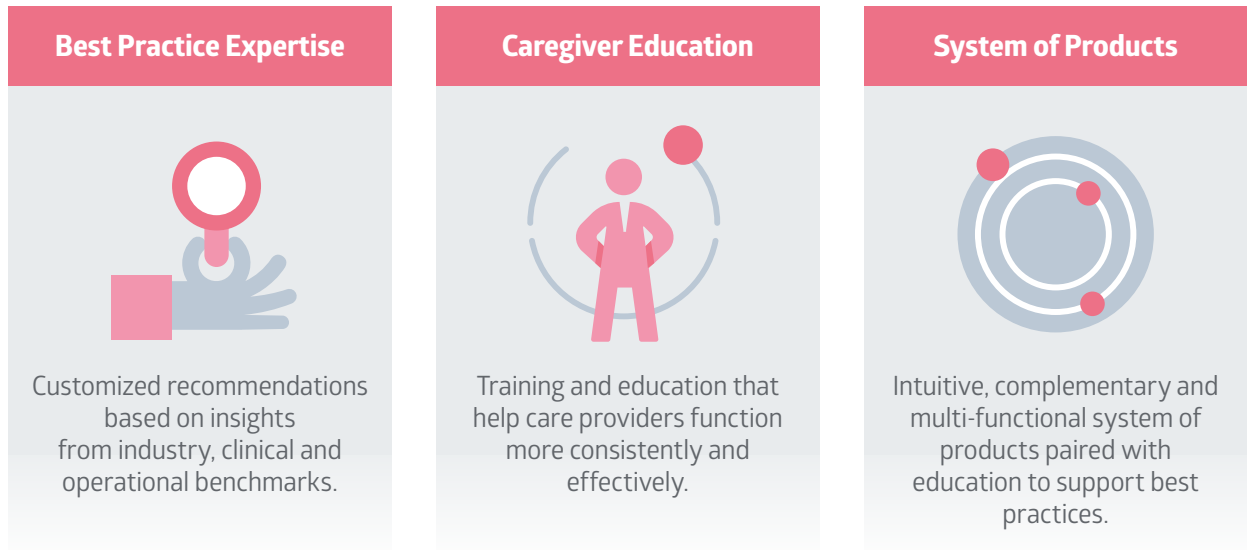

Clinical Support Specialists deliver customized education for Registered Staff and PSWs, reinforcing evidence-based practice. We are here to meet you virtually through Zoom!

Topics include:

- Wound Assessment and Documentation
- Pressure Injury Prevention and Management
- Diabetic Foot Ulcer
- Venous and Arterial Wounds
- Skin Tear Prevention and Management
- Skin Care
- Prevention and Management of Moisture Associated Skin Damage

Additional tools and support:

- Canadian E-learning courses now available through Medline University
- Clinical decision making tools and algorithms
- Wound identification guides

Make skin health second nature
medline.ca/skin-health

Index

AccuWrap™	49	OptiView	31
Bordered Gauze	57	Pinc™ Tape	58
Cover Dressings	57	PluroGel®	7
Dressing Retention Tapes	58	PrimaSeal® Post Op Ag+	24
Education and support	64	Product selector app	65
Educational packaging	64	Product support	64
Exuderm® family	39	Puracol®	42
FourFlex®	52	PureWash™	59
Gentac® EZ Silicone Tape	58	Qwick™	37
Gentac® Silicone Island Dressing	56	SilvaSorb®	25
Hydrogel Dressings	40	Skin Health Digital Platform	65
IoPlex®	13	Skin Health Hotline	66
Marathon®	45	Skintegrity® Eco Hydrogel	40
Maxorb® family	35	Skintegrity® Hydrogel Sheet	40
Maxorb® Ag+ family	19	Skintegrity® Wound Cleanser	59
Medfix Tapes	58	Stratasorb®	57
Medigrip™ and Medigrip LF	54	Sureprep®	48
Medline University	64	Tapes	58
NE1® Wound Assessment Tool	63	TheraHoney®	10
Opticell®	34	ThreeFlex®	52
Opticell® Ag+	16	TwoFlex®	51
OptiClose™	60	Unna-Z™/Unna-Z™ Stretch	53
Optifoam®	30	Versatel®/Versatel® One	55
Optifoam® Ag+	20	Wound Cleansers	59
Optifoam® Ag+ Post Op	22		
Optifoam® Gentle	27		
Optifoam® Gentle Ag+	21		
Optifoam® Gentle Ag+ Post Op	23		
OptiLock®	36		

References

PluroGel

1. Birchenough, S. A., Rodeheaver, G. T., Morgan, R. F., Peirce, S. M., & Katz, A. J. (2008). Topical poloxamer-188 improves blood flow following thermal injury in rat mesenteric microvasculature. *Annals of Plastic Surgery*, 60(5): 584-8. doi: 10.1097/SAP.0b013e3181651661.
2. Kim CK, Lee SW, Choi HG, et al. Trials of in situ gelling and mucoadhesive acetaminophen liquid suppository in human subjects. *International Journal of Pharmaceutics*. 1998;174(1-2):201-207.
3. Ratliff CR. Management of a groin wound using a concentrated surfactant-based gel dressing. *J Wound Ostomy Continence Nursing*. 2018;45(5):465-467.
4. Data on file. Caroline Fife, MD, FAAFP, CWS. Medline owns the rights to the photos in this case study.

Therahoney

5. Data on file.
6. Cimolai, N. Sweet success? Honey as a topical wound dressing *BCM J*. Vol. 49, No. 2, March 2007: 64-67.
7. Green AE. Wound healing properties of honey. *Br J Surg* 1988; 75(12): 1278.
8. Molan P. Debridement of Wounds with Honey. *J Wound Technology* 2009: 12-17.
9. Tovey FI. Honey and healing. *J R Soc Med* 1991; 84(7): 447.
10. Schultz, G; Gibson, D. "The Impact of Silver Impregnated Dressings and Wound Cleansers on the Osmotic Strength of 100% Manuka Honey Gel Dressing". Department of Obstetrics and Gynecology; Institute of Wound Research; University of Florida; Gainesville, FL. 2013.

IoPlex

11. Attinger C and Wolcott R. Clinically addressing biofilm in chronic wounds. *Advances in Wound Care*. 2012;1(3):127-132. Available at: <https://www.ncbi.nlm.nih.gov/pmc/articles/PMC3839004/>. Accessed September 16, 2019.
12. Phillips PL, Yang Q, Sampson E, Schultz G. Effects of antimicrobial agents on an in vitro biofilm model of skin wounds. *Advances in Wound Care*. 2010; 1:299-304.
13. Hill KE, Malic S, McKee R, Rennison T, Harding KG, et al. An in vitro model of chronic wound biofilms to test wound dressings and assess antimicrobial susceptibilities. *Journal of Antimicrobial Chemotherapy*. 2010; 65(6):195-206.
14. Thorn RMS, Austin AJ, Greenman J, Wilkins JPG, Davis PJ. In vitro comparison of antimicrobial activity of iodine and silver dressings against biofilms. *Journal of Wound Care*. 2009;18(8):343-346.
15. Phillips PL, Yang Q, Davis S, Sampson EM, Azeke JI, et al. Antimicrobial dressing efficacy against mature *Pseudomonas aeruginosa* biofilm on porcine skin explants. *International Wound Journal*. 2015; 12(4):469-483.
16. Schultz G, Bjarnsholt T, James GA, Leaper DJ, McBain AJ, Malone M, et al. Consensus guidelines for the identification and treatment of biofilms in chronic nonhealing wounds. *Wound Repair and Regeneration*. 2017;25(5):744-757. Available at: <https://onlinelibrary.wiley.com/doi/pdf/10.1111/wrr.12590>. Accessed September 16, 2019.
17. Sibbald RG, Leaper DJ, Queen D. Iodine made easy. *Wounds International*. 2011;2(2). Available at: <https://www.woundsinternational.com/resources/details/iodine-made-easy>. Accessed September 16, 2019.
18. Data on file.

Opticell Ag+

19. Data on file.

Maxorb Ag+

20. Yang Q, et al. Antibacterial and Flow Rate Assessment of a Nonadherent Layer Interface on a Silver Alginate Dressing. *Wounds*. 2015;27(1):1-4.
21. Chakravarthy D, Falconio-West M. A comparison of the antimicrobial efficacy of two silver ion containing absorptive fiber dressings. Presented at SAWC. Las Vegas, NV. 2006.
22. Chakravarthy D, Fleck C, Falconio-West M. An evaluation of two polysaccharide-silver based high absorbency wound dressings. Presented at SAWC. Las Vegas, NV. 2006.
23. Data on file.
24. DeLeon J, Nagel M, Hill D, Fudge M, Lucius A, Garcia B. Cost comparison of silver impregnated Hydrofiber® and alginate dressings. Presented at SAWC. Las Vegas, NV. 2006.
25. Chakravarthy D, Freeman R, Beele H, Meuleneire F, Nahuys M. Detailed clinical report: prospective randomized open label pilot study to evaluate a new silver alginate antimicrobial wound dressing on critically colonized chronic wounds. Internal report on file.
26. Freeman R, Beele H, Meuleneire F, Nahuys M. Results of a multicentre randomized prospective open label pilot study to evaluate a new silver alginate antimicrobial wound dressing on critically colonized chronic wounds. Presented at Wounds UK Conference. Harrogate. 2006.
27. Serena T, Chakravarthy D. Maxorb Ag in the treatment of burn wounds. Symposium on Advanced Wound Care. Tampa, FL. 2007.

Optifoam/Optifoam Gentle Ag+

28. Data on file.
29. Percival, S.L., et al. Antimicrobial Efficacy of a Silver Impregnated Hydrophilic PU Foam. *Surgical Technology International*. Vol. 32.

Optifoam/Optifoam Gentle Ag+ Post-Op

- 30.-31. Data on file.

PrimaSeal Post-Op Ag+

- 32.-33. Data on file.

SilvaSorb

34. Data on file.
35. Nametka M. Adventist Medical Center, Portland OR, Study Site Avamere LTC Facility, Portland OR. Presented at the Annual Symposium on Advanced Wound Care, 2002.
36. Castellano JJ, Shafil SM, Ko F, Donate G, Wright TE, Mannari RJ, Payne WG, Smith DJ, Robson MC. Comparative evaluation of silver containing antimicrobial dressings and drugs. *Int Wound Journal*. 4: 114-122.
37. Nametka M. Silver antimicrobial hydrophilic dressing benefits management of recurrent non-healing wounds. Symposium on Advanced Wound Care. Baltimore. 2002.
38. Nametka M. A hydrophilic silver antimicrobial wound dressing for site preparation and maintenance of human skin equivalent grafts to venous leg ulcers: Technical and clinical considerations. Clinical Symposium on Advances in Skin & Wound Care, Nashville. 2000.
39. Nametka M. Silver antimicrobial absorbent wound dressing can contribute to cost control in home care. Annual Clinical Symposium on Advances in Skin and Wound Care. Dallas. 2002.

Optifoam Gentle/Optifoam

40. Butcher M., Thompson G. Can the use of dressing materials actually prevent pressure ulcers; presenting the evidence. *Wounds UK*. 2010; 6(1): 119-125.

41.–42. Data on file.

OptiView

43. Testing data on file.

Opticell

44. Lab testing data on file.

45. Data on file.

Maxorb

46. Data on file.

OptiLock

47.–48. Data on file.

Qwick

49. Data on file.

Exuderm

50. Data on file.

Skintegrity

51. Independent cytotoxicity study performed by Thomas J. Stephens and Associates, Inc. Study Number 93-0070 (AOL).

Puracol

52. Böhm S, Strauß C, Stoiber S, Kasper C, Charwat V. Impact of Source and Manufacturing of Collagen Matrices on Fibroblast Cell Growth and Platelet Aggregation. *Materials* (Basel). 2017 Sep 15;10(9):1086. doi: 10.3390/ma10091086. PMID: 28914792; PMCID: PMC5615740.

53. Data on file.

54. Shah SS, Chakravarthy D. Evaluation of a bovine 100% native collagen for the treatment of chronic wounds. *J Wound Ostomy Continence Nursing*. 2015;42(3):226-234.

55. Picrosirius Assay to Determine Relative Nativity of Two Collagenous Dressings, internal report.

56. Comparative Physical Properties of Two Collagenous Dressings, Promogran® and Puracol Plus, data on file.

57. Scott, R; Chakravarthy, D. "The use of a 100% native MicroScaffold™ Collagen in conjunction with NPWT therapy". LifeCare Hospitals of Plano; Plano, TX. Presented at SAWC Fall Course, Baltimore, MD, September 2012.

Marathon

58. Study to Compare the Wash-off Resistance of Two Barrier Films Exposed to Synthetic Urine. Data on file.

59. Data on file.

Sureprep

60. Shannon RJ, Chakravarthy D. Effect of a water-based no-sting, protective barrier formulation and a solvent containing similar formulation on skin protection from medical adhesive trauma. *Int Wound J*. 2009 Feb;6(1):82-8.

61. Chakravarthy D, Falconio-West M. A Randomized, Controlled Trial of Two Sting-Free Polymeric Skin Barrier Products, One Water Based, the Other Solvent Based. Presented at Clinical Symposium on Advances in Skin and Wound Care. Nashville, TN. 2007.

62. Test data on file (independent lab).

63. 510(k) K051082, WOVE, 2005.

64. Data on file.

AccuWrap

65. Probst S, Allet L, Depeyre J, Colin S, Skinner MB. A targeted interprofessional educational intervention to address therapeutic adherence of venous leg ulcer persons (TIEIVLU): study protocol for a randomized controlled trial. *Trials*. 2019;20:243. Available at: <https://trialsjournal.biomedcentral.com/articles/10.1186/s13063-019-3333-4>. Accessed November 13, 2019.

66. Finlayson K, Edwards K, et al. (2010). The Impact of psychosocial factors on adherence to compression therapy to prevent recurrence of venous leg ulcers. *J. Clin Nurs*. 19(9-10): 1289-97.

67. Data on file.

ThreeFlex/FourFlex

68. Data on file: Compression values as measured at the ankle upon application, application per manufacturer's guidelines.

Versatel

69. Independent laboratory testing. Test reports on file.

Wound Cleansers

70. Hubner, N.O., and A. Kramer. "Review on the Efficacy, Safety and Clinical Applications of Polihexanide, a Modern Wound Antiseptic." *Skin Pharmacol Physiol Skin Pharmacology and Physiology* 23.1 (2010): 17-27.

71. International Wound Infection Institute (IWII) Therapeutic wound and skin cleansing: Clinical evidence and recommendations. Wounds International. 2025.

OptiClose

72. The-DERMABOND-Family-of-Topical-Skin-Adhesives information sheet (039696_171101).

73.–74. Data on file.

NE1 Wound Assessment Tool

75. Young DL, Estocado N, Landers MR, Black J. A Pilot Study Providing Evidence for the Validity of a New Tool to Improve Assignment of National Pressure Ulcer Advisory Panel Stage to Pressure Ulcers. *Advances in Skin & Wound Care*. April 2011; (24)4:168-75.

Make skin health second nature

Empower your frontline staff to confidently manage and treat wounds using best-in-class products.

Discover these tools and resources to help standardize care and reduce skin breakdown



Best Practice Guidance

Customized recommendations based on insights and analysis to help reduce variation.



Education and Training

Resources that fit every learning style to help staff work more consistently and effectively.



System of Products

Intuitively designed products paired with utilization recommendations to support best practices.

For more information
contact your Medline
Representative or visit
www.medline.ca



**We make
healthcare
run better®**

Medline Canada, Corporation | 1-800-268-2848 | medline.ca

©2026 Medline. All rights reserved. MEDLINE is a registered trademark of Medline Industries, LP. All product names, regardless of whether or not they appear with a trademark symbol, are the property of their respective companies unless otherwise noted.

MKT19W8101742 / eCA268 / 96