

# Cold Therapy Ankle Brace

## INSTRUCTIONS SHEET:

### FEATURES AND BENEFITS:

- Quick ankle wrap orthosis design with removable ice pack. (2 included)
- A quick inflation system that can improve outcomes by managing pain and swelling.

### CLEANING INSTRUCTIONS:

- Hand wash with mild detergent.
- Rinse and air dry.
- Do NOT use bleach.



## INSTRUCTIONS FOR USE:

1. Check packaging first for ankle brace/gel pack/pump with tube.
2. Freeze gel pack at least 2 hours before use.
3. Put gel pack back inside the brace after freezing.
4. Connect the tube to the ON/OFF switch and turn the switch to "ON".
5. Put the device on the affected ankle.
6. Stick the hook and loop fasteners together on the appropriate position and fix in place.
7. When turning the ON/OFF switch, please hold it with one hand and turn it with the other hand to ensure the switch is on the appropriate position, then pump.
8. Release the pressure before taking it off and be sure to freeze the gel pack so it is ready for the next use.

### WARNING:

The Cold Therapy Ankle Brace is designed for support and does not guarantee injury prevention. By using this product, consumers or patients acknowledge and agree to the following:

Innovative DME is not responsible for any injuries that may occur during the use of Innovative products.

Skin irritation is a potential side effect when using this device. If you experience an unusual skin reaction, it is essential to consult with your healthcare provider immediately.

Always consult your healthcare provider to ensure that this product is suitable for your specific medical condition and needs.

This product is intended for use by a single patient only.

Federal Law restricts the sale or distribution of this device to be carried out by or under the order of a licensed healthcare practitioner.

If you do not agree with these conditions, please contact your healthcare provider before using this product.

