

# INFRATens Red Light EMS Device

## INDICATIONS FOR USE:

- For temporary relief of pain associated with sore and aching muscles due to strain from exercise or normal household and work activities.
- To stimulate healthy muscles in order to improve and facilitate muscle performance.


## CLEANING AND MAINTENANCE:

- Use lightly damp cloth to clean device and then use a dry cloth to dry.
- Electrode pads are disposable and should be replaced once they lose their adhesiveness.



## INSTRUCTIONS FOR USE:

**Note:** Thoroughly clean the skin before each application. The pad may not adhere properly if lotion, makeup, natural oils, or dirt are present on the skin. Ensure the device is fully charged and powered off before attaching the electrode.

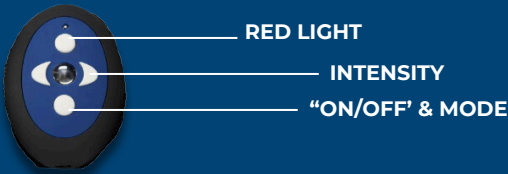

1. Securely snap the electrode onto the device.
2. Position the device with the electrode pad attached on the desired area.
3. Hold the "ON/OFF" button to turn on the power.
4. Select one of the stimulation modes by pressing the "ON/OFF" button. (there are two modes)
5. Turn on the red light therapy by pressing the red light button if desired.
6. Adjust the stimulation intensity by using the "+" and "-" buttons.
7. To turn off the device again, hold the "ON/OFF" when you're done.

## WARNING:

The INFRATens is for single patient use only and is designed for support and does not guarantee injury prevention. By using this product, consumers or patients acknowledge and agree to the following:

Innovative DME is not responsible for any injuries that may occur during the use of Innovative products.

If you experience increased pain, swelling, or any adverse reactions while using this product, immediately consult with your healthcare provider.

Always consult your healthcare provider to ensure that this product is suitable for your specific medical condition and needs.

This product is intended for use by a single patient only.

Federal Law restricts the sale or distribution of this device to be carried out by or under the order of a licensed healthcare practitioner.

If you do not agree with these conditions, please contact your healthcare provider before using this product.