

Foam Padded Shoulder Abduction Sling

INNOVATIVE
DME

The shoulder abduction sling combines convenience and craftsmanship with a breathable mesh and a 15-degree abduction pillow. Enhanced with adjustable straps and waist loops for safety and user-friendliness. It also includes an exercise handball for circulation and purposeful exercise.

INDICATION:


- Soft tissue repairs/strains.
- Shoulder or arm instabilities/injuries.

CLEANING INSTRUCTIONS:

- Hand wash straps and slings with mild detergent.
- Rinse and air dry.
- Do NOT use bleach.



INSTRUCTIONS FOR USE:

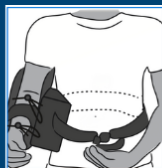

1. Slide the forearm into the sling. Wrap the adjustable shoulder strap around the neck and shoulders, and attach the strap end to the sling using the "alligator strap tab."
2. Adjust the shoulder strap to the required length.
3. Position the shoulder pad behind the neck. To secure the shoulder pad, unfold the hook material on the top of the pad, then press the shoulder strap back onto the hook material to secure the pad in place.
4. Attach the closure strap across the open top of the sling. Position the strap so that it holds the arm securely in the sling. Next, attach the thumb strap to the open end of the sling between the thumb and fingers.
5. After sling has been fit, it may be easily removed and reapplied using the quick release buckle on the shoulder strap.
6. Place the pillow at the waistline. Attach the sling to the pillow, aligning the hook material on the pillow with the loop on the sling.
7. If waist strap is too long, cut it to fit. Use the small piece of double sided hook material to secure the strap end. Place the double sided hook material on the inside of the cut strap end and secure it to the waist strap.

WARNING:

This Foam Padded Shoulder Abduction Sling is for single patient use only and is designed for support and does not guarantee injury prevention. By using this product, consumers or patients acknowledge and agree to the following:

Innovative DME is not responsible for any injuries that may occur during the use of Innovative products.

If you experience increased pain, swelling, or any adverse reactions while using this product, immediately consult with your healthcare provider.

Always consult your healthcare provider to ensure that this product is suitable for your specific medical condition and needs.

This product is intended for use by a single patient only.

Federal Law restricts the sale or distribution of this device to be carried out by or under the order of a licensed healthcare practitioner.

If you do not agree with these conditions, please contact your healthcare provider before using this product.



sales@innovatedme.com



(352) 587-7613



16034 US 19 Hudson, FL 34667