

Cold Therapy ROM Hip Brace

INSTRUCTIONS FOR USE:


QUICK FITTING, UNIVERSAL, HIP RANGE-OF-MOTION ORTHOSIS FITTING INSTRUCTIONS.

The brace is equipped with range of motion hinges that enable flexion and extension control. It features a user-friendly quick inflation system, which can enhance results by effectively managing pain and reducing swelling. Additionally, this hip brace comes with two high-quality ice packs.



CLEANING INSTRUCTIONS:

- Hand wash with mild detergent.
- Rinse and air dry.
- Do NOT use bleach.

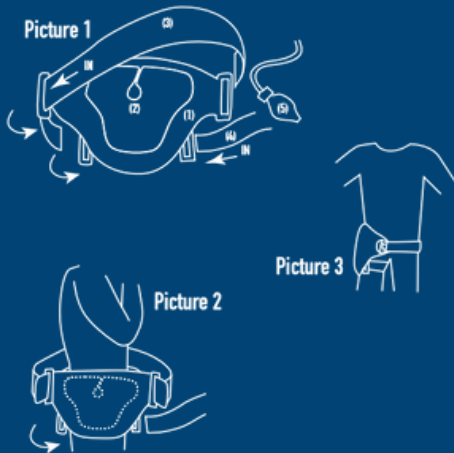

1. CHECK PACKAGING FIRST FOR BRACE, GEL PACK, AND PUMP WITH TUBE.
2. FREEZE THE GEL PACK FOR AT LEAST FOR 2 HOURS BEFORE USE.
3. PLACE GEL PACK BACK INSIDE THE BRACE AFTER FREEZING.
4. CONNECT THE TUBE TO THE VALVE AND TURN THE SWITCH TO "ON".
5. POSITION HIP INSIDE THE BRACE AND WRAP THE STRAPS OVER USING THE HOOK AND LOOP TABS TO SECURE IN PLACE (View images).
6. PUMP THE AIR BLADDER TO DESIRED COMPRESSION AND TURN THE VALVE TO "OFF".
7. RELEASE THE PRESSURE BEFORE REMOVING BRACE AND BE SURE TO FREEZE THE GEL PACK FOR NEXT USE.

WARNING:

This Cold Therapy ROM Hip Brace is designed for support and does not guarantee injury prevention. By using this product, consumers or patients acknowledge and agree to the following:

Innovative DME is not responsible for any injuries that may occur during the use of Innovative products.

Skin irritation is a potential side effect when using this device. If you experience an unusual skin reaction, it is essential to consult with your healthcare provider immediately.

Always consult your healthcare provider to ensure that this product is suitable for your specific medical condition and needs.

This product is intended for use by a single patient only.

Federal Law restricts the sale or distribution of this device to be carried out by or under the order of a licensed healthcare practitioner.

If you do not agree with these conditions, please contact your healthcare provider before using this product.

